


Rogue2 Transit

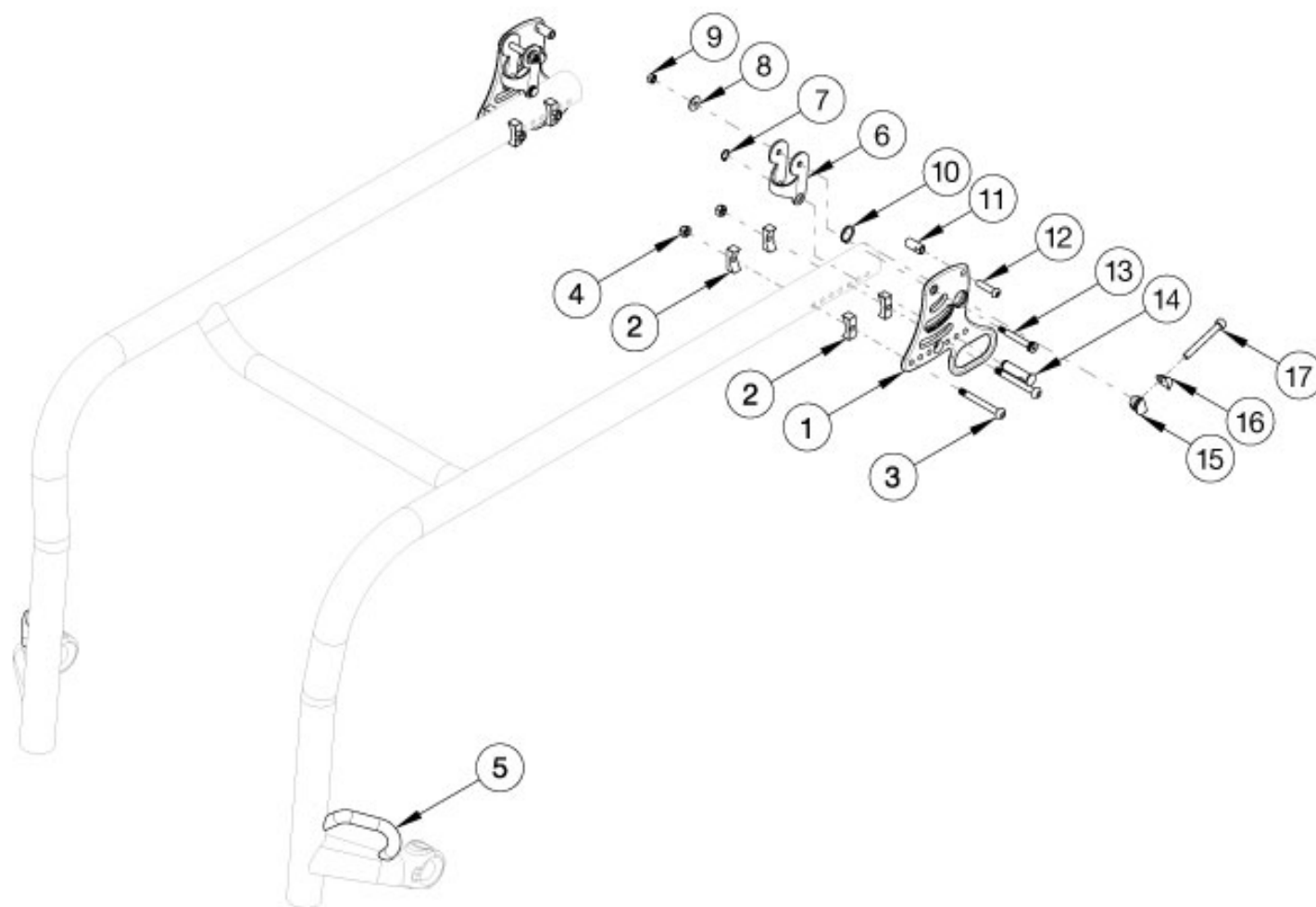

For further backrest hardware, please see the "Rogue2 Backrest Mount and Hardware" parts manual page. Rogue2 transit cannot be retrofitted. If needing to add transit, a new frame will be required through the Rogue2 Replacement Frame Kio configurator.

Item Number	Part Number	Description	Quantity
1a, 2-4, 6-17	117598	Transit Back Plate with Hardware, Black, Right Side, Rogue2	1
1b, 2-4, 6-17	117599	Transit Back Plate with Hardware, Black, Left Side, Rogue2	1
1a, 2-4, 6-17	117598_ _ _	Transit Back Plate with Hardware, Color Accent, Right Side, Rogue2	1
1b, 2-4, 6-17	117599_ _ _	Transit Back Plate with Hardware, Color Accent, Left Side, Rogue2	1
1a	115552	Back Plate, Transit, Right, Black	1
1a	115552_ _ _	Color Accent Back Plate, Transit, Right	1
1b	115553	Back Plate, Transit, Left, Black	1
1b	115553_ _ _	Color Accent Back Plate, Transit, Left	1

Item Number	Part Number	Description	Quantity
2	116329	Rigid Backrest Saddle, Narrow, 1.375"	4
3a	003232	M6 x 55 BHCS Blk Zc	2
3b	003241	M6 x 60 BHCS P/THD Custom Blk Zc <i>The part is required if the chair has swing away armrests</i>	2
4	100658	M6 Nylock Nut Blk Zn	2
7	115570	Eternal Retaining Ring, 3/8"	1
8	103022	M5 Washer, Blk Zn	1
9	103283	M5 Top Lock Nut BZ	1
10	115571	Eternal Retaining Ring, 5/8"	1
11	115546	Strike, Ø.364	1
12a	115573	M6 x 25, BHCS, CLS 10.9, NYL, BLZ	1
12b	115574	M6 x 30 BHCS, CLS 10.9, NYL, BLZ <i>The part is required if the chair has swing away armrests</i>	2
13	103284	M6 x 35 Shoulder Bolt BZ	1