


## Catalyst 4 Axle Bracket

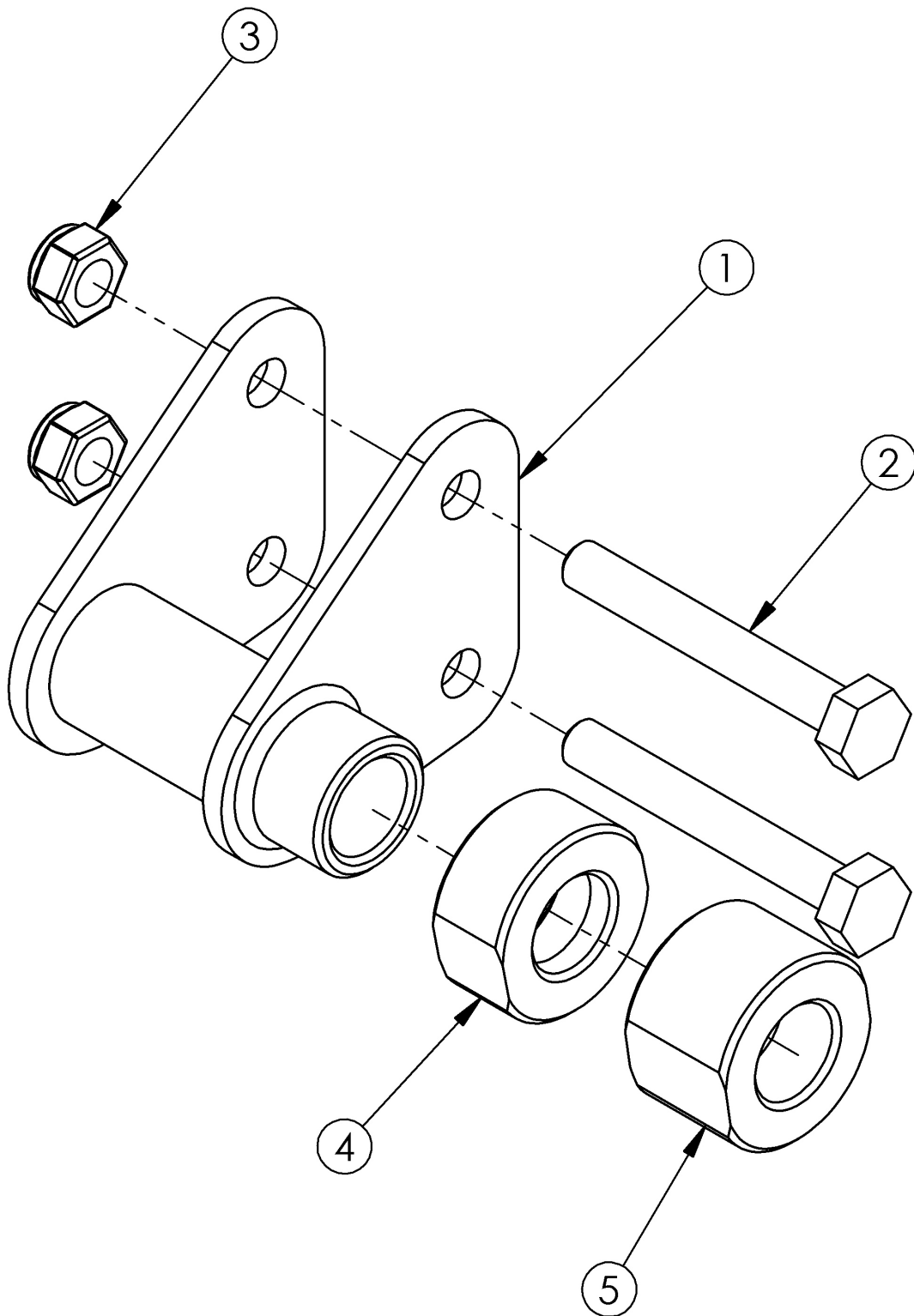

See Catalyst Anti-Tip Receivers page when reverse mounting Axle Bracket for all hardware used. See Axles and Receivers reference for axle bracket usage

| Item Number | Part Number | Description                                                                                       | Quantity |
|-------------|-------------|---------------------------------------------------------------------------------------------------|----------|
| 1a          | 000052R     | Axle Bracket Offset Right                                                                         | 1        |
| 1b          | 000052L     | Axle Bracket Offset Left                                                                          | 1        |
| 1c          | 000281R     | Extended Axle Bracket Offset Right                                                                | 1        |
| 1d          | 000281L     | Extended Axle Bracket Offset Left                                                                 | 1        |
| 1e          | 114491      | CAT 4 Axle Bracket, Right<br><i>Used when MAXX Mag wheels or Anti-Rollback System are present</i> | 1        |
| 1f          | 114492      | CAT 4 Axle Bracket, Left<br><i>Used when MAXX Mag wheels or Anti-Rollback System are present</i>  | 1        |
| 2a          | 100674      | M6 x 45 CLS10.9 DIN931 HHCS ZC<br><i>Used for Axle Plate only</i>                                 | 2        |
| 2b          | 101826      | M6 x 50 HHCS Blk Zn<br><i>Used for Anti-Tip or Swing Away Arm Bracket</i>                         | 2        |
| 2c          | 100676      | M6 x 55 CLS10.9 DIN931 HHCS ZC<br><i>Used for Anti-Tip and Swing Away Arm Bracket</i>             | 2        |
| 3           | 100658      | M6 Nylock Nut Blk Zn                                                                              | 2        |
| 4           | 101891      | Axle Spacer .15" CAT4<br><i>Used as needed to increase rear wheel spacing</i>                     | 1        |
| 5           | 101892      | Axle Spacer .35" CAT4<br><i>Used as needed to increase rear wheel spacing</i>                     | 1        |

## **Catalyst Axles and Receivers**

To find the appropriate axle, first establish the axle receiver according to the RECEIVERS section and then determine the axle using the AXLES section. On Catalyst 4 chairs, new axles and receivers may be required for proper fit.

### **RECEIVERS**

#### **Catalyst 4**

Receiver/Axle Bracket: 000052L and 000052R

Description: Axle Bracket Offset

Used On: Catalyst 4, 4C, 4TTL with no armrests, with X Core mag and Topolino wheels or with Superlight, Spinergy and 6-spoke mag wheels when standard flip back or swing away armrests are used.

Receiver/Axle Bracket: 000281L and 000281R

Description: Extended Axle Bracket Offset

Used On: Catalyst 4, 4C, 4TTL with Superlight, Spinergy and 6-spoke mag wheels when height adjustable armrests are used or with Ki Spoke and MAXX Performance Spoke Wheels when any armrest is used.

Receiver/Axle Bracket: 114491 and 114492

Description: CAT 4 Axle Bracket MAXX Mag

Used With: MAXX Mag wheels.

#### **Catalyst 5 / Spark**

Receiver: 100756

Description: Axle Receiver ½"

Used On: Catalyst 5, 5TTL, 5Ti, 5Vx, E, Spark when MAXX Mag wheels are not present.

Receiver: 114480

Description: CAT Axle Receiver 2.5"

Used On: Catalyst 5, 5TTL, 5Ti, 5Vx, E, Spark when MAXX Mag wheels are present.

### **AXLES**

000052L/R, 100756, and 114480 Receivers

Assembled 6-Spoke Mag – 100901 (Used with Quad Quick Release as well)

X Core Mag, MAXX Mag – 101191

All other wheels – 101108

Quad Quick Release (All wheels except 6-Spoke Mag) – 100755

000281L/R Receivers (not available with X Core Mag wheels)

6-Spoke Mag – 102432 (used with Quad Quick Release as well)

All other wheels – 101182

Quad Quick Release (All wheels except 6-Spoke Mag) – 100901

114491/114492 Receivers

MAXX Mag wheels - 101191