



# Catalyst / Focus / Flip Luggage Carrier

Frame Mount

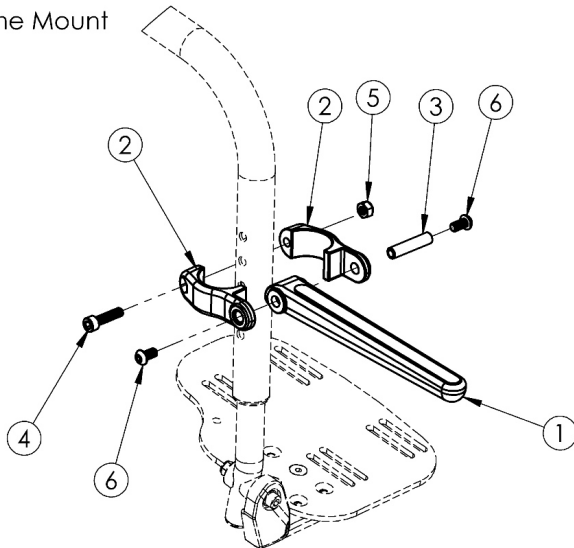

Extension Mount

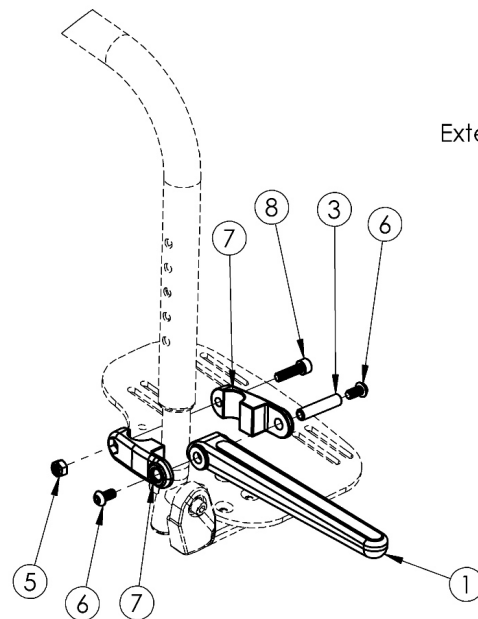

Item Number	Part Number	Description	Quantity
1 - 8	504464	Luggage Carrier Kit, Catalyst/Tilt	1
1	003430	Luggage Carrier Fork Assembly	2
2	003410	Luggage Carrier Clamp 1.00 <i>Used to mount Luggage Carrier to the Hangers</i>	4
3	003413	Luggage Carrier Barrel	2
4	100780	M6 x 25 SHCS ZC <i>Only used when Luggage Carrier is mounted to the Hangers</i>	2
5	100658	M6 Nylock Nut Blk Zn	2
6	100667	M6 x 12 BTN SHCS BLKZC w/Patch	4
7	003409	Luggage Carrier Clamp .75 <i>Used to mount Luggage Carrier to the Extension Tubes</i>	4
8	101845	M6 x 22 SHCS Blk Zn <i>Only used when Luggage Carrier is mounted to the Extension Tubes</i>	2