


Arc Single Hand Tilt Mechanism

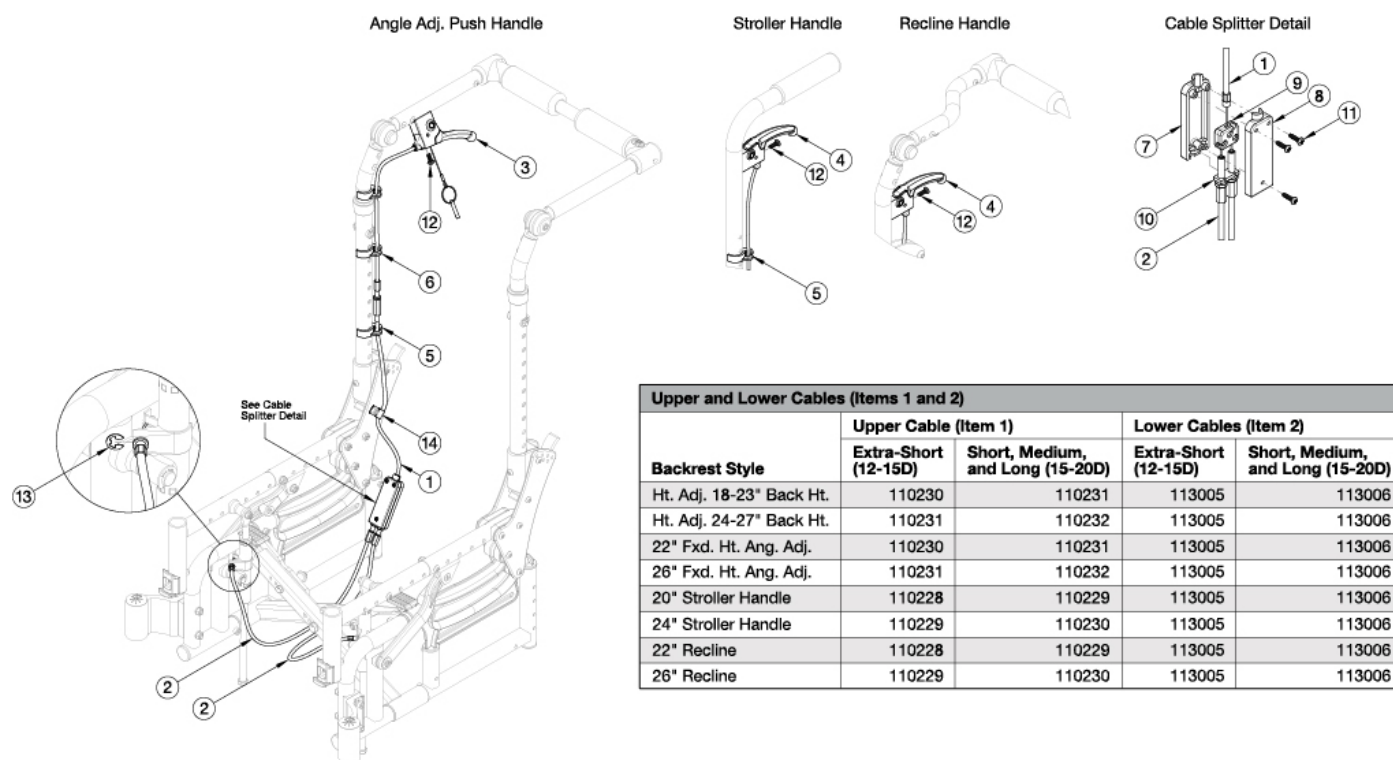

Please note kits 113005 and 113006 contain two cables and two e-rings.

Item Number	Part Number	Description	Quantity
1c, 2a, 3, 5-14	113010	Single Hand Tilt, Height Adjustable, 18-23" Height, Extra Short Depth Range, Arc	1
1d, 2b, 3, 5-14	113011	Single Hand Tilt, Height Adjustable, 18-23" Height, Short/Med/Long Depth Range, Arc	1
1d, 2a, 3, 5-14	113012	Single Hand Tilt, Height Adjustable, 24-27" Height, Extra Short Depth Range, Arc	1
1e, 2b, 3, 5-14	113013	Single Hand Tilt, Height Adjustable, 24-27" Height, Short/Med/Long Depth Range, Arc	1
1c, 2a, 3, 5, 7-14	113014	Single Hand Tilt, Fixed Height, 22" Height, Extra Short Depth Range, Arc	1
1d, 2b, 3, 5, 7-14	113015	Single Hand Tilt, Fixed Height, 22" Height, Short/Med/Long Depth Range, Arc	1
1d, 2a, 3, 5, 7-14	113016	Single Hand Tilt, Fixed Height, 26" Height, Extra Short Depth Range, Arc	1
1e, 2b, 3, 5, 7-14	113017	Single Hand Tilt, Fixed Height, 26" Height, Short/Med/Long Depth Range, Arc	1
1a, 2a, 4, 5, 7-14	113018	Single Hand Tilt, Stroller Handle, 20" Height, Extra Short Depth Range, Arc	1
1b, 2b, 4, 5, 7-14	113019	Single Hand Tilt, Stroller Handle, 20" Height, Short/Med/Long Depth Range, Arc	1
1b, 2a, 4, 5, 7-14	113020	Single Hand Tilt, Stroller Handle, 24" Height, Extra Short Depth Range, Arc	1

Item Number	Part Number	Description	Quantity
1c, 2b, 4, 5, 7-14	113021	Single Hand Tilt, Stroller Handle, 24" Height, Short/Med/Long Depth Range, Arc	1
1a, 2a, 4, 5, 7-14	113022	Single Hand Tilt, Reclining, 22" Height, Extra Short Depth Range, Arc	1
1b, 2b, 4, 5, 7-14	113023	Single Hand Tilt, Reclining, 22" Height, Short/Med/Long Depth Range, Arc	1
1b, 2a, 4, 5, 7-14	113024	Single Hand Tilt, Reclining, 26" Height, Extra Short Depth Range, Arc	1
1c, 2b, 4, 5, 7-14	113025	Single Hand Tilt, Reclining, 26" Height, Short/Med/Long Depth Range, Arc	1
1a	110228	Cable, Splitter, Upper, Short	1
1b	110229	Cable, Splitter, Upper, Medium	1
1c	110230	Cable, Splitter, Upper, Long	1
1d	110231	Cable, Splitter, Upper, Extra Long	1
1e	110232	Cable, Splitter, Upper, Extra Extra Long	1
2a, 13	113005	Cable, Tilt, Split, Medium, Arc	1
2b, 13	113006	Cable, Tilt, Split, Long, Arc	1
3	002801	Lever Assembly Horizontal Tilt	1
4	002800	Lever Assembly Vertical Tilt	1
5	001117	Cable Clip 1"	2
6	001118	Cable Clip 7/8"	2
7-11	113009	Cable Splitter Assembly, Arc	1
12	100734	M5 x 16 PHMS ZC	1
13	102822	E-Ring, M6, YZ	2
14	001239	P-Clip 3/16"	1