


Catalyst Cane and Crutch Holder

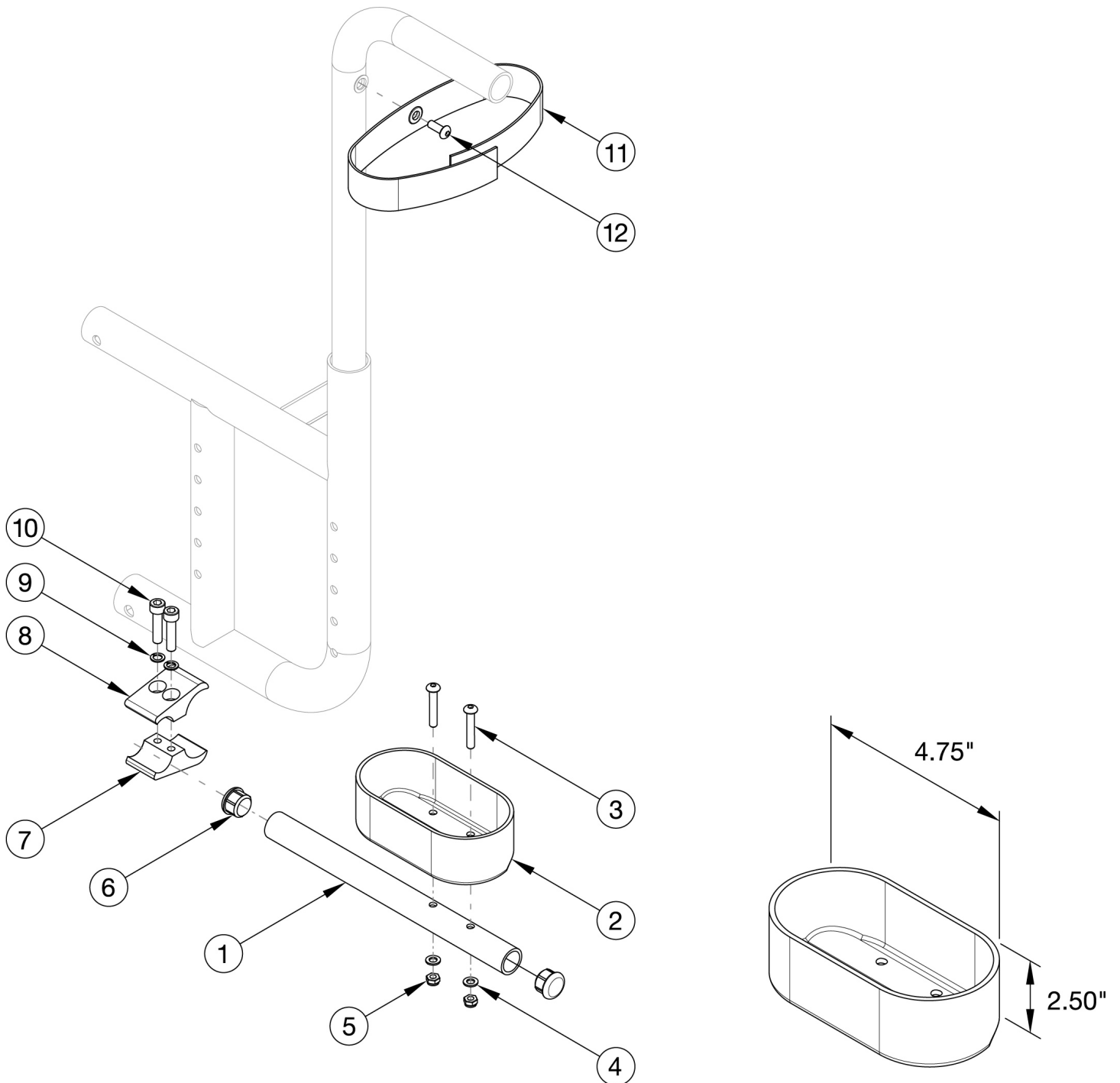

Item Number	Part Number	Description	Quantity	Retail
1 - 12	500710	Crutch Holder Assembly, Catalyst	1	\$195.00
1, 6	000919	Crutch Holder Tube CAT	1	\$21.44
2	000917	Crutch Holder Cup	1	\$57.78

Item Number	Part Number	Description	Quantity	Retail
3	101560	M5 x 30 BTN SHCS ZC	2	\$2.10
4	101856	Flat Washer, M5, Blk Zn	2	\$0.53
5	100657	M5 Nylock Nut Blk Zn	2	\$0.53
6	100638	Tube End Plug 3/4"	2	\$2.10
7	000379	Clamp Bottom 1.125" TSU TI	1	\$18.91
8	000378	Clamp Top 1.125" TSU TI	1	\$18.91
9	101835	M6 Lock Washer Blk Zn	2	\$0.53
10	100780	M6 x 25 SHCS ZC	2	\$2.10
11	101107	Crutch Holder Velcro Strap	1	\$21.01
12	100734	M5 x 16 PHMS ZC	1	\$1.58