


Double Rigid Stroller Handle

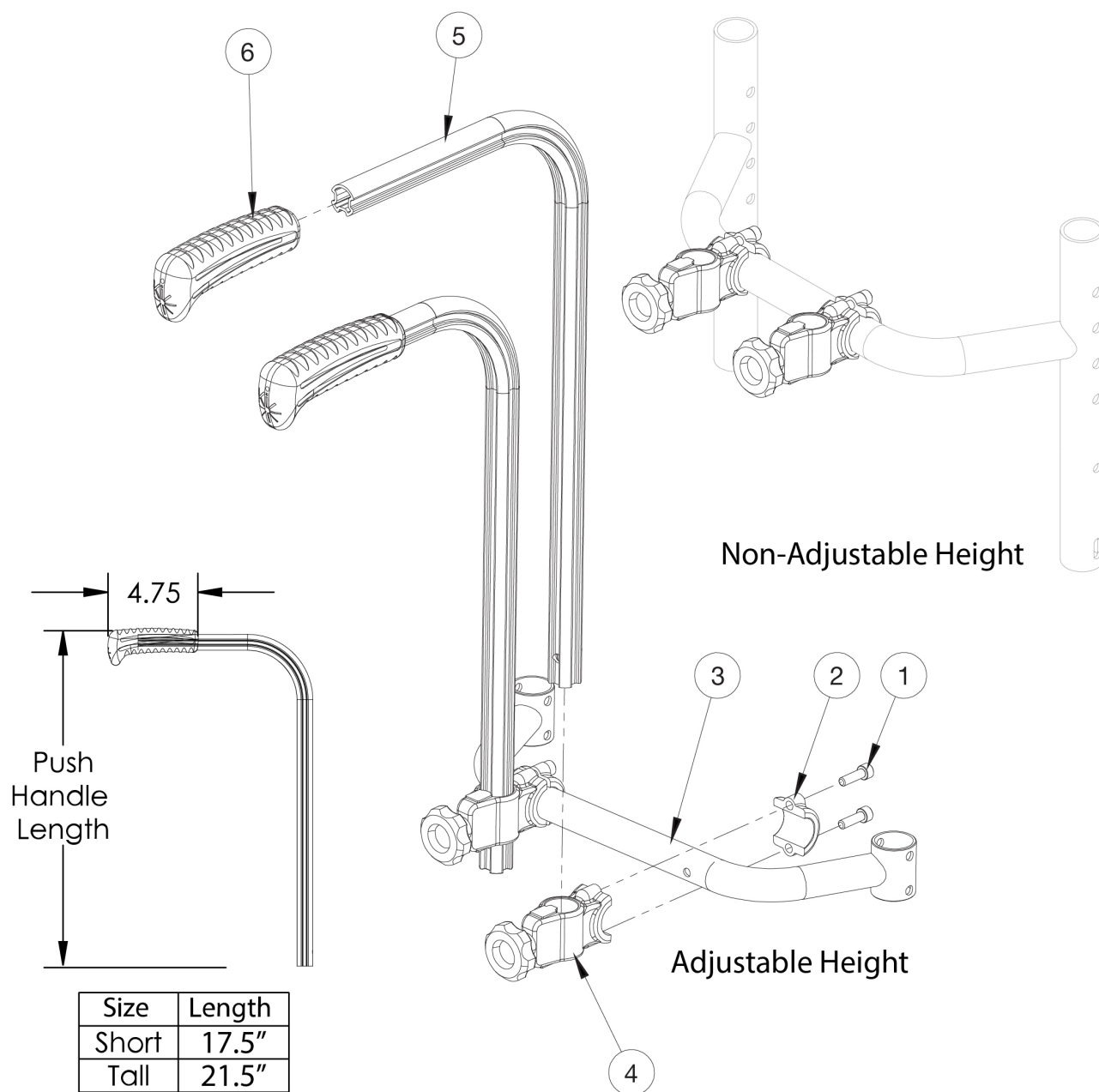

If changing from another style push handle and the Drum Brake is present, a new Drum Brake Cable Kit may be required. If retrofitting to the Key style Stroller Handle, please quote the corresponding Rigidizer or Backrest needed as well.

Item Number	Part Number	Description	Quantity
1,2,4	122138	Stroller Handle Clamp Assembly	2

Item Number	Part Number	Description	Quantity
1,2,4,5a,6	125269	Stroller Handle Short Assembly Each, Rigid <i>Used on chairs with 15" or greater rear seat height and drum brake is not present.</i>	2
1,2,4,5b,6	125270	Stroller Handle Tall Assembly Each, Rigid <i>Used on chairs with less than 15" rear seat height and drum brake is not present.</i>	2
5a,6	125271	Stroller Handle Tube Short, Rigid <i>Used on chairs with 15" or greater rear seat height and drum brake is not present. Replaced part number 506137.</i>	2
5a	122559	Stroller Handle with Snap Button Short <i>Used on chairs with 15" or greater rear seat height and drum brake is not present.</i>	2
5b	122560	Stroller Handle with Snap Button Tall <i>Used on chairs with less than 15" rear seat height and drum brake is not present.</i>	2
6.0	109433	Ergo Push Handle Grip 7/8" Standard with Instruction	2
3a	123701	ALX/LW HA Back Non HA R-Bar 9.25" Tall 12" Wide Dual Stroller Handle	1
3b	123702	ALX/LW HA Back Non HA R-Bar 9.25" Tall 13" Wide Dual Stroller Handle	1
3c	123703	ALX/LW HA Back Non HA R-Bar 9.25" Tall 14" Wide Dual Stroller Handle	1
3d	123704	ALX/LW HA Back Non HA R-Bar 9.25" Tall 15" Wide Dual Stroller Handle	1
3e	123705	ALX/LW HA Back Non HA R-Bar 9.25" Tall 16" Wide Dual Stroller Handle	1
3f	123706	ALX/LW HA Back Non HA R-Bar 9.25" Tall 17" Wide Dual Stroller Handle	1
3g	123707	ALX/LW HA Back Non HA R-Bar 9.25" Tall 18" Wide Dual Stroller Handle	1
3h	123708	ALX/LW HA Back Non HA R-Bar 9.25" Tall 19" Wide Dual Stroller Handle	1
3i	123709	ALX/LW HA Back Non HA R-Bar 9.25" Tall 20" Wide Dual Stroller Handle	1
3j	123710	ALX/LW HA Back Non HA R-Bar 8.25" Tall 12" Wide Dual Stroller Handle	1
3k	123711	ALX/LW HA Back Non HA R-Bar 8.25" Tall 13" Wide Dual Stroller Handle	1
3l	123712	ALX/LW HA Back Non HA R-Bar 8.25" Tall 14" Wide Dual Stroller Handle	1
3m	123713	ALX/LW HA Back Non HA R-Bar 8.25" Tall 15" Wide Dual Stroller Handle	1
3n	123714	ALX/LW HA Back Non HA R-Bar 8.25" Tall 16" Wide Dual Stroller Handle	1
3o	123715	ALX/LW HA Back Non HA R-Bar 8.25" Tall 17" Wide Dual Stroller Handle	1
3p	123716	ALX/LW HA Back Non HA R-Bar 8.25" Tall 18" Wide Dual Stroller Handle	1

Item Number	Part Number	Description	Quantity
3q	123717	ALX/LW HA Back Non HA R-Bar 8.25" Tall 19" Wide Dual Stroller Handle	1
3r	123718	ALX/LW HA Back Non HA R-Bar 8.25" Tall 20" Wide Dual Stroller Handle	1
3s	123652	Rogue/Ethos HA Back Non HA R-Bar 8.813" Tall 12" Wide Dual Stroller Handle	1
3t	123653	Rogue/Ethos HA Back Non HA R-Bar 8.813" Tall 13" Wide Dual Stroller Handle	1
3u	123654	Rogue/Ethos HA Back Non HA R-Bar 8.813" Tall 14" Wide Dual Stroller Handle	1
3v	123655	Rogue/Ethos HA Back Non HA R-Bar 8.813" Tall 15" Wide Dual Stroller Handle	1
3w	123656	Rogue/Ethos HA Back Non HA R-Bar 8.813" Tall 16" Wide Dual Stroller Handle	1
3x	123657	Rogue/Ethos HA Back Non HA R-Bar 8.813" Tall 17" Wide Dual Stroller Handle	1
3y	123658	Rogue/Ethos HA Back Non HA R-Bar 8.813" Tall 18" Wide Dual Stroller Handle	1
3z	123659	Rogue/Ethos HA Back Non HA R-Bar 8.813" Tall 19" Wide Dual Stroller Handle	1
3aa	123660	Rogue/Ethos HA Back Non HA R-Bar 8.813" Tall 20" Wide Dual Stroller Handle	1
3ab	123661	Rogue/Ethos HA Back Non HA R-Bar 7.813" Tall 12" Wide Dual Stroller Handle	1
3ac	123662	Rogue/Ethos HA Back Non HA R-Bar 7.813" Tall 13" Wide Dual Stroller Handle	1
3ad	123663	Rogue/Ethos HA Back Non HA R-Bar 7.813" Tall 14" Wide Dual Stroller Handle	1
3ae	123664	Rogue/Ethos HA Back Non HA R-Bar 7.813" Tall 15" Wide Dual Stroller Handle	1
3af	123665	Rogue/Ethos HA Back Non HA R-Bar 7.813" Tall 16" Wide Dual Stroller Handle	1
3ag	123666	Rogue/Ethos HA Back Non HA R-Bar 7.813" Tall 17" Wide Dual Stroller Handle	1
3ah	123667	Rogue/Ethos HA Back Non HA R-Bar 7.813" Tall 18" Wide Dual Stroller Handle	1
3ai	123668	Rogue/Ethos HA Back Non HA R-Bar 7.813" Tall 19" Wide Dual Stroller Handle	1
3aj	123669	Rogue/Ethos HA Back Non HA R-Bar 7.813" Tall 20" Wide Dual Stroller Handle	1
3ak	123670	Rogue/Ethos HA Back Non HA R-Bar 6.813" Tall 12" Wide Dual Stroller Handle	1
3al	123671	Rogue/Ethos HA Back Non HA R-Bar 6.813" Tall 13" Wide Dual Stroller Handle	1
3am	123672	Rogue/Ethos HA Back Non HA R-Bar 6.813" Tall 14" Wide Dual Stroller Handle	1
3an	123673	Rogue/Ethos HA Back Non HA R-Bar 6.813" Tall 15" Wide Dual Stroller Handle	1
3ao	123674	Rogue/Ethos HA Back Non HA R-Bar 6.813" Tall 16" Wide Dual Stroller Handle	1
3ap	123675	Rogue/Ethos HA Back Non HA R-Bar 6.813" Tall 17" Wide Dual Stroller Handle	1
3aq	123676	Rogue/Ethos HA Back Non HA R-Bar 6.813" Tall 18" Wide Dual Stroller Handle	1

Item Number	Part Number	Description	Quantity
3ar	123677	Rogue/Ethos HA Back Non HA R-Bar 6.813" Tall 19" Wide Dual Stroller Handle	1
3as	123678	Rogue/Ethos HA Back Non HA R-Bar 6.813" Tall 20" Wide Dual Stroller Handle	1
3at	122821_ _ _	Adjustable Height Rigidizer Bar 12" Dual Stroller Handle	1
3su	122822_ _ _	Adjustable Height Rigidizer Bar 13" Dual Stroller Handle	1
3sv	122823_ _ _	Adjustable Height Rigidizer Bar 14" Dual Stroller Handle	1
3aw	122824_ _ _	Adjustable Height Rigidizer Bar 15" Dual Stroller Handle	1
3ax	122825_ _ _	Adjustable Height Rigidizer Bar 16" Dual Stroller Handle	1
3ay	122826_ _ _	Adjustable Height Rigidizer Bar 17" Dual Stroller Handle	1
3az	122827_ _ _	Adjustable Height Rigidizer Bar 18" Dual Stroller Handle	1
3ba	122828_ _ _	Adjustable Height Rigidizer Bar 19" Dual Stroller Handle	1
3bb	122829_ _ _	Adjustable Height Rigidizer Bar 20" Dual Stroller Handle	1