


# Rogue XP Transit

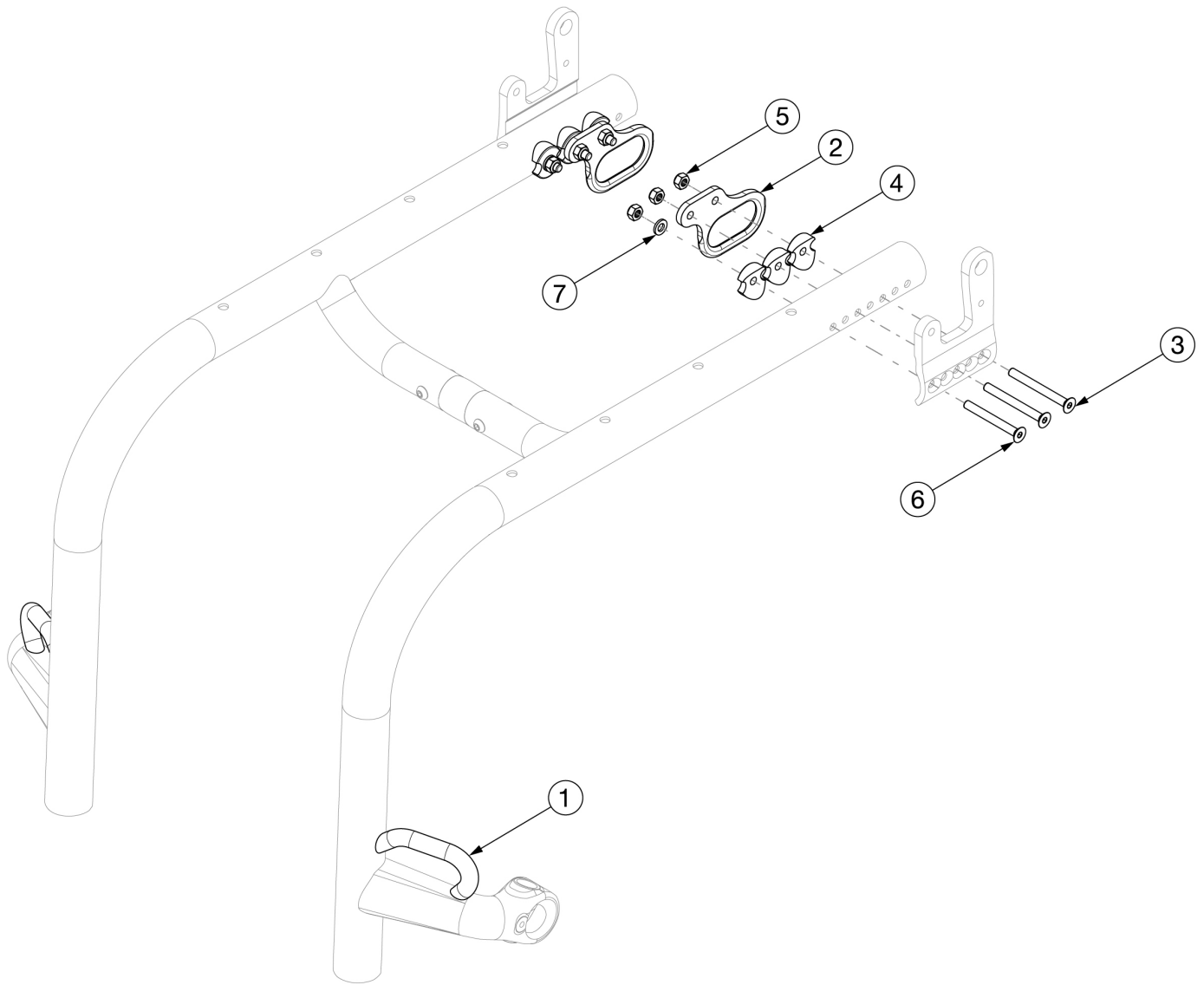

*Rogue XP Transit System cannot be retrofitted. If needing to add transit, please contact Customer Service. Used on chairs invoiced starting on 3/12/25*

Item Number	Part Number	Description	Quantity
2-7	124937	Rear Transit Brackets with Hardware	1
2	001782	Rear Transit Bracket	2
3	103068	M6 x 60mm FHCS Blk Zn	4
4	000248	Saddle 1.375"	6
5	100658	M6 Nylock Nut Blk Zn	6

<b>Item Number</b>	<b>Part Number</b>	<b>Description</b>	<b>Quantity</b>
6	101130	M6 x 55 FSHCS P/THD BLKZC	2
7	100746	M6 Flat Washer Blk Zn	2