


Liberty Stroller Handle Backrest

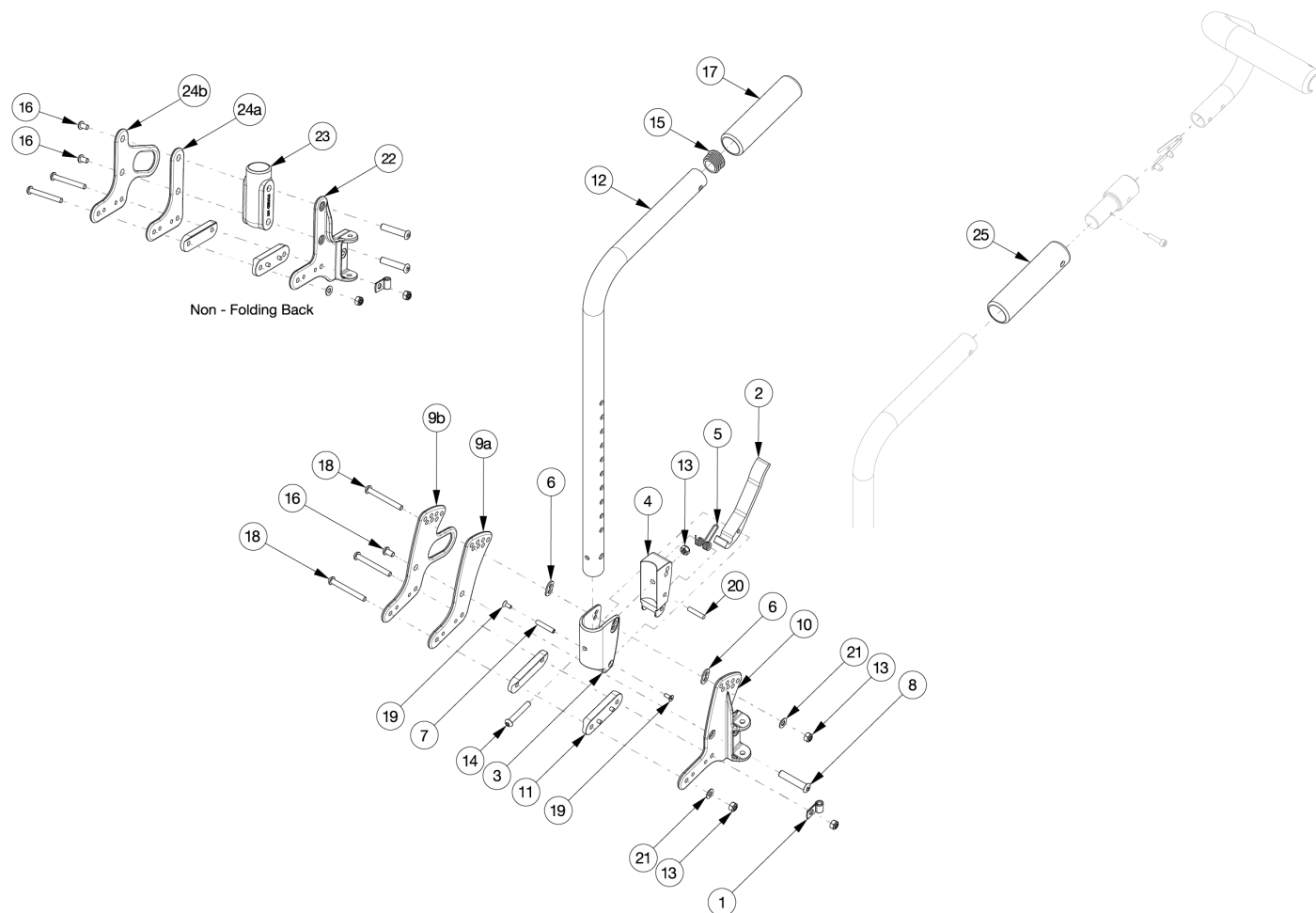

If changing to a new back post style or height, please see the Liberty Tilt Mechanism parts manual page to evaluate if a new tilt cable is necessary. If changing from a non-folding height adjustable backrest to a non-folding stroller handle backrest, #23 (003311) is required. To obtain the "Removable Stroller Handle Extensions", please reference the "Focus CR / Liberty Removable Stroller Handle" parts manual page.

Item Number	Part Number	Description	Quantity
2-8, 9a, 10a, 10b, 11, 12a, 13-21	120747	Short Fixed Height Folding Backrest PAIR Kit Liberty	1
2-8, 9b, 10a, 10b 11, 12a, 13-21	120755	Short Fixed Height Folding Backrest PAIR Kit Transit Liberty	1
2-8, 9a, 10a, 10b, 11, 12b, 13-21	120748	Tall Fixed Height Folding Backrest PAIR Kit Liberty	1
2-8, 9b, 10a, 10b, 11, 12b, 13-21	120756	Tall Fixed Height Folding Backrest PAIR Kit Transit Liberty	1

Item Number	Part Number	Description	Quantity
2-8, 9a, 10a, 10b, 11, 12a, 13-16, 18-22, 25	120749	Short Fixed Height Folding for Stroller Handle Extension Backrest PAIR Kit Liberty <i>Used with Stroller Handle Extensions</i>	1
2-8, 9b, 10a, 10b, 11, 12a, 13-16, 18-22, 25	120798	Short Fixed Height Folding for Stroller Handle Extension Backrest PAIR Kit Transit Liberty <i>Used with Stroller Handle Extensions</i>	1
2-8, 9a, 10a, 10b, 11, 12b, 13-16, 18-22, 25	120750	Tall Fixed Height Folding for Stroller Handle Extension Backrest PAIR Kit Liberty <i>Used with Stroller Handle Extensions</i>	1
2-8, 9b, 10a, 10b, 11, 12b, 13-16, 18-22, 25	120799	Tall Fixed Height Folding for Stroller Handle Extension Backrest PAIR Kit Tranist Liberty <i>Used with Stroller Handle Extensions</i>	1
8, 11, 12a, 15-18, 22a, 22b, 23, 24a	120751	Short Fixed Height Non-Folding Backrest PAIR Kit Liberty	1
8, 11, 12a, 15-18, 22a, 22b, 23, 24b	120759	Short Fixed Height Non-Folding Backrest PAIR Kit Tranist Liberty	1
8, 11, 12b, 15-18, 22a, 22b, 23, 24a	120752	Tall Fixed Height Non-Folding Backrest PAIR Kit Liberty	1
8, 11, 12b, 15-18, 22a, 22b, 23, 24b	120760	Tall Fixed Height Non-Folding Backrest PAIR Kit Tranist Liberty	1
8, 11, 12a, 15, 16, 18, 22a, 22b, 23, 24a, 25	120753	Short Fixed Height Non-Folding for Stroller Handle Extension Backrest PAIR Kit Liberty <i>Used with Stroller Handle Extensions</i>	1
1, 8, 11, 12a, 13, 15 - 18, 21, 22a, 22b, 23, 24b	120761	Short Fixed Height Non-Folding for Stroller Handle Extension Backrest PAIR Kit Transit Liberty <i>Used with Stroller Handle Extensions</i>	1
8, 11, 12b, 15, 16, 18, 22a, 22b, 23, 24a, 25	120754	Tall Fixed Height Non-Folding for Stroller Handle Extension Backrest PAIR Kit Liberty <i>Used with Stroller Handle Extensions</i>	1
1, 8, 11, 12b, 13, 15 - 18, 21, 22a, 22b, 23, 24b	120762	Tall Fixed Height Non-Folding for Stroller Handle Extension Backrest PAIR Kit Tranist Liberty <i>Used with Stroller Handle Extensions</i>	1
2-7, 12a, 13-15, 17, 19, 20	506324	Backpost Assembly Short Fixed Height, Folding, Liberty	1
2-7, 12b, 13-15, 17, 19, 20	506325	Backpost Assembly Tall Fixed Height, Folding, Liberty	1
2-7, 12a, 13, 14, 19, 20, 25	506650	Backpost Assembly Short Fixed Height Folding for Stroller Handle Extension Liberty	1

Item Number	Part Number	Description	Quantity
		<i>Used with Stroller Handle Extensions</i>	
2-7, 12b, 13, 14, 19, 20, 25	506651	Backpost Assembly Tall Fixed Height Folding for Stroller Handle Extension Liberty <i>Used with Stroller Handle Extensions</i>	1
12a, 15, 17, 23	506172	Backpost Assembly Short Fixed Height, Non-Folding, Liberty	1
12b, 15, 17, 23	506173	Backpost Assembly Tall Fixed Height, Non-Folding, Liberty	1
12a, 23, 25	506648	Backpost Assembly Short Fixed Height Non-Folding for Stroller Handle Extension Liberty <i>Used with Stroller Handle Extensions.</i>	1
12b, 23, 25	506649	Backpost Assembly Tall Fixed Height Non-Folding for Stroller Handle Extension Liberty <i>Used with Stroller Handle Extensions.</i>	1
12a, 15, 17	506807	Backpost with Grip Assembly Short Fixed Height, Liberty	1
12b, 15, 17	506808	Backpost with Grip Assembly Tall Fixed Height, Liberty	1
12a, 25	506809	Backpost with Stroller Handle Extension Grip Assembly Short Fixed Height, Liberty <i>Used with Stroller Handle Extensions.</i>	1
12b, 25	506810	Backpost with Stroller Handle Extension Grip Assembly Tall Fixed Height, Liberty <i>Used with Stroller Handle Extensions.</i>	1
2-8, 9a, 10a, 10b 11, 13, 14, 16, 18, 19, 20, 21	506164	Folding Backrest Hardware PAIR Kit, Liberty <i>Kit Includes Hardware for Left and Right side. Used to retrofit from non-folding to folding.</i>	1
2-8, 9b, 10a, 10b 11, 13, 14, 16, 18, 19, 20, 21	506165	Folding Backrest Hardware PAIR Kit, Transit, Liberty <i>Kit Includes Hardware for Left and Right side. Used to retrofit from non-folding to folding.</i>	1
8, 11, 16, 18, 22a, 22b, 23, 24a	506323	Non-Folding Backrest Hardware PAIR Kit, Fixed Height, Liberty <i>Kit Includes Hardware for Left and Right side. Used to retrofit from folding to non-folding.</i>	1
8, 11, 16, 18, 22a, 22b, 23, 24b	120763	Non-Folding Backrest Hardware PAIR Kit Fixed Height Transit Liberty <i>Kit Includes Hardware for Left and Right side. Used to retrofit from folding to non-folding.</i>	1
8, 9a, 10, 11, 13, 16, 18, 21	120795	Folding Backplates w/Hardware Liberty (Pair) <i>Kit Includes Hardware for Left and Right side.</i>	1
8, 9b, 10, 11, 13, 16, 18, 21	120796	Folding Backplates w/Hardware Transit Liberty (Pair) <i>Kit Includes Hardware for Left and Right side.</i>	1
8, 11, 16, 22a, 22b, 23, 24a	112924	Non-Folding Backplates w/Hardware Liberty (Pair) <i>Kit Includes Hardware for Left and Right side.</i>	1
8, 11, 16, 22a, 22b, 23, 24b	120797	Non-Folding Backplates w/Hardware Transit Liberty (Pair)	1

Item Number	Part Number	Description	Quantity
		<i>Kit Includes Hardware for Left and Right side.</i>	
3, 6	503834	Cover Plate, Latch Insert Kit	1
1	001239	P-Clip 3/16"	1
2	001893	Latch Lever, Backrest	1
4	001895	Latch Insert, Folding Backrest	1
5	001896	Spring, Backrest Latch	1
6	001897	Washer, Dual, PSA	2
7	002513	Threaded Barrel, 5mm OD x M4	1
8	003284	Binding Post, 8mm x 40mm x M6	1
9a	003307	Back Plate, Outer, Folding	1
9b	506322	Transit Back Plate Folding, Liberty	1
10a	003308	Back Plate, Inner, Folding, Right	1
10b	003309	Back Plate, Inner, Folding, Left	1
11	003312	Saddle, Seat Tube	2
13	100658	M6 Nylock Nut Blk Zn	8
14	100797	M6 x 40 BHCS BLK ZC	1
15	100804	Tube End Plug 1"	1
16	101184	M6 x 10 BTN SHCS BLKZC w/Patch	1
18	101781	M6 x 50mm Partial Thread BHCS Blk ZC	3
19	101818	M4 x 10 FHCS BLKZC w/Patch	2
20	102694	Dowel Pin, M5 X 24, Blk Zn	1
21	102803	Washer Flat .250 X .500 X .027 Hx Blz	2
22a	003305	Back Plate, Inner, Non-Folding, Right	1
22b	003306	Back Plate, Inner, Non-Folding, Left	1
23	003311	Receiver, Back Cane, Fixed Ht	1
24a	003304	Back Plate, Outer, Non-Folding	1
24b	506564	Transit Back Plate Non-Folding, Liberty	1