


Ethos Swing Away Armrest

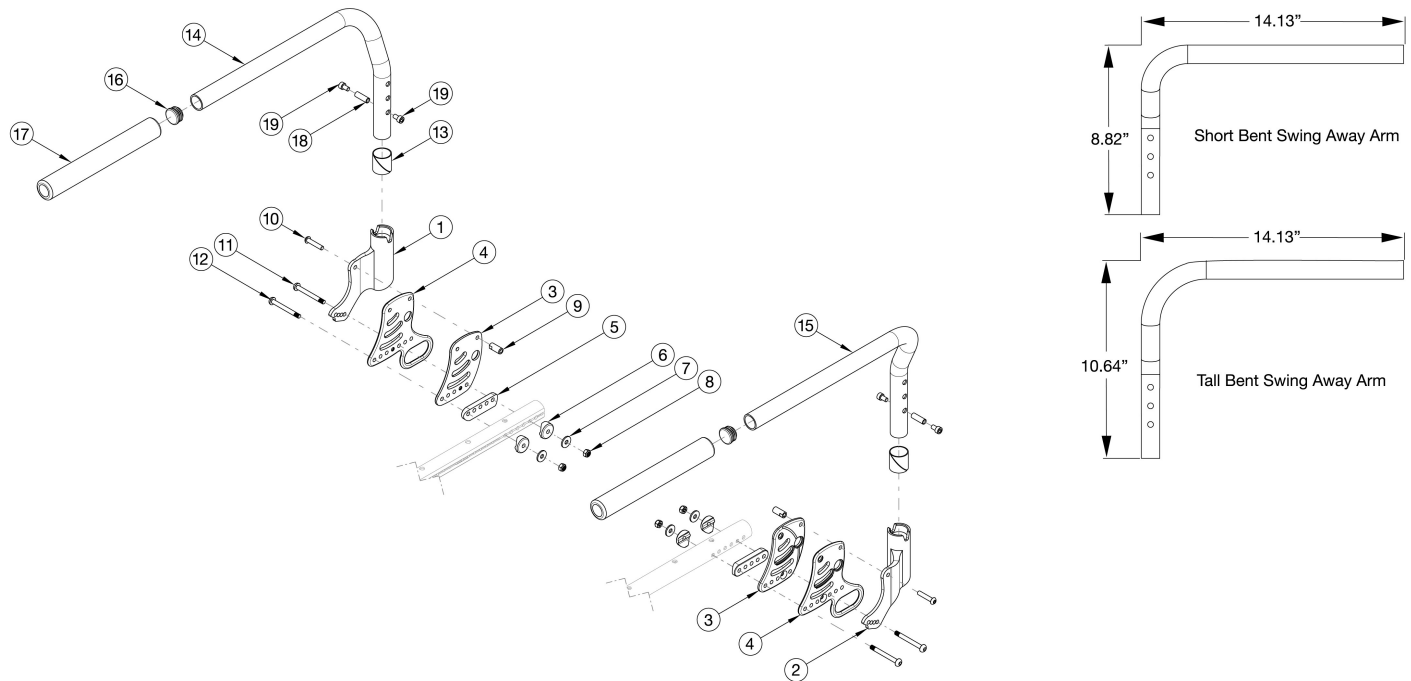

Effective 10/17/2022 If retrofitting, Receivers are needed. When to use Short vs. Tall Swing Away Arms 22" Wheel: Rear Seat Height = 13.5-15.5" Use Short // Rear Seat Height = 13.5-17.5" Use Tall 24" Wheel: Rear Seat Height = 14.5-17.5" Use Short // Rear Seat Height = 14.5-18.5" Use Tall 25" Wheel: Rear Seat Height = 14.5-18.5" Use Short // Rear Seat Height = 14.5-20" Use Tall 26" Wheel: Rear Seat Height = 15.5-19.5" Use Short // Rear Seat Height = 15.5-20" Use Tall

Item Number	Part Number	Description	Quantity
1, 6 - 8, 10 - 11, 13, 14a, 16 - 19	116901	Short Swing Away Arm with Receiver Assembly Right, Ethos	1
2, 6 - 8, 10 - 11, 13, 15a, 16 - 19	116902	Short Swing Away Arm with Receiver Assembly Left, Ethos	1
1 - 2, 6 - 8, 10 - 11, 13, 14a, 15a, 16 - 19	116903	Short Swing Away Arm with Receiver Assembly PAIR, Ethos	1
1, 6 - 8, 10 - 11, 13, 14b, 16 - 19	116904	Tall Swing Away Arm with Receiver Assembly Right, Ethos	1
2, 6 - 8, 10 - 11, 13, 15b, 16 - 19	116905	Tall Swing Away Arm with Receiver Assembly Left, Ethos	1
1 - 2, 6 - 8, 10 - 11, 13, 14b, 15a, 16 - 19	116906	Tall Swing Away Arm with Receiver Assembly PAIR, Ethos	1
1, 6 - 8, 10 - 11, 13	116907	Swing Away Arm Receiver Assembly Right, Ethos	1
2, 6 - 8, 10 - 11, 13	116908	Swing Away Arm Receiver Assembly Left, Ethos	1

Item Number	Part Number	Description	Quantity
14a, 16 - 19	108166	Short Swing Away Arm Right, Ethos / Rogue	1
15a, 16 - 19	108167	Short Swing Away Arm Left, Ethos / Rogue	1
14a, 15a, 16 - 19	108168	Short Swing Away Arm PAIR, Ethos / Rogue	1
14b, 16 - 19	108169	Tall Swing Away Arm Right, Ethos / Rogue	1
15b, 16 - 19	108170	Tall Swing Away Arm Left, Ethos / Rogue	1
14b, 15b, 16 - 19	108171	Tall Swing Away Arm PAIR, Ethos / Rogue	1
1, 13	117132	Swing Away Arm Receiver w/ Nyliner, Right	1
2, 13	117133	Swing Away Arm Receiver w/ Nyliner, Left	1
3a	115550	Back Plate, Right, Black	1
3b	115550_ _ _	Color Anodized/Color Accent Back Plate, Right	1
3c	115551	Back Plate, Left, Black	1
3d	115551_ _ _	Color Anodized/Color Accent Back Plate, Left	1
4a	115552	Back Plate, Transit, Right, Black	1
4b	115553	Back Plate, Transit, Left, Black	1
5	003848	Back Bracket Saddle 1 1/8"	2
6	003849	Saddle For Dimple 1 1/8"	4
7	101853	M6 Washer Blk Zn	4
8	100658	M6 Nylock Nut Blk Zn	4
9	115546	Strike, Ø.364	2
10	115574	M6 x 30 BHCS, CLS 10.9, NYL, BLZ	2
11	003232	M6 x 55 BHCS Blk Zc	2
12	003846	M6 x 50 BHCS, Custom, Blk Zc	2
13	103517	Nyliner 1" ID x 1.094" OD x 1.125" Long	2
16	100804	Tube End Plug 1"	2
18	000065	M6 x 23.68 Threaded Barrel	2
19	110061	M6 x 10 Stainless Steel Armrest Screw with Patch	4