


Focus CR Tilt Stop

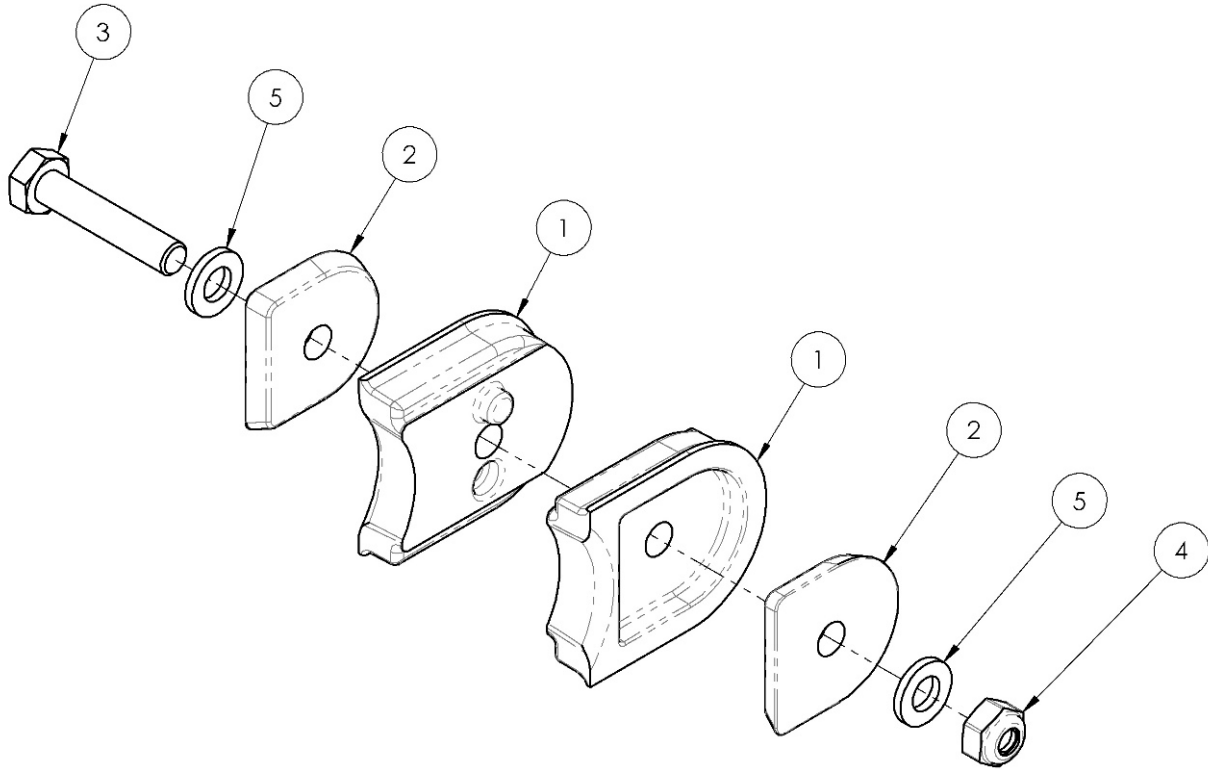

| Item Number | Part Number | Description | Quantity |
|-------------|-------------|-------------------------------|----------|
| 1 - 5 | 501072 | Tilt Stop PAIR Assembly Focus | 1 |
| 1 | 001036 | Pad Tilt Stop TILT | 2 |
| 2 | 001037 | Thrust Plate Tilt Stop TILT | 2 |
| 3 | 101824 | M6 X 30 Hhcs Blk Zn | 1 |
| 4 | 100658 | M6 Nylock Nut Blk Zn | 1 |
| 5 | 100746 | M6 Flat Washer Blk Zn | 2 |