


Focus CR Height Adjustable Dual Post Flip Back Armrest

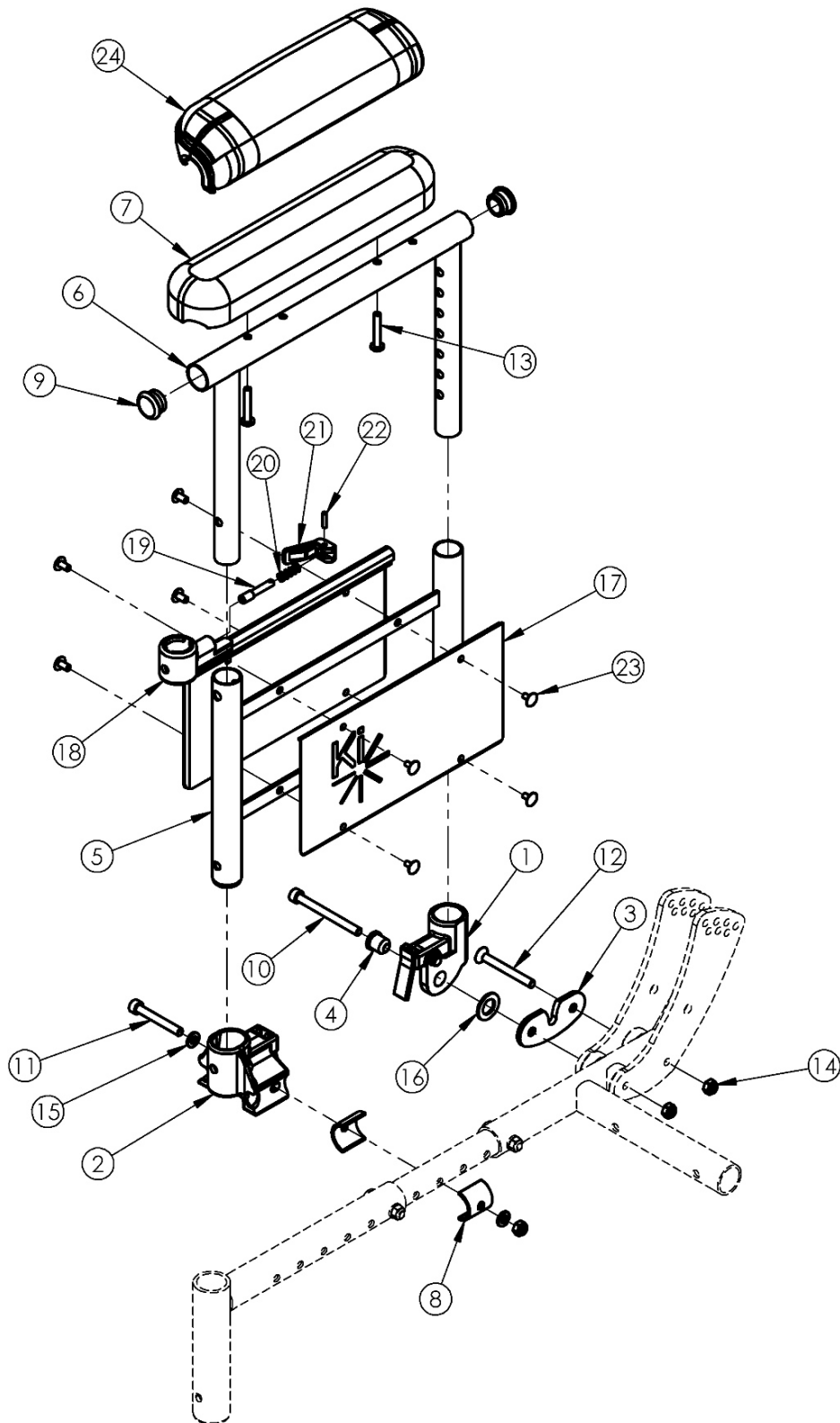

If retrofitting, Receivers are needed.

Item Number	Part Number	Description	Quantity
1-5, 6a, 7a, 8-12, 13a, 14-23	501139	Armrest Assembly Dual Post Desk Pad PAIR w/ Receivers Focus	1
1-5, 6b, 7b, 8-12, 13a, 14-23	501138	Armrest Assembly Dual Post Full Pad PAIR w/ Receivers Focus	1
1-5, 6a, 8-12, 13b, 14-23, 24b	110036	Armrest Assembly Dual Post Desk Waterfall Pad PAIR w/ Receivers Focus	1
1-5, 6b, 8-12, 13b, 14-23, 24b	110037	Armrest Assembly Dual Post Full Waterfall Pad PAIR w/ Receivers Focus	1
1a, 2a, 3, 4, 5a, 6a, 7a, 8-12, 13a, 14-23	502294	Armrest Dual Post Kit Right Desk Pad w/ Receivers Focus	1
1b, 2b, 3, 4, 5b, 6a, 7a, 8-12, 13a, 14-23	502295	Armrest Dual Post Kit Left Desk Pad w/ Receivers Focus	1
1a, 2a, 3, 4, 5a, 6b, 7b, 8-12, 13a, 14-23	502296	Armrest Dual Post Kit Right Full Pad w/ Receivers Focus	1
1b, 2b, 3, 4, 5b, 6b, 7b, 8-12, 13a, 14-23	502297	Armrest Dual Post Kit Left Full Pad w/ Receivers Focus	1
1a, 2a, 3, 4, 5a, 6a, 8-12, 13b, 14-23, 24b	110032	Armrest Dual Post Kit Right Desk Waterfall Pad w/ Receivers Focus	1
1b, 2b, 3, 4, 5b, 6a, 8-12, 13b, 14-23, 24b	110033	Armrest Dual Post Kit Left Desk Waterfall Pad w/ Receivers Focus	1
1a, 2a, 3, 4, 5a, 6b, 8-12, 13b, 14-23, 24b	110034	Armrest Dual Post Kit Right Full Waterfall Pad w/ Receivers Focus	1
1b, 2b, 3, 4, 5b, 6b, 8-12, 13b, 14-23, 24b	110035	Armrest Dual Post Kit Left Full Waterfall Pad w/ Receivers Focus	1
5b, 6a, 7a, 8, 9, 13a, 17-23	503301	Armrest Dual Post Kit Right w/ Desk Pad Focus <i>Does Not Include Receivers or Mounting Hardware</i>	1
5a, 6a, 7a, 8, 9, 13a, 17-23	503302	Armrest Dual Post Kit Left w/ Desk Pad Focus <i>Does Not Include Receivers or Mounting Hardware</i>	1
5b, 6b, 7b, 8, 9, 13a, 17-23	503303	Armrest Dual Post Kit Right w/ Full Pad Focus <i>Does Not Include Receivers or Mounting Hardware</i>	1
5a, 6b, 7b, 8, 9, 13a, 17-23	503304	Armrest Dual Post Kit Left w/ Full Pad Focus <i>Does Not Include Receivers or Mounting Hardware</i>	1
5b, 6a, 8, 9, 13b, 17-23, 24b	503305	Armrest Dual Post Kit Right w/ Desk Waterfall Pad Focus <i>Does Not Include Receivers or Mounting Hardware</i>	1

Item Number	Part Number	Description	Quantity
5a, 6a, 8, 9, 13b, 17-23, 24b	503306	Armrest Dual Post Kit Left w/ Desk Waterfall Pad Focus <i>Does Not Include Receivers or Mounting Hardware</i>	1
5b, 6b, 8, 9, 13b, 17-23, 24a	503307	Armrest Dual Post Kit Right w/ Full Waterfall Pad Focus <i>Does Not Include Receivers or Mounting Hardware</i>	1
5a, 6b, 8, 9, 13b, 17-23, 24a	503308	Armrest Dual Post Kit Left w/ Full Waterfall Pad Focus <i>Does Not Include Receivers or Mounting Hardware</i>	1
1a, 3, 4, 10, 12, 14-16	502290	Dual Post Armrest Rear Receiver Assembly w/ Hardware Right Tilt	1
1b, 3, 4, 10, 12, 14-16	502291	Dual Post Armrest Rear Receiver Assembly w/ Hardware Left Tilt	1
2a, 8, 11, 14, 15	502292	Dual Post Armrest Front Receiver Assembly w/ Hardware Right Tilt	1
2b, 8, 11, 14, 15	502293	Dual Post Armrest Front Receiver Assembly w/ Hardware Left Tilt	1
1a	001173	Dual Post Armrest Rear Receiver Assembly Right Tilt	1
1b	001174	Dual Post Armrest Rear Receiver Assembly Left Tilt	1
2a	001175	Dual Post Armrest Front Receiver Assembly Right Tilt	1
2b	001176	Dual Post Armrest Front Receiver Assembly Left Tilt	1
3	001182	Mount Plate Dual Post Armrest Tilt	1
4	001183	Bushing Dual Post Armrest Tilt	1
5a, 17-23	001232	Dual Post Armrest Frame Assembly Lower Left Tilt	1
5b, 17-23	001231	Dual Post Armrest Frame Assembly Lower Right Tilt	1
6a	001213	Dual Post Armrest Frame Upper Desk Tilt	1
6b	001214	Dual Post Armrest Frame Upper Full Tilt	1
7a	100501	Arm Pad Desk	1
7b	100500	Arm Pad Full	1
8	114443	Tube Spacer, Black, 1" OD x .083" x .80 LG	2
9	100803	Tube End Plug 7/8"	2
10	101849	M6 x 65 SHCS Blk Zn	1
11	101472	M6 x 45 SHCS BLK ZC	1
12	101130	M6 x 55 FSHCS P/THD BLKZC	1
13a	100696	M5 x 30mm PPH Capscrew	2

Item Number	Part Number	Description	Quantity
		<i>Used with Standard Arm Pads</i>	
13b	100654	M5 x 40 DIN7985 PH Pan HD M/S ZC <i>Used with Waterfall Arm Pads</i>	2
14	100658	M6 Nylock Nut Blk Zn	3
15	100746	M6 Flat Washer Blk Zn	4
16	102003	Washer, Black Nylon .51 ID X .875 OD X .062	1
24a	102452	Waterfall Arm Pad Full	1
24b	102521	Waterfall Arm Pad Desk	1