

## INSTRUCTION SHEET Yamaha Plate Assembly Replacement


(Please read and understand instructions before beginning installation)

## Tools Required:

2 Flat head screwdrivers 8mm open end wrench 2mm hex key

## Parts Required:

Yamaha Plate Assembly (Fig1) (Sunrise Part Number 133107)

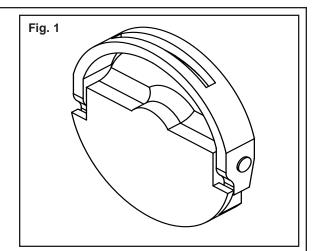

## Removal

 Using a flat head screw driver remove the c-clip securing the Xtender Plastic Cover. The c-clip can be removed by locating the flat spot in the cover and inserting the flat blade of the screw driver into the space between the flat spot and the c-clip and gently prying it out. Once the c-clip is removed lift the cover off over the Plate Assembly.



2) Use a 2mm Hex key to loosen the set screw securing the Plate Assembly to the axle.



SUNRISE MEDICAL 107003 REV. A March 2008 PAGE 1

- 3) Hold the flat spots on the axle with an 8mm Wrench, while turning the Plate Assembly counter clockwise. This will remove the Plate Assembly from the axle.
- 4) Reverse the above procedure to install the new Plate Assembly.



Fig. 3

SUNRISE MEDICAL 107003 REV. A March 2008 PAGE 2