


Flip for x:panda Tilt Mechanism

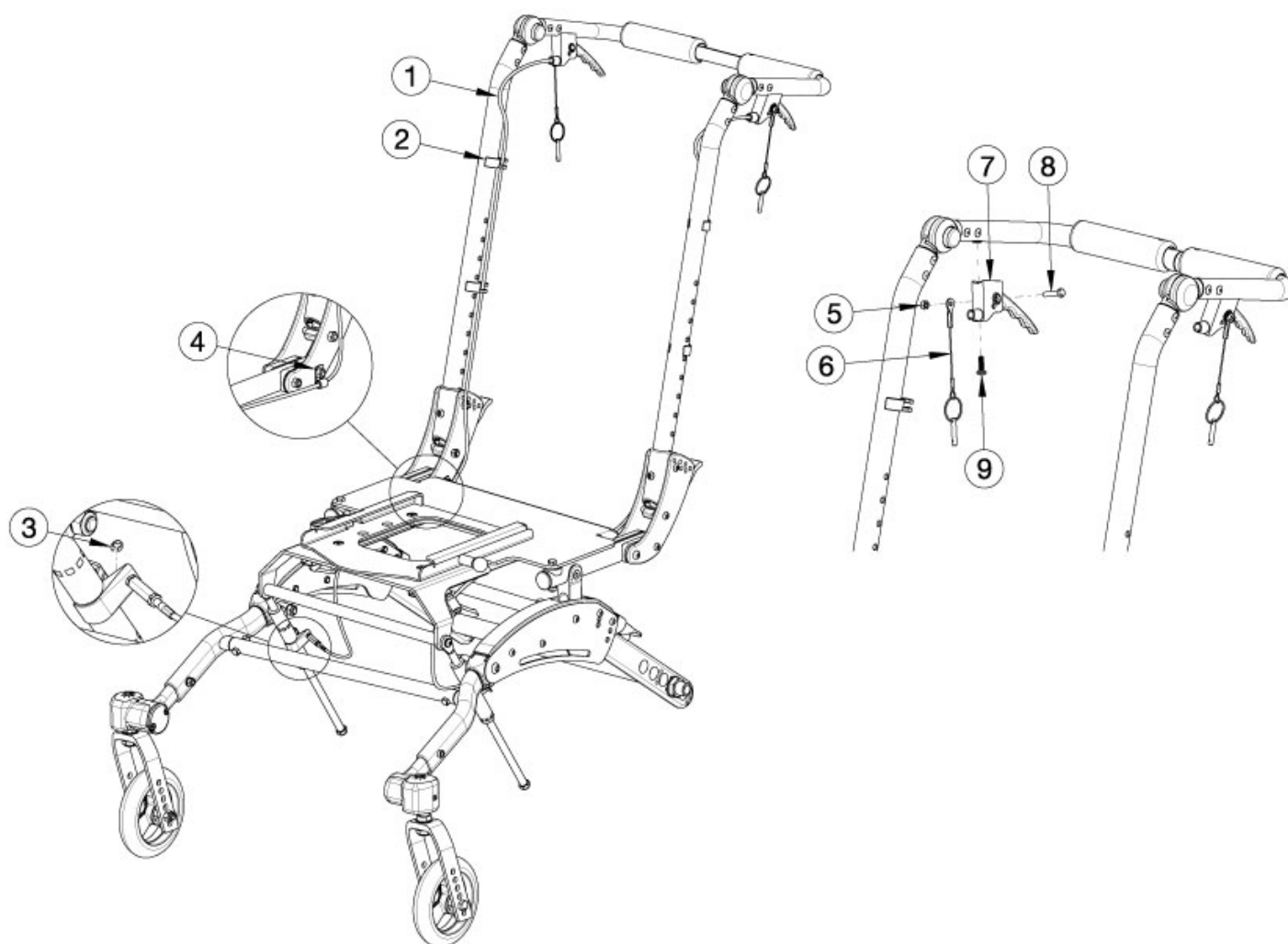

Item Number	Part Number	Description	Quantity
1a, 2 - 9	111267	Hand Tilt Kit, Fixed Height with Adjustable Handle, Short, Flip for Leckey	2
1b, 2 - 9	111268	Hand Tilt Kit, Fixed Height with Adjustable Handle, Tall, Flip for Leckey	2
5 - 9	501112	Trigger Assembly Horizontal Tilt	1
1a, 3	502832	Cable, Hand Tilt, Long FLIP <i>Used with short fixed height with adjustable handle backrest</i>	2
1b, 3	502833	Cable, Hand Tilt, Extra Long FLIP <i>Used with tall fixed height with adjustable handle backrest</i>	2
2	001117	Cable Clip 1"	4
3	102822	E-Ring, M6, YZ	2

Item Number	Part Number	Description	Quantity
4	001239	P-Clip 3/16"	2
5	100657	M5 Nylock Nut Blk Zn	2
6	001124	Pin with Lanyard	2
7	111270	Horizontal Trigger Mount, No Lockout	2
8	101820	M5 x 22 HHCS Blk Zn	2
9	100734	M5 x 16 PHMS ZC	2