


(Discontinued) Catalyst Depth Adjustable Backrest with Hardware

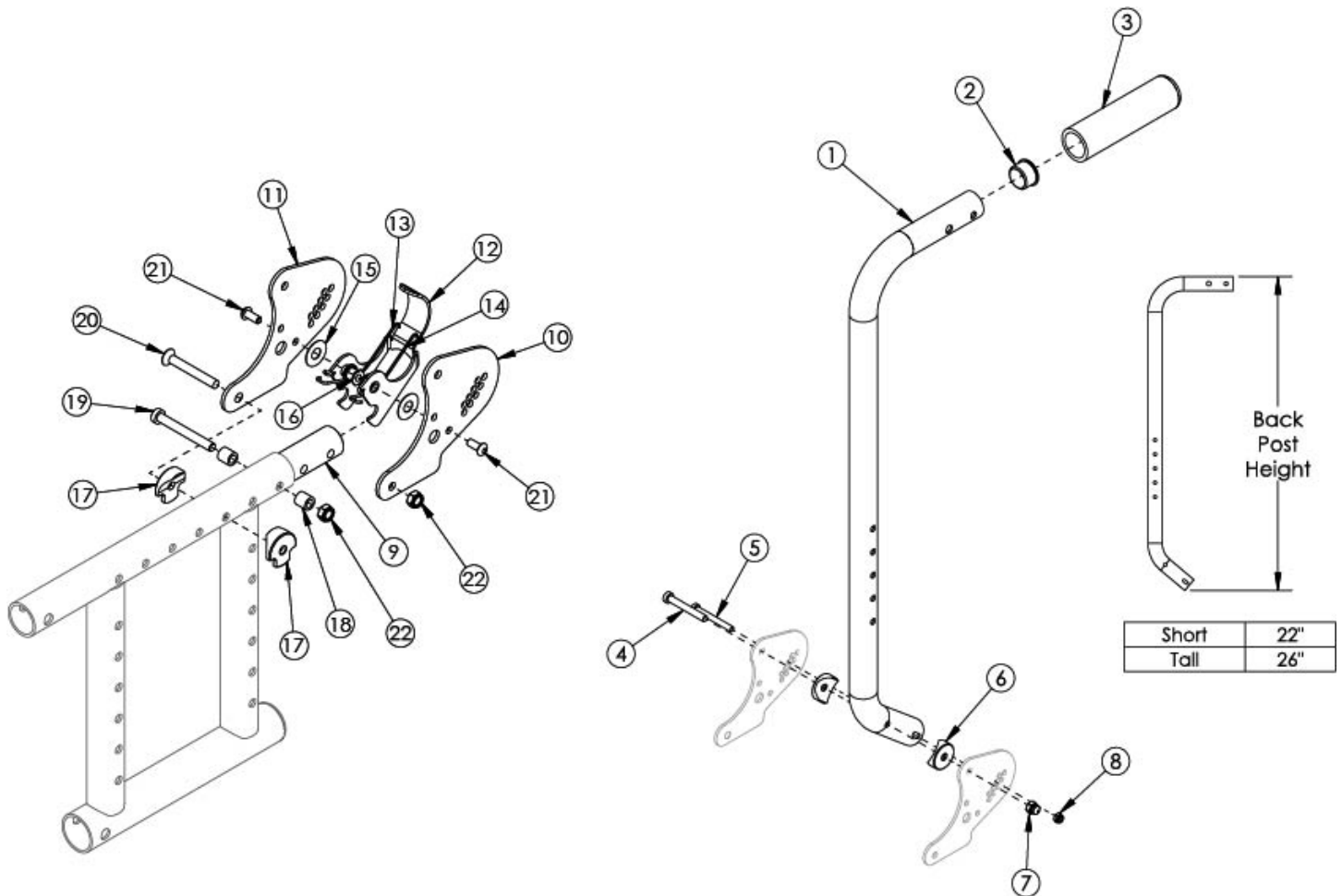

This backrest style discontinued February 8th, 2021 <p>

Not compatible with Drum Brake. A backrest retrofit to the Catalyst Stroller Handle Back Post (Depth Adjustable) is required if retrofitting Drum Brake.

Item Number	Part Number	Description	Quantity	Retail
1a - 9, 10b, 11-22	111330	Back Post Depth Adjustable with Push Handle Short Assm with All Hardware CAT/ SPARK - Right Side	1	\$213.81
1b - 9, 10b, 11-22	111332	Back Post Depth Adjustable with Push Handle Tall Assm with All Hardware CAT/ SPARK - Right Side	1	\$213.81
1a, 2, 3	500613	Back Post Depth Adjustable with Push Handle Short Assm CAT/ SPARK	1	\$48.74
1b, 2, 3	500614	Back Post Depth Adjustable with Push Handle Tall Assm CAT/ SPARK	1	\$48.74
1a - 8	501206	Back Post Depth Adjustable with Push Handle Short Assm with Hardware CAT/ SPARK	1	\$59.24
1b - 8	501207	Back Post Depth Adjustable with Push Handle Tall Assm with Hardware CAT/ SPARK	1	\$59.24

Item Number	Part Number	Description	Quantity	Retail
9, 10b, 11-22	500739	Depth Adjustable Backrest Hardware Assm Right CAT	1	\$148.03
9, 11-22	501227	Depth Adjustable Backrest Hardware Assm SPARK/ OPEN SEAT	1	\$148.03
4-8	111164	Depth Adjustable Backrest Attaching Hardware Kit	1	\$9.48
2	100804	Tube End Plug 1"	1	\$1.58
4	101123	M6 x 45 Low SHCS BLK ZC	1	\$1.58
5	101186	M5 X 45 CLS12.9 DIN912 SHCS P/THD BLZ	1	\$3.15
6	000055	Saddle 1"	2	\$1.58
7	100558	M6 NYL INS Domed Cap L/N ZC	1	\$1.05
8	100657	M5 Nylock Nut Blk Zn	1	\$0.53
9	000605	Depth Adjustable Frame Extension Tube, Catalyst/Spark	1	\$37.14
10b	000604R	Depth Adjustable Backrest Plate Inside Right CAT/ SPARK	1	\$23.21
11	000611	Depth Adjustable Backrest Side Plate CAT/ SPARK	2	\$23.21
12	000670	Depth Adjustable Backrest Latch CAT/ SPARK	1	\$32.49
13	000613R	Depth Adjustable Backrest Latch Spring Right CAT/ SPARK	1	\$2.76
14	000613L	Depth Adjustable Backrest Latch Spring Left CAT/ SPARK	1	\$2.76
15	000428	Wear Washer	2	\$2.63
16	000420	Backrest Latch Pivot Bushing	2	\$2.55
17	000612	1" Half Saddle Washer	2	\$1.92
18	000240	Backrest Latch Strike Bushing	2	\$3.57
19	101124	M6 x 50 Low SHCS BLK ZC	1	\$1.58
20	101592	M6 x 45 CLS10.9 DIN7991 FHCS BLKZC	1	\$2.63
21	100669	M5 x 10 BTN SHCS BLKZC w/Patch	2	\$1.58
22	100658	M6 Nylock Nut Blk Zn	2	\$1.05