


(Discontinued) Ethos Swing Away Armrest

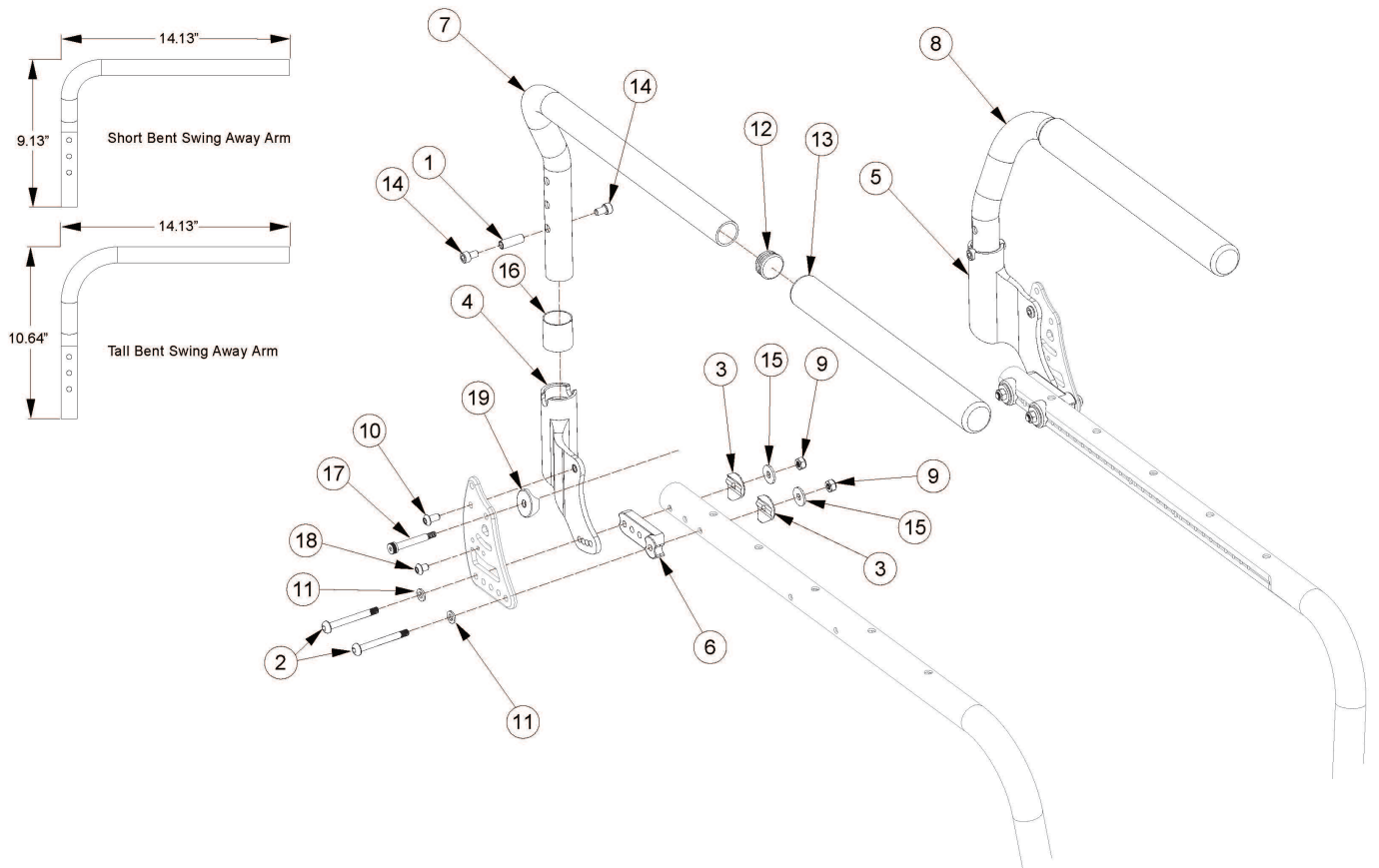

Discontinued 10/17/2022 If retrofitting, Receivers are needed.

Item Number	Part Number	Description	Quantity
1, 14	114440	Ethos Swing Away Armrest Attaching Hardware <i>One side only</i>	1
2	003232	M6 x 55 BHCS Blk Zc	2
3	003849	Saddle For Dimple 1 1/8"	2
5, 16	108163	Swing Away Arm Receiver with Nyliner, Left	1
1, 7a, 12 - 14	108166	Short Swing Away Arm Right, Ethos / Rogue	1
1, 8a, 12 - 14	108167	Short Swing Away Arm Left, Ethos / Rogue	1
1, 7a, 8a, 12 - 14	108168	Short Swing Away Arm PAIR, Ethos / Rogue	1
1, 7b, 12 - 14	108169	Tall Swing Away Arm Right, Ethos / Rogue	1
1, 8b, 12 - 14	108170	Tall Swing Away Arm Left, Ethos / Rogue	1

Item Number	Part Number	Description	Quantity
1, 7b, 8b, 12 - 14	108171	Tall Swing Away Arm PAIR, Ethos / Rogue	1
9	100658	M6 Nylock Nut Blk Zn	2
10	100667	M6 x 12 BTN SHCS BLKZC w/Patch	1
11	100746	M6 Flat Washer Blk Zn	2
12	100804	Tube End Plug 1"	1
15	101853	M6 Washer Blk Zn	2
16	103517	Nyliner 1" ID x 1.094" OD x 1.125" Long	1
17	103397	M6 x 40 Shoulder Screw BLK ZC	1
18	102621	M5 x 8 BHCS BLK ZC	1
19	004117	Saddle M6 ID x 1 OD x .25 T	1