



Ethos Luggage Carrier

Frame Mount

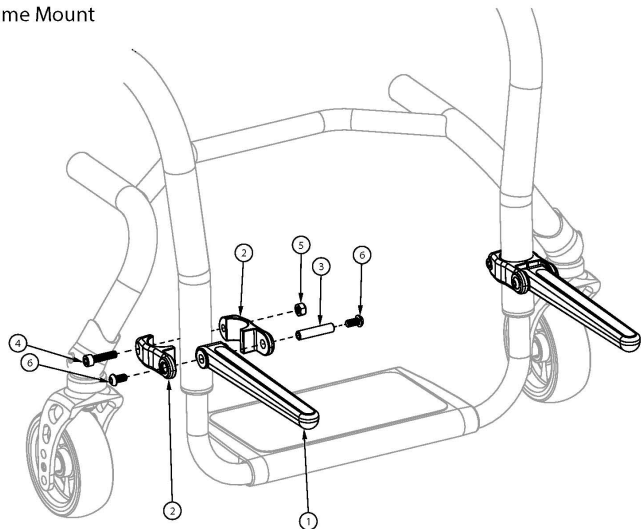

Extension Mount

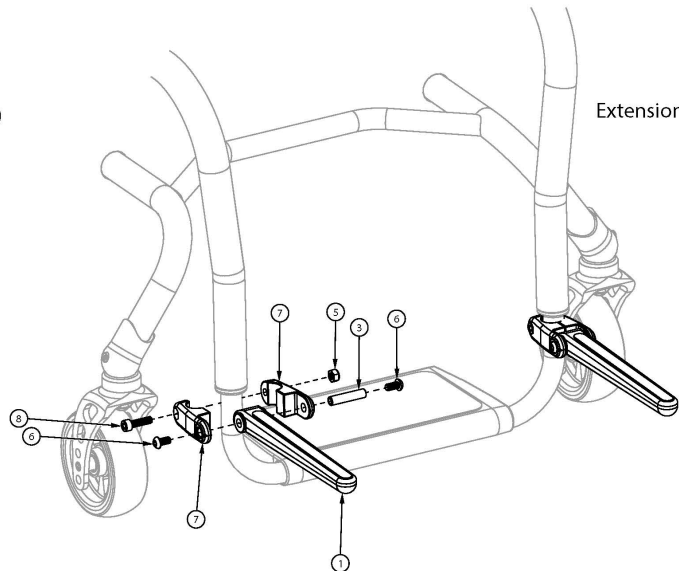

| Item Number | Part Number | Description | Quantity |
|-------------|-------------|--|----------|
| 1 - 8 | 504466 | Luggage Carrier Kit, Klik/Ethos | 1 |
| 1 | 003430 | Luggage Carrier Fork Assembly | 2 |
| 2 | 003411 | Luggage Carrier Clamp 1.125 <i>Used to mount Luggage Carrier to the frame</i> | 4 |
| 3 | 003413 | Luggage Carrier Barrel | 2 |
| 4 | 100780 | M6 x 25 SHCS ZC <i>Only used when Luggage Carrier is mounted to the frame</i> | 2 |
| 5 | 100658 | M6 Nylock Nut Blk Zn | 2 |
| 6 | 100667 | M6 x 12 BTN SHCS BLKZC w/Patch | 4 |
| 7 | 003409 | Luggage Carrier Clamp .75 <i>Used to mount Luggage Carrier to the footrest tubes/extensions</i> | 4 |
| 8 | 101845 | M6 x 22 SHCS Blk Zn <i>Only used when Luggage Carrier is mounted to the footrest tubes/extensions</i> | 2 |