


INSTRUCTION SHEET

Handgrip Replacement

WARNING: Please read the following assembly instructions completely before attempting to install handgrips.

These instructions are provided as a guideline for the installation of the Quickie handgrips. Please read them carefully and follow the step by step instructions. Sunrise Medical only recommends 90% Isopropyl Alcohol, to be used as a lubricant to install the Quickie handgrips.


Step 1. REMOVING OLD GRIP

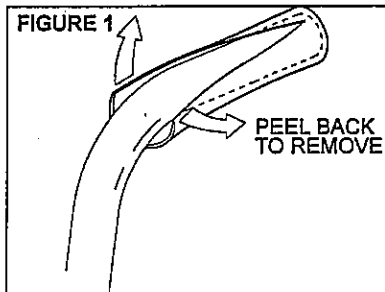

Using a small blade, cut a slit down the length of the foam handgrip. Always cut away from yourself and utilize protective gloves. Remove the foam handgrip by peeling back and pulling off (Figure 1). If excess foam or adhesive is present on the tube, remove by scraping the tube with a scraper or chisel point.

Residue may be left behind which can adversely affect the handgrip's ability to secure to the tube. Clean the tube with a clean cloth and Isopropyl Alcohol (NOTE: Do not use any other cleaner except Isopropyl Alcohol). Make sure the cloth used is clean and free of any oils or solvents.

Step 2. APPLYING LUBRICANT

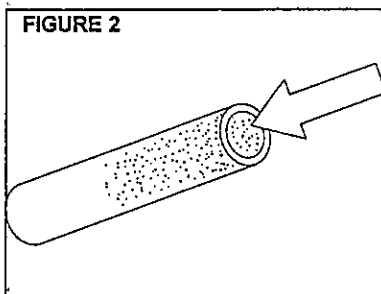


CAUTION: Do not apply alcohol until ready to install handgrip.

The 90% isopropyl alcohol has a 2- to 3-minute work time and will evaporate if installation is not immediate. Do not apply alcohol until installation instructions are read and understood.

CAUTION: Use proper eye and skin protection when handling alcohol.

Spray alcohol lubricant to the inside of the Quickie handgrip enough to ensure proper lubrication (Figure 2). Hold the Quickie handgrip with the open end facing up to allow for the alcohol to spread downward.

Step 3. INSTALLING GRIP

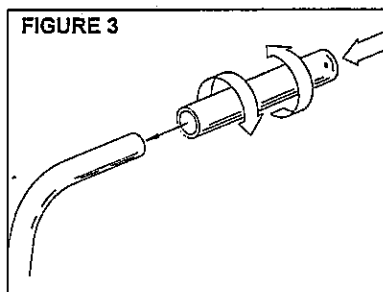

Immediately install by sliding the Quickie handgrip onto the tube. Rotate the Quickie handgrip back and forth, while concurrently using a steady force to slide handgrip onto the back tube (Figure 3). Push and cover the small hole at the end of the grip. This will create an air bubble to aid in the installation of the handgrip. Slide the handgrip until it is completely on the back tube. Wipe off excess alcohol with a clean cloth.

Allow one hour for alcohol to evaporate before any attempts to pull or rotate the handgrip.

Any questions or concerns relating to the replacement of any foam grip can be handled by our customer service representatives at 1-800-456-8165.

FOR DEALER USE Installed by: _____ Date: _____ (retain as record of installation)