


Focus CR Height Adjustable Dual Post Flip Back Armrest

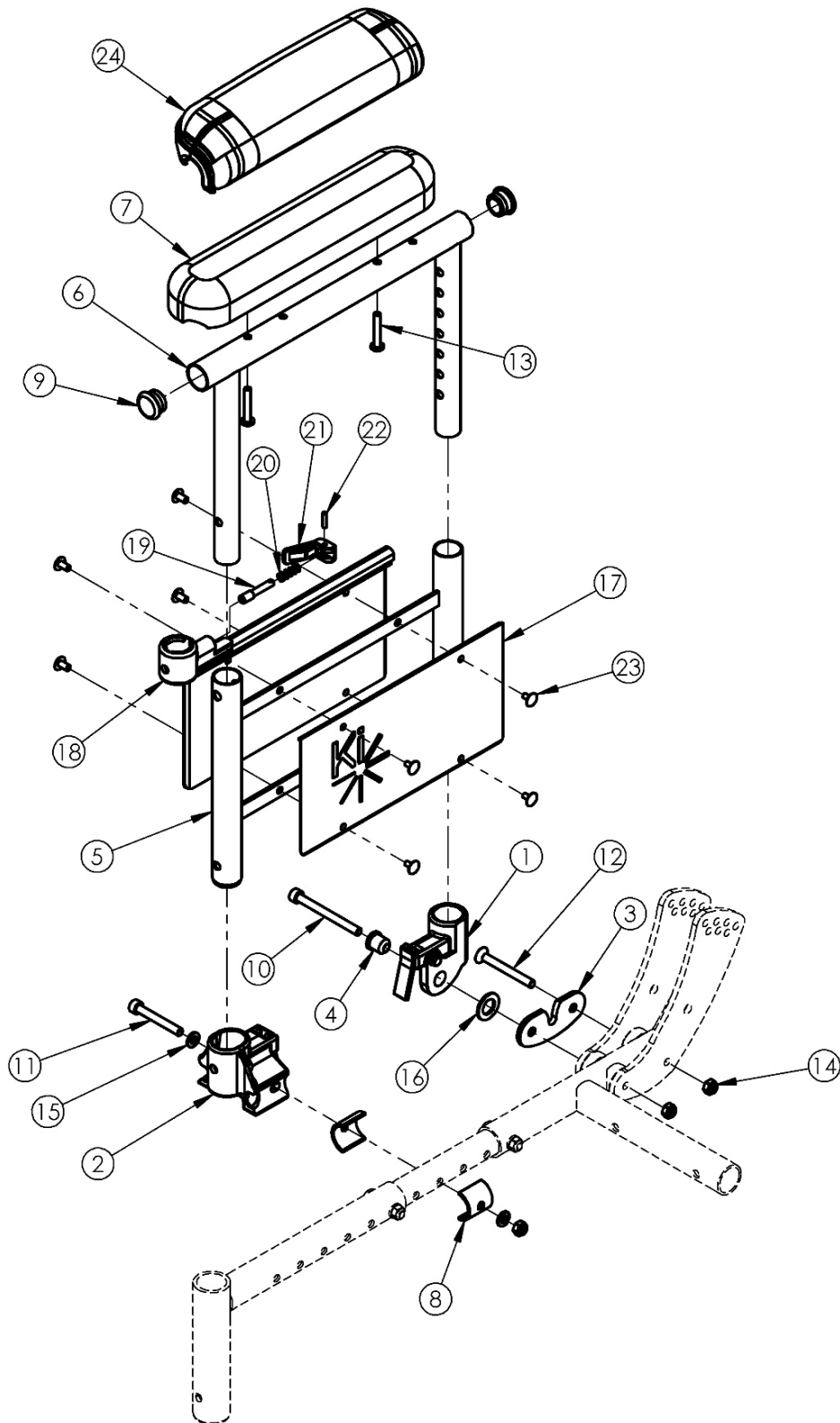

Item Number	Part Number	Description	Quantity	Retail
1a, 3, 4, 10, 12, 14-16	502290	Dual Post Armrest Rear Receiver Assembly w/ Hardware Right Tilt	1	\$121.17
1b, 3, 4, 10, 12, 14-16	502291	Dual Post Armrest Rear Receiver Assembly w/ Hardware Left Tilt	1	\$121.17
2a, 8, 11, 14, 15	502292	Dual Post Armrest Front Receiver Assembly w/ Hardware Right Tilt	1	\$96.37
2b, 8, 11, 14, 15	502293	Dual Post Armrest Front Receiver Assembly w/ Hardware Left Tilt	1	\$96.37
5b, 6a, 7a, 8, 9, 13a, 17-23	503301	Armrest Dual Post Kit Right w/ Desk Pad Focus <i>Does Not Include Receivers or Mounting Hardware</i>	1	\$94.10
5a, 6a, 7a, 8, 9, 13a, 17-23	503302	Armrest Dual Post Kit Left w/ Desk Pad Focus <i>Does Not Include Receivers or Mounting Hardware</i>	1	\$94.10
5b, 6b, 7b, 8, 9, 13a, 17-23	503303	Armrest Dual Post Kit Right w/ Full Pad Focus <i>Does Not Include Receivers or Mounting Hardware</i>	1	\$94.10
5a, 6b, 7b, 8, 9, 13a, 17-23	503304	Armrest Dual Post Kit Left w/ Full Pad Focus <i>Does Not Include Receivers or Mounting Hardware</i>	1	\$94.10
5b, 6a, 8, 9, 13b, 17-23, 24b	503305	Armrest Dual Post Kit Right w/ Desk Waterfall Pad Focus <i>Does Not Include Receivers or Mounting Hardware</i>	1	\$103.72
5a, 6a, 8, 9, 13b, 17-23, 24b	503306	Armrest Dual Post Kit Left w/ Desk Waterfall Pad Focus <i>Does Not Include Receivers or Mounting Hardware</i>	1	\$103.72
5b, 6b, 8, 9, 13b, 17-23, 24a	503307	Armrest Dual Post Kit Right w/ Full Waterfall Pad Focus <i>Does Not Include Receivers or Mounting Hardware</i>	1	\$103.72
5a, 6b, 8, 9, 13b, 17-23, 24a	503308	Armrest Dual Post Kit Left w/ Full Waterfall Pad Focus <i>Does Not Include Receivers or Mounting Hardware</i>	1	\$103.72
1a, 2a, 3, 4, 5a, 6a, 7a, 8- 12, 13a, 14-23	502294	Armrest Dual Post Kit Right Desk Pad w/ Receivers Focus	1	\$292.22
1b, 2b, 3, 4, 5b, 6a, 7a, 8- 12, 13a, 14-23	502295	Armrest Dual Post Kit Left Desk Pad w/ Receivers Focus	1	\$292.22
1a, 2a, 3, 4, 5a, 6b, 7b, 8- 12, 13a, 14-23	502296	Armrest Dual Post Kit Right Full Pad w/ Receivers Focus	1	\$292.22
1b, 2b, 3, 4, 5b, 6b, 7b, 8- 12, 13a, 14-23	502297	Armrest Dual Post Kit Left Full Pad w/ Receivers Focus	1	\$292.22
1a, 2a, 3, 4, 5a, 6a, 8-12, 13b, 14-23, 24b	110032	Armrest Dual Post Kit Right Desk Waterfall Pad w/ Receivers Focus	1	\$301.83
1b, 2b, 3, 4, 5b, 6a, 8-12, 13b, 14-23, 24b	110033	Armrest Dual Post Kit Left Desk Waterfall Pad w/ Receivers Focus	1	\$301.83

Item Number	Part Number	Description	Quantity	Retail
1a, 2a, 3, 4, 5a, 6b, 8-12, 13b, 14-23, 24b	110034	Armrest Dual Post Kit Right Full Waterfall Pad w/ Receivers Focus	1	\$301.83
1b, 2b, 3, 4, 5b, 6b, 8-12, 13b, 14-23, 24b	110035	Armrest Dual Post Kit Left Full Waterfall Pad w/ Receivers Focus	1	\$301.83
1-5, 6a, 7a, 8-12, 13a, 14-23	501139	Armrest Assembly Dual Post Desk Pad PAIR w/ Receivers Focus	1	\$584.41
1-5, 6b, 7b, 8-12, 13a, 14-23	501138	Armrest Assembly Dual Post Full Pad PAIR w/ Receivers Focus	1	\$584.41
1-5, 6a, 8-12, 13b, 14-23, 24b	110036	Armrest Assembly Dual Post Desk Waterfall Pad PAIR w/ Receivers Focus	1	\$603.65
1-5, 6b, 8-12, 13b, 14-23, 24b	110037	Armrest Assembly Dual Post Full Waterfall Pad PAIR w/ Receivers Focus	1	\$603.65
1a	001173	Dual Post Armrest Rear Receiver Assembly Right Tilt	1	\$60.67
1b	001174	Dual Post Armrest Rear Receiver Assembly Left Tilt	1	\$60.67
2a	001175	Dual Post Armrest Front Receiver Assembly Right Tilt	1	\$68.40
2b	001176	Dual Post Armrest Front Receiver Assembly Left Tilt	1	\$68.40
3	001182	Mount Plate Dual Post Armrest Tilt	1	\$9.44
4	001183	Bushing Dual Post Armrest Tilt	1	\$27.41
5a, 17-23	001232	Dual Post Armrest Frame Assembly Lower Left Tilt	1	\$113.62
5b, 17-23	001231	Dual Post Armrest Frame Assembly Lower Right Tilt	1	\$113.62
6a	001213	Dual Post Armrest Frame Upper Desk Tilt	1	\$71.70
6b	001214	Dual Post Armrest Frame Upper Full Tilt	1	\$90.90
7a	100501	Arm Pad Desk	1	\$24.82
7b	100500	Arm Pad Full	1	\$24.82
8	001230	Tube Spacer 1 x .083	2	\$3.54
9	100803	Tube End Plug 7/8"	2	\$1.66
10	101849	M6 x 65 SHCS Blk Zn	1	\$2.21
11	101472	M6 x 45 SHCS BLK ZC	1	\$1.66
12	101130	M6 x 55 FSHCS P/THD BLKZC	1	\$2.21
13a	100696	M5 x 30mm PPH Capscrew <i>Used with Standard Arm Pads</i>	2	\$1.66

Item Number	Part Number	Description	Quantity	Retail
13b	100654	M5 x 40 DIN7985 PH Pan HD M/S ZC <i>Used with Waterfall Arm Pads</i>	2	\$1.66
14	100658	M6 Nylock Nut Blk Zn	3	\$1.10
15	100746	M6 Flat Washer Blk Zn	4	\$1.66
16	102003	Washer, Nylon .51 ID X .875 OD X .062	1	\$0.56
24a	102452	Waterfall Arm Pad Full	1	\$34.44
24b	102521	Waterfall Arm Pad Desk	1	\$34.44