


Angle Adjustable Footplate - Extension Mount

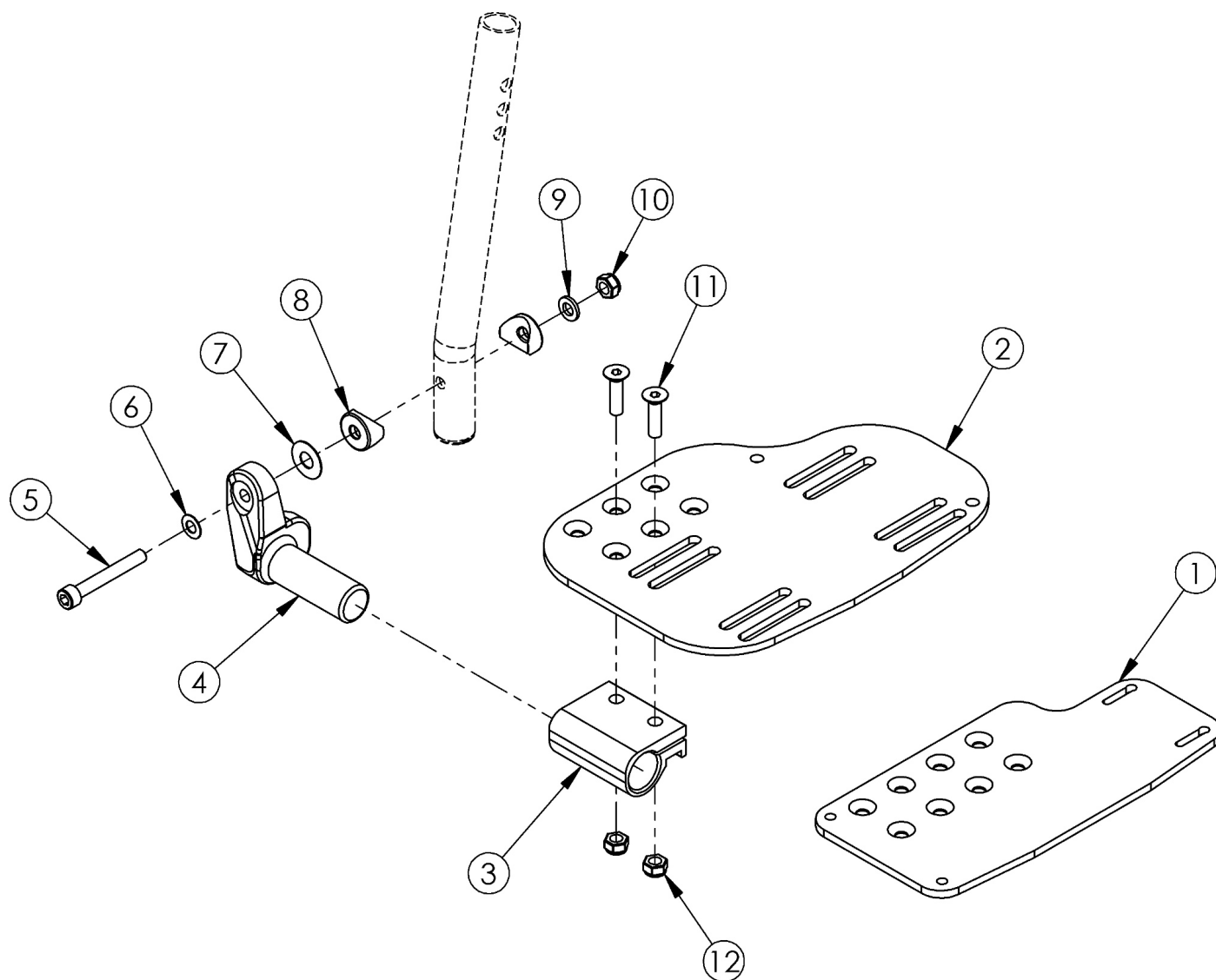

Sizes for footplates/assemblies are based on seat width with straight frame; if Tilt chair has Offset Frame, add 2" to seat width. If changing footplate style, new extension tubes may be needed.

| Item Number | Part Number | Description | Quantity |
|-----------------|-------------|------------------------------------------------------------------------|----------|
| 3, 4b, 5-12 | 502202 | Footplate Hardware Assembly Angle Adjustable Extension Mount/ELR Left | 1 |
| 3, 4a, 5-12 | 502203 | Footplate Hardware Assembly Angle Adjustable Extension Mount/ELR Right | 1 |
| 3, 4a, 4b, 5-12 | 502204 | Footplate Hardware Assembly Angle Adjustable Extension Mount/ELR Pair | 1 |
| 1a, 3, 4b, 5-12 | 501049 | Footplate Angle Adjustable Extension Mount Assembly 10-11 Wide Left | 1 |
| 1b, 3, 4a, 5-12 | 501050 | Footplate Angle Adjustable Extension Mount Assembly 10-11 Wide Right | 1 |

| Item Number | Part Number | Description | Quantity |
|-------------------------|-------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------|----------|
| 1a, 1b, 3, 4a, 4b, 5-12 | 501051 | Footplate Angle Adjustable Extension Mount Assembly 10-11 Wide PAIR | 1 |
| 1c, 3, 4b, 5-12 | 501052 | Footplate Angle Adjustable Extension Mount Assembly 12-13 Wide Left | 1 |
| 1d, 3, 4a, 5-12 | 501053 | Footplate Angle Adjustable Extension Mount Assembly 12-13 Wide Right | 1 |
| 1c, 1d, 3, 4a, 4b, 5-12 | 501054 | Footplate Angle Adjustable Extension Mount Assembly 12-13 Wide Pair | 1 |
| 2a, 3, 4b, 5-12 | 100252 | Footplate Angle Adjustable 14-15 Wide Extension Mount Left/ELR Right Assembly | 1 |
| 2b, 3, 4a, 5-12 | 100253 | Footplate Angle Adjustable 14-15 Wide Extension Mount Right/ELR Left Assembly | 1 |
| 2a, 2b, 3, 4a, 4b, 5-12 | 100254 | Footplate Angle Adjustable 14-15 Wide Extension Mount/ELR PAIR Assembly | 1 |
| 2c, 3, 4b, 5-12 | 100255 | Footplate Angle Adjustable 16-17 Wide Extension Mount Left/ELR Right Assembly | 1 |
| 2d, 3, 4a, 5-12 | 100256 | Footplate Angle Adjustable 16-17 Wide Extension Mount Right/ELR Left Assembly | 1 |
| 2c, 2d, 3, 4a, 4b, 5-12 | 100257 | Footplate Angle Adjustable 16-17 Wide Extension Mount/ELR PAIR Assembly | 1 |
| 2e, 3, 4b, 5-12 | 100258 | Footplate Angle Adjustable 18-24 Wide Extension Mount Left/ELR Right Assembly | 1 |
| 2f, 3, 4a, 5-12 | 100259 | Footplate Angle Adjustable 18-24 Wide Extension Mount Right/ELR Left Assembly | 1 |
| 2e, 2f, 3, 4a, 4b, 5-12 | 100260 | Footplate Angle Adjustable 18-24 Wide Extension Mount/ELR PAIR Assembly | 1 |
| 2a, 3, 4b, 5-12 | 502184 | Footplate Kit Angle Adjustable w/ Heel Loop for Ext. Mount Hanger 14-15 Wide Left <i>Heel Loop with attaching hardware is included but not pictured</i> | 1 |
| 2b, 3, 4a, 5-12 | 502185 | Footplate Kit Angle Adjustable w/ Heel Loop for Ext. Mount Hanger 14-15 Wide Right <i>Heel Loop with attaching hardware is included but not pictured</i> | 1 |
| 2a, 2b, 3, 4a, 4b, 5-12 | 502186 | Footplate Kit Angle Adjustable w/ Heel Loop for Ext. Mount Hanger 14-15 Wide Pair <i>Heel Loop with attaching hardware is included but not pictured</i> | 1 |
| 2c, 3, 4b, 5-12 | 502187 | Footplate Kit Angle Adjustable w/ Heel Loop for Ext. Mount Hanger 16-17 Wide Left <i>Heel Loop with attaching hardware is included but not pictured</i> | 1 |
| 2d, 3, 4a, 5-12 | 502188 | Footplate Kit Angle Adjustable w/ Heel Loop for Ext. Mount Hanger 16-17 Wide Right <i>Heel Loop with attaching hardware is included but not pictured</i> | 1 |
| 2c, 2d, 3, 4a, 4b, 5-12 | 502189 | Footplate Kit Angle Adjustable w/ Heel Loop for Ext. Mount Hanger 16-17 Wide Pair <i>Heel Loop with attaching hardware is included but not pictured</i> | 1 |
| 2e, 3, 4b, 5-12 | 502190 | Footplate Kit Angle Adjustable w/ Heel Loop for Ext. Mount Hanger 18-24 Wide Left <i>Heel Loop with attaching hardware is included but not pictured</i> | 1 |
| 2f, 3, 4a, 5-12 | 502191 | Footplate Kit Angle Adjustable w/ Heel Loop for Ext. Mount Hanger 18-24 Wide Right | 1 |

| Item Number | Part Number | Description | Quantity |
|-------------------------|-------------|------------------------------------------------------------------------------------------------------------------------------------------------------------|----------|
| | | <i>Heel Loop with attaching hardware is included but not pictured</i> | |
| 2e, 2f, 3, 4a, 4b, 5-12 | 502192 | Footplate Kit Angle Adjustable w/ Heel Loop for Ext. Mount Hanger 18-24 Wide Pair <i>Heel Loop with attaching hardware is included but not pictured</i> | 1 |
| 1a | 001304 | Footplate Angle Adjustable 10-11" Wide CAT/TILT Left | 1 |
| 1b | 001305 | Footplate Angle Adjustable 10-11" Wide CAT/TILT Right | 1 |
| 1c | 001306 | Footplate Angle Adjustable 12-13" Wide CAT/TILT Left | 1 |
| 1d | 001307 | Footplate Angle Adjustable 12-13" Wide CAT/TILT Right | 1 |
| 2a | 000105 | Footplate Angle Adjustable 14-15" Wide CAT/TILT Left (Std ELR Right) | 1 |
| 2b | 000106 | Footplate Angle Adjustable 14-15" Wide CAT/TILT Right (Std ELR Left) | 1 |
| 2c | 000107 | Footplate Angle Adjustable 16-17" Wide CAT/TILT Left (Std ELR Right) | 1 |
| 2d | 000108 | Footplate Angle Adjustable 16-17" Wide CAT/TILT Right (Std ELR Left) | 1 |
| 2e | 000109 | Footplate Angle Adjustable 18-24" Wide CAT/TILT Left (Std ELR Right) | 1 |
| 2f | 000110 | Footplate Angle Adjustable 18-24" Wide CAT/TILT Right (Std ELR Left) | 1 |
| 3 | 100502 | Angle Adj Footplate Clamp 2" | 1 |
| 4a | 000047 | Footplate Angle Adjustable Casting 3/4 Right | 1 |
| 4b | 000049 | Footplate Angle Adjustable Casting 3/4" Left | 1 |
| 5 | 101472 | M6 x 45 SHCS BLK ZC | 1 |
| 6 | 101455 | M6 Belleville Washer (.252 x .551 x .050) | 1 |
| 7 | 000428 | Wear Washer | 1 |
| 8 | 000053 | Saddle 3/4" | 2 |
| 9 | 100746 | M6 Flat Washer Blk Zn | 1 |
| 10 | 120030 | M6 Top Lock Nut | 1 |
| 11 | 100662 | M6 x 25 FSHCS BLK ZC | 2 |
| 12 | 100658 | M6 Nylock Nut Blk Zn | 2 |