


# Liberty Caster Forks and Stems

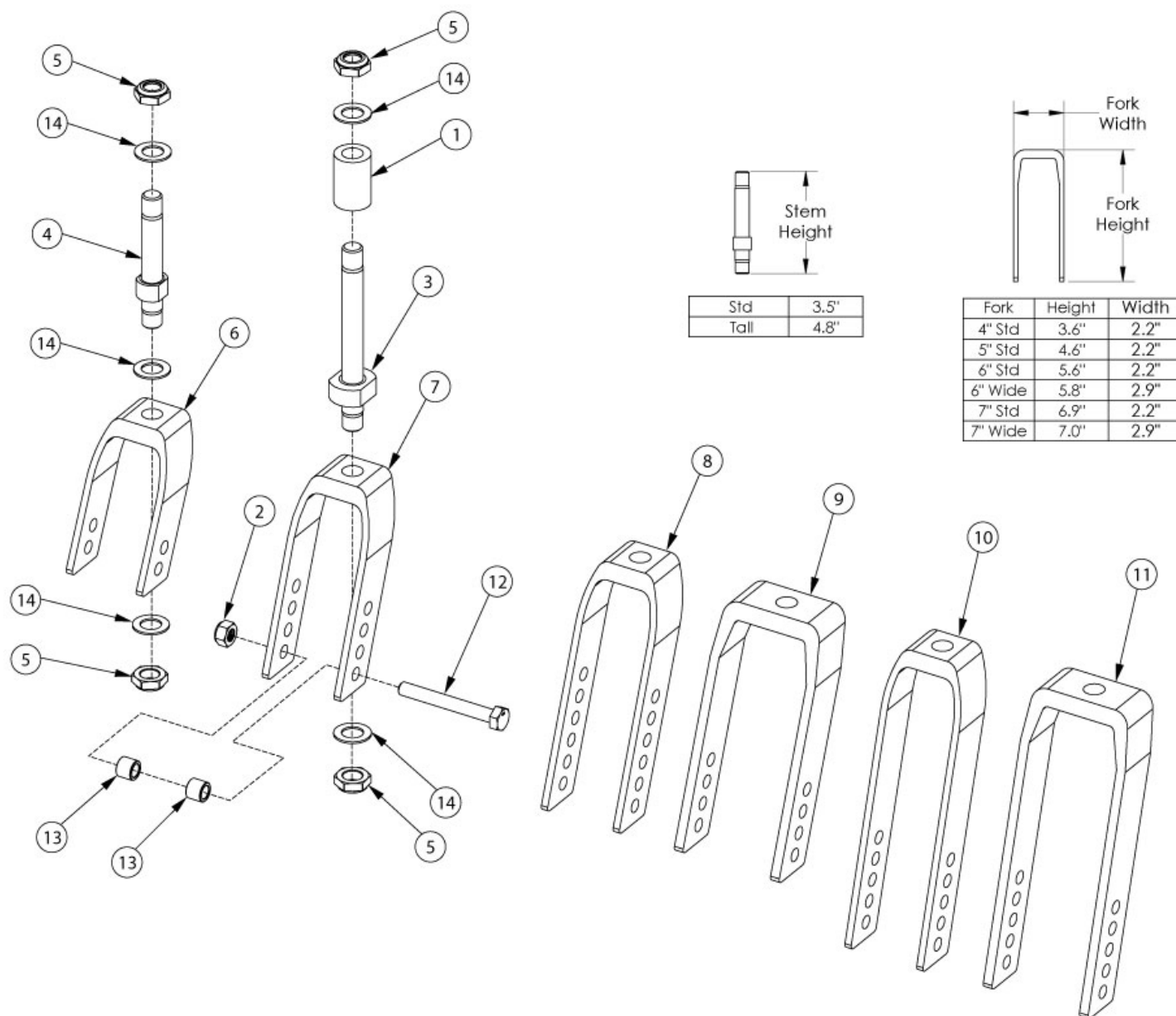

If changing to 2" casters, wide forks and hardware needed.

| Item Number | Part Number | Description                                                                   | Quantity |
|-------------|-------------|-------------------------------------------------------------------------------|----------|
| 4, 5, 6, 14 | 112904      | Fork Assembly, Black, 4" Std Stem, Tilt                                       | 1        |
| 4, 5, 7, 14 | 503816      | Fork Assembly 5" Std Stem, Tilt                                               | 1        |
| 4, 5, 8, 14 | 503814      | Fork Assembly 6" Std Stem, Tilt                                               | 1        |
| 4, 5, 9, 14 | 503817      | Fork Assembly 6" Wide Std Stem, Tilt<br><i>Only used with 2" wide casters</i> | 1        |

| Item Number     | Part Number | Description                                                                                                      | Quantity |
|-----------------|-------------|------------------------------------------------------------------------------------------------------------------|----------|
| 4, 5, 10, 14    | 503815      | Fork Assembly 7" Std Stem, Tilt                                                                                  | 1        |
| 4, 5, 11, 14    | 503818      | Fork Assembly 7" Wide Std Stem, Tilt<br><i>Only used with 2" wide casters</i>                                    | 1        |
| 1, 3, 5, 6, 14  | 506153      | Fork Assembly 4" Tall Stem, Tilt                                                                                 | 1        |
| 1, 3, 5, 7, 14  | 506154      | Fork Assembly 5" Tall Stem, Tilt                                                                                 | 1        |
| 1, 3, 5, 8, 14  | 506155      | Fork Assembly 6" Tall Stem, Tilt                                                                                 | 1        |
| 1, 3, 5, 9, 14  | 506157      | Fork Assembly 6" Wide Tall Stem, Tilt<br><i>Only used with 2" wide casters</i>                                   | 1        |
| 1, 3, 5, 10, 14 | 506156      | Fork Assembly 7" Tall Stem, Tilt                                                                                 | 1        |
| 1, 3, 5, 11, 14 | 506158      | Fork Assembly 7" Wide Tall Stem, Tilt<br><i>Only used with 2" wide casters</i>                                   | 1        |
| 1               | 002476      | Spacer, 3/4" OD x 1/2" ID x 1.23<br><i>Only used with Tall Stems</i>                                             | 1        |
| 2               | 100659      | M8 Nylock Nut Blk Zn                                                                                             | 1        |
| 3, 5            | 506490      | Caster Stem Tall, Liberty                                                                                        | 1        |
| 4, 5            | 506491      | Caster Stem Standard, Liberty                                                                                    | 1        |
| 5               | 100529      | M12 x 1.25mm Lock Nut Black                                                                                      | 2        |
| 5, 6            | 506492      | Caster Fork 4", Tilt                                                                                             | 1        |
| 5, 7            | 506493      | Caster Fork 5", Liberty                                                                                          | 1        |
| 5, 8            | 506494      | Caster Fork 6", Liberty                                                                                          | 1        |
| 5, 9            | 506495      | Caster Fork 6", Wide Liberty<br><i>Only used with 2" wide casters</i>                                            | 1        |
| 5, 10           | 506496      | Caster Fork 7", Liberty                                                                                          | 1        |
| 5, 11           | 506497      | Caster Fork 7", Wide Liberty<br><i>Only used with 2" wide casters</i>                                            | 1        |
| 12a             | 101829      | M8 x 65 HHCS Blk Zn                                                                                              | 1        |
| 12b             | 101831      | M8 x 80 HHCS Blk Zn<br><i>Only used with 2" wide casters</i>                                                     | 1        |
| 13a             | 100792      | Caster Spacer D8 x D12 x 10mm / Close Mount<br>Handrim Spacer<br><i>Used with all casters except 6x2 and 8x2</i> | 2        |
| 13b             | 001246      | Caster Spacer Black 6 x 2 Poly<br>.472ODX.319IDX.370LG<br><i>Only used with 6" x 2" casters</i>                  | 2        |

| Item Number | Part Number | Description                                                                           | Quantity |
|-------------|-------------|---------------------------------------------------------------------------------------|----------|
| 14          | 100682      | Washer 1/2" .518 x .875 x .047 F/W Black Zinc                                         | 3        |
| 2, 12a, 13a | 111379      | Caster Attaching Hardware Kit<br><i>Used will all casters except 6x2 and 8x2</i>      | 1        |
| 2, 12b, 13b | 111380      | Caster Attaching Hardware Kit for Wide Forks<br><i>Only used with 2" wide casters</i> | 1        |