


(Discontinued 1) Rogue Backrest Mount and Hardware

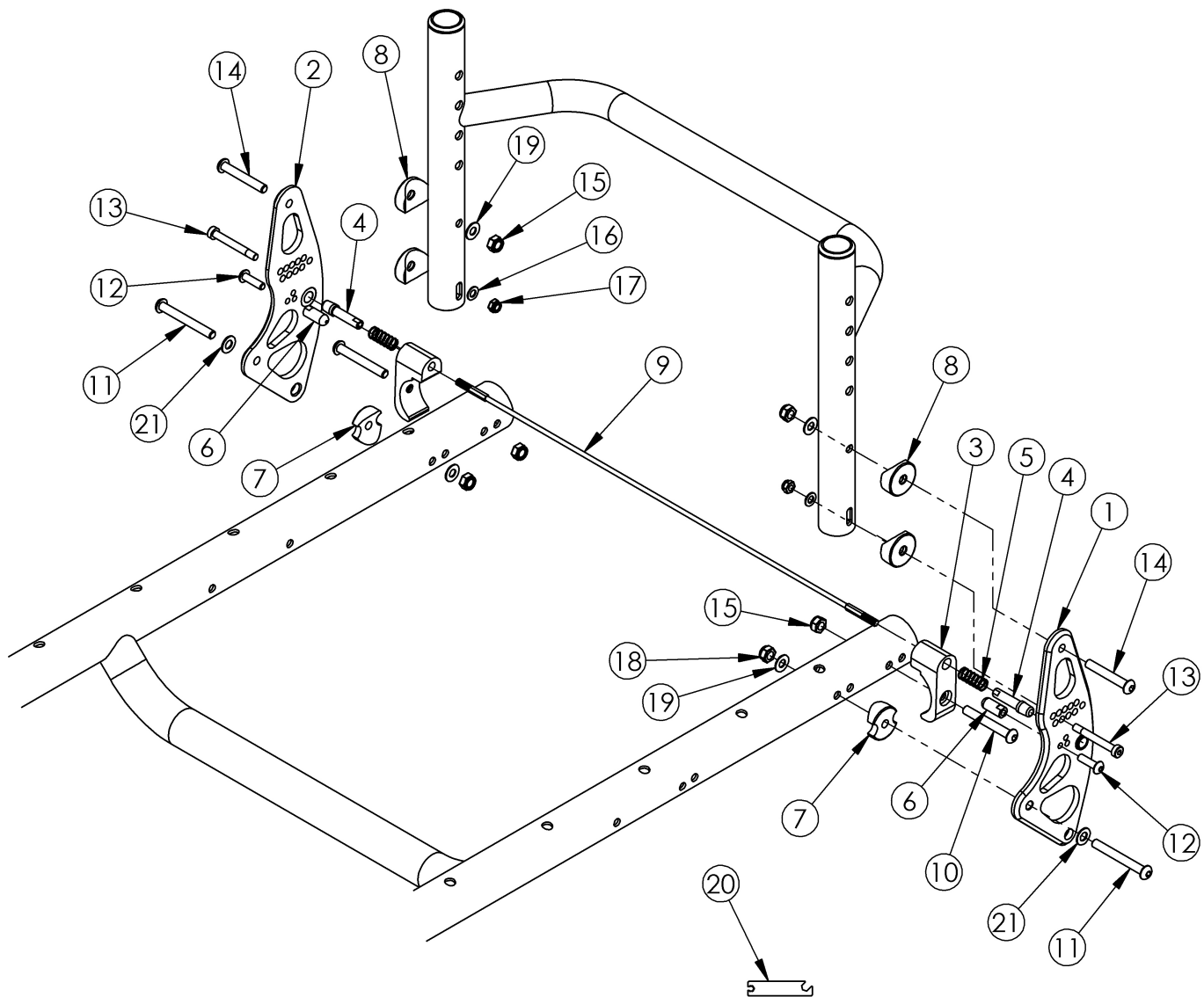

Adjustment Tool required when ordering Spinergy Back Release Cable

This Rogue Backrest Mount is discontinued and was used on serial numbers prior to RG0016612 This backrest mount and hardware is used on the Discontinued Rogue with XP option frame. If you have a Rogue XP, use the Rogue XP Backrest Mount pages for replacement parts.

Item Number	Part Number	Description	Quantity
1b, 2b, 6-8, 10, 11a, 12-19, 21	500759_ _ _	Color Anodized Backrest Plate Kit	1
1b	102600_ _ _	Color Anodized Backrest Mounting Plate Left RG	1
2a, 6	000958R	Backrest Mounting Plate w/ Pin Right Black Anodized RG	1

Item Number	Part Number	Description	Quantity
2b	102601_ _ _	Color Anodized Backrest Mounting Plate Right RG	1
3a	000910	Backrest Pin Lock Housing Black Anodized RG	2
4	000909	Backrest Lockout Pin	2
5	000965	Backrest Cable Release Spring RG	2
6	000922	Backrest Stop RG	2
7	000248	Saddle 1.375"	2
8	000055	Saddle 1"	4
9c	200262	Spinergy Back Release Cable 12" Wide Red	1
9d	200263	Spinergy Back Release Cable 12" Wide Yellow	1
9h	200267	Spinergy Back Release Cable 12" Wide White	1
9t	200271	Spinergy Back Release Cable 13" Wide Yellow	1
9w	200274	Spinergy Back Release Cable 13" Wide Orange	1
9ag	200276	Spinergy Back Release Cable 14" Wide Black	1
9am	200282	Spinergy Back Release Cable 14" Wide Orange	1
9az	200287	Spinergy Back Release Cable 15" Wide Yellow	1
9bc	200290	Spinergy Back Release Cable 15" Wide Orange	1
9bp	200295	Spinergy Back Release Cable 16" Wide Yellow	1
9bq	200296	Spinergy Back Release Cable 16" Wide Green	1
9bt	200299	Spinergy Back Release Cable 16" Wide White	1
9ce	200302	Spinergy Back Release Cable 17" Wide Red	1
9cf	200303	Spinergy Back Release Cable 17" Wide Yellow	1
9cg	200304	Spinergy Back Release Cable 17" Wide Green	1
9ci	200306	Spinergy Back Release Cable 17" Wide Orange	1
9cu	200310	Spinergy Back Release Cable 18" Wide Red	1
9cv	200311	Spinergy Back Release Cable 18" Wide Yellow	1
9cw	200312	Spinergy Back Release Cable 18" Wide Green	1
9cx	200313	Spinergy Back Release Cable 18" Wide Pink	1
9cy	200314	Spinergy Back Release Cable 18" Wide Orange	1

Item Number	Part Number	Description	Quantity
9cz	200315	Spinergy Back Release Cable 18" Wide White	1
9di	200316	Spinergy Back Release Cable 19" Wide Black	1
9dy	200324	Spinergy Back Release Cable 20" Wide Black	1
9eo	200332	Spinergy Back Release Cable 21" Wide Black	1
9fe	200340	Spinergy Back Release Cable 22" Wide Black	1
10	100670	M6 x 45 BTN SHCS BLK ZC	2
11a	003232	M6 x 55 BHCS Blk Zc <i>Used on chairs with 0" or positive center of gravity</i>	2
11b	003241	M6 x 60 BHCS P/THD Custom Blk Zc <i>Used on chairs with negative center of gravity</i>	2
12	101716	M5 x 16 SHCS Blk Zn	2
13	101121	M5 x 40 CLS8.8 DIN7984 Low SHCS BLKZC	2
14	100797	M6 x 40 BHCS BLK ZC	2
15	100658	M6 Nylock Nut Blk Zn	4
16	101856	Flat Washer, M5, Blk Zn	2
17	100657	M5 Nylock Nut Blk Zn	2
18	101456	M6 Ovl Top L/N ZC	2
19	102072	Washer Nylon .234 ID x .406 OD x .062 Thick	2
20	101991	Spinergy Release Cable/SPOX Adjustment Tool <i>No charge when ordered with Spinergy Release Cable</i>	1
21	000369	Flat Washer .5 x .257 x .0032 Plastic	2

Color Anodizing Samples

