


# Catalyst 5 Performance Caster Housing for Swing Away Frame

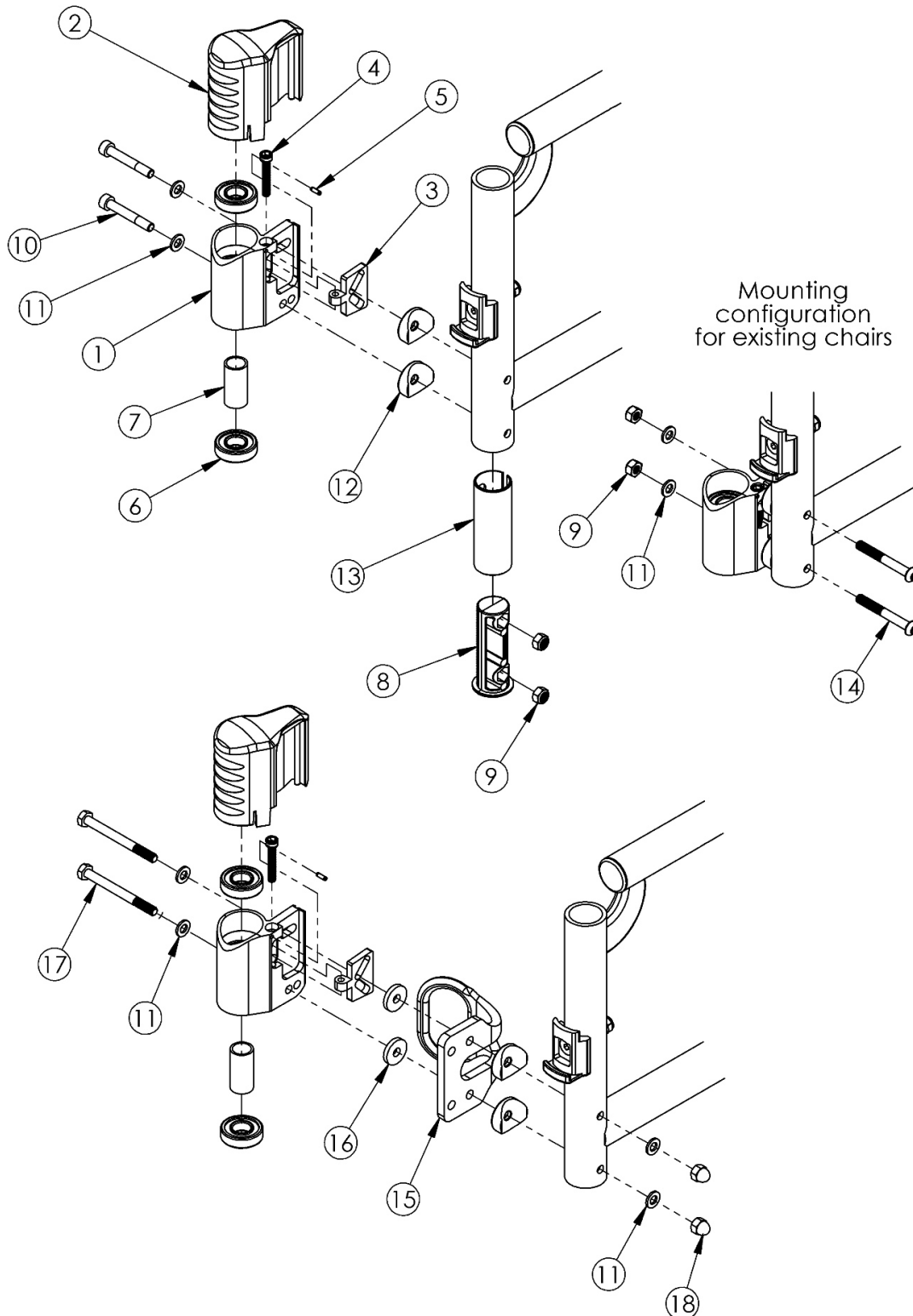

Not available with Heavy Duty Option Please Note: The shown orientation of the caster housing above is in the "Forward" position. If only 1 side caster housing is needed, customer must confirm the orientation of the caster housing.

| Item Number                           | Part Number | Description  | Quantity |
|---------------------------------------|-------------|--|----------|
| 1a, 2a, 3 - 12                        | 502414      | Performance Caster Housing Kit Right                                   | 1        |
| 1b, 2b, 3 - 12                        | 502415      | Performance Caster Housing Kit Left                                    | 1        |
| 1a, 1b, 2a, 2b, 3 - 12                | 502416      | Performance Caster Housing Kit PAIR                                    | 1        |
| 1a, 2a, 3 - 13                        | 502417      | Performance Caster Housing Kit Titanium Right                          | 1        |
| 1b, 2b, 3 - 13                        | 502418      | Performance Caster Housing Kit Titanium Left                           | 1        |
| 1a, 1b, 2a, 2b, 3 - 13                | 502419      | Performance Caster Housing Kit Titanium PAIR                           | 1        |
| 1a, 2a, 3 - 12, 14                    | 502426      | Retro Fit Performance Caster Housing Kit Right                         | 1        |
| 1b, 2b, 3 - 12, 14                    | 502427      | Retro Fit Performance Caster Housing Kit Left                          | 1        |
| 1a, 1b, 2a, 2b, 3 - 12, 14            | 502428      | Retro Fit Performance Caster Housing Kit PAIR                          | 1        |
| 1a, 2a, 3-7, 9, 11, 12, 15-18         | 502429      | Performance Caster Housing Kit for Transit Right                       | 1        |
| 1b, 2b, 3-7, 9, 11, 12, 15-18         | 502430      | Performance Caster Housing Kit for Transit Left                        | 1        |
| 1a, 1b, 2a, 2b, 3-7, 9, 11, 12, 15-18 | 502431      | Performance Caster Housing Kit for Transit PAIR                        | 1        |
| 1a, 3, 4, 5                           | 502522      | Caster Barrel RP Right w/ Adjustment Plate CAT                         | 1        |
| 1b, 3, 4, 5                           | 502523      | Caster Barrel RP Left w/ Adjustment Plate CAT                          | 1        |
| 2a                                    | 002442      | Cover Caster Barrel Right CAT  | 1        |
| 2b                                    | 002443      | Cover Caster Barrel Left CAT   | 1        |
| 6                                     | 100902      | Bearing 1 1/8" x 1/2" R8RS   | 2        |
| 7                                     | 002447      | Spacer Caster Barrel   | 1        |
| 8                                     | 002478      | Front Frame Plug CAT   | 1        |
| 9                                     | 100658      | M6 Nylock Nut Blk Zn   | 2        |
| 10                                    | 002575      | M6 X 1.0 X 40 MM SHCS Blk  | 2        |
| 11                                    | 100746      | M6 Flat Washer Blk Zn  | 2        |
| 12                                    | 000056      | Saddle 1 1/8"  | 2        |
| 13                                    | 002768      | Front Frame Insert for Ti Frame<br><i>Only used on Titanium Frames</i> | 1        |
| 14                                    | 101781      | M6 x 50mm Partial Thread BHCS Blk ZC                                   | 2        |

| <b>Item Number</b> | <b>Part Number</b> | <b>Description</b>                        | <b>Quantity</b> |
|--------------------|--------------------|---|-----------------|
| 16                 | 102814             | Washer Nylon 1/4 x 3/4 x 1/8              | 2               |
| 17                 | 100679             | M6 x 65 CLS 8.8 DIN931 HHCS ZC            | 2               |
| 18                 | 100558             | M6 NYL INS Domed Cap L/N ZC               | 2               |
| 1a, 6, 7           | 002888             | RP Tuning Caster Barrel w/ Bearings Right | 1               |
| 1b, 6, 7           | 002889             | RP Tuning Caster Barrel w/ Bearings Left  | 1               |