

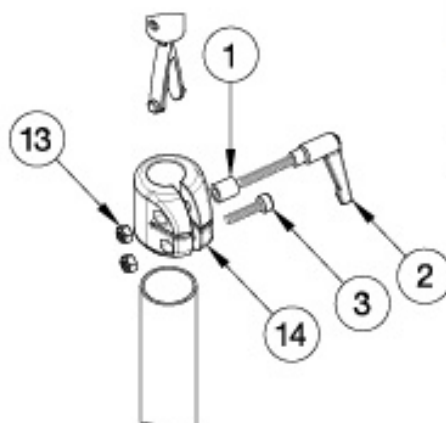



Catalyst IV Pole

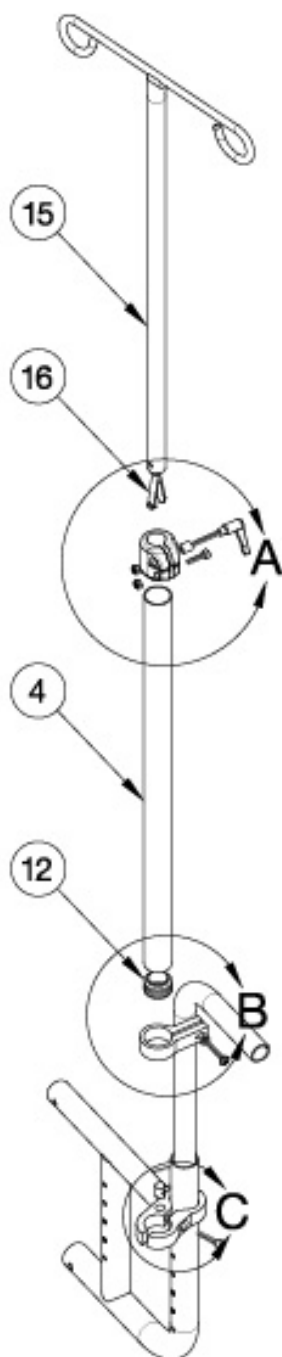

DETAIL A

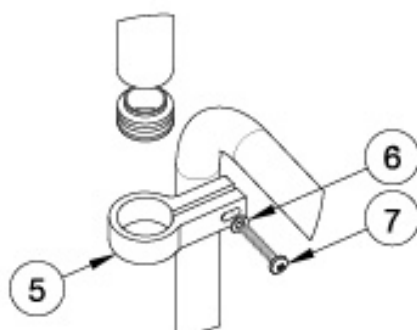

DETAIL B

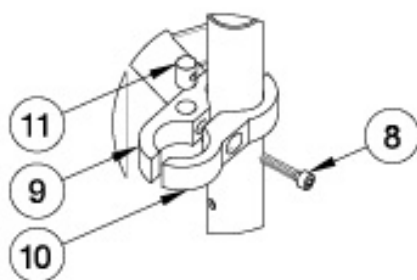

DETAIL C

Part #	Length	Back Height
002670	20"	9" - 12"
002669	24"	13" - 16"
002581	28"	17"+

If retro fitting a Cat4, straight back canes will be needed.

Item Number	Part Number	Description	Quantity
1-3, 4a, 5-7, 8a, 8b, 9-16	502404	IV Pole Assembly 9-12" Back Height, Catalyst	1
1-3, 4b, 5-7, 8a, 8b, 9-17	502405	IV Pole Assembly 13-16" Back Height, Catalyst/Liberty	1
1-3, 4c, 5-7, 8a, 8b, 9-18	502406	IV Pole Assembly 17-20" Back Height, Catalyst/Liberty	1
1, 2, 3, 13, 14	502401	IV Pole Top Clamp Assembly	1
5, 6, 7	502402	IV Pole Middle Clamp Assembly, Catalyst	1
8a, 8b, 9, 10, 11	502403	IV Pole Bottom Clamp Assembly, Catalyst	1
1	102766	Spacer 3/8R. X 7/16 x .218 ID.	1
2	102767	Handle wM5 x 32L Threaded Stud	1
3	101843	M5 x 20 SHCS Blk Zn	1
4a	002670	IV Pole Outer Tube 20" Height	1
4b	002669	IV Pole Outer Tube 24" Height	1
4c	002581	IV Pole Outer Tube 28" Height	1
5	002707	IV Pole Clamp Single 1-1/8"	1
6	101856	Flat Washer, M5, Blk Zn	1
7	100696	M5 x 30mm PPH Capscrew	1
8a	100780	M6 x 25 SHCS ZC <i>Used on Catalyst 5 chairs</i>	1
8b	102627	M6 X 30 SHCS BLK ZC <i>Used on Catalyst 4 chairs</i>	1
9	002685	IV Pole Clamp 1-1/8" 1" Thru 1-1/4" w/ Cross Dowel Hole	1
10	002686	IV Pole Clamp 1-1/8" 1" Thru 1-1/4"	1
11	102768	M6 Barrel Nut	1
12	100532	Tube End Plug 1 1/8"	1
13	100657	M5 Nylock Nut Blk Zn	2
14	002395	Footrest Clamp Left LW	1
15	002585	IV Pole Upper	1
16	102769	Snap Button Plastic	1