


CR45 Forks and Stems

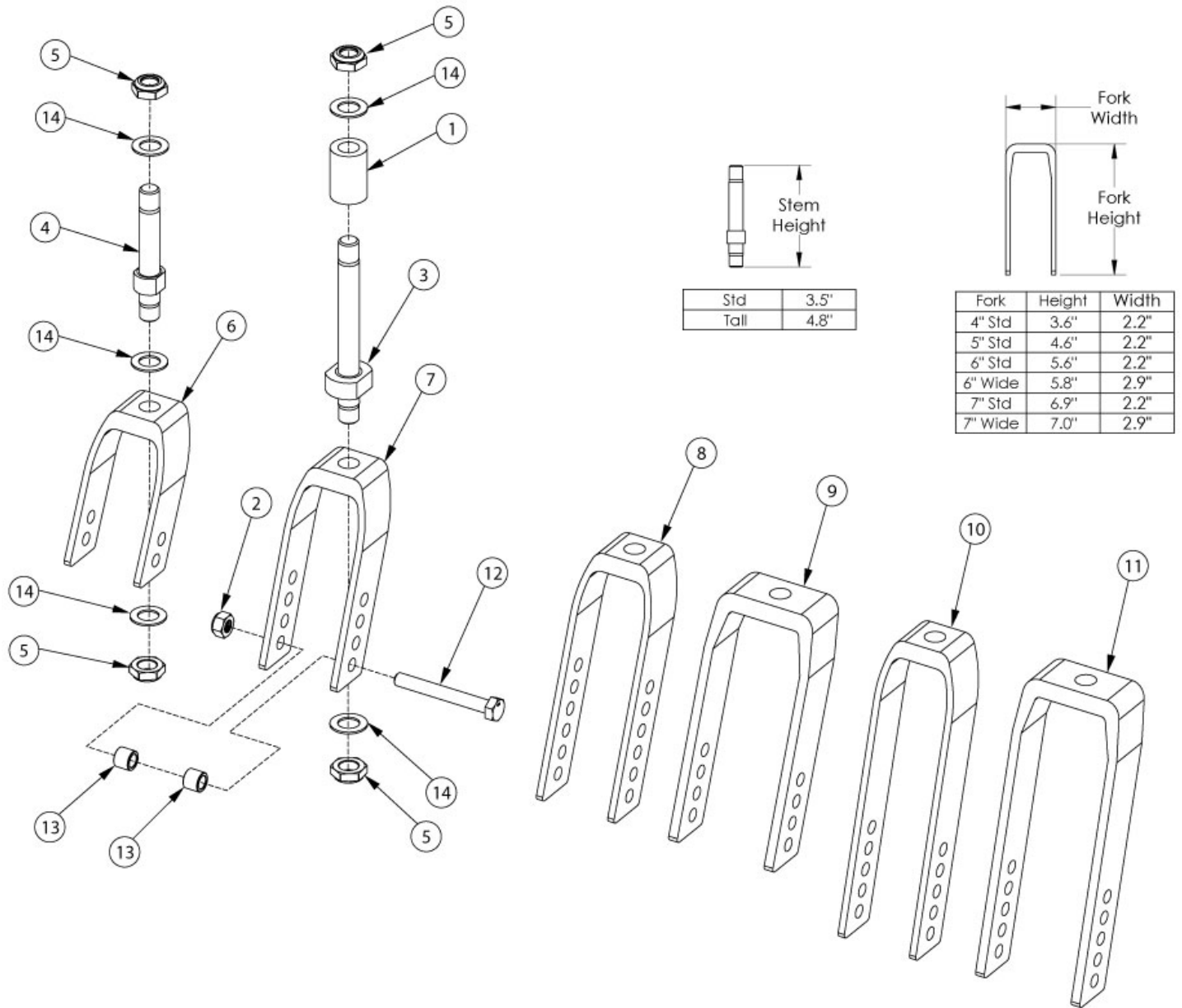

Item Number	Part Number	Description	Quantity
4, 5, 6, 14	112904	Fork Assembly, Black, 4" Std Stem, Tilt	1
4, 5, 7, 14	503816	Fork Assembly 5" Std Stem, Tilt	1
4, 5, 8, 14	503814	Fork Assembly 6" Std Stem, Tilt	1
4, 5, 9, 14	503817	Fork Assembly 6" Wide Std Stem, Tilt <i>Only used with 2" wide casters.</i>	1
4, 5, 10, 14	503815	Fork Assembly 7" Std Stem, Tilt	1

Item Number	Part Number	Description	Quantity
4, 5, 11, 14	503818	Fork Assembly 7" Wide Std Stem, Tilt <i>Only used with 2" wide casters.</i>	1
1, 3, 5, 6, 14	506153	Fork Assembly 4" Tall Stem, Tilt	1
1, 3, 5, 7, 14	506369	Fork Assembly 5" Tall Stem Focus	1
1, 3, 5, 8, 14	506370	Fork Assembly 6" Tall Stem Focus	1
1, 3, 5, 9, 14	506371	Fork Assembly 6" Wide Tall Stem Focus <i>Only used with 2" wide casters.</i>	1
1, 3, 5, 10, 14	506372	Fork Assembly 7" Tall Stem Focus	1
2, 12a, 13a	502653	Caster Attachment Hardware Kit CAT/FCS	1
2, 12b, 13b	502654	Caster Attachment Hardware Kit for 6" and 7" Wide Fork, CAT/FCS <i>Only used with 2" wide casters</i>	1
1	002476	Spacer, 3/4" OD x 1/2" ID x 1.23	1
2	100659	M8 Nylock Nut Blk Zn	1
1, 3, 5, 14	506374	Caster Stem Tall, Adult Tilt	1
4, 5, 14	503819	Caster Stem Standard, Focus	1
5	100529	M12 x 1.25mm Lock Nut Black	2
5, 6	506492	Caster Fork 4", Liberty	1
5, 7	503823	Caster Fork 5" Focus	1
5, 8	503824	Caster Fork 6" Focus	1
5, 9	503825	Caster Fork 6" Wide Focus <i>Only used with 2" wide casters</i>	1
5, 10	503826	Caster Fork 7" Focus	1
5, 11	503827	Caster Fork 7" Wide Focus <i>Only used with 2" wide casters</i>	1
12a	101829	M8 x 65 HHCS Blk Zn	1
12b	101831	M8 x 80 HHCS Blk Zn <i>Only used with 2" wide casters</i>	1
13a	100792	Caster Spacer D8 x D12 x 10mm / Close Mount Handrim Spacer <i>Used with all casters except 6x2</i>	2
13b	001246	Caster Spacer Black 6 x 2 Poly .472ODX.319IDX.370LG <i>Only used with 6" x 2" casters</i>	2

Item Number	Part Number	Description	Quantity
14	100682	Washer 1/2" .518 x .875 x .047 F/W Black Zinc	3