






<u>/!</u>\

Please read these instructions carefully before beginning the assembly. Failure to understand and follow assembly instructions may result in injury to technician and/or end user and may void the warranty. If you have any questions call Sunrise Medical Technical Support at +1 (800) 333-4000.

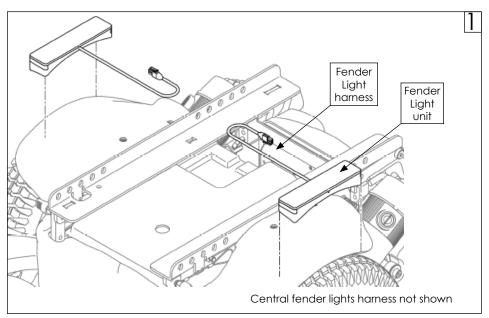

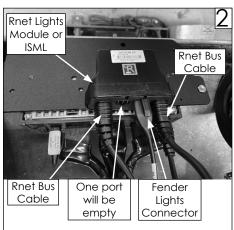

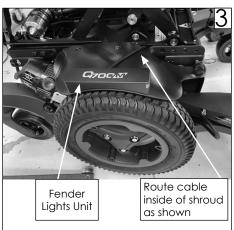

A. Mounting instructions (Fig. 1)

- Fender lights mount using adhesive backing. Peel off paper strip from bottom of fender light unit and place on main shroud as shown.
- Lights should be mounted so that the harness goes toward the center of the base. White LED light will be facing the front of the wheelchair.

B. Harness routing (Fig. 2 and Fig. 3)

- Fender light harnesses should route into the base to meet the central fender lights harness.
- The central fender lights harness will connect the fender lights to the ISML or Lights module for use.
 - a. Part number for ISML harness is 20011250-001.
 - b. Part number for Lights Module harness is 20011250-002.

▲ DEALER/TECHNICIAN WARNING

Attention dealers and qualified technicians, do not operate or service this device without first reading the owners manual. If you do not understand the instructions and warnings in the owners manual please contact the Quickie Technical Service Department before operating and/or servicing the Quickie device. Failure to do so may result in damage and/or injury.

Find more information and important warnings in the wheelchair owner's manual or at: www.SunriseMedical.com.

Sunrise Medical 2842 N. Business Park Ave Fresno, CA 93727 USA In Canada: (800) 263-3390



Customer Service: +1(800) 333-4000 or visit www.SunriseMedical.com