


Rogue XP Tipping Lever

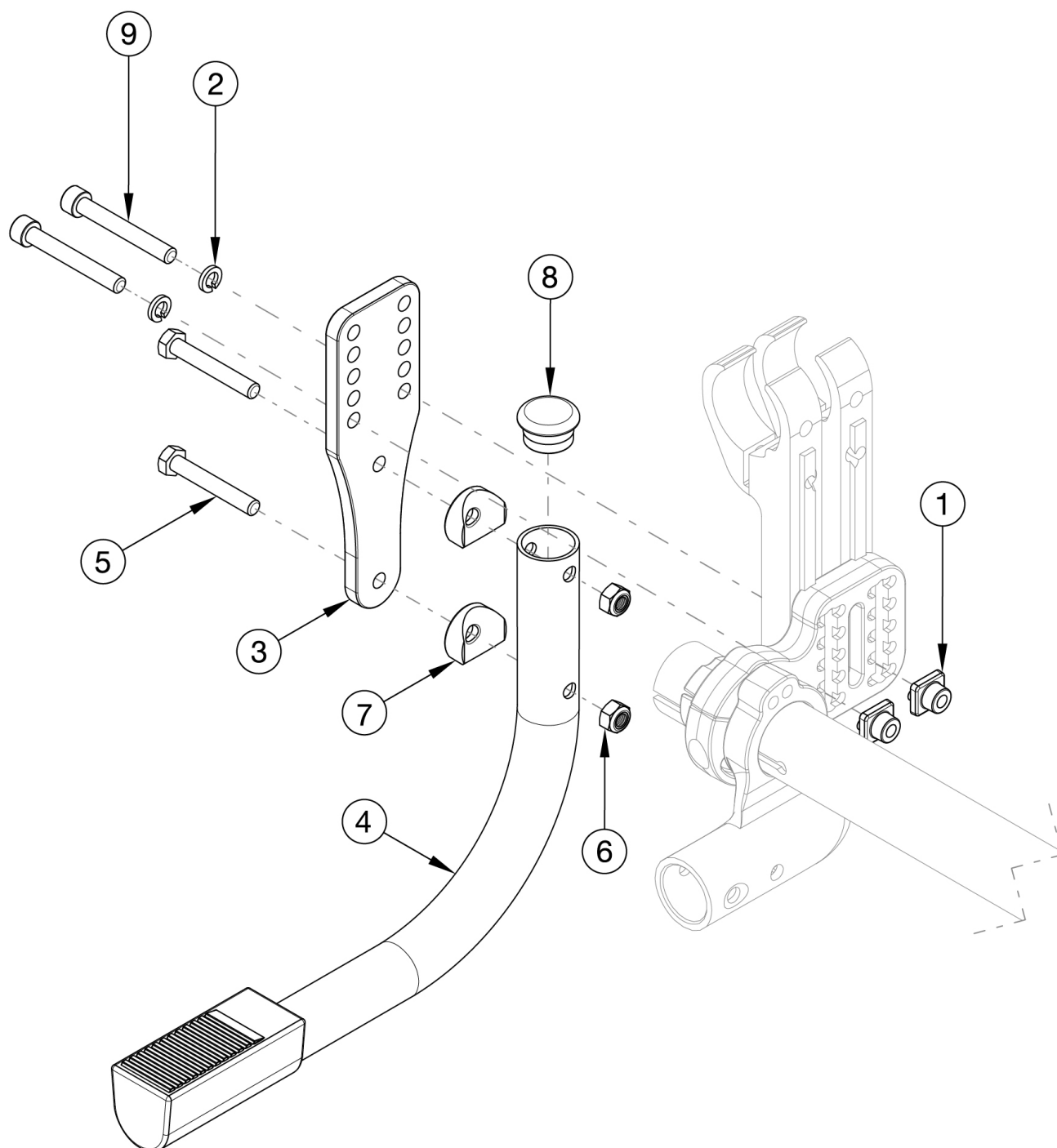

Not available if right and left User Activated Anti Tips are present Used on chairs invoiced starting on 3/12/25

Item Number	Part Number	Description	Quantity
1-9	120812	Tipping Lever Amputee Camber Mount Short Kit Rogue2 / Rogue XP	1

Item Number	Part Number	Description	Quantity
1	114014	Rear Tower Square Nut	2
2	102810	M6 Lockwasher, Hi-Collar, ZC	2
3	119607	Rogue2 / Rogue XP Tipping Lever Adapter Plate (Amputee)	1
4, 8	121383	Tipping Lever Tube Short Rogue2 / Rogue XP	1
5	100681	M6 x 40 CLS8.8 DIN931 HHCS ZC	2
6	100658	M6 Nylock Nut Blk Zn	2
7	000054	Saddle 7/8"	2
9	101472	M6 x 45 SHCS BLK ZC	2