


# Arc Caster Housing

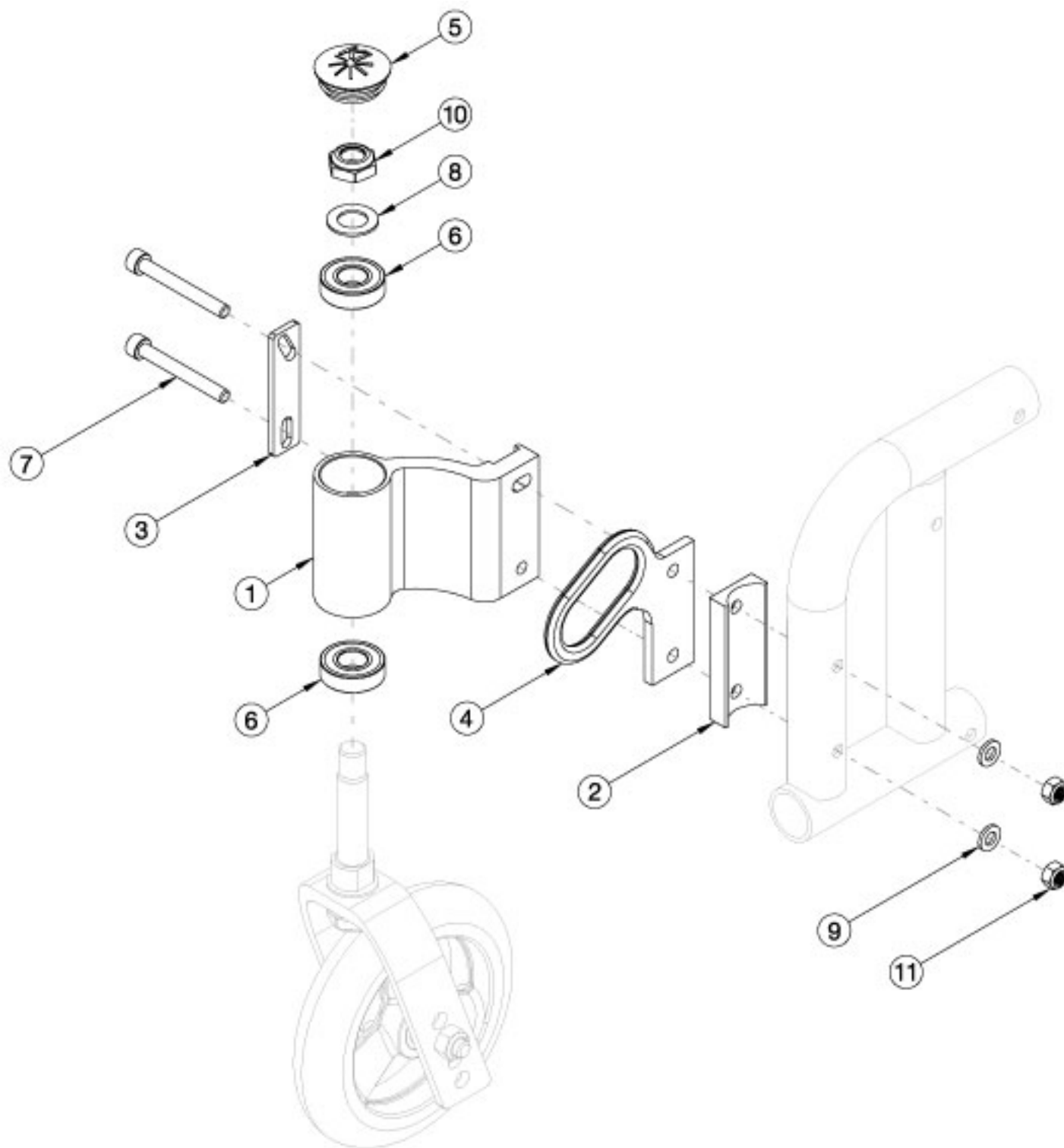

*The front transit bracket is pictured above for reference, however not available on this page. Please navigate to the "Arc Transit" parts manual page if the front transit bracket is needed.*

Item Number	Part Number	Description	Quantity
1a, 2, 3, 5, 6, 7a, 8-11	113183	Caster Housing Assembly, No Transit, Right, Arc	1
1b, 2, 3, 5, 6, 7a, 8-11	113184	Caster Housing Assembly, No Transit, Left, Arc	1
1a, 2, 3, 5, 6, 7b, 8-11	113185	Caster Housing Assembly, Transit, Right, Arc <i>Transit Bracket not included in assembly</i>	1
1b, 2, 3, 5, 6, 7b, 8-11	113186	Caster Housing Assembly, Transit, Left, Arc <i>Transit Bracket not included in assembly</i>	1
1a, 6	113187	Caster Mount, Right, Arc	1
1b, 6	113188	Caster Mount, Left, Arc	1
2	110204	Saddle, Caster Mount	1
3	110199	Caster Adjuster	1
5	102005	Ki Caster Housing Cap	1
7a	102473	M6 X 50 CLS12.9 SHCS BLK ZC	2
7b	100739	M6 X 55 SHCS CLS 12.96 DIN912 BLKZC	2
8	100682	Washer 1/2" .518 x .875 x .047 F/W Black Zinc	1
9	100746	M6 Flat Washer Blk Zn	2
10	100529	M12 x 1.25mm Lock Nut Black	1
11	100658	M6 Nylock Nut Blk Zn	2