


(Discontinued) Rogue XP Height Adjustable T-Arm

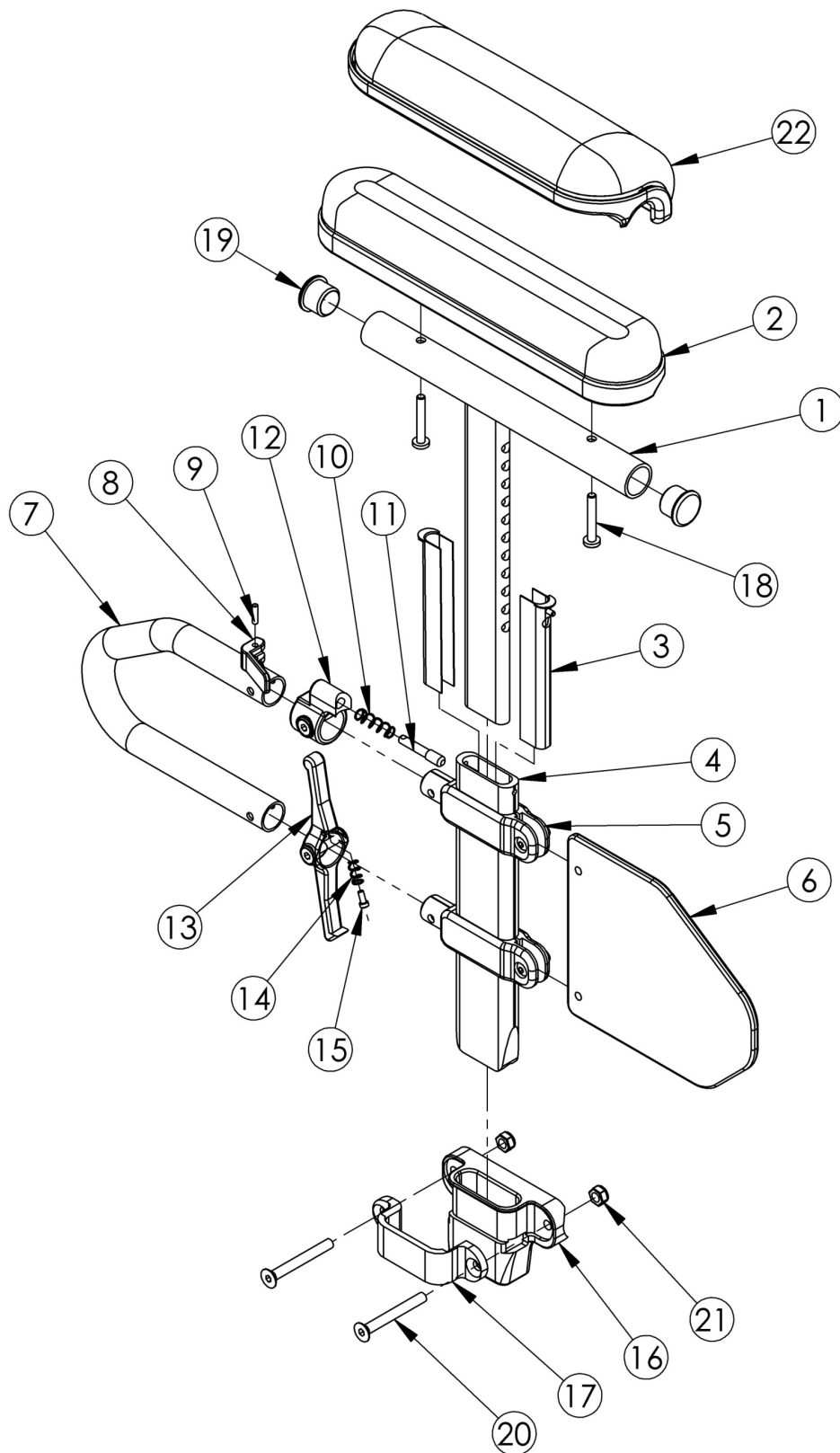

Height Adjustable T-Arms cannot be retrofitted

| Item Number | Part Number | Description | Quantity |
|----------------------------------|-------------|---|----------|
| 1a, 2b, 3-5, 6, 7-15, 18, 19 | 501682 | Height Adj T-Arm Assm Short Desk w/ Std Pad Left <i>Replaced with 505646</i> | 1 |
| 1a, 2b, 3-5, 6, 7-15, 18, 19 | 501683 | Height Adj T-Arm Assm Short Desk w/ Std Pad Right <i>Replaced with 505646</i> | 1 |
| 1b, 2a, 3-5, 6, 7-15, 18, 19 | 501684 | Height Adj T-Arm Assm Short Full w/ Std Pad Left <i>Replaced with 505644</i> | 1 |
| 1b, 2a, 3-5, 6, 7-15, 18, 19 | 501685 | Height Adj T-Arm Assm Short Full w/ Std Pad Right <i>Replaced with 505644</i> | 1 |
| 1a, 2d, 3-5, 6, 7-15, 18, 19 | 501690 | Height Adj T-Arm Assm Short Desk w/ Waterfall Pad Left <i>Replaced with 505650</i> | 1 |
| 1a, 2d, 3-5, 6, 7-15, 18, 19 | 501691 | Height Adj T-Arm Assm Short Desk w/ Waterfall Pad Right <i>Replaced with 505650</i> | 1 |
| 1b, 2c, 3-5, 6, 7-15, 18, 19 | 501692 | Height Adj T-Arm Assm Short Full w/ Waterfall Pad Left <i>Replaced with 505648</i> | 1 |
| 1b, 2c, 3-5, 6, 7-15, 18, 19 | 501693 | Height Adj T-Arm Assm Short Full w/ Waterfall Pad Right <i>Replaced with 505648</i> | 1 |
| 1a, 2b, 3-5, 6, 7-15, 16b, 17-21 | 501714 | Height Adj T-Arm Assm Short Desk w/ Std Pad and 1-3/8" Receiver Left TSU/RG <i>Replaced with 506081</i> | 1 |
| 1a, 2b, 3-5, 6, 7-15, 16b, 17-21 | 501715 | Height Adj T-Arm Assm Short Desk w/ Std Pad and 1-3/8" Receiver Right TSU/RG <i>Replaced with 506081</i> | 1 |
| 1b, 2a, 3-5, 6, 7-15, 16b, 17-21 | 501716 | Height Adj T-Arm Assm Short Full w/ Std Pad and 1-3/8" Receiver Left TSU/RG <i>Replaced with 506093</i> | 1 |
| 1b, 2a, 3-5, 6, 7-15, 16b, 17-21 | 501717 | Height Adj T-Arm Assm Short Full w/ Std Pad and 1-3/8" Receiver Right TSU/RG <i>Replaced with 506093</i> | 1 |
| 16, 17, 20, 21 | 501731 | Height Adj T-Arm Receiver Assembly 1-3/8", RG ALX/LW XP | 1 |
| 8-12 | 500541 | Arm Height Adjustable T Upper Lever Assembly | 1 |
| 1a, 19 | 000091 | Height Adj T Armrest Upper Desk | 1 |
| 1b, 19 | 000101 | Height Adj T Armrest Upper Full | 1 |
| 2a | 100500 | Arm Pad Full | 1 |
| 2b | 100501 | Arm Pad Desk | 1 |
| 3 | 000090 | Height Adjustable T Armrest Shim | 2 |

| Item Number | Part Number | Description | Quantity |
|----------------------|-------------|---|----------|
| 4 | 000089 | Height Adjustable T Armrest Post | 1 |
| 6 | 000097 | Height Adj T Armrest Side Guard Short | 1 |
| 7 | 000095 | Height Adjustable T Armrest Transfer Tube | 1 |
| 9 | 200257 | Dowel Pin 1/8 x 1/2 | 1 |
| 10 | 000134 | Height Adjustable Armrest Lever Spring | 1 |
| 11 | 000099 | Height Adjustable Armrest Lever Pin | 1 |
| 12 | 000096 | Height Adjustable T Armrest Pin Lock Housing | 1 |
| 14 | 000135 | Height Adjustable T Armrest Release Lever Spring | 1 |
| 15 | 000152 | Height Adjustable T Armrest Release Spring Button | 1 |
| 16 | 004578 | Height Adjustable T-Arm Receiver 1.375" | 1 |
| 17 | 000088 | Height Adjustable T Armrest Bracket | 1 |
| 18a | 100696 | M5 x 30mm PPH Capscrew <i>Used with Standard Arm Pads</i> | 2 |
| 18b | 100654 | M5 x 40 DIN7985 PH Pan HD M/S ZC <i>Used with Waterfall Arm Pads</i> | 2 |
| 19 | 100803 | Tube End Plug 7/8" | 2 |
| 20 | 101130 | M6 x 55 FSHCS P/THD BLKZC | 2 |
| 21 | 100658 | M6 Nylock Nut Blk Zn | 2 |
| 22a | 102452 | Waterfall Arm Pad Full | 1 |
| 22b | 102521 | Waterfall Arm Pad Desk | 1 |
| 1a, 3-5, 6, 7-15, 19 | 100238 | Height Adj T Armrest Short Desk Right <i>Replaced with 003463</i> | 1 |
| 1a, 3-5, 6, 7-15, 19 | 100242 | Height Adj T Armrest Short Desk Left <i>Replaced with 003463</i> | 1 |
| 1b, 3-5, 6, 7-15, 19 | 100239 | Height Adj T Armrest Short Full Right <i>Replaced with 003464</i> | 1 |
| 1b, 3-5, 6, 7-15, 19 | 100243 | Height Adj T Armrest Short Full Left <i>Replaced with 003464</i> | 1 |