


eBack[™]

The back system of choice for clients requiring more support than a sling back can provide.

Therapists recommend the eBack for clients in need of a lightweight, comfortable back support with minimal adjustment in recline or depth. The eBack offers two angles of recline and is easy to install. The eBack is portable and can be mounted or removed with one hand.

Users and caregivers value the eBack's comfort and effective support, as well as its lightweight, user-friendly design.



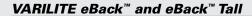

Cover: Mesh

See price list for size availability HCPCS Code E2611



eBack**

Effective postural support, exceptional comfort and simplicity have made the eBack a favorite of seating specialists. **VARILITE** eBack systems are available in a range of widths, from pediatric to bariatric, and in regular and tall sizes.

1. Hardware

All-metal Evolution Back hardware offers adjustability at all four attachment points. Adjustment can be made while client is seated in the wheelchair. To allow for client growth, aluminum brackets permit 1 inch (2.5 cm) of lateral adjustment. Steel pin bars slide anteriorly and posteriorly to allow for back angle and seat depth adjustment. Pin bars simply slide in the receiving cane clip and click the system into place. Quick, easy, and no unsightly J-hooks.

2. Shell

Made from aluminum and trimmed with protective edging, the *VARILITE* Evolution Back shell is one of the lightest in the industry. Back panel is flat for flush mounting of headrest hardware. Contoured sides provide lateral support and attachment area for hip belts, laterals and hip guides. Shell has a handle cutout for easy shell removal by individuals with limited hand function. Slots accommodate different chair and mounting configurations. Shell supports users up to 250 lbs (115 kg).

3. Cushion


VARILITE Air-Foam Floatation. A two-way valve allows air in and out of the cushion for a custom fit. Fabric is puncture and water-resistant. Cushion is oversized to protect the user during reaching or weight-shifting activities.


4. Cover

Four-way-stretch mesh top fabric wicks moisture away from the body. A large loop panel on the backside of the cover securely attaches to hook strips on the shell, while a wrap around handle flap prevents downward slippage for even the most active user. Machine washable.

5. Pin Bar

Pin bars for *VARILITE* eBack systems offer both a neutral and a recline position.

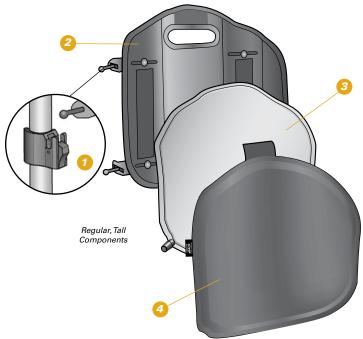

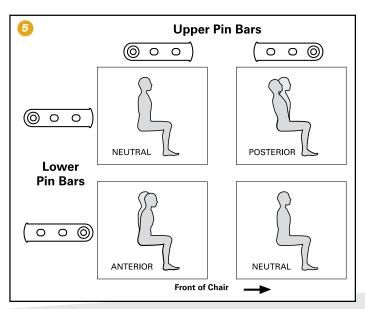



4000 1st Avenue South Seattle, WA 98134 Phone: (800) 827-4548 Fax: (206) 343-5795 www.yarilite.com Authorized EU Representative: Cascade Designs, Limited Dwyer Road, Midleton, Co. Cork Republic of Ireland

