



# Rigid Padded Velcro Adjustable Back Upholstery

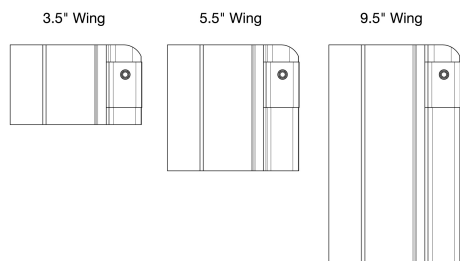

Backrest Range and Height	Wings 10\"/>			
Short 11\"/>	003081 003086	003083 003088	002746 002749	1
Short 13\"/>	002857 002865	002859 002867	002861 002869	1
Medium 15\"/>	002857 002865	002859 002867	002861 002869	2
Medium 17\"/>	002857 002865	002859 002867	002861 002869	3
Tall 17\"/>	002857 002865	N/A	N/A	3
Tall 19\"/>	002857 002865	N/A	N/A	4
Tall 17-18\"/>	N/A	002875 002883	002877 002885	1
Tall 19\"/>	N/A	002875 002883	002877 002885	2
Tall 20\"/>	N/A	002875 002883	002877 002885	3

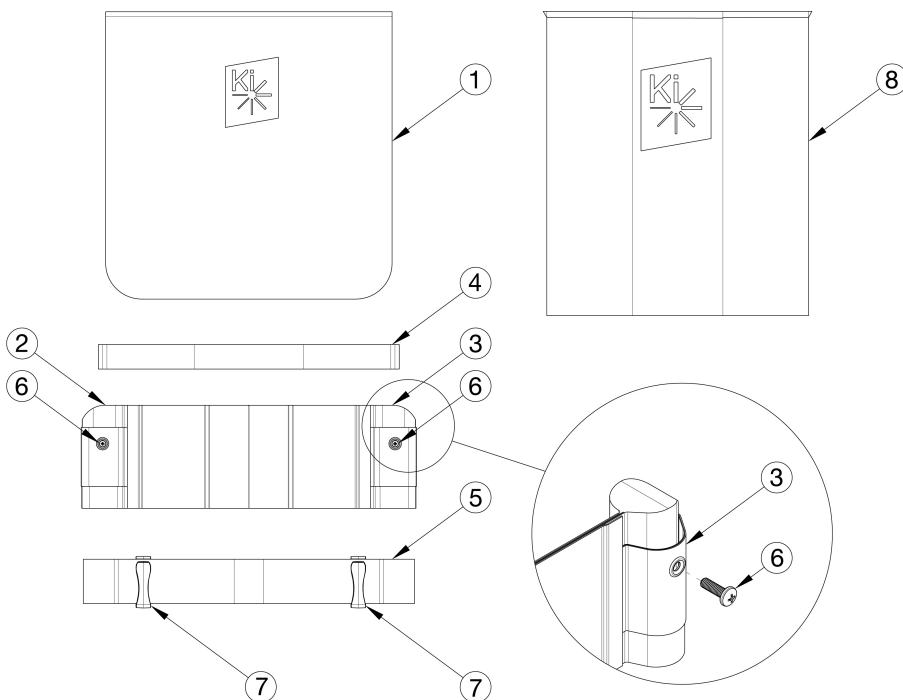

Assemblies contain the cover, amount of straps and d-rings used for the tallest height listed in the assembly. Attaching hardware is not included with the back upholstery assemblies. Tension Straps (4a-4f) are based on seat width and used with the combination of Fold Down Push Handle and Padded Velcro Adjustable Back Upholstery. Not available with Heavy Duty Option

Item Number	Part Number	Description	Quantity
1b, 2a, 3a, 5a, 7	111174	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 10" Wide, 11-12 Height, Rigid	1
1b, 2d, 3d, 5a, 7	506595	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 10" Wide, 13-14 Height, Rigid	1
1c, 2d, 3d, 5a, 7	506596	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 10" Wide, 15-16 Height, Rigid	1
1d, 2d, 3d, 5a, 7	506597	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 10" Wide, 17-18 Height, Rigid	1
1f, 2a, 3a, 5b, 7	111175	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 11" Wide, 11-12 Height, Rigid	1
1f, 2d, 3d, 5b, 7	506598	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 11" Wide, 13-14 Height, Rigid	1
1g, 2d, 3d, 5b, 7	506599	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 11" Wide, 15-16 Height, Rigid	1
1h, 2d, 3d, 5b, 7	506600	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 11" Wide, 17-18 Height, Rigid	1

Item Number	Part Number	Description	Quantity
1j, 2a, 3a, 5c, 7	111176	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 12" Wide, 11-12 Height, Rigid	1
1j, 2d, 3d, 5c, 7	506601	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 12" Wide, 13-14 Height, Rigid	1
1k, 2d, 3d, 5c, 7	506602	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 12" Wide, 15-16 Height, Rigid	1
1l, 2d, 3d, 5c, 7	506603	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 12" Wide, 17-18 Height, Rigid	1
1n, 2b, 3b, 5d, 7	111177	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 13" Wide, 11-12 Height, Rigid	1
1n, 2e, 3e, 5d, 7	506604	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 13" Wide, 13-14 Height, Rigid	1
1o, 2e, 3e, 5d, 7	506605	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 13" Wide, 15-16 Height, Rigid	1
1q, 2g, 3g, 5d, 7	506606	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 13" Wide, 17-20 Height, Rigid	1
1s, 2b, 3b, 5e, 7	111178	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 14" Wide, 11-12 Height, Rigid	1
1s, 2e, 3e, 5e, 7	506607	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 14" Wide, 13-14 Height, Rigid	1
1t, 2e, 3e, 5e, 7	506608	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 14" Wide, 15-16 Height, Rigid	1
1v, 2g, 3g, 5e, 7	506609	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 14" Wide, 17-20 Height, Rigid	1
1x, 2b, 3b, 5f, 7	111179	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 15" Wide, 11-12 Height, Rigid	1
1x, 2e, 3e, 5f, 7	506610	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 15" Wide, 13-14 Height, Rigid	1
1y, 2e, 3e, 5f, 7	506611	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 15" Wide, 15-16 Height, Rigid	1
1aa, 2g, 3g, 5f, 7	506612	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 15" Wide, 17-20 Height, Rigid	1
1ac, 2b, 3b, 5g, 7	111180	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 16" Wide, 11-12 Height, Rigid	1
1ac, 2e, 3e, 5g, 7	506613	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 16" Wide, 13-14 Height, Rigid	1
1ad, 2e, 3e, 5g, 7	506614	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 16" Wide, 15-16 Height, Rigid	1

Item Number	Part Number	Description	Quantity
1af, 2g, 3g, 5g, 7	506615	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 16" Wide, 17-20 Height, Rigid	1
1ah, 2b, 3b, 5h, 7	111181	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 17" Wide, 11-12 Height, Rigid	1
1ah, 2e, 3e, 5h, 7	506616	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 17" Wide, 13-14 Height, Rigid	1
1ai, 2e, 3e, 5h, 7	506617	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 17" Wide, 15-16 Height, Rigid	1
1ak, 2g, 3g, 5h, 7	506618	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 17" Wide, 17-20 Height, Rigid	1
1am, 2c, 3c, 5i, 7	111182	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 18" Wide, 11-12 Height, Rigid	1
1am, 2f, 3f, 5i, 7	506619	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 18" Wide, 13-14 Height, Rigid	1
1an, 2f, 3f, 5i, 7	506620	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 18" Wide, 15-16 Height, Rigid	1
1ap, 2h, 3h, 5i, 7	506621	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 18" Wide, 17-20 Height, Rigid	1
1ar, 2c, 3c, 5j, 7	111183	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 19" Wide, 11-12 Height, Rigid	1
1ar, 2f, 3f, 5j, 7	506622	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 19" Wide, 13-14 Height, Rigid	1
1as, 2f, 3f, 5j, 7	506623	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 19" Wide, 15-16 Height, Rigid	1
1au, 2h, 3h, 5j, 7	506624	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 19" Wide, 17-20 Height, Rigid	1
1aw, 2c, 3c, 5k, 7	111184	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 20" Wide, 11-12 Height, Rigid	1
1aw, 2f, 3f, 5k, 7	506625	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 20" Wide, 13-14 Height, Rigid	1
1ax, 2f, 3f, 5k, 7	506626	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 20" Wide, 15-16 Height, Rigid	1
1az, 2h, 3h, 5k, 7	506627	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 20" Wide, 17-20 Height, Rigid	1
1ba, 2f, 3f, 5l, 7	506628	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 21" Wide, 13-16 Height, Rigid	1
1bc, 2h, 3h, 5l, 7	506629	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 21" Wide, 17-20 Height, Rigid	1

Item Number	Part Number	Description	Quantity
1bd, 2f, 3f, 5m, 7	506630	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 22" Wide, 13-16 Height, Rigid	1
1bf, 2h, 3h, 5m, 7	506631	Back Upholstery Assembly Padded Velcro Adjustable with Standard Cover 22" Wide, 17-20 Height, Rigid	1
8b, 2a, 3a, 5c, 7	111185	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 12" Wide, 11-12 Height, Rigid	1
8b, 2d, 3d, 5c, 7	107751	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 12" Wide, 13-14 Height, Rigid	1
8c, 2d, 3d, 5c, 7	107752	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 12" Wide, 15-16 Height, Rigid	1
8d, 2d, 3d, 5c, 7	107753	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 12" Wide, 17-18 Height, Rigid	1
8f, 2b, 3b, 5d, 7	111186	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 13" Wide, 11-12 Height, Rigid	1
8f, 2e, 3e, 5d, 7	107754	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 13" Wide, 13-14 Height, Rigid	1
8g, 2e, 3e, 5d, 7	107755	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 13" Wide, 15-16 Height, Rigid	1
8i, 2g, 3g, 5d, 7	107756	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 13" Wide, 17-20 Height, Rigid	1
8k, 2b, 3b, 5e, 7	111187	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 14" Wide, 11-12 Height, Rigid	1
8k, 2e, 3e, 5e, 7	107757	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 14" Wide, 13-14 Height, Rigid	1
8l, 2e, 3e, 5e, 7	107758	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 14" Wide, 15-16 Height, Rigid	1
8n, 2g, 3g, 5e, 7	107759	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 14" Wide, 17-20 Height, Rigid	1
8p, 2b, 3b, 5f, 7	111188	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 15" Wide, 11-12 Height, Rigid	1
8p, 2e, 3e, 5f, 7	107760	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 15" Wide, 13-14 Height, Rigid	1
8q, 2e, 3e, 5f, 7	107761	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 15" Wide, 15-16 Height, Rigid	1
8s, 2g, 3g, 5f, 7	107762	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 15" Wide, 17-20 Height, Rigid	1
8u, 2b, 3b, 5g, 7	111189	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 16" Wide, 11-12 Height, Rigid	1

Item Number	Part Number	Description	Quantity
8u, 2e, 3e, 5g, 7	107763	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 16" Wide, 13-14 Height, Rigid	1
8v, 2e, 3e, 5g, 7	107764	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 16" Wide, 15-16 Height, Rigid	1
8x, 2g, 3g, 5g, 7	107765	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 16" Wide, 17-20 Height, Rigid	1
8z, 2b, 3b, 5h, 7	111190	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 17" Wide, 11-12 Height, Rigid	1
8z, 2e, 3e, 5h, 7	107766	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 17" Wide, 13-14 Height, Rigid	1
8aa, 2e, 3e, 5h, 7	107767	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 17" Wide, 15-16 Height, Rigid	1
8ac, 2g, 3g, 5h, 7	107768	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 17" Wide, 17-20 Height, Rigid	1
8ae, 2c, 3c, 5i, 7	111191	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 18" Wide, 11-12 Height, Rigid	1
8ae, 2f, 3f, 5i, 7	107769	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 18" Wide, 13-14 Height, Rigid	1
8af, 2f, 3f, 5i, 7	107770	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 18" Wide, 15-16 Height, Rigid	1
8ah, 2h, 3h, 5i, 7	107771	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 18" Wide, 17-20 Height, Rigid	1
8aj, 2c, 3c, 5j, 7	111192	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 19" Wide, 11-12 Height, Rigid	1
8aj, 2f, 3f, 5j, 7	107772	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 19" Wide, 13-14 Height, Rigid	1
8ak, 2f, 3f, 5j, 7	107773	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 19" Wide, 15-16 Height, Rigid	1
8am, 2h, 3h, 5j, 7	107774	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 19" Wide, 17-20 Height, Rigid	1
8ao, 2c, 3c, 5k, 7	111193	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 20" Wide, 11-12 Height, Rigid	1
8ao, 2f, 3f, 5k, 7	107775	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 20" Wide, 13-14 Height, Rigid	1
8ap, 2f, 3f, 5k, 7	107776	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 20" Wide, 15-16 Height, Rigid	1
8ar, 2h, 3h, 5k, 7	107777	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 20" Wide, 17-20 Height, Rigid	1

Item Number	Part Number	Description	Quantity
8as, 2f, 3f, 5l, 7	107778	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 21" Wide, 13-16 Height, Rigid	1
8au, 2h, 3h, 5l, 7	107779	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 21" Wide, 17-20 Height, Rigid	1
8av, 2f, 3f, 5m, 7	107780	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 22" Wide, 13-16 Height, Rigid	1
8ax, 2h, 3h, 5m, 7	107781	Back Upholstery Assembly Padded Velcro Adjustable with Air Mesh Cover 22" Wide, 17-20 Height, Rigid	1
1a	002485	Backrest Cover 10" Rigid x 9-12"	1
1b	002486	Backrest Cover 10" Rigid x 11-14"	1
1c	002487	Backrest Cover 10" Rigid x 13-16"	1
1d	002488	Backrest Cover 10" Rigid x 15-18"	1
1e	002489	Backrest Cover 11" Rigid x 9-12"	1
1f	002490	Backrest Cover 11" Rigid x 11-14"	1
1g	002491	Backrest Cover 11" Rigid x 13-16"	1
1h	002492	Backrest Cover 11" Rigid x 15-18"	1
1i	002493	Backrest Cover 12" Rigid x 9-12"	1
1j	002494	Backrest Cover 12" Rigid x 11-14"	1
1k	002495	Backrest Cover 12" Rigid x 13-16"	1
1l	002496	Backrest Cover 12" Rigid x 15-18"	1
1m	002497	Backrest Cover 13" Rigid / 14" Cat x 9-12"	1
1n	002498	Backrest Cover 13" Rigid / 14" Cat x 11-14"	1
1o	002499	Backrest Cover 13" Rigid / 14" Cat x 13-16"	1
1p	002500	Backrest Cover 13" Rigid - 14" CAT/Liberty x 15-18"	1
1q	003043	Backrest Cover 13" Rigid - 14" CAT/Liberty x 17-20"	1
1r	002501	Backrest Cover 14" Rigid / 15" Cat x 9-12"	1
1s	002502	Backrest Cover 14" Rigid / 15" Cat x 11-14"	1
1t	002503	Backrest Cover 14" Rigid / 15" Cat x 13-16"	1
1u	002504	Backrest Cover 14" Rigid - 15" CAT/Liberty x 15-18"	1
1v	003044	Backrest Cover 14" Rigid - 15" CAT/Liberty x 17-20"	1

Item Number	Part Number	Description	Quantity
1w	002505	Backrest Cover 15" Rigid / 16" Cat x 9-12"	1
1x	002506	Backrest Cover 15" Rigid / 16" Cat x 11-14"	1
1y	002507	Backrest Cover 15" Rigid / 16" Cat x 13-16"	1
1z	002508	Backrest Cover 15" Rigid - 16" CAT/Liberty x 15-18"	1
1aa	003045	Backrest Cover 15" Rigid - 16" CAT/Liberty x 17-20"	1
1ab	002509	Backrest Cover 16" Rigid / 17" Cat x 9-12"	1
1ac	002510	Backrest Cover 16" Rigid / 17" Cat x 11-14"	1
1ad	002511	Backrest Cover 16" Rigid / 17" Cat x 13-16"	1
1ae	002512	Backrest Cover 16" Rigid - 17" CAT/Liberty x 15-18"	1
1af	003046	Backrest Cover 16" Rigid - 17" CAT/Liberty x 17-20"	1
1ag	003048	Backrest Cover 17" Rigid / 18" Cat x 9-12"	1
1ah	003049	Backrest Cover 17" Rigid / 18" Cat x 11-14"	1
1ai	003050	Backrest Cover 17" Rigid / 18" Cat x 13-16"	1
1aj	003051	Backrest Cover 17" Rigid - 18" CAT/Liberty x 15-18"	1
1ak	003052	Backrest Cover 17" Rigid - 18" CAT/Liberty x 17-20"	1
1al	003054	Backrest Cover 18" Rigid / 19" Cat x 9-12"	1
1am	003055	Backrest Cover 18" Rigid / 19" Cat x 11-14"	1
1an	003056	Backrest Cover 18" Rigid / 19" Cat x 13-16"	1
1ao	003057	Backrest Cover 18" Rigid - 19" CAT/Liberty x 15-18"	1
1ap	003058	Backrest Cover 18" Rigid - 19" CAT/Liberty x 17-20"	1
1aq	003060	Backrest Cover 19" Rigid / 20" Cat x 9-12"	1
1ar	003061	Backrest Cover 19" Rigid / 20" Cat x 11-14"	1
1as	003062	Backrest Cover 19" Rigid / 20" Cat x 13-16"	1
1at	003063	Backrest Cover 19" Rigid - 20" CAT/Liberty x 15-18"	1
1au	003064	Backrest Cover 19" Rigid - 20" CAT/Liberty x 17-20"	1
1av	003066	Backrest Cover 20" Rigid / 21" Cat x 9-12"	1
1aw	003067	Backrest Cover 20" Rigid / 21" Cat x 11-14"	1

Item Number	Part Number	Description	Quantity
1ax	003068	Backrest Cover 20" Rigid / 21" Cat x 13-16"	1
1ay	003069	Backrest Cover 20" Rigid - 21" CAT/Liberty x 15-18"	1
1az	003070	Backrest Cover 20" Rigid - 21" CAT/Liberty x 17-20"	1
1ba	003072	Backrest Cover 21" Rigid / 22" Cat x 13-16"	1
1bb	003073	Backrest Cover 21" Rigid - 22" CAT/Liberty x 15-18"	1
1bc	003074	Backrest Cover 21" Rigid - 22" CAT/Liberty x 17-20"	1
1bd	003076	Backrest Cover 22" Rigid x 13-16"	1
1be	003077	Backrest Cover 22" Rigid x 15-18"	1
1bf	003078	Backrest Cover 22" Rigid x 17-20"	1
2a	003086	Standard Upholstery Wing 3.5" Height 10" - 12" Wide, Left	1
2b	003088	Standard Upholstery Wing 3.5" Height 13" - 17" Wide, Left	1
2c	002749	Standard Upholstery Wing 3.5" Height 18" - 22" Wide, Left	1
2d	002865	Standard Upholstery Wing 5.5" Height 10-12" Wide Left	1
2e	002867	Standard Upholstery Wing 5.5" Height 13-17" Wide Left	1
2f	002869	Standard Upholstery Wing 5.5" Height 18-22" Wide Left	1
2g	002883	Standard Upholstery Wing 9.5" Height 13-17" Wide Left	1
2h	002885	Standard Upholstery Wing 9.5" Height 18-22" Wide Left	1
3a	003081	Standard Upholstery Wing 3.5" Height 10" - 12" Wide, Right	1
3b	003083	Standard Upholstery Wing 3.5" Height 13" - 17" Wide, Right	1
3c	002746	Standard Upholstery Wing 3.5" Height 18" - 22" Wide, Right	1
3d	002857	Standard Upholstery Wing 5.5" Height 10-12" Wide Right	1
3e	002859	Standard Upholstery Wing 5.5" Height 13-17" Wide Right	1
3f	002861	Standard Upholstery Wing 5.5" Height 18-22" Wide Right	1
3g	002875	Standard Upholstery Wing 9.5" Height 13-17" Wide Right	1
3h	002877	Standard Upholstery Wing 9.5" Height 18-22" Wide Right	1
4a	001748	Tension Strap Folding Handle 9-11 Wide	1
4b	001749	Tension Strap Folding Handle 12-13 Wide	1



Item Number	Part Number	Description	Quantity
4c	001750	Tension Strap Folding Handle 14-15 Wide	1
4d	001751	Tension Strap Folding Handle 16-17 Wide	1
4e	001752	Tension Strap Folding Handle 18-19 Wide	1
4f	001753	Tension Strap Folding Handle 20-22 Wide	1
5a	002533	Back Upholstery Strap 19.3" <i>Used on 10" chair widths</i>	1
5b	002534	Back Upholstery Strap 21.3" <i>Used on 11" chair widths</i>	1
5c	002535	Back Upholstery Strap 23.3" <i>Used on 12" chair widths</i>	1
5d	002536	Back Upholstery Strap 25.3" <i>Used on 13" chair widths</i>	1
5e	002537	Back Upholstery Strap 27.3" <i>Used on 14" chair widths</i>	1
5f	002538	Back Upholstery Strap 29.3" <i>Used on 15" chair widths</i>	1
5g	002539	Back Upholstery Strap 31.3" <i>Used on 16" chair widths</i>	1
5h	002540	Back Upholstery Strap 33.3" <i>Used on 17" chair widths</i>	1
5i	002741	Back Upholstery Strap 35.3" <i>Used on 18" chair widths</i>	1
5j	002742	Back Upholstery Strap 37.3" <i>Used on 19" chair widths</i>	1
5k	002743	Back Upholstery Strap 39.3" <i>Used on 20" chair widths</i>	1
5l	002744	Back Upholstery Strap 41.3" <i>Used on 21" chair widths</i>	1
5m	002745	Back Upholstery Strap 43.3" <i>Used on 22" chair widths</i>	1
6	100734	M5 x 16 PHMS ZC	2
7	102736	Quick-Slip Keeper	2
8a	003892	Air Mesh Backrest Cover 12R Width x 9-12" Height	1
8b	003893	Air Mesh Backrest Cover 12R Width x 11-14" Height	1

Item Number	Part Number	Description	Quantity
8c	003894	Air Mesh Backrest Cover 12R Width x 13-16" Height	1
8d	003895	Air Mesh Backrest Cover 12R Width x 15-18" Height	1
8e	003896	Air Mesh Backrest Cover 14C 13R Width x 9-12" Height	1
8f	003897	Air Mesh Backrest Cover 14C 13R Width x 11-14" Height	1
8g	003898	Air Mesh Backrest Cover 14C 13R Width x 13-16" Height	1
8h	003899	Air Mesh Backrest Cover 14C 13R Width x 15-18" Height	1
8i	003913	Air Mesh Backrest Cover 14C 13R Width x 17-20" Height	1
8j	003900	Air Mesh Backrest Cover 15C 14R Width x 9-12" Height	1
8k	003901	Air Mesh Backrest Cover 15C 14R Width x 11-14" Height	1
8l	003902	Air Mesh Backrest Cover 15C 14R Width x 13-16" Height	1
8m	003903	Air Mesh Backrest Cover 15C 14R Width x 15-18" Height	1
8n	003914	Air Mesh Backrest Cover 15C 14R Width x 17-20" Height	1
8o	003904	Air Mesh Backrest Cover 16C 15R Width x 9-12" Height	1
8p	003905	Air Mesh Backrest Cover 16C 15R Width x 11-14" Height	1
8q	003906	Air Mesh Backrest Cover 16C 15R Width x 13-16" Height	1
8r	003907	Air Mesh Backrest Cover 16C 15R Width x 15-18" Height	1
8s	003915	Air Mesh Backrest Cover 16C 15R Width x 17-20" Height	1
8t	003909	Air Mesh Backrest Cover 17C 16R Width x 9-12" Height	1
8u	003910	Air Mesh Backrest Cover 17C 16R Width x 11-14" Height	1
8v	003911	Air Mesh Backrest Cover 17C 16R Width x 13-16" Height	1
8w	003912	Air Mesh Backrest Cover 17C 16R Width x 15-18" Height	1
8x	003916	Air Mesh Backrest Cover 17C 16R Width x 17-20" Height	1
8y	003918	Air Mesh Backrest Cover 18C 17R Width x 9-12" Height	1
8z	003919	Air Mesh Backrest Cover 18C 17R Width x 11-14" Height	1
8aa	003920	Air Mesh Backrest Cover 18C 17R Width x 13-16" Height	1
8ab	003921	Air Mesh Backrest Cover 18C 17R Width x 15-18" Height	1
8ac	003922	Air Mesh Backrest Cover 18C 17R Width x 17-20" Height	1

Item Number	Part Number	Description	Quantity
8ad	003924	Air Mesh Backrest Cover 19C 18R Width x 9-12" Height	1
8ae	003925	Air Mesh Backrest Cover 19C 18R Width x 11-14" Height	1
8af	003926	Air Mesh Backrest Cover 19C 18R Width x 13-16" Height	1
8ag	003927	Air Mesh Backrest Cover 19C 18R Width x 15-18" Height	1
8ah	003928	Air Mesh Backrest Cover 19C 18R Width x 17-20" Height	1
8ai	003930	Air Mesh Backrest Cover 20C 19R Wide x 9-12" Height	1
8aj	003931	Air Mesh Backrest Cover 20C 19R Width x 11-14" Height	1
8ak	003932	Air Mesh Backrest Cover 20C 19R Width x 13-16" Height	1
8al	003933	Air Mesh Backrest Cover 20C 19R Width x 15-18" Height	1
8am	003934	Air Mesh Backrest Cover 20C 19R Width x 17-20" Height	1
8an	003936	Air Mesh Backrest Cover 21C 20R Width x 9-12" Height	1
8ao	003937	Air Mesh Backrest Cover 21C 20R Width x 11-14" Height	1
8ap	003938	Air Mesh Backrest Cover 21C 20R Width x 13-16" Height	1
8aq	003939	Air Mesh Backrest Cover 21C 20R Width x 15-18" Height	1
8ar	003940	Air Mesh Backrest Cover 21C 20R Width x 17-20" Height	1
8as	003942	Air Mesh Backrest Cover 22C 21R Width x 13-16" Height	1
8at	003943	Air Mesh Backrest Cover 22C 21R Width x 15-18" Height	1
8au	003944	Air Mesh Backrest Cover 22C 21R Width x 17-20" Height	1
8av	003946	Air Mesh Backrest Cover 22R Width x 13-16" Height	1
8aw	003947	Air Mesh Backrest Cover 22R Width x 15-18" Height	1
8ax	003948	Air Mesh Backrest Cover 22R Width x 17-20" Height	1