
User manual

Crocodile


R82[®]
by Etac

M1220 - 2023-06-26 - rev. 15.0

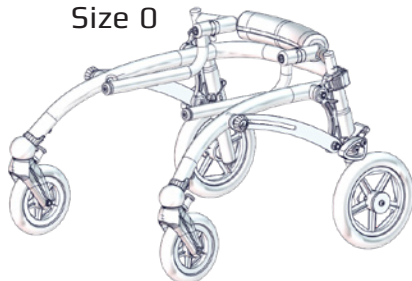
Size 0

Size 1

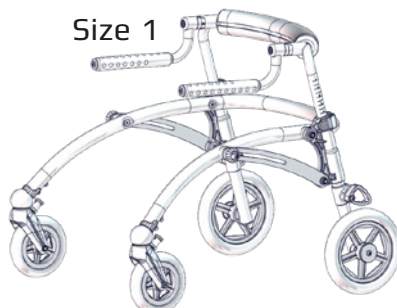Size 2

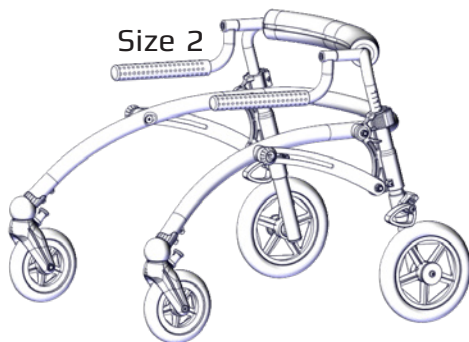
Size 3


www.etac.com

	CROCODILE 0 & 1 & 2	CROCODILE 3		
	3	13	ENGLISH GERMAN	36 - 38 39 - 43
	4 - 5	14 - 15	NEDERLANDS DANSK	44 - 47 48 - 50
	6	16	NORSK SVENSKA	51 - 53 54 - 56
	7	17	SUOMI ESPAÑOL	57 - 59 60 - 63
	8	18	FRANÇAIS ITALIANO	64 - 67 68 - 71
	9	19	PORTUGUÊS PORTUGUÊS DO BRASIL	72 - 75 76 - 79
	10	20	РОССИЯ POLSKI	80 - 84 85 - 88
	11	21	ČESKY 中國	89 - 91 92 - 94
	12	22	日本 ΕΛΛΗΝΙΚΗ	95 - 98 99 - 103
mm (inch) kg (lb)	23 - 24	23 - 24		
	25 - 26	25 - 26		
	27 - 35	27 - 35		

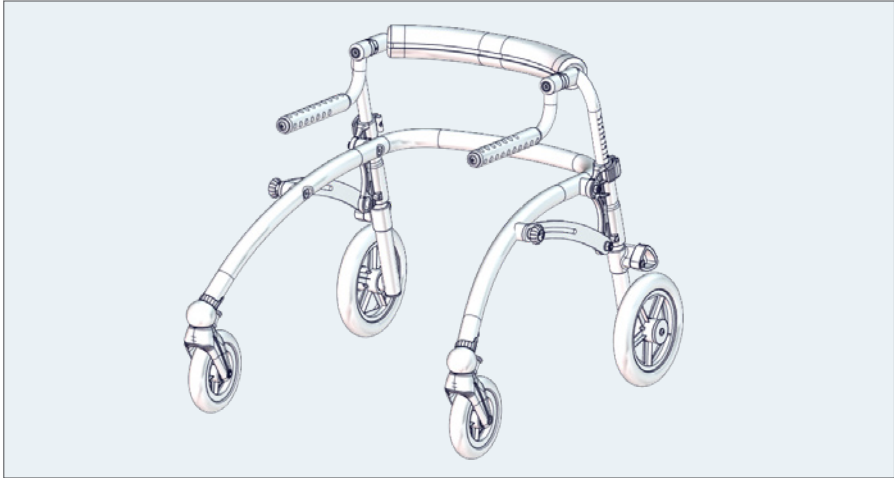CROCODILE 0

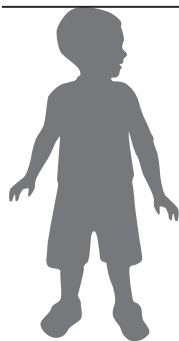
<1050 mm
<41¼"

<30 kg
<66 lb

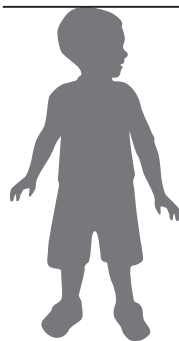CROCODILE 1


<1150 mm
<45¼"

<30 kg
<66 lb




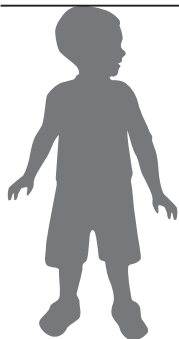
CROCODILE 2

<1500 mm
<59"


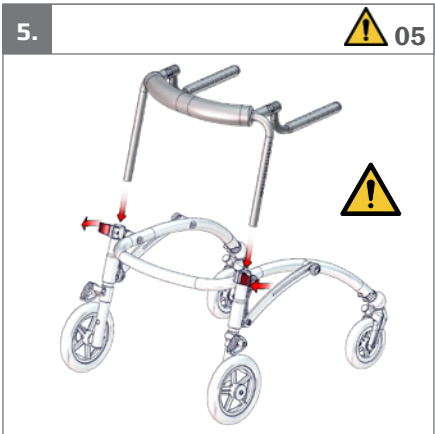
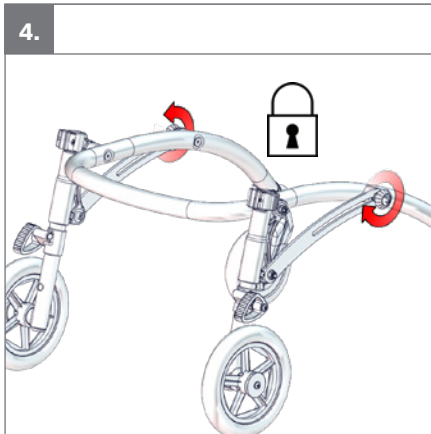
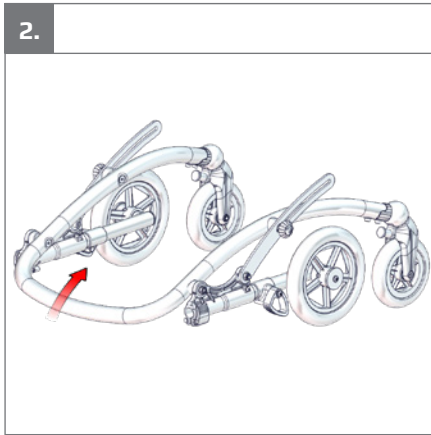
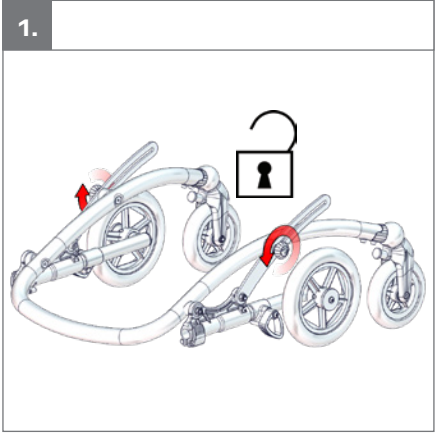
<45 kg
<99 lb

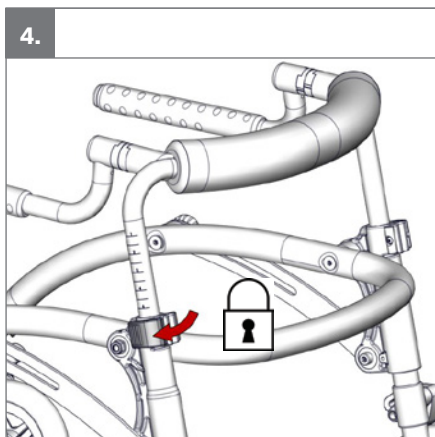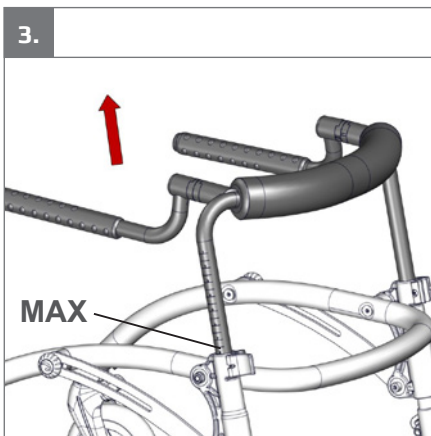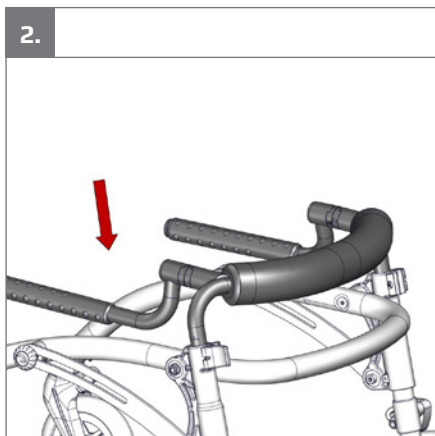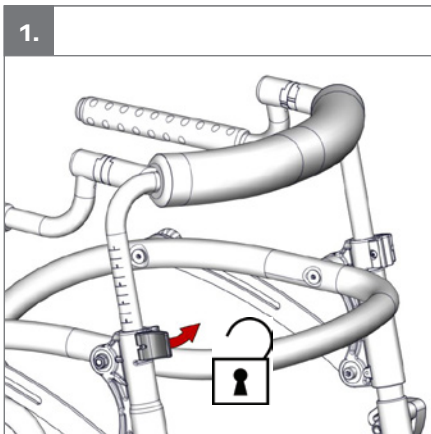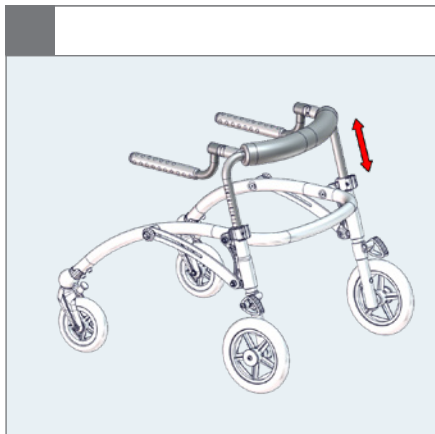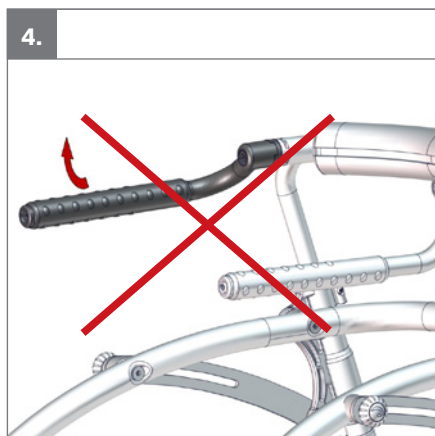
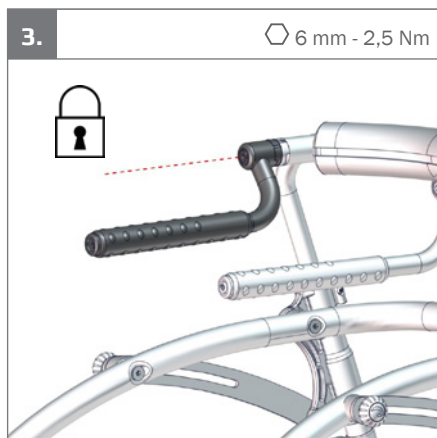
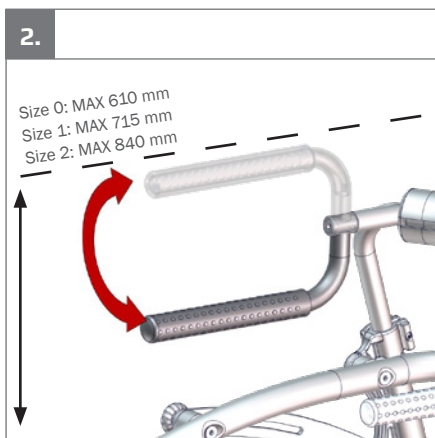
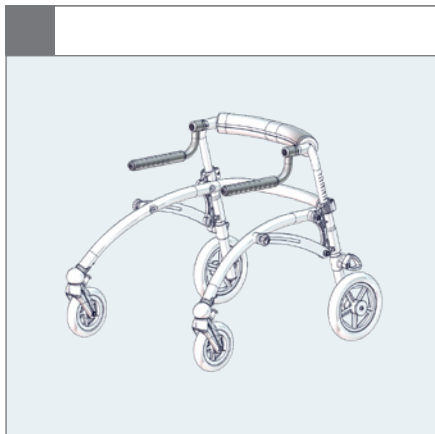
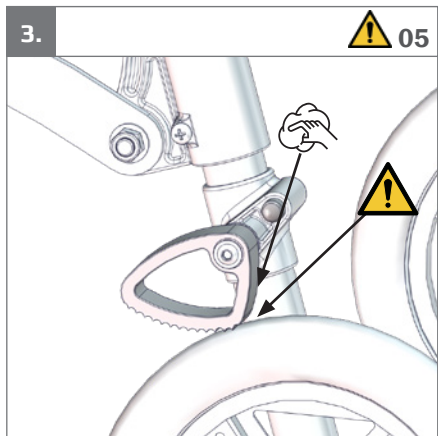
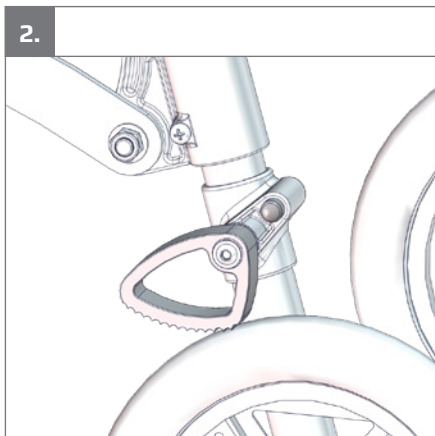
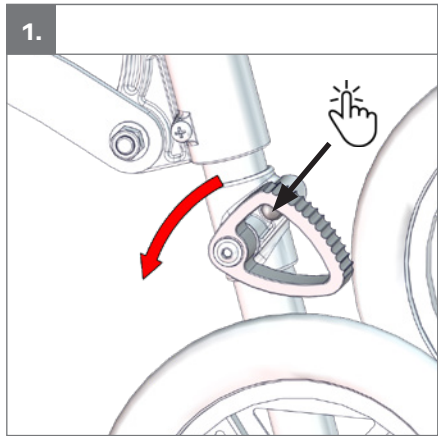
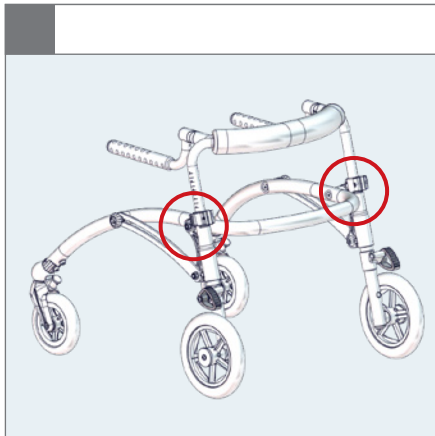
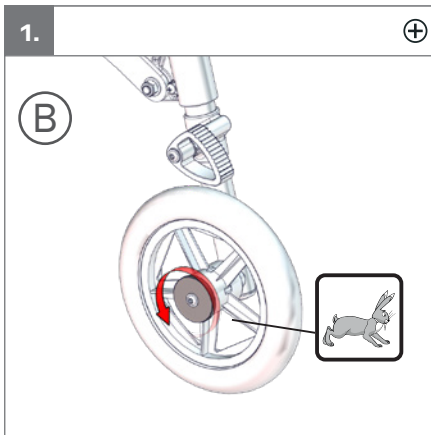
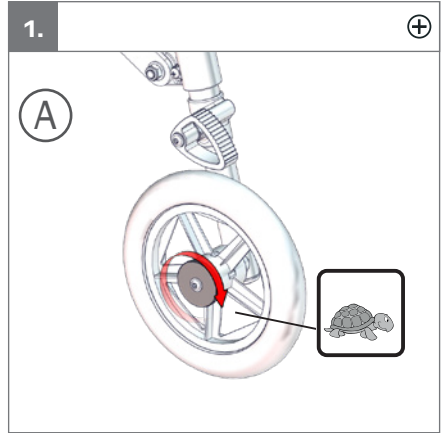
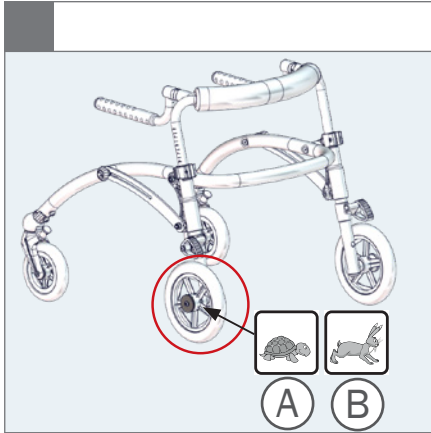
6.






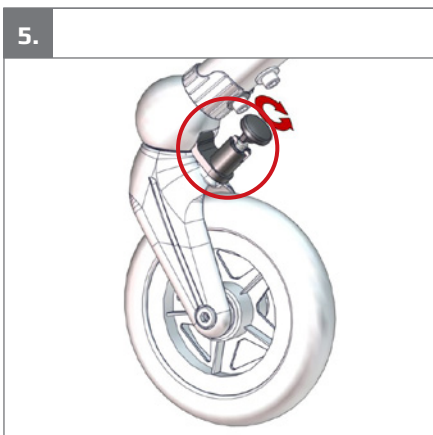
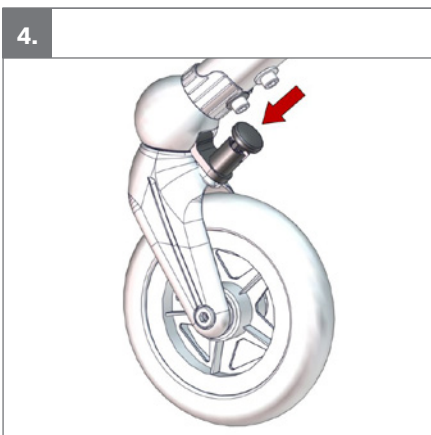
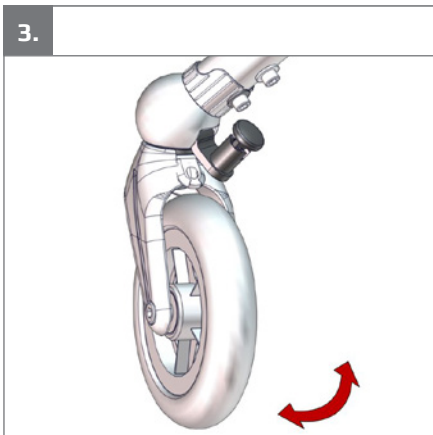
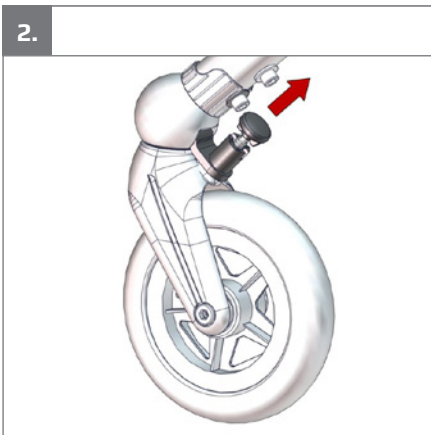
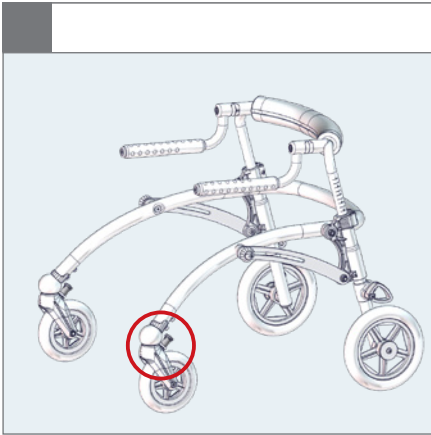
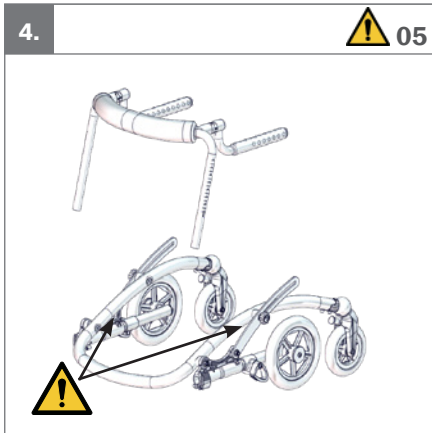
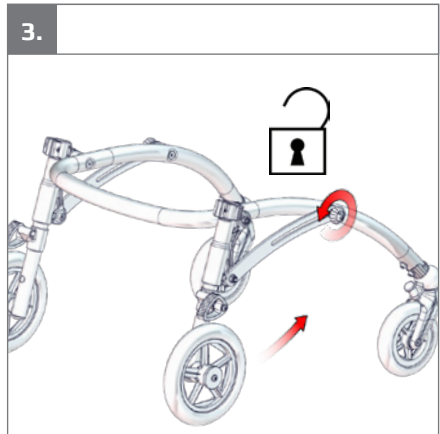
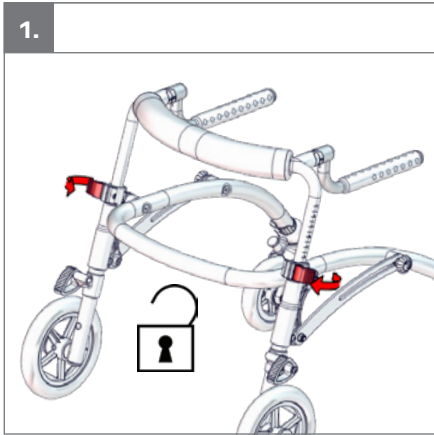
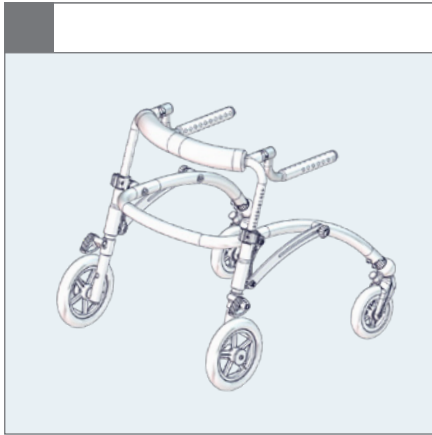
CROCODILE 3
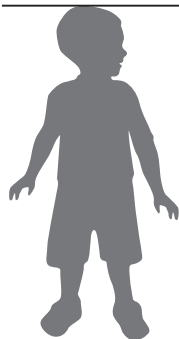
<1800 mm
< 71³/₄"

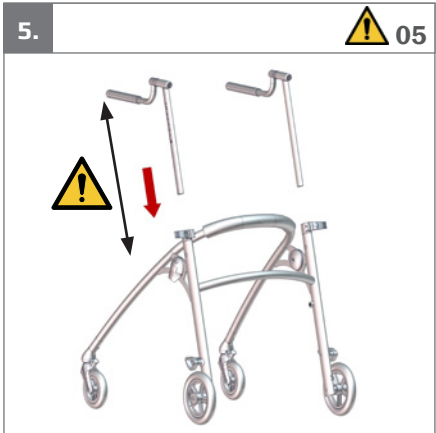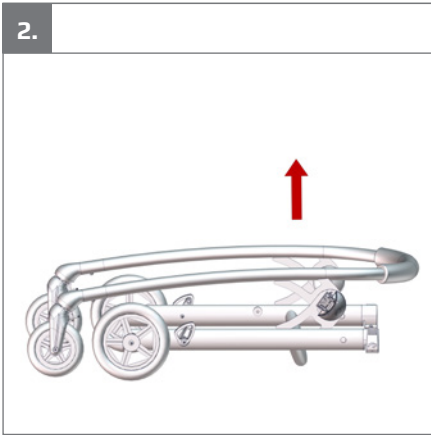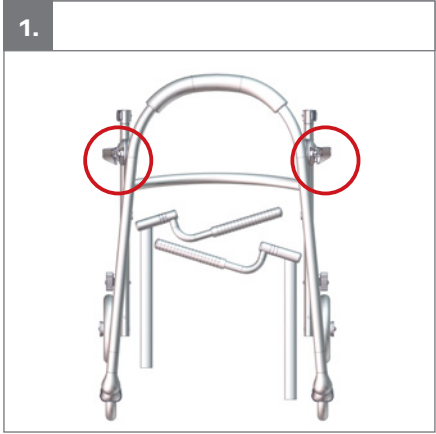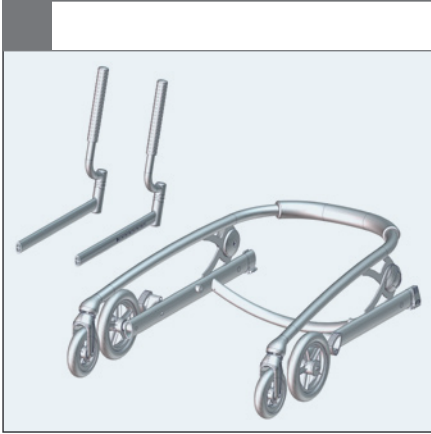
<80 kg
<176 lb







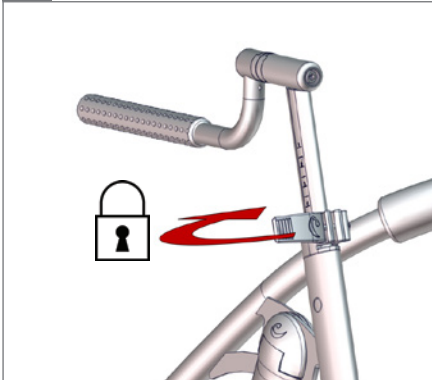
6.

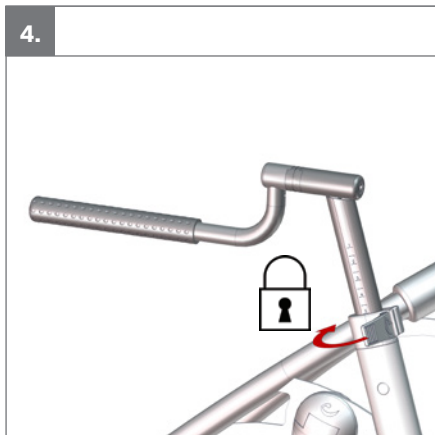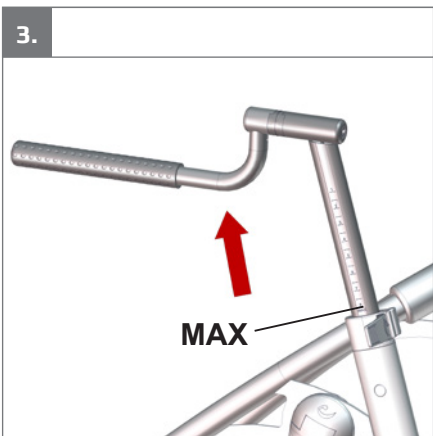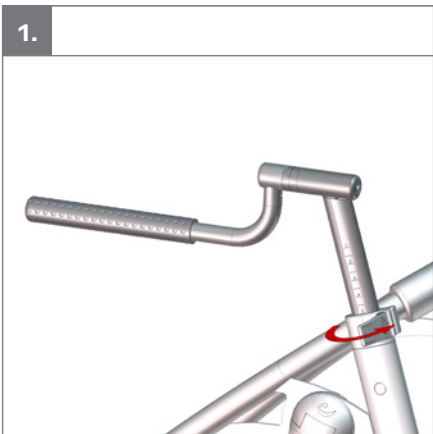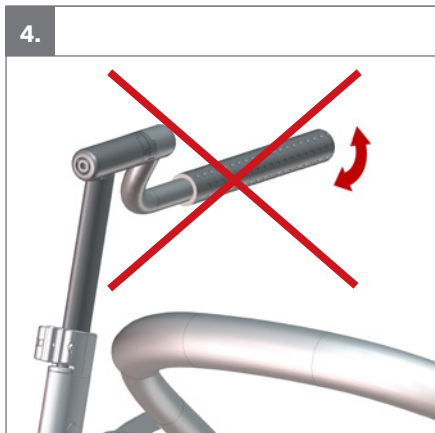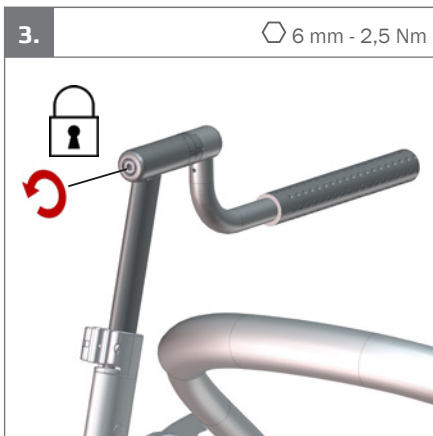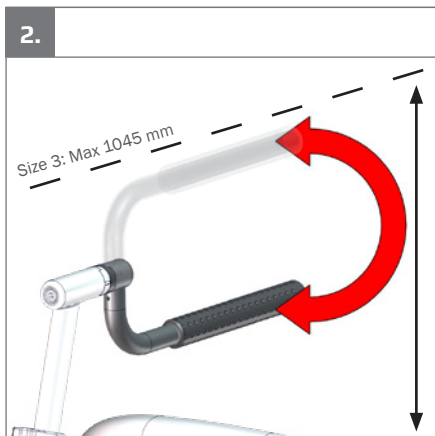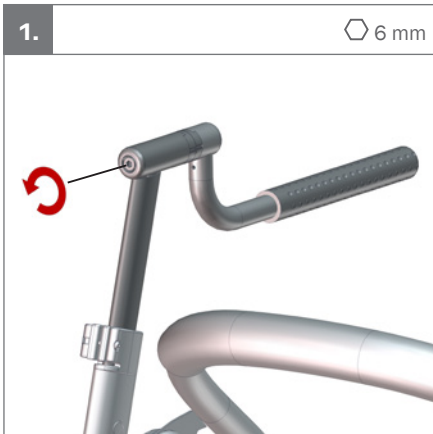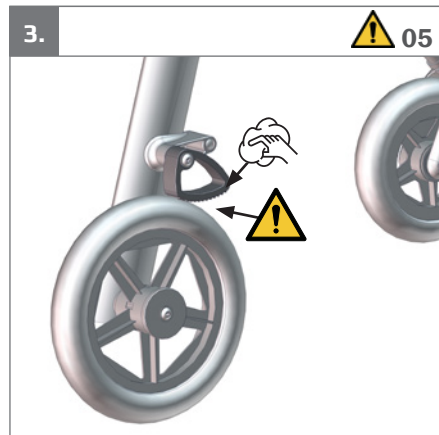
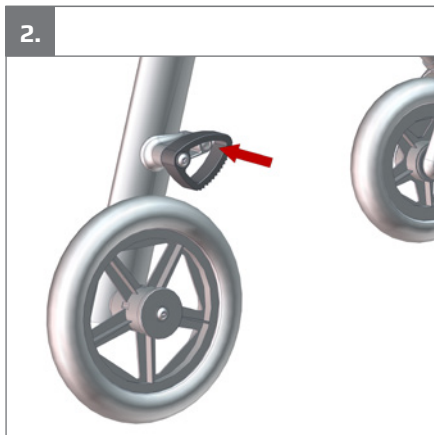
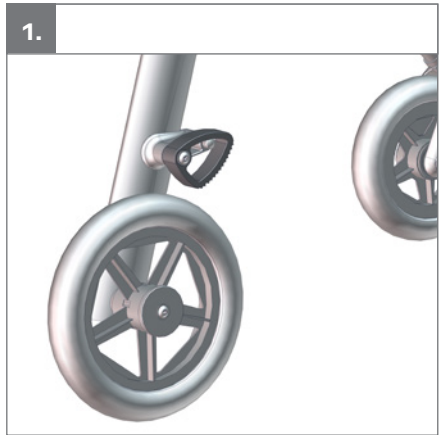
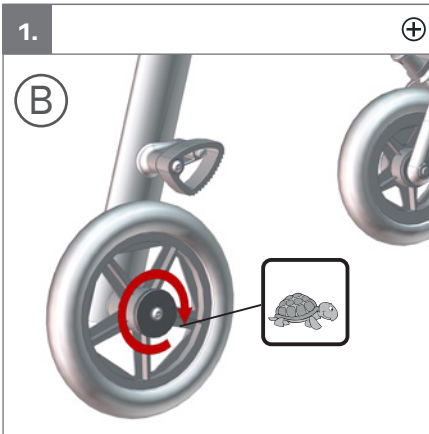
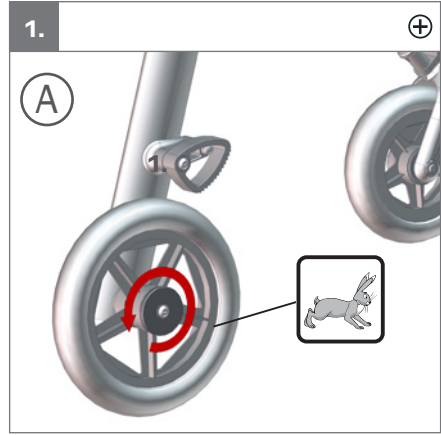
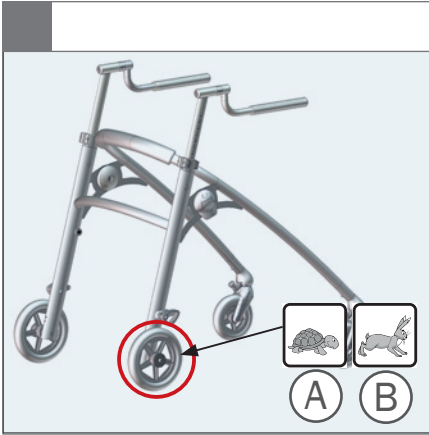
7.

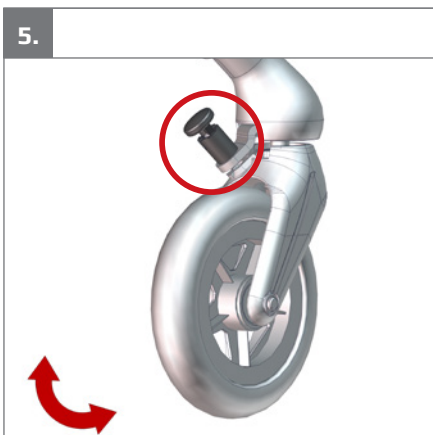
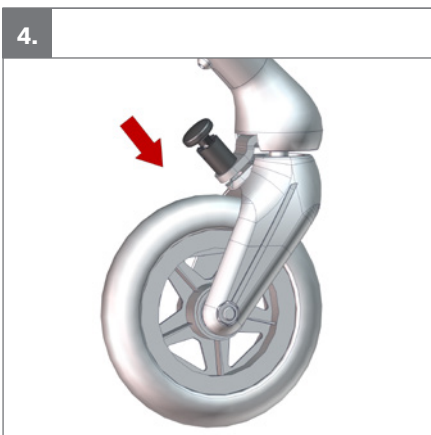
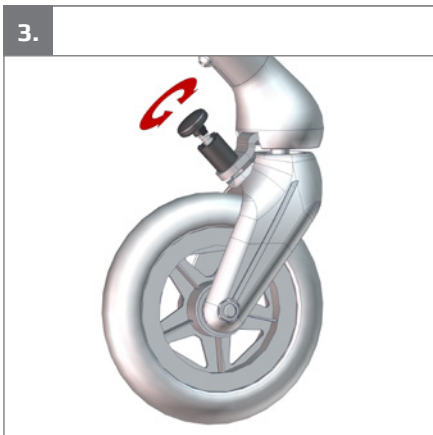
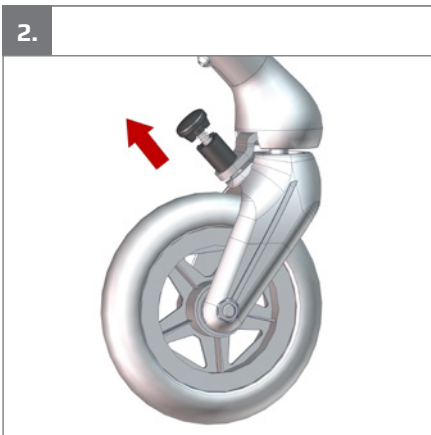
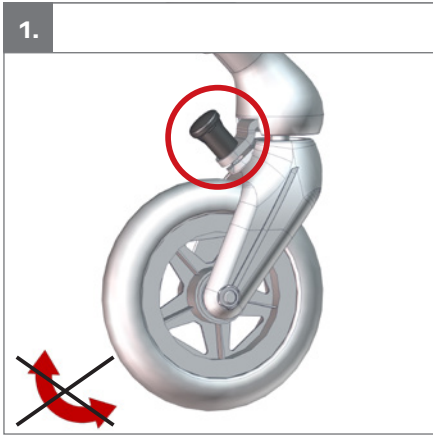
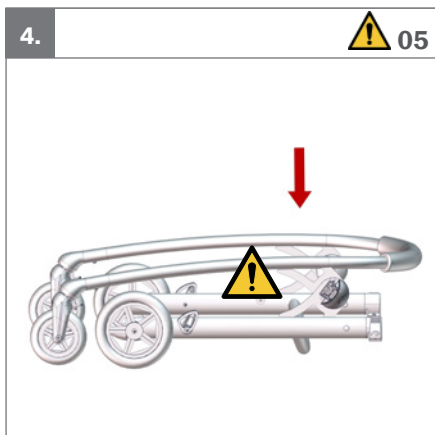
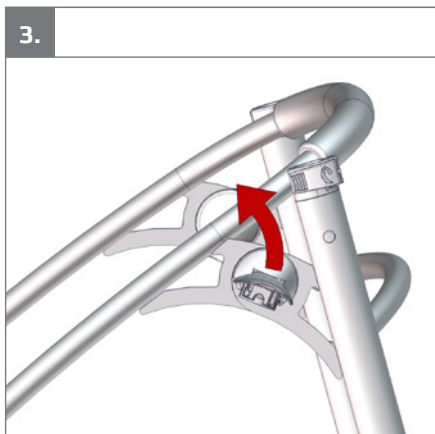
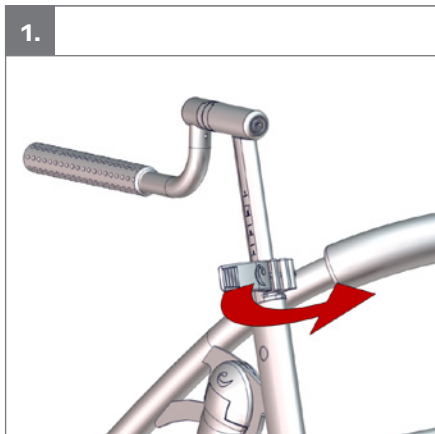
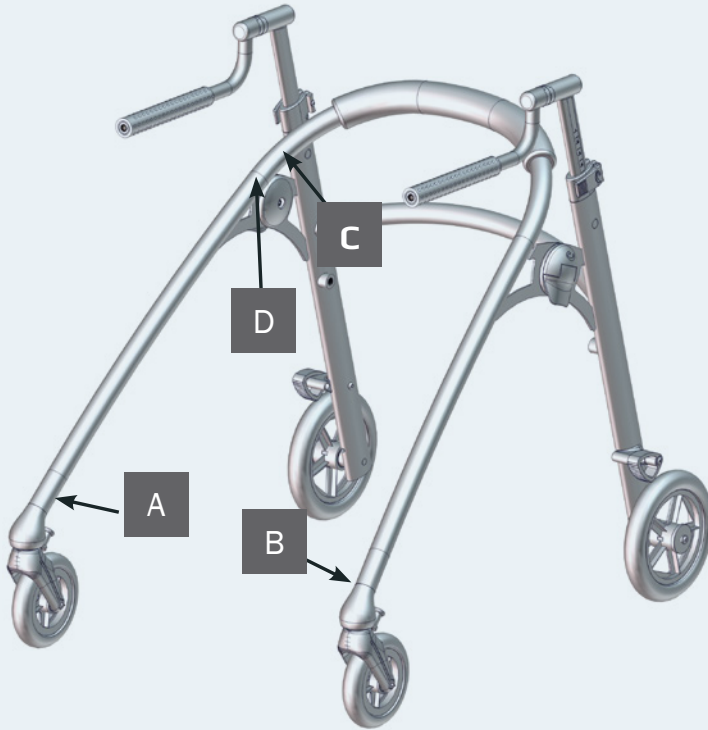
A

R82[®]
by Etac
R82 A/S Parallelvej 3
DK-8751 Gedved
etac.com
Made in: XXXXXX YYY-MM-DD

B

Product: xxxx - Size x
Art. no.: xxxxxxxx Max user mass xxx kg
Art. XXXXXXX
DMR ver.: XXXX Mass device: xxx kg
Max rated load: xxx kg

C

D


XXX mm XXX mm Ø 140 mm Ø 200 mm

mm
(inch)
kg
(lb)

mm (inch), kg (lb)	0	1	2	3
 Width between handles	100 - 250 mm (4" - 9¾")	190 - 350 mm (7½" - 13¾")	240 - 420 mm (9½" - 16½")	330 - 480 mm (13" - 19")
 Height from floor to handles	300 - 610 mm (11¾" - 24")	420 - 715 mm (16½" - 28¼")	550 - 840 mm (21¾" - 33")	750 - 1045 mm (29½" - 41¼")
 Height from floor to handles with starter kit	N/A	380 - 540 mm (15" - 21¼")	N/A	N/A
 Overall width	580 mm (22¾")	630 mm (24¾")	690 mm (27¼")	690 mm (27¼")
 Length	640 mm (25¼")	720 mm (28¼")	840 mm (33")	870 mm (34¼")
 Length with starter kit	N/A	660 mm (26")	N/A	N/A
 Turning width	1440 mm (56¾")	1600 mm (63")	1900 mm (75")	2000 mm (78¾")
 Max. user height	1050 mm (41¼")	1150 mm (45¼")	1500 mm (59")	1800 mm (70¾")
 Mass device	6 kg (13.4 lb)	7 kg (15.7 lb)	7 kg (15.7 lb)	12 kg (26.7 lb)
 Max. user mass	30 kg (66 lb)	30 kg (66 lb)	45 kg (99 lb)	80 kg (176 lb)
 Max. rated load	38 kg (83.8 lb)	39 kg (85.9 lb)	54 kg (119 lb)	94 kg (207.2 lb)
 Small castor (Width/Diameter)	W=28 mm/ D=140 mm (W=1"/D=5½")	W=28 mm/ D=140 mm (W=1"/D= 5½")	W=28 mm/ D=140 mm (W=1"/D= 5½")	W=28 mm/ D=140 mm (W=1"/D =5½")
 Large castor (Width/Diameter)	W=30 mm/ D=200 mm (W=1¼"/D=8")	W=30 mm/ D=200 mm (W=1¼"/D=8")	W=30 mm/ D=200 mm (W=1¼"/D=8")	W=30 mm/ D=200 mm (W=1¼"/D =8")
 Handgrip width	25 mm (1")	32 mm (1¼")	32 mm (1¼")	32 mm (1¼")
 Handgrip length	190 mm (7½")	190 mm (7½")	190 mm (7½")	250 mm (9¾")
 Brake grip distance	74 mm (3")	74 mm (3")	74 mm (3")	74 mm (3")

mm
(inch)
kg
(lb)


 Static stability downhill	15°	15°	15°	15°
 Static stability uphill	7°	7°	7°	7°
 Static stability sideways	4.5°	4.5°	4.5°	4.5°
 Folded length	650 mm (25¾")	720 mm (28½")	860 mm (34")	1000 mm (39½")
 Folded width	580 mm (22¾")	620 mm (24½")	690 mm (27¼")	690 mm (27¼")
 Folded height	300 mm (12")	310 mm (12¼")	270 mm (10¾")	290 mm (11½")

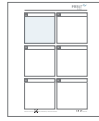9996097201
9996097218

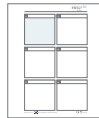
9996097199

9996097208

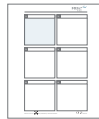9996097211

9996097229

9996097202

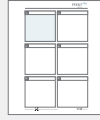
9996097205


9996097212

9996097203


9996097204

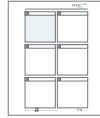9996097200

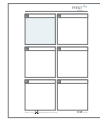
9996097207

9996097206

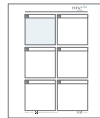
9996097210
9996097209

9996097217

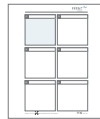9996098117

9996098122

9996098123

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ENGLISH

General

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product please read this User Guide before use and save it for future reference.

Intended purpose

The product is an assistive device for walking, intended for children and adults with disabilities.

Indications

The device can be used by children and adults with disabilities, who experience limitations in their ability to walk independently. The target group for the device is based on functional ability and anthropometry. It is not based on a specific health condition or age.

The user is the person walking in the device. The caregiver is the person operating the device. Never leave the user unattended in this device. Ensure permanent supervision by a caregiver.

The use of the device will be discontinued more times during the day where the user will be sitting, standing or lying down in another assistive device or a conventional product.

Contra indications

There are no known contraindications.

Precautions

Special attention should be given to the considerations about the configuration of the device and the use of accessories for users with limitation in their:

- Sensory functions and the sensation of pain, consider extra checks of the pressure caused by the support provided by the product and its accessories.
- ability to maintain a standing position, consider using accessories such as back and side support with a strap or a sling seat.
- ability to maintain the position of their head, consider using an assistive device for walking that provide support to the positioning of the head.

Application areas

The product is for indoor and outdoor use

Enclosed tools

- 2½+3+4+5+6 mm Allen key

CE Declaration of Conformity

This product conforms to the requirements of the Medical Devices Regulation (2017/745). Further, this product conforms to the requirements according to:

- EN 12182
- ISO 11199-2

- ISO 11199-3
 - EN 1985
- The CE mark must be removed when:
- Rebuilding the product
 - Using the product outside the intended purpose and declaration of conformity
 - Using other than original R82 spare parts and fittings

When combining the device with other products, the combination must not change the intended purpose of the products or modify the products in such a way that compliance with the applicable requirements may be affected. The person / company responsible for the combination of the products must ensure that the applicable requirements are fulfilled.

Applicable requirements examples:

- Material safety (biocompatibility, flammability), mechanical safety (stability, squeezing hazards, inter connection), electrical safety

Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

Accessories and spare parts

The products from R82 can be supplied with a variety of accessories which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

R82 Warranty

www.etac.com/support

Presales information and instructions

Presales information and instructions are available at www.etac.com, under your product, in the documents section.

Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions. Any adverse event in relation to the device must be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer.

Caregiver

- Read the "User manual" carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user
- This product contains small parts which could present a choking hazard if removed from their intended placing.
- Settings and adjustments to the product or accessories are only to be made by persons who have read the user guide.
- Always use the correct lifting techniques and aids for this purpose
- Never leave the user unattended in this product. Ensure permanent supervision by a caregiver
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible
- Ensure that moving parts does not squeeze or trap body parts when adjusting them
- Ensure that body parts are not trapped in openings for mounting of supports/accessories.
- Ensure fingers are not trapped when adjusting the handle position (ensure free clearance distance >25 mm)
- Be aware of any MAX mark indications before adjustment

Environment

- Observe the surface temperature on the product, before placing an user in it. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41 °C then allow the product to cool before use
- Use this product on a level surface without any obstacles. Be aware of reduced maneuverability on slippery surfaces
- When using on inclines, please refer to measurements

User

- If the user is close to the max. load limit and/or has a lot of involuntary move-

ments, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered

- Be aware of the user's capacity to exceed walking speed or ability to complete sharp turns, which may cause the product to become unstable

Product

- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children
- Activate the brakes before placing the user into the product
- Ensure the stability of the product is maintained before placing the user in it.
- Ensure the castors are fully-functional and secure before use
- Inspect the product and all of its accessories and replace any worn out parts before use
- This product is not to be occupied in a motor vehicle. The user must be positioned in a vehicle seat and the product must be stored in a luggage compartment
- The user must not occupy the product while it is being transported into or out of a vehicle by a lift
- Unoccupied device is suitable for land and air transport

Service & maintenance information

Service and maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to service and maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to service and maintain a device may compromise the clinical condition or safety of users and/or their caregivers. Do not perform service and maintenance while the user is in the product. Contact the local dealer for assistance, if needed, in setting up, using or how to perform service and maintenance of the device.

Lifetime

The lifetime of this product in normal use is 5 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded. The device may be reissued within its lifetime.

Service interval

The frequency of inspection may be altered depending upon use and wear. It is recommended that the product is inspected yearly, every time the product is re-issued for use and after a long-term storage (more than 4 months). The inspection should be carried out by a person who understands the use of the product.

Service checklist:

Check and adjust the following as per their intended use:

- Castors are moving freely and all castor locks/brakes are easily activated
 - Directional lock on each castor
 - Height and angle for handles are adjustable and stays in position after adjustment
 - Nuts and bolts on the product are tightened
 - Product for cracks or signs for worn-out parts
 - All labels on the product are intact
- Repair or replace any damaged or worn out parts.

Maintenance

Before every use it is recommended to: Wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before use.

Carry out a visual inspection, for any damaged or worn-out parts.

Washing

Main product

This product can be washed at 60° using a mild detergent for 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product. Unmount all accessories and wash them separately.

Fabric and covers

The cover can be washed at max. 60°C using a mild detergent in a washing machine.

Handwash

This product and its accessories can also be washed by hand. Use warm water and a mild detergent/ soap without chlorine and let all parts dry before use.

Disinfection

The product can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

Materials

- Aluminium
- Rubber
- Plastic
- Steel
- Fabric

Surface treatment

The following surface treatment processes were used to protect against corrosion:

- Lacquered surfaces = Polyester powder coating or ED-coating
- Non-lacquered aluminium parts = Anodized coating
- Non-lacquered steel surfaces = Galvanized

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Etac Ltd.
Gelbart AG