


# Rogue ALX Adjustable Height Backrest with Non-Height Adjustable Rigidizer Bar - Growth (Formerly Tsunami)

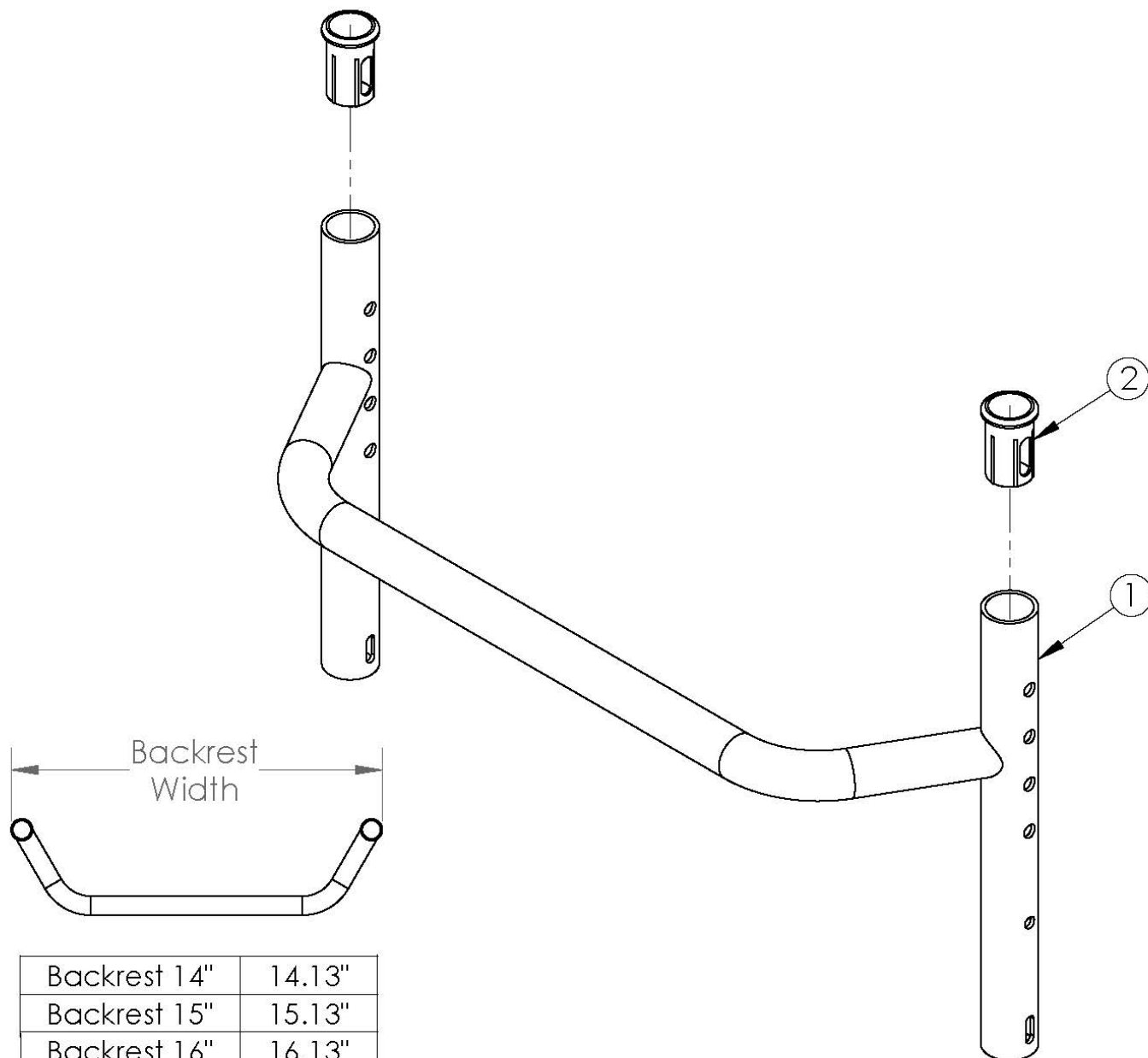

Backrest 14"	14.13"
Backrest 15"	15.13"
Backrest 16"	16.13"
Backrest 17"	17.13"
Backrest 18"	18.13"
Backrest 19"	19.13"
Backrest 20"	20.13"
Backrest 21"	21.13"
Backrest 22"	22.13"

If chair shipped after 7/3/2025 and Rigidizer Mounted Stroller Handles are present, please refer to quote the appropriate Stroller Handle Page for the Rigidizer.

Item Number	Part Number	Description	Quantity
1a, 2	110641	14" Rigid Backrest with Sleeves <i>Used with standard push handles, low profile push handles or omit push handles in any back height range. Also used with fold down push handles with a tall back height range.</i>	1
1b, 2	110642	15" Rigid Backrest with Sleeves <i>Used with standard push handles, low profile push handles or omit push handles in any back height range. Also used with fold down push handles with a tall back height range.</i>	1
1c, 2	110643	16" Rigid Backrest with Sleeves <i>Used with standard push handles, low profile push handles or omit push handles in any back height range. Also used with fold down push handles with a tall back height range.</i>	1
1d, 2	110644	17" Rigid Backrest with Sleeves <i>Used with standard push handles, low profile push handles or omit push handles in any back height range. Also used with fold down push handles with a tall back height range.</i>	1
1e, 2	110645	18" Rigid Backrest with Sleeves <i>Used with standard push handles, low profile push handles or omit push handles in any back height range. Also used with fold down push handles with a tall back height range.</i>	1
1f, 2	110646	19" Rigid Backrest with Sleeves <i>Used with standard push handles, low profile push handles or omit push handles in any back height range. Also used with fold down push handles with a tall back height range.</i>	1
1g, 2	110647	20" Rigid Backrest with Sleeves <i>Used with standard push handles, low profile push handles or omit push handles in any back height range. Also used with fold down push handles with a tall back height range.</i>	1
1h, 2	110648	21" Rigid Backrest with Sleeves <i>Used with standard push handles, low profile push handles or omit push handles in any back height range. Also used with fold down push handles with a tall back height range.</i>	1
1i, 2	110649	22" Rigid Backrest with Sleeves <i>Used with standard push handles, low profile push handles or omit push handles in any back height range. Also used with fold down push handles with a tall back height range.</i>	1
1j, 2	110654	14" Rigid Backrest with Sleeves for Fold Down Push Handles, Short/Medium <i>Used with fold down push handles with a short or medium back height range</i>	1
1k, 2	110655	15" Rigid Backrest with Sleeves for Fold Down Push Handles, Short/Medium <i>Used with fold down push handles with a short or medium back height range</i>	1
1l, 2	110656	16" Rigid Backrest with Sleeves for Fold Down Push Handles, Short/Medium <i>Used with fold down push handles with a short or medium back height range</i>	1

Item Number	Part Number	Description	Quantity
1m, 2	110657	17" Rigid Backrest with Sleeves for Fold Down Push Handles, Short/Medium <i>Used with fold down push handles with a short or medium back height range</i>	1
1n, 2	110658	18" Rigid Backrest with Sleeves for Fold Down Push Handles, Short/Medium <i>Used with fold down push handles with a short or medium back height range</i>	1
1o, 2	110659	19" Rigid Backrest with Sleeves for Fold Down Push Handles, Short/Medium <i>Used with fold down push handles with a short or medium back height range</i>	1
1p, 2	110660	20" Rigid Backrest with Sleeves for Fold Down Push Handles, Short/Medium <i>Used with fold down push handles with a short or medium back height range</i>	1
1q, 2	110661	21" Rigid Backrest with Sleeves for Fold Down Push Handles, Short/Medium <i>Used with fold down push handles with a short or medium back height range</i>	1
1r, 2	110662	22" Rigid Backrest with Sleeves for Fold Down Push Handles, Short/Medium <i>Used with fold down push handles with a short or medium back height range</i>	1