


Rogue ALX Heavy Duty Backrest Mount and Hardware (Formerly Tsunami)

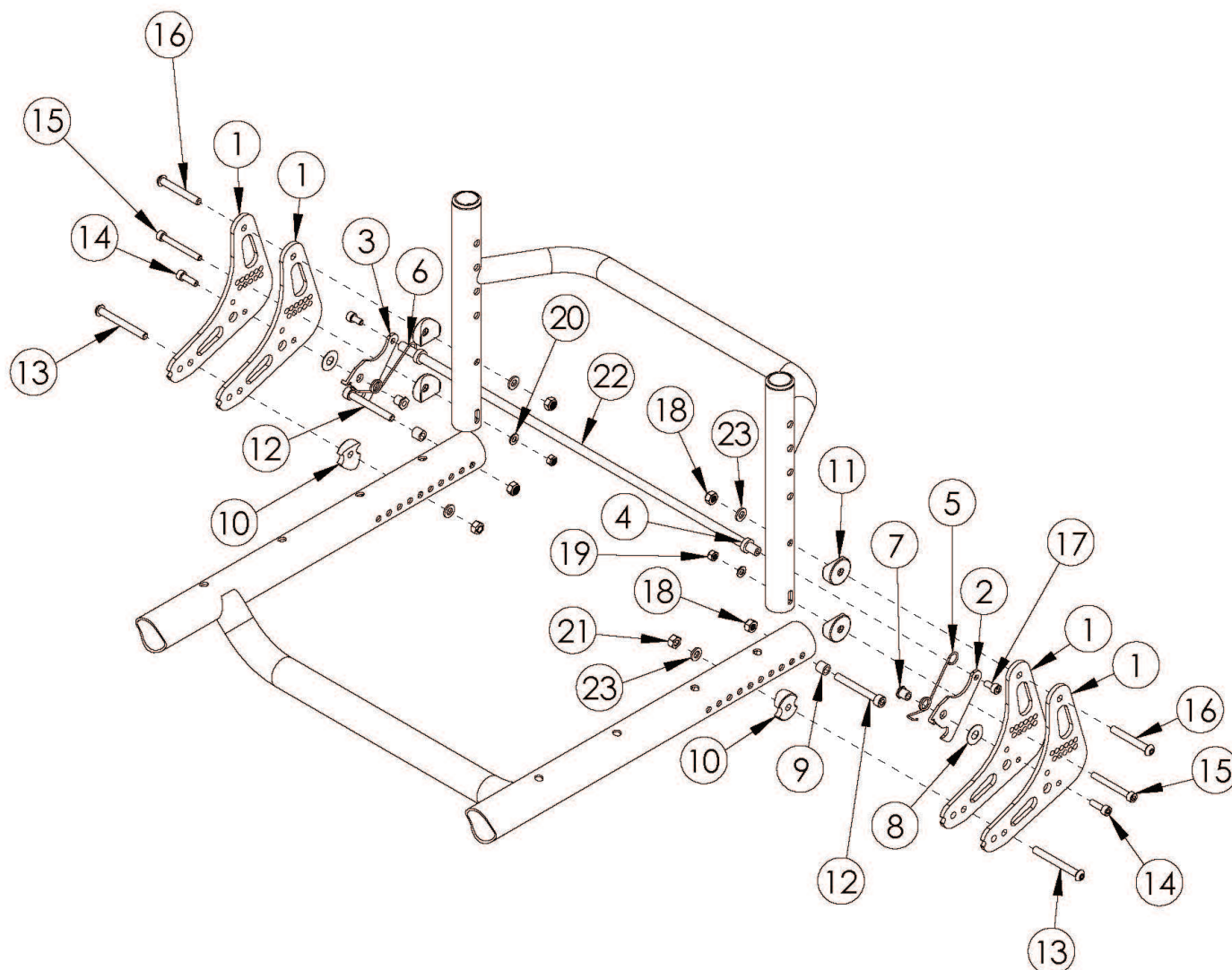

Heavy Duty option cannot be retrofitted This Heavy Duty Backrest Mount and Hardware is only used with the Tsunami Style Backrest Backrest Release Bar based on seat width

Item Number	Part Number	Description	Quantity
1-3, 5-21, 23	111344	Heavy Duty Backrest Plate Kit, RG ALX	1
1	000448	Backrest Plate, RG ALX/LW XP	2
2	000228-L	Backrest Latch Left, RG ALX/LW XP	1
3	000228-R	Backrest Latch Right, RG ALX/LW XP	1
4	000432	Backrest Latch Spring Retainer, RG ALX/LW XP	2

Item Number	Part Number	Description	Quantity
5	000418L	Backrest Latch Spring Left, RG ALX/LW XP	1
6	000418R	Backrest Latch Spring Right, RG ALX/LW XP	1
7	000420	Backrest Latch Pivot Bushing	2
8	000428	Wear Washer	2
9	000240	Backrest Latch Strike Bushing / Close Mount Handrim Spacer	2
10	000248	Saddle 1.375"	2
11	000055	Saddle 1"	4
12	102473	M6 X 50 CLS12.9 SHCS BLK ZC	2
13a	101125	M6 x 55 BTN SHCS BLKZC <i>Used when bolt mounts the back plate to the frame, no engagement to the tower</i>	2
13b	101126	M6 x 60 BTN SHCS BLKZC <i>Used when bolt goes through tower and slide, but not backrest plate</i>	2
13c	101127	M6 x 65 BTN SHCS BLKZC <i>Used when bolt goes through backrest plate and tower</i>	2
13d	101128	M6 x 70 BTN SHCS BLKZC <i>Used when bolt goes through backrest plate, tower and slide and/or when bolt goes through backrest plate and tower</i>	2
14	101120	M5 x 16 SHCS BLZ	2
15	101186	M5 X 45 CLS12.9 DIN912 SHCS P/THD BLZ	2
16	100670	M6 x 45 BTN SHCS BLK ZC	2
17	101766	M5 X 10 Cls12.9 Din912 Shcs Pln	2
18	100658	M6 Nylock Nut Blk Zn	4
19	100657	M5 Nylock Nut Blk Zn	2
20	101856	Flat Washer, M5, Blk Zn	2
21	101456	M6 Ovl Top L/N ZC	2
22a, 4	503347	Backrest Release Bar with Retainers 14"	1
22b, 4	503348	Backrest Release Bar with Retainers 15"	1
22c, 4	503349	Backrest Release Bar with Retainers 16"	1
22d, 4	503350	Backrest Release Bar with Retainers 17"	1

Item Number	Part Number	Description	Quantity
22e, 4	503351	Backrest Release Bar with Retainers 18"	1
22f, 4	503352	Backrest Release Bar with Retainers 19"	1
22g, 4	503353	Backrest Release Bar with Retainers 20"	1
22h, 4	503354	Backrest Release Bar with Retainers 21"	1
22i, 4	503355	Backrest Release Bar with Retainers 22"	1
23	100746	M6 Flat Washer Blk Zn	2