


Focus CR Caster Forks and Stems

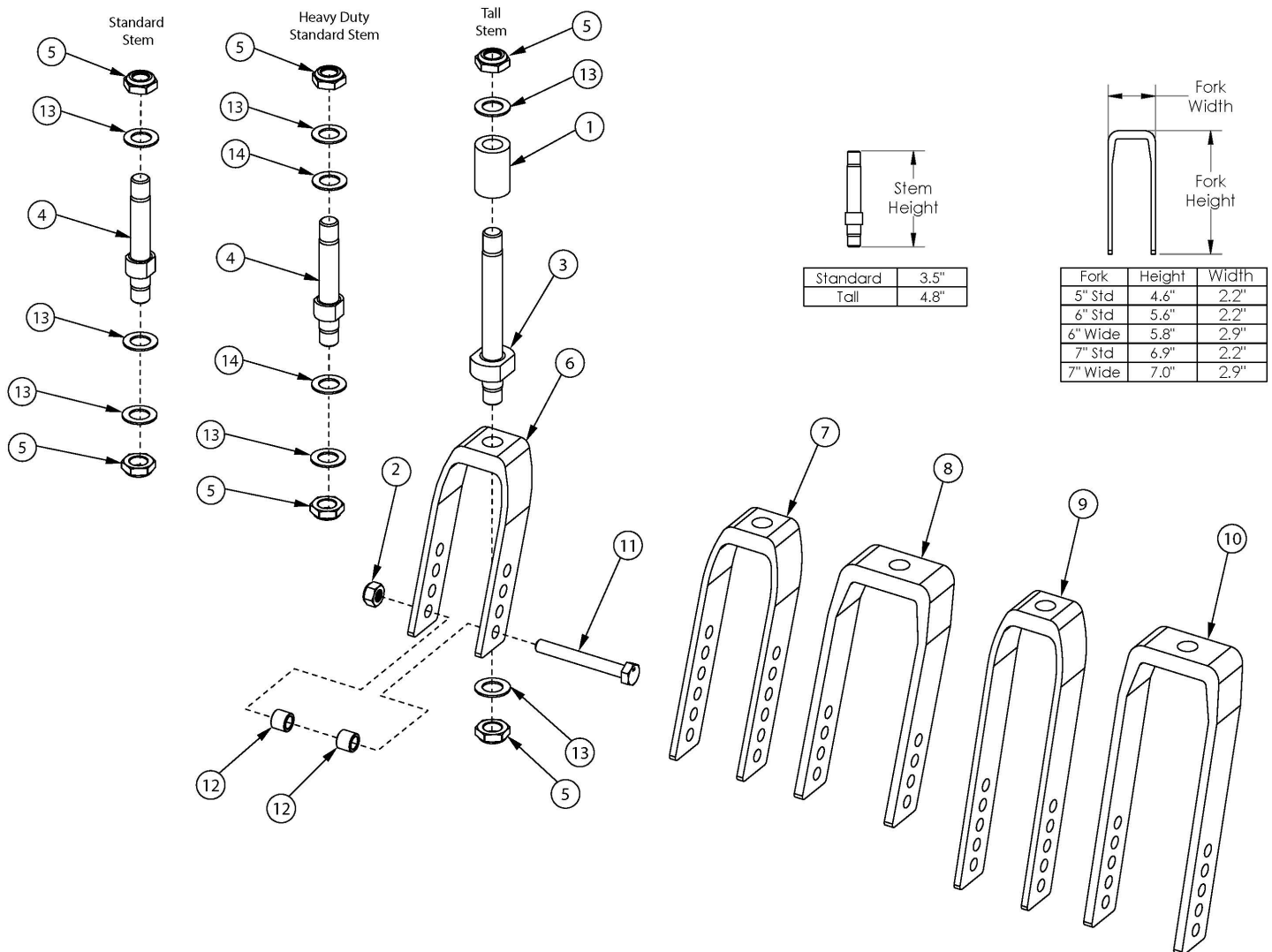

Item Number	Part Number	Description	Quantity	Retail
4a, 5, 6, 13	503816	Fork Assembly 5" Std Stem, Tilt <i>Used with standard weight capacity</i>	1	\$58.65
4a, 5, 7, 13	503814	Fork Assembly 6" Std Stem, Tilt <i>Used with standard weight capacity</i>	1	\$58.65
4a, 5, 8, 13	503817	Fork Assembly 6" Wide Std Stem, Tilt <i>Used with standard weight capacity. Only used with 2" wide casters.</i>	1	\$82.14
4b, 5, 8, 13, 14	506368	Fork Assembly 6" Wide Heavy Duty Std Stem Focus <i>Used with heavy duty weight capacity. Only used with 2" wide casters.</i>	1	\$213.00
4a, 5, 9, 13	503815	Fork Assembly 7" Std Stem, Tilt <i>Used with standard weight capacity</i>	1	\$58.65

Item Number	Part Number	Description	Quantity	Retail
4a, 5, 10, 13	503818	Fork Assembly 7" Wide Std Stem, Tilt <i>Used with standard weight capacity. Only used with 2" wide casters.</i>	1	\$82.14
4b, 5, 10, 13, 14	503122	Fork Assembly 7" Wide Heavy Duty Std Stem Focus <i>Used with heavy duty weight capacity. Only used with 2" wide casters.</i>	1	\$213.00
1, 3, 5, 6, 13	506369	Fork Assembly 5" Tall Stem Focus <i>Used with standard and heavy duty weight capacity</i>	1	\$124.31
1, 3, 5, 7, 13	506370	Fork Assembly 6" Tall Stem Focus <i>Used with standard and heavy duty weight capacity</i>	1	\$124.31
1, 3, 5, 8, 13	506371	Fork Assembly 6" Wide Tall Stem Focus <i>Used with standard and heavy duty weight capacity. Only used with 2" wide casters.</i>	1	\$144.47
1, 3, 5, 9, 13	506372	Fork Assembly 7" Tall Stem Focus <i>Used with standard and heavy duty weight capacity.</i>	1	\$124.31
1, 3, 5, 10, 13	506373	Fork Assembly 7" Wide Tall Stem Focus <i>Used with standard and heavy duty weight capacity. Only used with 2" wide casters.</i>	1	\$144.47
2, 11a, 12a	502653	Caster Attachment Hardware Kit CAT/FCS	1	\$7.88
2, 11b, 12b	502654	Caster Attachment Hardware Kit for 6" and 7" Wide Fork, CAT/FCS <i>Only used with 2" wide casters</i>	1	\$17.29
1	002476	Spacer, 3/4" OD x 1/2" ID x 1.23	1	\$2.14
2	100659	M8 Nylock Nut Blk Zn	1	\$1.05
1, 3, 5, 13	506374	Caster Stem Tall, Adult Tilt <i>Used with standard and heavy duty weight capacity</i>	1	\$26.93
4a, 5, 13	503819	Caster Stem Standard, Focus <i>Used with standard weight capacity</i>	1	\$17.82
4b, 5, 13	503820	Caster Stem Standard HD, Focus <i>Used with heavy duty weight capacity</i>	1	\$108.41
5	100529	M12 x 1.25mm Lock Nut Black	2	\$1.34
5, 6	503823	Caster Fork 5" Focus	1	\$44.95
5, 7	503824	Caster Fork 6" Focus	1	\$44.95
5, 8	503825	Caster Fork 6" Wide Focus <i>Only used with 2" wide casters</i>	1	\$65.12
5, 9	503826	Caster Fork 7" Focus	1	\$44.95
5, 10	503827	Caster Fork 7" Wide Focus	1	\$65.12

Item Number	Part Number	Description	Quantity	Retail
		<i>Only used with 2" wide casters</i>		
11a	101829	M8 x 65 HHCS Blk Zn	1	\$3.15
11b	101831	M8 x 80 HHCS Blk Zn <i>Only used with 2" wide casters</i>	1	\$3.07
12a	100792	Caster Spacer D8 x D12 x 10mm <i>Used with all casters except 6x2 and 8x2</i>	2	\$2.14
12b	001246	Caster Spacer Black 6 x 2 Poly .472ODX.319IDX.370LG <i>Only used with 6" x 2" casters</i>	2	\$5.84
13	100682	Washer 1/2" .518 x .875 x .047 F/W Black Zinc	3	\$1.05
14	002593	Caster Stem Washer Heavy Duty <i>Used with standard stem in combination with heavy duty option; Use qty 3 of 002593 and qty 1 of 100682</i>	3	\$19.27