


Rogue XP Drum Brake

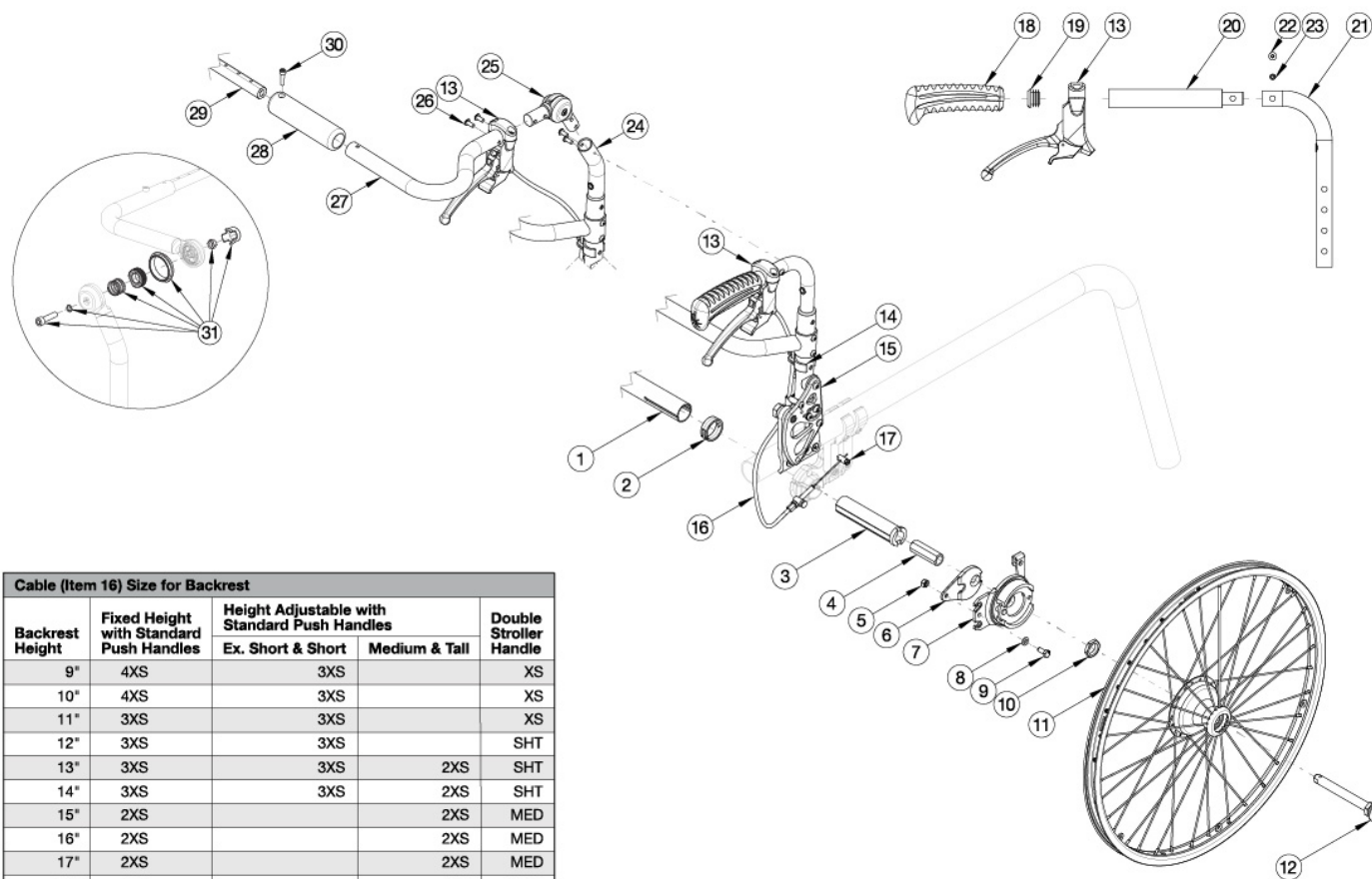

Cable (Item 16) Size for Backrest				
Backrest Height	Fixed Height with Standard Push Handles	Height Adjustable with Standard Push Handles		Double Stroller Handle
		Ex. Short & Short	Medium & Tall	
9"	4XS	3XS		XS
10"	4XS	3XS		XS
11"	3XS	3XS		XS
12"	3XS	3XS		SHT
13"	3XS	3XS	2XS	SHT
14"	3XS	3XS	2XS	SHT
15"	2XS		2XS	MED
16"	2XS		2XS	MED
17"	2XS		2XS	MED
18"	2XS		2XS	MED

For a complete retrofit, quote a drum brake retrofit kit (qty 1), backtube / stroller handle kit (qty 2), drum brake cable kit (qty 2) and new rear towers (qty 2). Replacement Camber Mount Towers may be quoted, but must be ordered through Customer Service. If chair includes both push handles and rigidizer mounted handles, Drum Brake handles are installed on the rigidizer mounted handles. Most Drum Brake Retrofit Kits contain two wheels, but do not contain rear tires or handdrims. If 24" Pneumatic w/ Airless Insert tires are needed, kits 124260 - 124264 contain Pneumatic w/ Airless Insert tires and the inserts but no handdrims. For item number 1, a new camber tube is required when changing the degree of camber by more than 2 degrees. See Rigid Camber Tube and Adapters parts manual page if needing a new camber tube. Used on chairs invoiced starting on 3/12/25

Item Number	Part Number	Description	Quantity
3a, 4-6, 7a, 7b, 8-10, 11a, 12	124240	Drum Brake Retrofit Kit 20" MAXX Spoke, 0° Camber Rogue XP	1
3b, 4-6, 7a, 7b, 8-10, 11a, 12	124241	Drum Brake Retrofit Kit 20" MAXX Spoke, 2° Camber Rogue XP	1
3c, 4-6, 7a, 7b, 8-10, 11a, 12	124242	Drum Brake Retrofit Kit 20" MAXX Spoke, 4° Camber Rogue XP	1
3d, 4-6, 7a, 7b, 8-10, 11a, 12	124243	Drum Brake Retrofit Kit 20" MAXX Spoke, 6° Camber Rogue XP	1
3e, 4-6, 7a, 7b, 8-10, 11a, 12	124244	Drum Brake Retrofit Kit 20" MAXX Spoke, 8° Camber Rogue XP	1
3a, 4-6, 7a, 7b, 8-10, 11b, 12	124245	Drum Brake Retrofit Kit 22" MAXX Spoke, 0° Camber Rogue XP	1

Item Number	Part Number	Description	Quantity
3b, 4-6, 7a, 7b, 8-10, 11b, 12	124246	Drum Brake Retrofit Kit 22" MAXX Spoke, 2° Camber Rogue XP	1
3c, 4-6, 7a, 7b, 8-10, 11b, 12	124247	Drum Brake Retrofit Kit 22" MAXX Spoke, 4° Camber Rogue XP	1
3d, 4-6, 7a, 7b, 8-10, 11b, 12	124248	Drum Brake Retrofit Kit 22" MAXX Spoke, 6° Camber Rogue XP	1
3e, 4-6, 7a, 7b, 8-10, 11b, 12	124249	Drum Brake Retrofit Kit 22" MAXX Spoke, 8° Camber Rogue XP	1
3a, 4-6, 7a, 7b, 8-10, 11c, 12	124250	Drum Brake Retrofit Kit 24" MAXX Spoke, 0° Camber Rogue XP	1
3b, 4-6, 7a, 7b, 8-10, 11c, 12	124251	Drum Brake Retrofit Kit 24" MAXX Spoke, 2° Camber Rogue XP	1
3c, 4-6, 7a, 7b, 8-10, 11c, 12	124252	Drum Brake Retrofit Kit 24" MAXX Spoke, 4° Camber Rogue XP	1
3d, 4-6, 7a, 7b, 8-10, 11c, 12	124253	Drum Brake Retrofit Kit 24" MAXX Spoke, 6° Camber Rogue XP	1
3e, 4-6, 7a, 7b, 8-10, 11c, 12	124254	Drum Brake Retrofit Kit 24" MAXX Spoke, 8° Camber Rogue XP	1
3a, 4-6, 7a, 7b, 8-10, 11c, 12	124260	Drum Brake Retrofit Kit 24" MAXX Spoke, Pneumatic w/ Airless Insert Tire (Black), 0° Camber Rogue XP	1
3b, 4-6, 7a, 7b, 8-10, 11c, 12	124261	Drum Brake Retrofit Kit 24" MAXX Spoke, Pneumatic w/ Airless Insert Tire (Black), 2° Camber Rogue XP	1
3c, 4-6, 7a, 7b, 8-10, 11c, 12	124262	Drum Brake Retrofit Kit 24" MAXX Spoke, Pneumatic w/ Airless Insert Tire (Black), 4° Camber Rogue XP	1
3d, 4-6, 7a, 7b, 8-10, 11c, 12	124263	Drum Brake Retrofit Kit 24" MAXX Spoke, Pneumatic w/ Airless Insert Tire (Black), 6° Camber Rogue XP	1
3e, 4-6, 7a, 7b, 8-10, 11c, 12	124264	Drum Brake Retrofit Kit 24" MAXX Spoke, Pneumatic w/ Airless Insert Tire (Black), 8° Camber Rogue XP	1
18, 19, 20a, 21a, 22a, 23a	113221	Backtube w/ Push Handle Short with Backcane Extension, Drum Brake, Rigid <i>Used with extra short and short back height ranges with height adjustable backposts on Klik and Rogue XP. Used with short back height range with height adjustable backposts on Little Wave XP. Used with fixed height back posts on Klik, Rogue XP and Little Wave XP.</i>	2
18, 19, 20b, 21b, 22a, 23a	113222	Backtube w/ Push Handle Tall with Backcane Extension, Drum Brake, Rigid <i>Used with medium and tall back height ranges with height adjustable back posts on Klik and Rogue XP. Used with medium back height range with height adjustable backposts on Little Wave XP.</i>	2
18, 19, 20b, 21c, 22a, 23a	113223	Backtube w/ Push Handle Tall with Backcane Extension, Drum Brake, Rigid <i>Used with tall back height range with height adjustable back posts on Little Wave XP.</i>	2
18, 19, 20c, 21d, 22b, 23b	113197	Stroller Handle Back Post 8° Bend Short with Backcane Extension, Drum Brake <i>Used on chairs with 15" or greater rear seat height</i>	2
18, 19, 20c, 21d, 22b, 23b	113242	Stroller Handle Short Assembly with Backcane Extension, Drum Brake, Each, Rigid <i>Used on chairs with 15" or greater rear seat height. This kit also contains the stroller handle clamp assembly.</i>	2

Item Number	Part Number	Description	Quantity
18, 19, 20c, 21e, 22b, 23b	113224	Stroller Handle Back Post 8° Bend Tall with Backcane Extension, Drum Brake <i>Used on chairs with less than 15" rear seat height</i>	2
18, 19, 20c, 21e, 22b, 23b	113243	Stroller Handle Tall Assembly with Backcane Extension, Drum Brake, Each, Rigid <i>Used on chairs with less than 15" rear seat height. This kit also contains the stroller handle clamp assembly.</i>	2
13-16, 17a, 18	113216	Drum Brake Cable Kit with Brake Lever, 4XS, Rigid	2
13-16, 17b, 18	113217	Drum Brake Cable Kit with Brake Lever, 3XS, Rigid	2
13-16, 17c, 18	113218	Drum Brake Cable Kit with Brake Lever, 2XS, Rigid	2
13-16, 17d, 18	113219	Drum Brake Cable Kit with Brake Lever, XS, Rigid	2
13-16, 17e, 18	113220	Drum Brake Cable Kit with Brake Lever, Short, Rigid	2
13-16, 17f, 18	113196	Drum Brake Cable Kit with Brake Lever, Medium, Rigid	2
2	111808	Detent Clip For Camber Tube	2
3a	119688	0 Degree Spud Rogue2 / Rogue XP	2
4-6 7a, 7b, 8- 10, 12	113214	Drum Brake Rotor Kit with Hardware PAIR, Rigid	1
4-6, 9-10, 12	113215	Hardware Kit for Drum Brake Rotor PAIR, Rigid	1
4-6, 7a, 9-10, 12	113239	Drum Brake Rotor Kit with Hardware, Side B, Rigid	1
4-6, 7b, 9-10, 12	113240	Drum Brake Rotor Kit with Hardware, Side A, Rigid	1
4-6, 9-10, 12	113241	Hardware Kit for Drum Brake Rotor, One Side, Rigid	1
6	110635	Adapter, Drum Brake, Rigid	1
11a	111731	Wheel 20" MAXX Spoke, Drum Brake	2
11b	111732	Wheel 22" MAXX Spoke, Drum Brake	2
11c	111733	Wheel 24" MAXX Spoke, Drum Brake	2
11c	118358	24" Spoke for Drum Brake, Pneumatic w/ Airless Insert Tire (Black) <i>Replaced part number 117432. Does not include the handrim. Quote handrim hardware if needed.</i>	2
12	101108	Axle Quick Release w/ Button 1/2" x 99MM	2
14	111735	Brake Lever, Lock, Ang, 22mm	2
14	001117	Cable Clip 1"	2
15	001239	P-Clip 3/16"	2
16a	111605	Drum Brake / Hemi Wheel Lock 4XS Cable	2

Item Number	Part Number	Description	Quantity
16b	111606	Drum Brake / Hemi Wheel Lock 3XS Cable	2
16c	111545	Drum Brake / Hemi Wheel Lock 2XS Cable	2
16d	111546	Drum Brake / Hemi Wheel Lock XS Cable	2
16e	002468	Drum Brake / Hemi Wheel Lock Short Cable	2
16f	002469	Drum Brake / Hemi Wheel Lock Medium Cable	2
17	111598	Pinch Bolt	2