

Fresno, California 93727-1328 (559) 292-2171 • (800) 456-8168

RISE
I C A L
(7-1328
456-8168INSTRUCTION SHEET
PEDAL EXTENSION KIT
INSTALLATION



• Please read and understand these instructions before beginning the installation.

Allen Wrench set

Tools Required: Torque Wrench 1/2" Socket 1/2" End Wrench

Parts Required: 119504

1. (Fig 1) Remove base frame end plug (item A) from right base frame tube. Remove the Pedal Assembly (item M) by unscrewing the two hex head screws (item F). Set screws and washers(item G) aside for reinstallation.

2. (Fig 2)Insert Adapter Tube (item E) into end of base tube, aligning hole with rear hole in base frame. Fasten in place with screw (item B), flat washers (item C) and lock nut (item D). Tighten to 120 in-lb.

3. (Fig 3) Loosen SHCS (item L) on Linkage Clamp (item K) just enough that the Linkage Rod (item J) Slides easily. Slide Extension Tube (item H) over adapter tube, aligning the front hole in the extension tube with the exposed hole in the adapter tube. Insert screws (item F) with lock washers (item G) through extension tube holes and into pedal assembly (item M) and tighten to 120 in-Ib. Re tighten SHCS (item L) when Linkage Rod is properly adjusted.

4. (Fig 4) Re-Install plug (item A) into rear of extension tube.



