


Little Wave XP Adjustable Height Backrest with Adjustable Height Rigidizer Bar

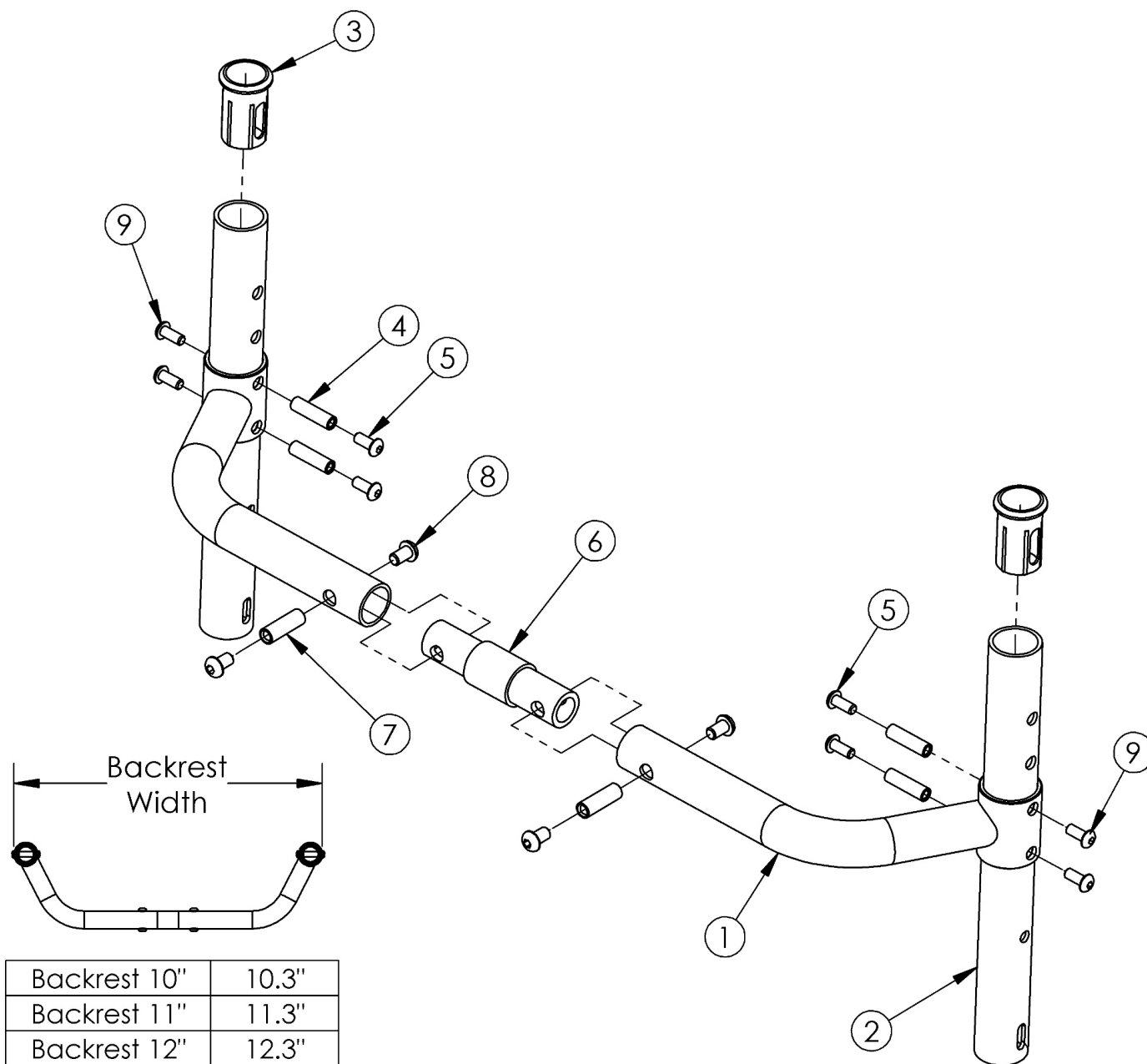

Backrest 10"	10.3"
Backrest 11"	11.3"
Backrest 12"	12.3"
Backrest 13"	13.3"
Backrest 14"	14.3"
Backrest 15"	15.3"
Backrest 16"	16.3"

Item Number	Part Number	Description	Quantity
1a, 6a, 7, 8	109150	Adjustable Height Rigidizer Bar 10", 0" Connector and Hardware	1

Item Number	Part Number	Description	Quantity
1a, 6b, 7, 8	109151	Adjustable Height Rigidizer Bar 10", 1" Connector and Hardware	1
1a, 6c, 7, 8	109152	Adjustable Height Rigidizer Bar 10", 2" Connector and Hardware	1
1a, 6d, 7, 8	109153	Adjustable Height Rigidizer Bar 10", 3" Connector and Hardware	1
1b, 6a, 7, 8	109154	Adjustable Height Rigidizer Bar 11", 0" Connector and Hardware	1
1b, 6b, 7, 8	109155	Adjustable Height Rigidizer Bar 11", 1" Connector and Hardware	1
1b, 6c, 7, 8	109156	Adjustable Height Rigidizer Bar 11", 2" Connector and Hardware	1
1b, 6d, 7, 8	109157	Adjustable Height Rigidizer Bar 11", 3" Connector and Hardware	1
1c, 6a, 7, 8	109158	Adjustable Height Rigidizer Bar 12", 0" Connector and Hardware	1
1c, 6b, 7, 8	109159	Adjustable Height Rigidizer Bar 12", 1" Connector and Hardware	1
1c, 6c, 7, 8	109160	Adjustable Height Rigidizer Bar 12", 2" Connector and Hardware	1
1c, 6d, 7, 8	109161	Adjustable Height Rigidizer Bar 12", 3" Connector and Hardware	1
1d, 6a, 7, 8	109162	Adjustable Height Rigidizer Bar 13", 0" Connector and Hardware	1
1d, 6b, 7, 8	109163	Adjustable Height Rigidizer Bar 13", 1" Connector and Hardware	1
1d, 6c, 7, 8	109164	Adjustable Height Rigidizer Bar 13", 2" Connector and Hardware	1
1d, 6d, 7, 8	109165	Adjustable Height Rigidizer Bar 13", 3" Connector and Hardware	1
1e, 6a, 7, 8	109166	Adjustable Height Rigidizer Bar 14", 0" Connector and Hardware	1
1e, 6b, 7, 8	109167	Adjustable Height Rigidizer Bar 14", 1" Connector and Hardware	1
1e, 6c, 7, 8	109168	Adjustable Height Rigidizer Bar 14", 2" Connector and Hardware	1
1e, 6d, 7, 8	109169	Adjustable Height Rigidizer Bar 14", 3" Connector and Hardware	1
1f, 6a, 7, 8	109170	Adjustable Height Rigidizer Bar 15", 0" Connector and Hardware	1
1f, 6b, 7, 8	109171	Adjustable Height Rigidizer Bar 15", 1" Connector and Hardware	1
1f, 6c, 7, 8	109172	Adjustable Height Rigidizer Bar 15", 2" Connector and Hardware	1
1f, 6d, 7, 8	109173	Adjustable Height Rigidizer Bar 15", 3" Connector and Hardware	1
1g, 6a, 7, 8	109174	Adjustable Height Rigidizer Bar 16", 0" Connector and Hardware	1
1g, 6b, 7, 8	109175	Adjustable Height Rigidizer Bar 16", 1" Connector and Hardware	1
1g, 6c, 7, 8	109176	Adjustable Height Rigidizer Bar 16", 2" Connector and Hardware	1
1g, 6d, 7, 8	109177	Adjustable Height Rigidizer Bar 16", 3" Connector and Hardware	1

Item Number	Part Number	Description	Quantity
6b, 7, 8	502318	Backrest Connector Assembly 1" w/ Hardware, Rigid	1
6c, 7, 8	502319	Backrest Connector Assembly 2" w/ Hardware, Rigid	1
6d, 7, 8	502320	Backrest Connector Assembly 3" w/ Hardware, Rigid	1
2a	000177	Rigid Backrest Tube, 9.25" <i>Used with standard push handles, low profile push handles or omit push handles in any back height range. Also used with fold down push handles with a tall back height range.</i>	2
2b	109099	Rigid Backrest Tube, 8.25" <i>Used with fold down push handles with a short or medium back height range</i>	2
2a, 3	110663	Rigid Backrest Tube, 9.25" with Sleeve <i>Used with standard push handles, low profile push handles or omit push handles in any back height range. Also used with fold down push handles with a tall back height range.</i>	2
2b, 3	110664	Rigid Backrest Tube, 8.25" with Sleeve <i>Used with fold down push handles with a short or medium back height range</i>	2
3	000034	Sleeve 3/4" x 1 1/2"	2
4	000073	M5 x 24 Threaded Barrel	4
5	100669	M5 x 10 BTN SHCS BLKZC w/Patch	4
6a	000589	Backrest Connector 0" LW/RG	1
6b	000590	Backrest Connector 1" LW/RG	1
6c	000591	Backrest Connector 2" LW/RG	1
6d	000592	Backrest Connector 3" LW/RG	1
7	000669	M6 x 20.5 Threaded Barrel	2
8	101184	M6 x 10 BTN SHCS BLKZC w/Patch	4
9	103524	M5 x 10 BTN SHCS BLK ZN	8