


Liberty Drum Brake - Growth

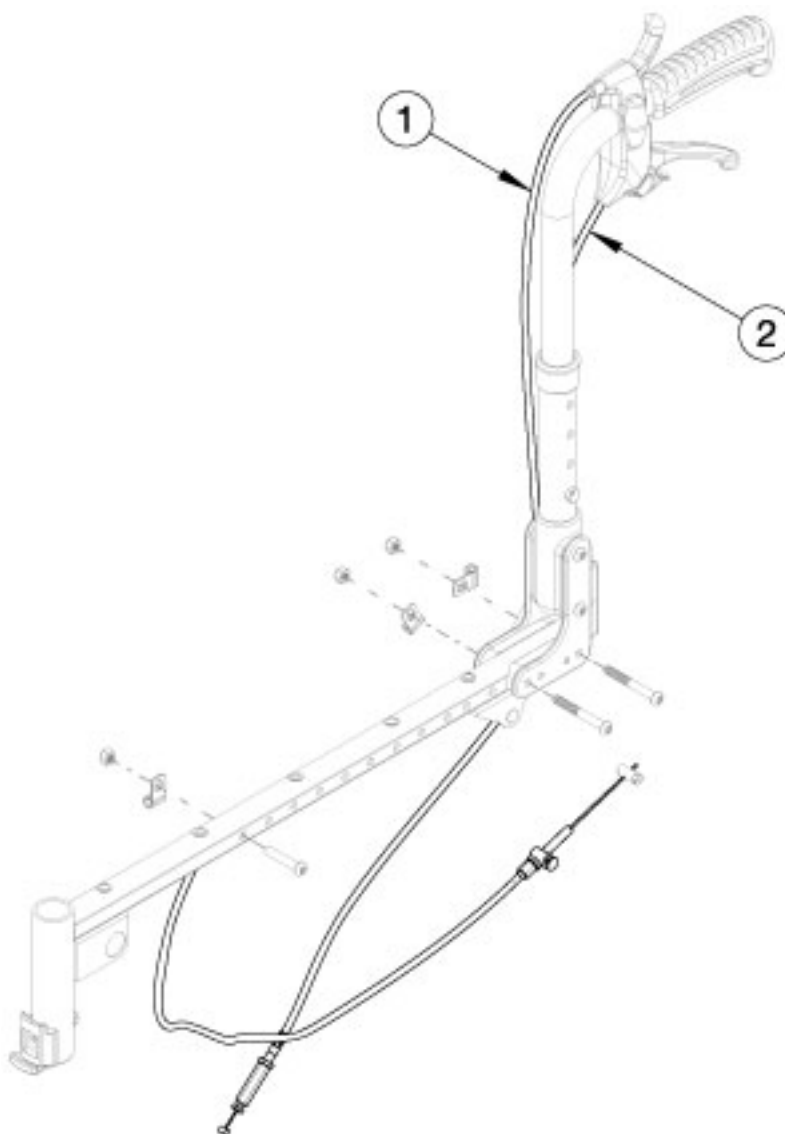

Did your depth change? If yes, evaluate if a different size cable is necessary.

Item Number	Part Number	Description	Quantity
1a	112928	Cable, Tilt, for Drum Brake, Long, Liberty	2
1b	112929	Cable, Tilt, for Drum Brake, XL	2
1c	112930	Cable, Tilt, for Drum Brake, XXL	2
1d	112931	Cable, Tilt, for Drum Brake, XXXL	2
1e	112568	Cable, Tilt, for Drum Brake, 4XL, Liberty	2
1f	112569	Cable, Tilt, for Drum Brake, 5XL, Liberty	2

Item Number	Part Number	Description	Quantity
2a	002471	Drum Brake / Hemi Wheel Lock XL Cable	2
2b	002472	Cable, Drum Brake, XXL	2
2c	111547	Cable, Drum Brake, 3XL	2
2d	111548	Cable, Drum Brake, 4XL	2
2e	111549	Cable, Drum Brake, 5XL	2

Liberty FT Drum Brake Cable Usage							
Backrest Type & Height Range	Seat Depth						
	14"	15"	16"	17"	18"	19"	20"
Height Adjustable Straight, Short Height Adjustable 8° Bend, Short	XL (002471)	XL (002471)	2XL (002472)	2XL (002472)	3XL (111547)	3XL (111547)	4XL (111548)
Height Adjustable Straight, Medium Height Adjustable 8° Bend, Medium Stroller Handle, Short	XL (002471)	2XL (002472)	2XL (002472)	3XL (111547)	3XL (111547)	4XL (111548)	4XL (111548)
Height Adjustable Straight, Tall Height Adjustable 8° Bend, Tall	2XL (002472)	2XL (002472)	3XL (111547)	3XL (111547)	4XL (111548)	4XL (111548)	5XL (111549)
Stroller Handle, Tall	2XL (002472)	3XL (111547)	3XL (111547)	4XL (111548)	4XL (111548)	5XL (111549)	5XL (111549)

Liberty FT Tilt Cable Usage			
Backrest Type & Height Range	Seat Depth Range		
	Short (14" - 16")	Medium (16" - 18")	Long (18" - 20")
Height Adjustable Straight, Short Height Adjustable 8° Bend, Short	Long (112928)	XL (112929)	XXL (112930)
Height Adjustable Straight, Medium Height Adjustable 8° Bend, Medium Stroller Handle, Short	XL (112929)	XXL (112930)	XXXL (112931)
Height Adjustable Straight, Tall Height Adjustable 8° Bend, Tall	XXL (112930)	XXXL (112931)	4XL (112568)
Stroller Handle, Tall	XXXL (112931)	4XL (112568)	5XL (112569)