




(Discontinued) Klik Caster Forks and Stems

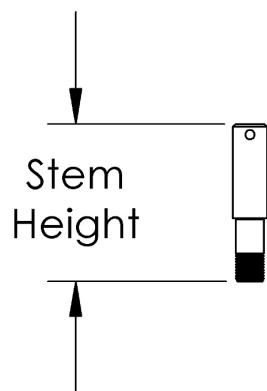

Std	2.0"
+ 3/4"	2.8"

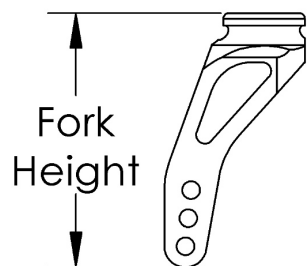

Ex. Short	3.6"
Short	4.5"
Tall	4.9"

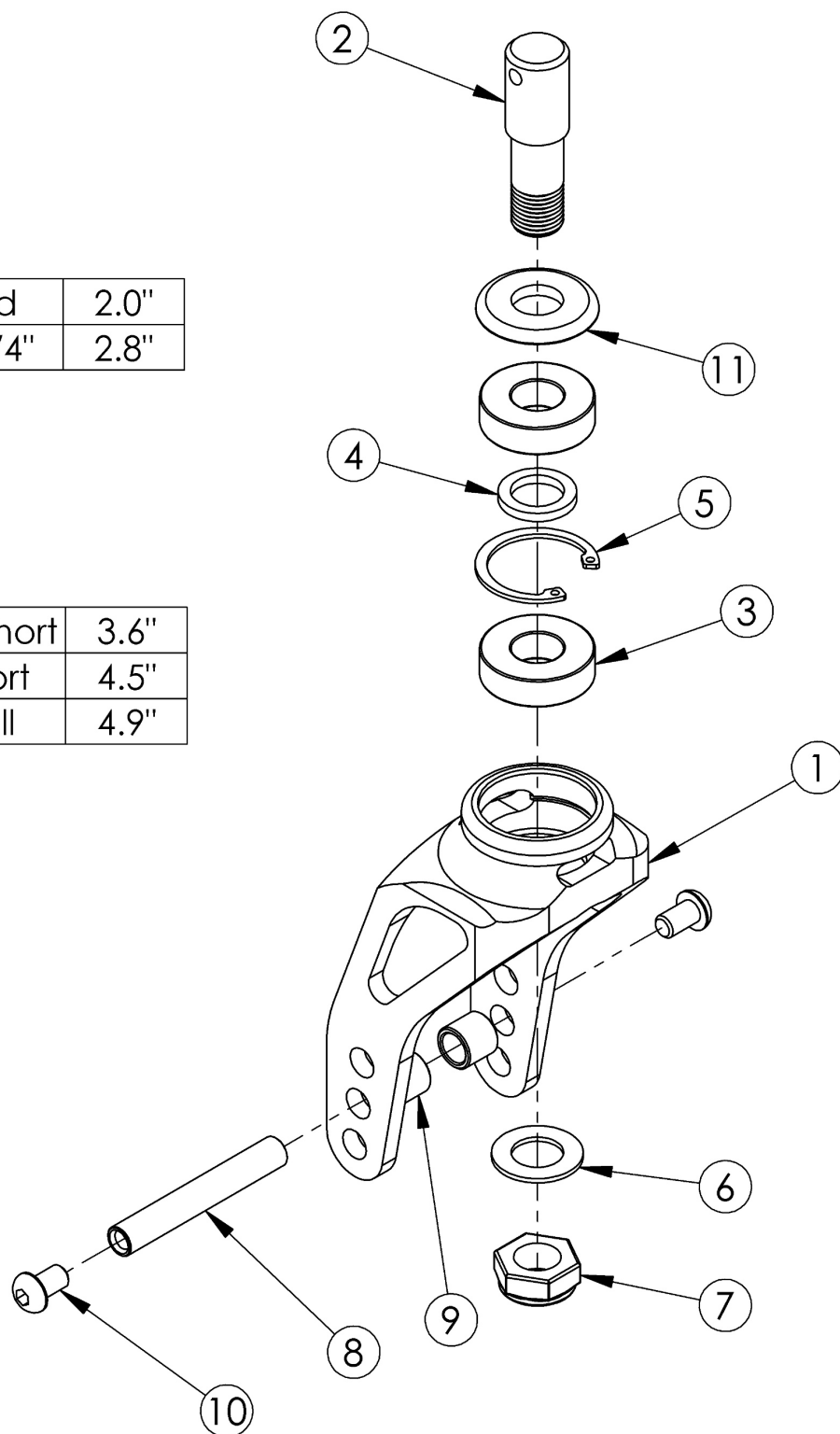

Item Number	Part Number	Description	Quantity
1b, 2b, 3-7, 11	502646_ _ _	Color Anodized Fork and Stem Assembly Extra Short 3/4	1
1b, 3-5	001812_ _ _	Color Anodized Extra Short Caster Fork w/ Bearing Housing LW <i>Replaced with 004361_ _ _</i>	1
1f, 3-5	000928_ _ _	Color Anodized Caster Fork Tall w/ Bearing Housing RG <i>Replaced with 004362_ _ _</i>	1
2b	000930INST	Caster Stem In Line +3/4 Rogue/Clik <i>Replaced with 004371INST</i>	1
3	100902	Bearing 1 1/8" x 1/2" R8RS	2
6	100682	Washer 1/2" .518 x .875 x .047 F/W Black Zinc	1
7	101459	1/2-20 NTE JAM NYL INS L/N BLK ZC	1
8	000422	M6 x 56 Threaded Barrel	1
9	100792	Caster Spacer D8 x D12 x 10mm / Close Mount Handrim Spacer	2
10	100667	M6 x 12 BTN SHCS BLKZC w/Patch	2
11a	001355	Caster Bearing Cap Black Anodized <i>Used with 3/4" stem only</i>	1
11b	001355_ _ _	Color Anodized Caster Bearing Cap	1

Color Anodizing Samples

