


Instructions for Use

Supplier: These instructions must be given to the user of this product.
Operator (Individual or Caregiver): Before using this product, read instructions and save for future reference.

Parts Included:

- 2 - ¼-20 x 1” or ¼-20 x 1¼” Philips head screw
- 2 - ¼” x 1¼” OD Fender washer

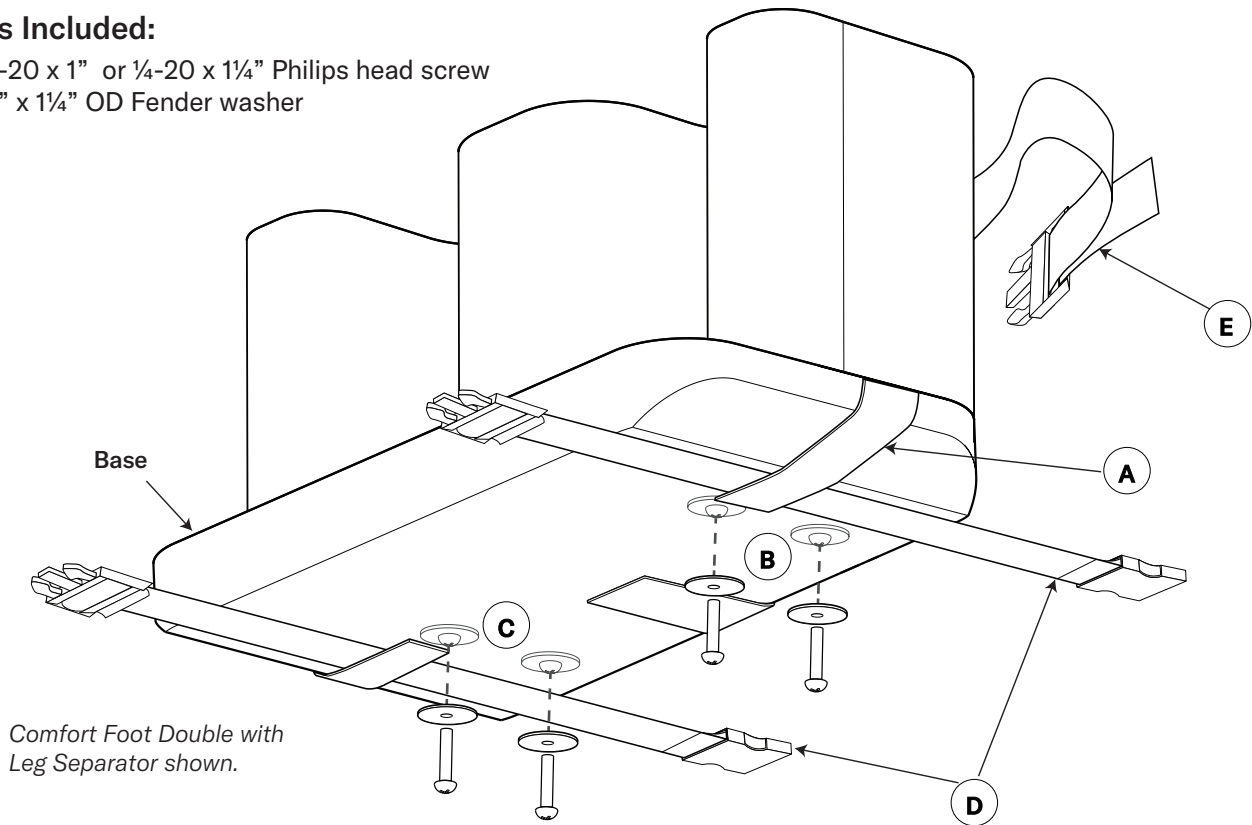

- ⚠ WARNINGS:**
- After installation is complete, check the product for proper functioning. Follow all instructions and safety information provided with the product and provided by the manufacturer of the wheelchair or other accessories.
 - Check daily to make sure that all hardware is fully tightened. DO NOT use the product if any part cannot be tightened or is broken.
 - If you are unable to perform any tasks described in this document, seek assistance by contacting the clinician, equipment provider, distributor, or Customer Support.

- 1** Place the template* on the wheelchair footplate in the desired position.

**See the template on the other side of this page.*
- 2** Drill two 5/16” diameter holes through the wheelchair footplate, as marked on the template.
- 3** Adjust the angle of the product, as desired, using the base straps (A).
- 4** From the bottom of the wheelchair footplate, install the screws** with the washers (position B for left side / position C for right side), into the base of the product.
- 5** Wrap the base straps (D) around the wheelchair footplates and secure with the buckles.
- 6** Wrap the rear strap (E) around the legrest tubing and secure with the buckle.

Notes:

The "Tall" style is secured with two rear straps.

Bolt-on installation with the securing buckle allows the product to swing away for patient transfers. The product can also be installed using only the securing buckles.

** The thickness of the wheelchair footplate determines the correct length of screws. Screws are either ¼-20 x 1” or ¼-20 x 1¼”. For Comfort Foot Double with Gel, use ¼-20 x 1” screws.

Comfort Foot Double
Instructions for Use and Template



Comfort Foot Double with Leg Separator shown.

permobil


 The Comfort Companies, LLC, 1960 South Calhoun Rd., New Berlin, WI 53151 U.S.A.

Contact Information: 800.736.0925
Customer Support: orders.comfort@permobil.com

permobil.com


The Comfort Companies, LLC is a part of Permobil. Product information is changed as needed; current product information is available at permobil.com. Contact Customer Support for previous versions of product information.
© 2013, 2021 Permobil

Comfort Foot Double
Bolt-On Template
Left footplate

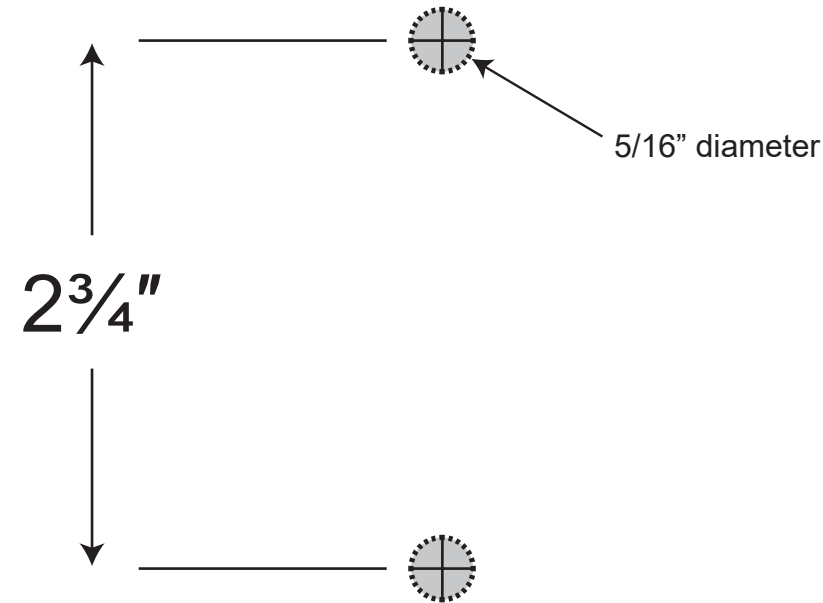

- 1 Place this template on the wheelchair footplate in the desired position.
- 2 Drill two 5/16" holes, where indicated.

Comfort Foot Double
Bolt-On Template
Right footplate

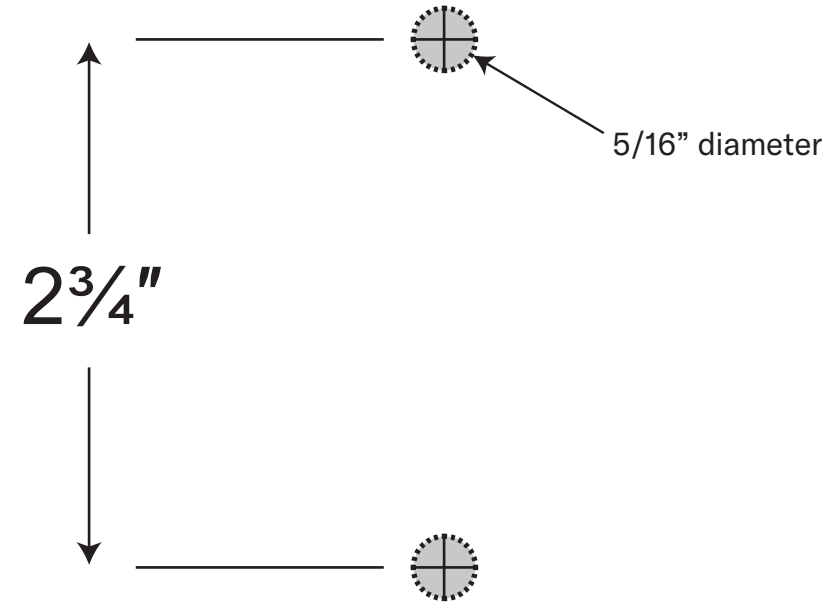

- 1 Place this template on the wheelchair footplate in the desired position.
- 2 Drill two 5/16" holes, where indicated.

