


Flip Fixed Height with Adjustable Handle Backrest - Growth

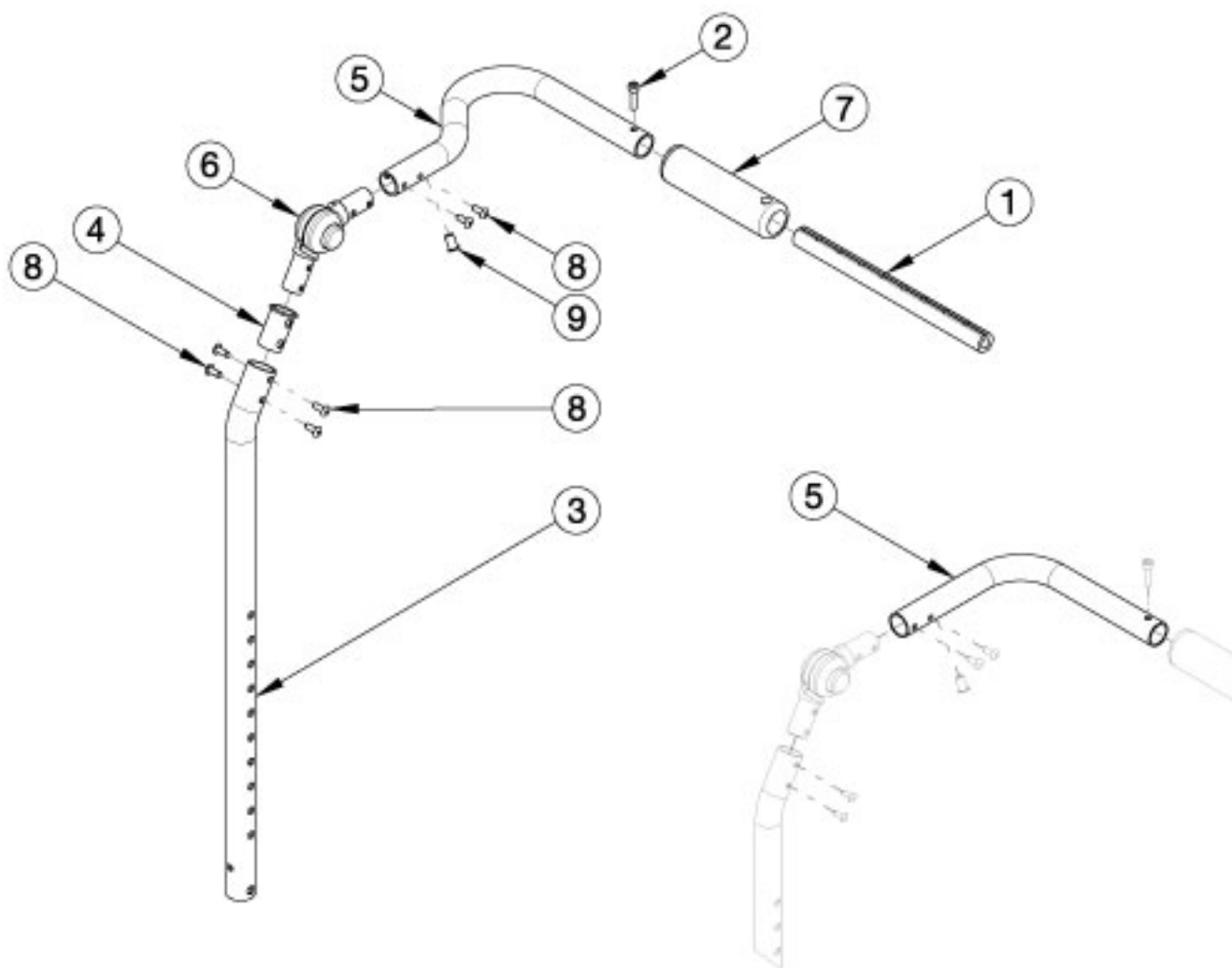

Has your width changed? Will the push handle (#5) accommodate your new width? If no, please select the appropriate back assembly. Callout #5: 10"-13" widths receive the curved handle and 14"-18" widths receive the "L" shaped handle. If chair width is 14"-18" wide and has a serial number of FLP010784 through FLP011079, see the Discontinued Flip Fixed Height with Adjustable Handle Backrest page.

Item Number	Part Number	Description	Quantity
3a, 4, 5a, 6, 7, 8, 9	112172	Back Assembly, Fixed Height w/ Adjustable Handle, Short, 10-13" Wide, Left Flip	1
3a, 4, 5b, 6, 7, 8, 9	112173	Back Assembly, Fixed Height w/ Adjustable Handle, Short, 10-13" Wide, Right Flip	1
3a, 4, 5a, 6, 7, 8, 9	112174	Back Assembly, Fixed Height w/ Adjustable Handle, Tall, 10-13" Wide, Left Flip	1
3a, 4, 5b, 6, 7, 8, 9	112175	Back Assembly, Fixed Height w/ Adjustable Handle, Tall, 10-13" Wide, Right Flip	1
3b, 4, 5c, 6, 7, 8, 9	112176	Back Assembly, Fixed Height w/ Adjustable Handle, Short, 14-18" Wide, Left Flip <i>Used with serial numbers prior to FLP010784 and serial numbers starting with FLP011080</i>	1

Item Number	Part Number	Description	Quantity
3b, 4, 5d, 6, 7, 8, 9	112177	Back Assembly, Fixed Height w/ Adjustable Handle, Short, 14-18" Wide, Right Flip <i>Used with serial numbers prior to FLP010784 and serial numbers starting with FLP011080</i>	1
3b, 4, 5c, 6, 7, 8, 9	112178	Back Assembly, Fixed Height w/ Adjustable Handle, Tall, 14-18" Wide, Left Flip <i>Used with serial numbers prior to FLP010784 and serial numbers starting with FLP011080</i>	1
3b, 4, 5d, 6, 7, 8, 9	112179	Back Assembly, Fixed Height w/ Adjustable Handle, Tall, 14-18" Wide, Right Flip <i>Used with serial numbers prior to FLP010784 and serial numbers starting with FLP011080</i>	1
1a	002578	Growth Strut, Stroller, Short	1
1a, 2	503107	Growth Strut Assembly Short, Focus/Flip	1
1b	002579	Growth Strut, Stroller, Long <i>Used only if growing chair from 13" wide or less to 14" wide or greater</i>	1
2	101843	M5 x 20 SHCS Blk Zn	1