


Rigid Aluminum Seat Pan

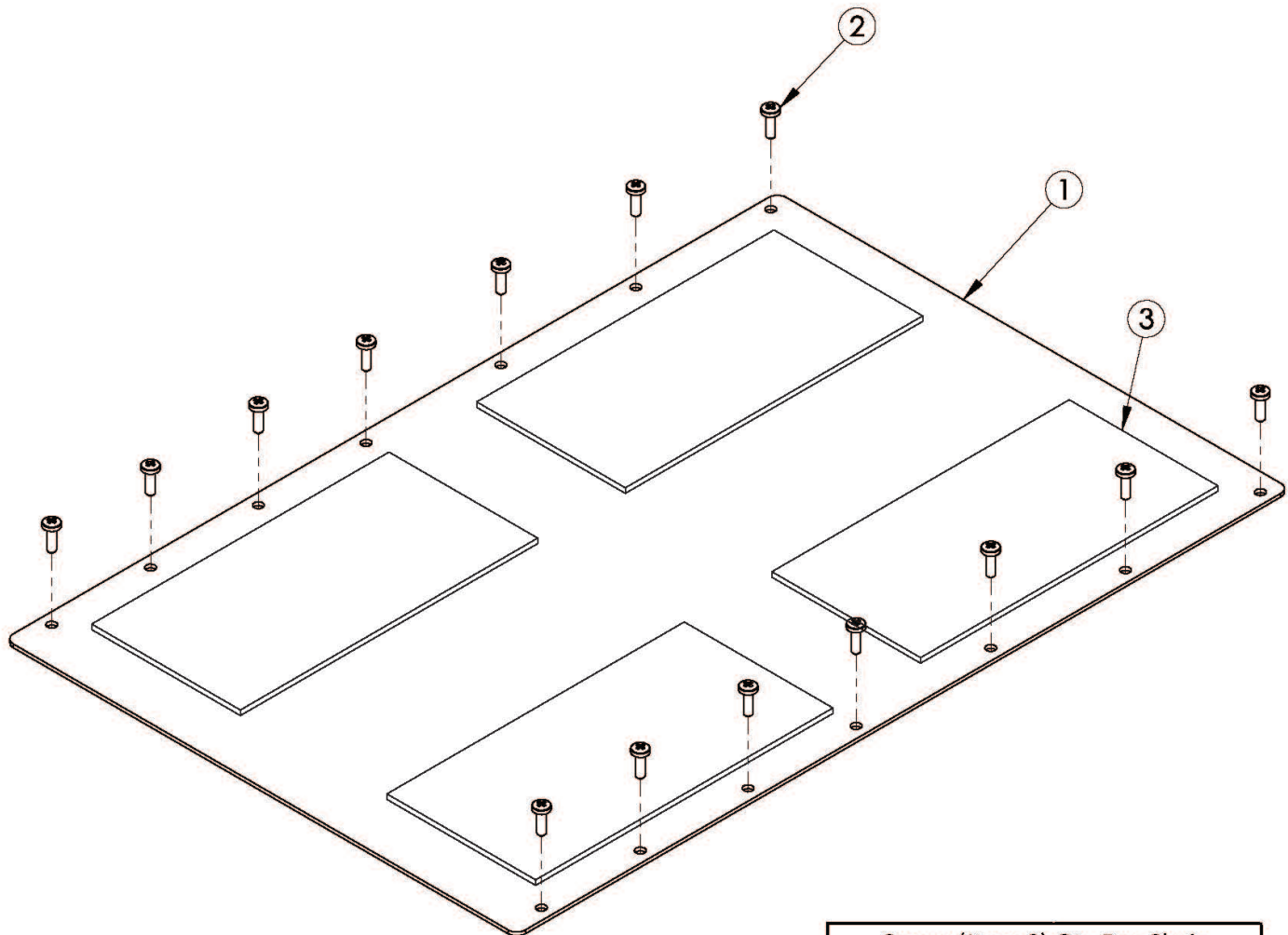

Part Number	Tape Size	Seat Pan Depth
004458	3" X 4"	10" - 11"
004458 & 004459	3" X 4" & 3" X 5"	12"
004459	3" X 5"	13"
004460	3" X 6"	14" - 22"

Screw (Item 2) Qty Per Chair	
Seat Depth	Screw Qty
10" - 12"	6
13" - 15"	8
16" - 18"	10
19" - 21"	12
22"	14

Rigid Seat Pans may be quoted, but must be ordered through Customer Service. If your chair was ordered prior to February 2013 a custom seat pan may be required. The seat pan width measures approximately 0.75" and the seat pan depth measures approximately 0.50" less (however a depth of 21" will be closer to 0.75") than the chair size.

Item Number	Part Number	Description	Quantity
1a, 3a	505952	Seat Pan Rigid with AB Tape 10 x 10	1
1b, 3a	505953	Seat Pan Rigid with AB Tape 10 x 11	1
1c, 3a, 3b	505954	Seat Pan Rigid with AB Tape 10 x 12	1

Item Number	Part Number	Description	Quantity
1d, 3b	505955	Seat Pan Rigid with AB Tape 10 x 13	1
1e, 3c	505956	Seat Pan Rigid with AB Tape 10 x 14	1
1f, 3c	505957	Seat Pan Rigid with AB Tape 10 x 15	1
1g, 3c	505958	Seat Pan Rigid with AB Tape 10 x 16	1
1h, 3c	505959	Seat Pan Rigid with AB Tape 10 x 17	1
1i, 3c	505960	Seat Pan Rigid with AB Tape 10 x 18	1
1j, 3c	505961	Seat Pan Rigid with AB Tape 10 x 19	1
1k, 3a	505962	Seat Pan Rigid with AB Tape 11 x 10	1
1l, 3a	505963	Seat Pan Rigid with AB Tape 11 x 11	1
1m, 3a, 3b	505964	Seat Pan Rigid with AB Tape 11 x 12	1
1n, 3b	505965	Seat Pan Rigid with AB Tape 11 x 13	1
1o, 3c	505966	Seat Pan Rigid with AB Tape 11 x 14	1
1p, 3c	505967	Seat Pan Rigid with AB Tape 11 x 15	1
1q, 3c	505968	Seat Pan Rigid with AB Tape 11 x 16	1
1r, 3c	505969	Seat Pan Rigid with AB Tape 11 x 17	1
1s, 3c	505970	Seat Pan Rigid with AB Tape 11 x 18	1
1t, 3c	505971	Seat Pan Rigid with AB Tape 11 x 19	1
1u, 3a	505972	Seat Pan Rigid with AB Tape 12 x 10	1
1v, 3a	505973	Seat Pan Rigid with AB Tape 12 x 11	1
1w, 3a, 3b	505974	Seat Pan Rigid with AB Tape 12 x 12	1
1x, 3b	505975	Seat Pan Rigid with AB Tape 12 x 13	1
1y, 3c	505976	Seat Pan Rigid with AB Tape 12 x 14	1
1z, 3c	505977	Seat Pan Rigid with AB Tape 12 x 15	1
1aa, 3c	505978	Seat Pan Rigid with AB Tape 12 x 16	1
1ab, 3c	505979	Seat Pan Rigid with AB Tape 12 x 17	1
1ac, 3c	505980	Seat Pan Rigid with AB Tape 12 x 18	1
1ad, 3c	505981	Seat Pan Rigid with AB Tape 12 x 19	1
1ae, 3c	505982	Seat Pan Rigid with AB Tape 12 x 20	1

Item Number	Part Number	Description	Quantity
1af, 3a	505983	Seat Pan Rigid with AB Tape 13 x 10	1
1ag, 3a	505984	Seat Pan Rigid with AB Tape 13 x 11	1
1ah, 3a, 3b	505985	Seat Pan Rigid with AB Tape 13 x 12	1
1ai, 3b	505986	Seat Pan Rigid with AB Tape 13 x 13	1
1aj, 3c	505987	Seat Pan Rigid with AB Tape 13 x 14	1
1ak, 3c	505988	Seat Pan Rigid with AB Tape 13 x 15	1
1al, 3c	505989	Seat Pan Rigid with AB Tape 13 x 16	1
1am, 3c	505990	Seat Pan Rigid with AB Tape 13 x 17	1
1an, 3c	505991	Seat Pan Rigid with AB Tape 13 x 18	1
1ao, 3c	505992	Seat Pan Rigid with AB Tape 13 x 19	1
1ap, 3c	505993	Seat Pan Rigid with AB Tape 13 x 20	1
1aq, 3a	505994	Seat Pan Rigid with AB Tape 14 x 10	1
1ar, 3a	505995	Seat Pan Rigid with AB Tape 14 x 11	1
1as, 3a, 3b	505996	Seat Pan Rigid with AB Tape 14 x 12	1
1at, 3b	505997	Seat Pan Rigid with AB Tape 14 x 13	1
1au, 3c	505859	Seat Pan Rigid with AB Tape 14 x 14	1
1av, 3c	505860	Seat Pan Rigid with AB Tape 14 x 15	1
1aw, 3c	505861	Seat Pan Rigid with AB Tape 14 x 16	1
1ax, 3c	505862	Seat Pan Rigid with AB Tape 14 x 17	1
1ay, 3c	505863	Seat Pan Rigid with AB Tape 14 x 18	1
1az, 3c	505864	Seat Pan Rigid with AB Tape 14 x 19	1
1ba, 3c	505865	Seat Pan Rigid with AB Tape 14 x 20	1
1bb, 3c	505866	Seat Pan Rigid with AB Tape 14 x 21	1
1bc, 3c	505867	Seat Pan Rigid with AB Tape 14 x 22	1
1bd, 3a	505998	Seat Pan Rigid with AB Tape 15 x 10	1
1be, 3a	505999	Seat Pan Rigid with AB Tape 15 x 11	1
1bf, 3a, 3b	506000	Seat Pan Rigid with AB Tape 15 x 12	1
1bg, 3b	506001	Seat Pan Rigid with AB Tape 15 x 13	1

Item Number	Part Number	Description	Quantity
1bh, 3c	505868	Seat Pan Rigid with AB Tape 15 x 14	1
1bi, 3c	505869	Seat Pan Rigid with AB Tape 15 x 15	1
1bj, 3c	505870	Seat Pan Rigid with AB Tape 15 x 16	1
1bk, 3c	505871	Seat Pan Rigid with AB Tape 15 x 17	1
1bl, 3c	505872	Seat Pan Rigid with AB Tape 15 x 18	1
1bm, 3c	505873	Seat Pan Rigid with AB Tape 15 x 19	1
1bn, 3c	505874	Seat Pan Rigid with AB Tape 15 x 20	1
1bo, 3c	505875	Seat Pan Rigid with AB Tape 15 x 21	1
1bp, 3c	505876	Seat Pan Rigid with AB Tape 15 x 22	1
1bq, 3a	506002	Seat Pan Rigid with AB Tape 16 x 10	1
1br, 3a	506003	Seat Pan Rigid with AB Tape 16 x 11	1
1bs, 3a, 3b	506004	Seat Pan Rigid with AB Tape 16 x 12	1
1bt, 3b	506005	Seat Pan Rigid with AB Tape 16 x 13	1
1bu, 3c	505877	Seat Pan Rigid with AB Tape 16 x 14	1
1bv, 3c	505878	Seat Pan Rigid with AB Tape 16 x 15	1
1bw, 3c	505879	Seat Pan Rigid with AB Tape 16 x 16	1
1bx, 3c	505880	Seat Pan Rigid with AB Tape 16 x 17	1
1by, 3c	505881	Seat Pan Rigid with AB Tape 16 x 18	1
1bz, 3c	505882	Seat Pan Rigid with AB Tape 16 x 19	1
1ca, 3c	505883	Seat Pan Rigid with AB Tape 16 x 20	1
1cb, 3c	505884	Seat Pan Rigid with AB Tape 16 x 21	1
1cc, 3c	505885	Seat Pan Rigid with AB Tape 16 x 22	1
1cd, 3a, 3b	506006	Seat Pan Rigid with AB Tape 17 x 12	1
1ce, 3b	506007	Seat Pan Rigid with AB Tape 17 x 13	1
1cf, 3c	505886	Seat Pan Rigid with AB Tape 17 x 14	1
1cg, 3c	505887	Seat Pan Rigid with AB Tape 17 x 15	1
1ch, 3c	505888	Seat Pan Rigid with AB Tape 17 x 16	1
1ci, 3c	505889	Seat Pan Rigid with AB Tape 17 x 17	1

Item Number	Part Number	Description	Quantity
1cj, 3c	505890	Seat Pan Rigid with AB Tape 17 x 18	1
1ck, 3c	505891	Seat Pan Rigid with AB Tape 17 x 19	1
1cl, 3c	505892	Seat Pan Rigid with AB Tape 17 x 20	1
1cm, 3c	505893	Seat Pan Rigid with AB Tape 17 x 21	1
1cn, 3c	505894	Seat Pan Rigid with AB Tape 17 x 22	1
1co, 3b	506008	Seat Pan Rigid with AB Tape 18 x 13	1
1cp, 3c	505895	Seat Pan Rigid with AB Tape 18 x 14	1
1cq, 3c	505896	Seat Pan Rigid with AB Tape 18 x 15	1
1cr, 3c	505897	Seat Pan Rigid with AB Tape 18 x 16	1
1cs, 3c	505898	Seat Pan Rigid with AB Tape 18 x 17	1
1ct, 3c	505899	Seat Pan Rigid with AB Tape 18 x 18	1
1cu, 3c	505900	Seat Pan Rigid with AB Tape 18 x 19	1
1cv, 3c	505901	Seat Pan Rigid with AB Tape 18 x 20	1
1cw, 3c	505902	Seat Pan Rigid with AB Tape 18 x 21	1
1cx, 3c	505903	Seat Pan Rigid with AB Tape 18 x 22	1
1cy, 3c	505904	Seat Pan Rigid with AB Tape 19 x 14	1
1cz, 3c	505905	Seat Pan Rigid with AB Tape 19 x 15	1
1da, 3c	505906	Seat Pan Rigid with AB Tape 19 x 16	1
1db, 3c	505907	Seat Pan Rigid with AB Tape 19 x 17	1
1dc, 3c	505908	Seat Pan Rigid with AB Tape 19 x 18	1
1dd, 3c	505909	Seat Pan Rigid with AB Tape 19 x 19	1
1de, 3c	505910	Seat Pan Rigid with AB Tape 19 x 20	1
1df, 3c	505911	Seat Pan Rigid with AB Tape 19 x 21	1
1dg, 3c	505912	Seat Pan Rigid with AB Tape 19 x 22	1
1dh, 3c	505913	Seat Pan Rigid with AB Tape 20 x 14	1
1di, 3c	505914	Seat Pan Rigid with AB Tape 20 x 15	1
1dj, 3c	505915	Seat Pan Rigid with AB Tape 20 x 16	1
1dk, 3c	505916	Seat Pan Rigid with AB Tape 20 x 17	1

Item Number	Part Number	Description	Quantity
1dl, 3c	505917	Seat Pan Rigid with AB Tape 20 x 18	1
1dm, 3c	505918	Seat Pan Rigid with AB Tape 20 x 19	1
1dn, 3c	505919	Seat Pan Rigid with AB Tape 20 x 20	1
1do, 3c	505920	Seat Pan Rigid with AB Tape 20 x 21	1
1dp, 3c	505921	Seat Pan Rigid with AB Tape 20 x 22	1
1dq, 3c	505922	Seat Pan Rigid with AB Tape 21 x 14	1
1dr, 3c	505923	Seat Pan Rigid with AB Tape 21 x 15	1
1ds, 3c	505924	Seat Pan Rigid with AB Tape 21 x 16	1
1dt, 3c	505925	Seat Pan Rigid with AB Tape 21 x 17	1
1du, 3c	505926	Seat Pan Rigid with AB Tape 21 x 18	1
1dv, 3c	505927	Seat Pan Rigid with AB Tape 21 x 19	1
1dw, 3c	505928	Seat Pan Rigid with AB Tape 21 x 20	1
1dx, 3c	505929	Seat Pan Rigid with AB Tape 21 x 21	1
1dy, 3c	505930	Seat Pan Rigid with AB Tape 21 x 22	1
1dz, 3c	505931	Seat Pan Rigid with AB Tape 22 x 14	1
1ea, 3c	505932	Seat Pan Rigid with AB Tape 22 x 15	1
1eb, 3c	505933	Seat Pan Rigid with AB Tape 22 x 16	1
1ec, 3c	505934	Seat Pan Rigid with AB Tape 22 x 17	1
1ed, 3c	505935	Seat Pan Rigid with AB Tape 22 x 18	1
1ee, 3c	505936	Seat Pan Rigid with AB Tape 22 x 19	1
1ef, 3c	505937	Seat Pan Rigid with AB Tape 22 x 20	1
1eg, 3c	505938	Seat Pan Rigid with AB Tape 22 x 21	1
1eh, 3c	505939	Seat Pan Rigid with AB Tape 22 x 22	1
2	100734	M5 x 16 PHMS ZC	14
3a	004458	AB Tape, 3" x 4"	4
3b	004459	AB Tape, 3" x 5"	4
3c	004460	AB Tape, 3" x 6"	4