


Catalyst Push and Pull to Lock

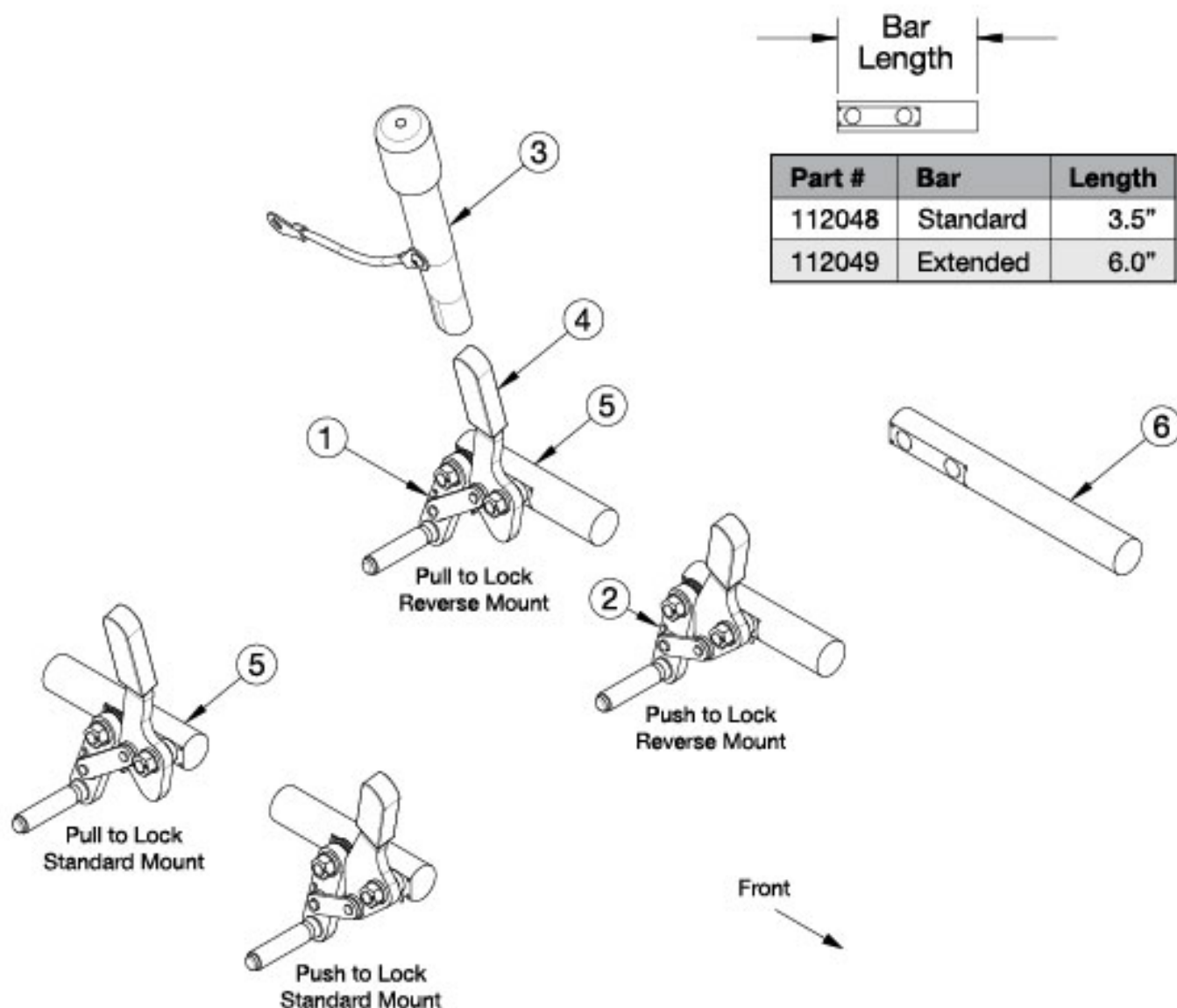

Implemented August 2022 Notice: Chair configuration review required prior to parts being quoted or ordered Please see the "Catalyst Wheel Lock Orientation" reference to determine if you need standard or reversed wheel locks. If adding Wheel Locks, Wheel Lock Clamps will be needed.

Item Number	Part Number	Description	Quantity
2, 5	124094	Thru Post Wheel Lock Push Right Standard <i>If invoiced prior to 7/2025 pair of wheel lock will be needed</i>	1
2, 5	124095	Thru Post Wheel Lock Push Left Standard <i>If invoiced prior to 7/2025 pair of wheel lock will be needed</i>	1
2, 5	124896	Thru Post Wheel Lock, Push PAIR	1

Item Number	Part Number	Description	Quantity
		<i>If invoiced prior to 7/2025 pair of wheel lock will be needed</i>	
2, 5	115614	Wheel Lock, Push, Right, Reversed	1
2, 5	115615	Wheel Lock, Push, Left, Reversed	1
2, 5	124897	Wheel Lock, Push, Reversed PAIR	1
1, 5	124098	Thru Post Wheel Lock Pull Right Standard <i>If invoiced prior to 7/2025 pair of wheel lock will be needed</i>	1
1, 5	124099	Thru Post Wheel Lock Pull Left Standard <i>If invoiced prior to 7/2025 pair of wheel lock will be needed</i>	1
1, 5	124899	Thru Post Wheel Lock, Pull PAIR <i>If invoiced prior to 7/2025 pair of wheel lock will be needed</i>	1
1, 5	115618	Wheel Lock, Pull, Right, Reversed	1
1, 5	115619	Wheel Lock, Pull, Left, Reversed	1
1, 5	124900	Wheel Lock, Pull, Reversed PAIR	1
2, 3, 5	124096	Thru Post Wheel Lock Push Right with Extension Handle <i>If invoiced prior to 7/2025 pair of wheel lock will be needed</i>	1
2, 3, 5	124097	Thru Post Wheel Lock Push Left with Extension Handle <i>If invoiced prior to 7/2025 pair of wheel lock will be needed</i>	1
2, 3, 5	124901	Thru Post Wheel Lock, Push w/ Extension PAIR <i>If invoiced prior to 7/2025 pair of wheel lock will be needed</i>	1
2, 3, 5	115616	Wheel Lock, Push w/ Extension, Right, Reversed	1
2, 3, 5	115617	Wheel Lock, Push w/ Extension, Left, Reversed	1
2, 3, 5	124902	Wheel Lock, Push w/ Extension, Reversed PAIR	1
1, 3, 5	124100	Thru Post Wheel Lock Pull Right with Extension Handle <i>If invoiced prior to 7/2025 pair of wheel lock will be needed</i>	1
1, 3, 5	124101	Thru Post Wheel Lock Pull Left with Extension Handle <i>If invoiced prior to 7/2025 pair of wheel lock will be needed</i>	1
1, 3, 5	124903	Thru Post Wheel Lock, Pull w/ Extension PAIR <i>If invoiced prior to 7/2025 pair of wheel lock will be needed</i>	1
1, 3, 5	115620	Wheel Lock, Pull w/ Extension, Right, Reversed	1
1, 3, 5	115621	Wheel Lock, Pull w/ Extension, Left, Reversed	1
1, 3, 5	124904	Wheel Lock, Pull w/ Extension, Reversed PAIR	1
3	200369	Wheel Lock Extension	1

Item Number	Part Number	Description	Quantity
4	111247	Wheel Lock Cover Plastic	1
6	112049	Wheel Lock Mount Bar Extended <i>Used as needed</i>	1

Catalyst Wheel Lock Orientation					
				Rear Wheel Size	
Center Of Gravity	Rear Seat Height	T-Arm	Seat Depth	Less than or equal to 22"	Greater than 22"
Rear	Greater than 17"	Yes	Less than 18"	Reverse	Reverse
Rear	Greater than 17"	Yes	Greater than or equal to 18"	Reverse	Reverse
Rear	Greater than 17"	No	Less than 18"	Reverse	Standard
Rear	Greater than 17"	No	Greater than or equal to 18"	Reverse	Standard
Rear	Less than or equal to 17"	Yes	Less than 18"	Reverse	Reverse
Rear	Less than or equal to 17"	Yes	Greater than or equal to 18"	Reverse	Reverse
Rear	Less than or equal to 17"	No	Less than 18"	Reverse	Standard
Rear	Less than or equal to 17"	No	Greater than or equal to 18"	Reverse	Standard
Center	Greater than 17"	Yes	Less than 18"	Reverse	Standard
Center	Greater than 17"	Yes	Greater than or equal to 18"	Reverse	Reverse
Center	Greater than 17"	No	Less than 18"	Reverse	Standard
Center	Greater than 17"	No	Greater than or equal to 18"	Reverse	Standard
Center	Less than or equal to 17"	Yes	Less than 18"	Reverse	Standard
Center	Less than or equal to 17"	Yes	Greater than or equal to 18"	Reverse	Standard
Center	Less than or equal to 17"	No	Less than 18"	Reverse	Standard
Center	Less than or equal to 17"	No	Greater than or equal to 18"	Reverse	Standard
Forward	Greater than 17"	Yes	Less than 18"	Reverse	Standard
Forward	Greater than 17"	Yes	Greater than or equal to 18"	Reverse	Reverse
Forward	Greater than 17"	No	Less than 18"	Reverse	Standard
Forward	Greater than 17"	No	Greater than or equal to 18"	Reverse	Standard
Forward	Less than or equal to 17"	Yes	Less than 18"	Reverse	Standard
Forward	Less than or equal to 17"	Yes	Greater than or equal to 18"	Reverse	Standard
Forward	Less than or equal to 17"	No	Less than 18"	Reverse	Standard
Forward	Less than or equal to 17"	No	Greater than or equal to 18"	Reverse	Standard

Center of Gravity Position 1 = Forward

Center of Gravity Position 2 and 3 = Center

Center of Gravity Position 4 = Rear

Catalyst 4 models utilize the Center Position

Effective 7/11/2023