



ORTHO-MED, INC.
Commitment to Caring

User Guide 9400



9400

1. Remove chair from box
2. Swing arm rails forward
3. Match pins and snap into place
4. Place red hook to the front leg of the rail (small bar underneath)
5. Attach backrest onto frame by guiding the tubing through the holes on each side of the plastic backrest.

* To use the push-button mechanism – push the gold button inward and slide tube until the button pops out the hole provided.

LIFETIME LIMITED WARRANTY

Nova Ortho-Med, Inc. warrants this product to be free from defects in material or workmanship for a period of lifetime from date of purchase. If within the specified warranty period, this product is proven to be defective, it will be repaired or replaced at NOVA O.M.'s option. This warranty does not cover shipping costs incurred in replacement or repair of this product.

NOVA O.M. stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This Lifetime Limited Warranty represents our confidence in the materials and workmanship of our products. Do not return any products to our office without prior consent.

This warranty does not cover problems due to user negligence or misuse of the product. This warranty shall not extend to non-durable components, including but not limited to rubber tips, handgrips, casters, and other parts subject to normal "wear and tear"

THIS FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA O.M. SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.