



SEDEO PRO FIXED BACK QUICK RELEASE INSTALLATION INSTRUCTION SHEET



Please read these instructions carefully before beginning the installation. Failure to understand and follow installation instructions may result in injury to installer and/or end user and may void the warranty. If you have any questions call Sunrise Medical Technical Support at (800) 333-4000.

Quick Release parts:

1. 2 ea M6x65 thumb screw (D)
2. 2 ea bushing (E)
3. 2 ea quick release receiver (F)
4. 2 ea M6 nylock nut (G)

Required tools:

1. 5mm hex key
2. 10mm open box wrench


A. Pre-installation: remove locking receiver

1

1. Using the 5mm hex key, remove existing M6 bolt (A), bushing (B) and receiver (C). Discard existing hardware (A,B,C) shown in figure 1.

⚠ Warning



Note: once the receiver (C) is removed, the back is free to fold forward towards the front of the seat. Make sure to have a firm hold on the backrest upon removing this component.

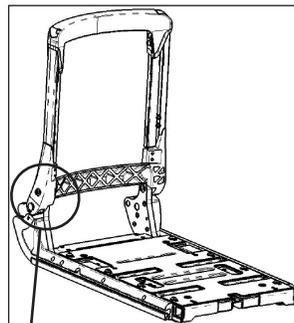

B. Install Quick Release Components

2/3

1. Install the quick release components (D, E, F, G) as shown in Fig 2.
2. Hand tighten the thumbscrew (D) through the receiver (F).
3. Using the 10mm open box wrench, tighten the nylock nut (G) to the thumbscrew (D) until the nylock nut (G) is flush with the end of the thumbscrew threads (D), shown in Fig 3.

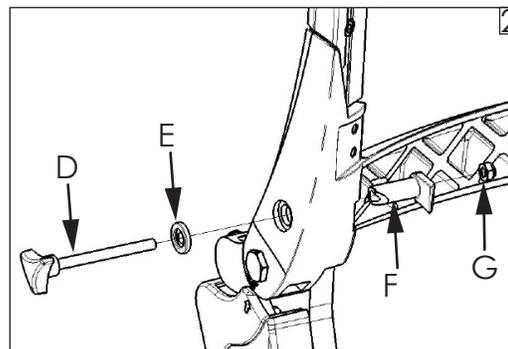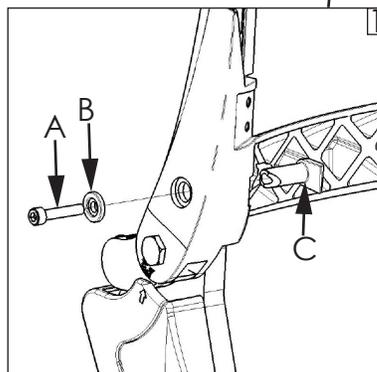

C. Inspect


1. Verify the folding back is locked once the quick release components are completely installed.
2. Verify that the nylon on the nylock nut (G) is threaded through by the threads on the thumbscrew.

D. Operation

1. While firmly grasping the backrest, loosen the thumbscrew (D), until the hex nut (G) is against the receiver (F).
2. Slide the receiver (F) out of the backrest, then lower the backrest forward.

⚠ Do not loosen item G

Note: The quick release components will remain on the backrest, as long as the hex nut (G) remains on the thumbscrew (D)

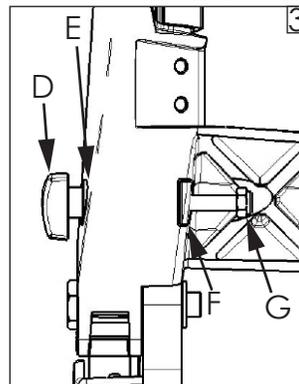

⚠ DEALER/TECHNICIAN WARNING

Attention dealers and qualified technicians, do not operate or service this device without first reading the owners manual. If you do not understand the instructions and warnings in the owners manual please contact the Quickie Technical Service Department before operating and/or servicing the Quickie device. Failure to do so may result in damage and/or injury.

Find more information and important warnings see the wheelchair owner's manual or at: www.SunriseMedical.com.

Sunrise Medical 2842 N. Business Park Ave Fresno, CA 93727 USA
In Canada (800) 263-3390



Customer service: (800) 333-4000
or visit www.SunriseMedical.com

© 2019 Sunrise Medical (US) LLC
9/2019, 252229 Rev.A