


Rigid Tension Adjustable Back Upholstery

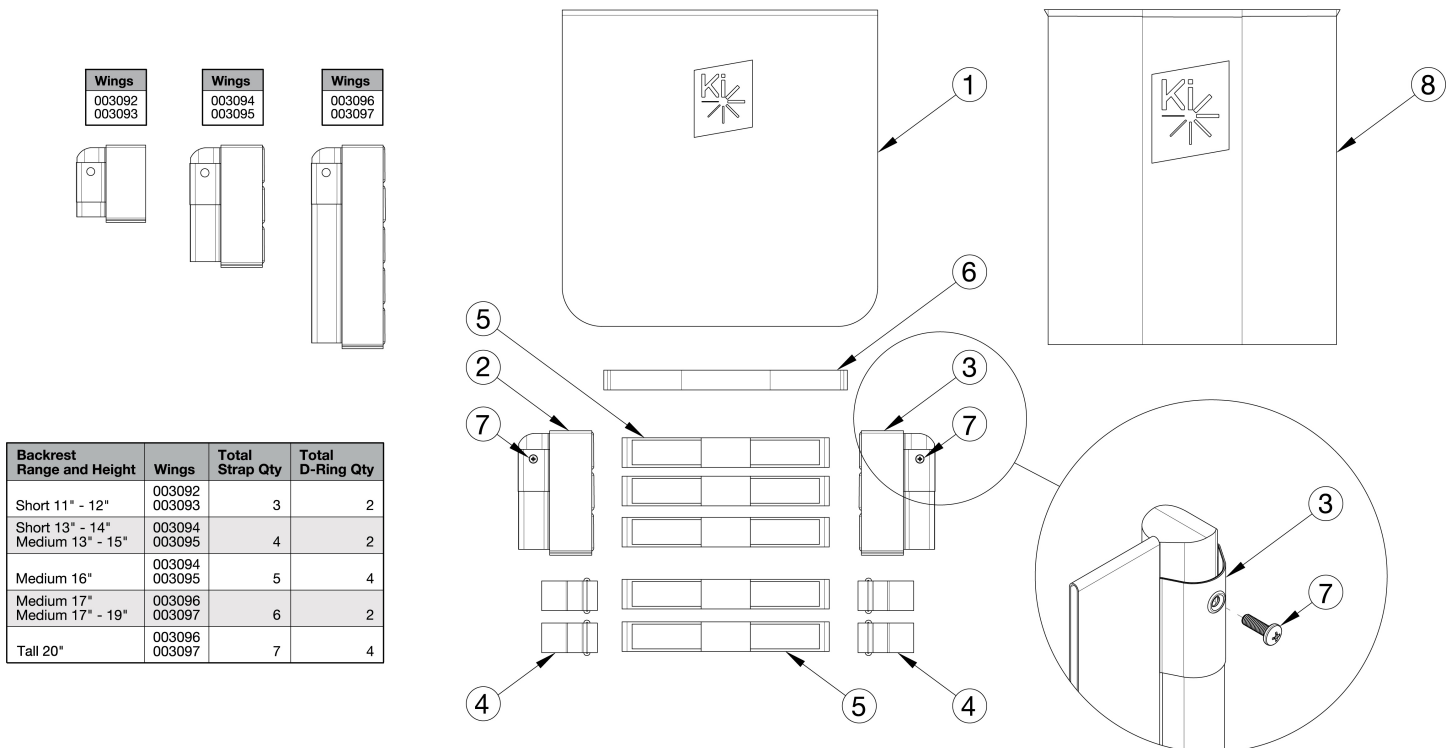

Backrest Range and Height	Wings	Total Strap Qty	Total D-Ring Qty
Short 11" - 12"	003092 003093	3	2
Short 13" - 14" Medium 13" - 15"	003094 003095	4	2
Medium 16"	003094 003095	5	4
Medium 17" Medium 17" - 19"	003096 003097	6	2
Tall 20"	003096 003097	7	4

Assemblies contain the cover, amount of straps and d-rings used for the tallest height listed in the assembly. Attaching hardware is not included with the back upholstery assemblies. Tension Straps (6a-6f) are based on seat width and used with the combination of Fold Down Push Handle and Tension Adjustable Back Upholstery.

Item Number	Part Number	Description	Quantity
1a, 2a, 3a, 4, 5a	111194	Back Upholstery Assembly Tension Adjustable with Standard Cover 10" Wide, 11-12 Height, Rigid	1
1a, 2b, 3b, 4, 5a	506451	Back Upholstery Assembly Tension Adjustable with Standard Cover 10" Wide, 13-14 Height, Rigid	1
1b, 2b, 3b, 4, 5a	506452	Back Upholstery Assembly Tension Adjustable with Standard Cover 10" Wide, 15-16 Height, Rigid	1
1c, 2c, 3c, 4, 5a	506453	Back Upholstery Assembly Tension Adjustable with Standard Cover 10" Wide, 17-20 Height, Rigid	1
1d, 2a, 3a, 4, 5b	111195	Back Upholstery Assembly Tension Adjustable with Standard Cover 11" Wide, 11-12 Height, Rigid	1
1d, 2b, 3b, 4, 5b	506454	Back Upholstery Assembly Tension Adjustable with Standard Cover 11" Wide, 13-14 Height, Rigid	1
1e, 2b, 3b, 4, 5b	506455	Back Upholstery Assembly Tension Adjustable with Standard Cover 11" Wide, 15-16 Height, Rigid	1
1f, 2c, 3c, 4, 5b	506456	Back Upholstery Assembly Tension Adjustable with Standard Cover 11" Wide, 17-20 Height, Rigid	1

Item Number	Part Number	Description	Quantity
1g, 2a, 3a, 4, 5c	111196	Back Upholstery Assembly Tension Adjustable with Standard Cover 12" Wide, 11-12 Height, Rigid	1
1g, 2b, 3b, 4, 5c	506457	Back Upholstery Assembly Tension Adjustable with Standard Cover 12" Wide, 13-14 Height, Rigid	1
1h, 2b, 3b, 4, 5c	506458	Back Upholstery Assembly Tension Adjustable with Standard Cover 12" Wide, 15-16 Height, Rigid	1
1i, 2c, 3c, 4, 5c	506459	Back Upholstery Assembly Tension Adjustable with Standard Cover 12" Wide, 17-20 Height, Rigid	1
1j, 2a, 3a, 4, 5d	111197	Back Upholstery Assembly Tension Adjustable with Standard Cover 13" Wide, 11-12 Height, Rigid	1
1j, 2b, 3b, 4, 5d	506460	Back Upholstery Assembly Tension Adjustable with Standard Cover 13" Wide, 13-14 Height, Rigid	1
1k, 2b, 3b, 4, 5d	506461	Back Upholstery Assembly Tension Adjustable with Standard Cover 13" Wide, 15-16 Height, Rigid	1
1l, 2c, 3c, 4, 5d	506462	Back Upholstery Assembly Tension Adjustable with Standard Cover 13" Wide, 17-20 Height, Rigid	1
1m, 2a, 3a, 4, 5e	111198	Back Upholstery Assembly Tension Adjustable with Standard Cover 14" Wide, 11-12 Height, Rigid	1
1m, 2b, 3b, 4, 5e	506463	Back Upholstery Assembly Tension Adjustable with Standard Cover 14" Wide, 13-14 Height, Rigid	1
1n, 2b, 3b, 4, 5e	506464	Back Upholstery Assembly Tension Adjustable with Standard Cover 14" Wide, 15-16 Height, Rigid	1
1o, 2c, 3c, 4, 5e	506465	Back Upholstery Assembly Tension Adjustable with Standard Cover 14" Wide, 17-20 Height, Rigid	1
1p, 2a, 3a, 4, 5f	111199	Back Upholstery Assembly Tension Adjustable with Standard Cover 15" Wide, 11-12 Height, Rigid	1
1p, 2b, 3b, 4, 5f	506466	Back Upholstery Assembly Tension Adjustable with Standard Cover 15" Wide, 13-14 Height, Rigid	1
1q, 2b, 3b, 4, 5f	506467	Back Upholstery Assembly Tension Adjustable with Standard Cover 15" Wide, 15-16 Height, Rigid	1
1r, 2c, 3c, 4, 5f	506468	Back Upholstery Assembly Tension Adjustable with Standard Cover 15" Wide, 17-20 Height, Rigid	1
1s, 2a, 3a, 4, 5g	111200	Back Upholstery Assembly Tension Adjustable with Standard Cover 16" Wide, 11-12 Height, Rigid	1
1s, 2b, 3b, 4, 5g	506469	Back Upholstery Assembly Tension Adjustable with Standard Cover 16" Wide, 13-14 Height, Rigid	1
1t, 2b, 3b, 4, 5g	506470	Back Upholstery Assembly Tension Adjustable with Standard Cover 16" Wide, 15-16 Height, Rigid	1

Item Number	Part Number	Description	Quantity
1u, 2c, 3c, 4, 5g	506471	Back Upholstery Assembly Tension Adjustable with Standard Cover 16" Wide, 17-20 Height, Rigid	1
1v, 2a, 3a, 4, 5h	111201	Back Upholstery Assembly Tension Adjustable with Standard Cover 17" Wide, 11-12 Height, Rigid	1
1v, 2b, 3b, 4, 5h	506472	Back Upholstery Assembly Tension Adjustable with Standard Cover 17" Wide, 13-14 Height, Rigid	1
1w, 2b, 3b, 4, 5h	506473	Back Upholstery Assembly Tension Adjustable with Standard Cover 17" Wide, 15-16 Height, Rigid	1
1x, 2c, 3c, 4, 5h	506474	Back Upholstery Assembly Tension Adjustable with Standard Cover 17" Wide, 17-20 Height, Rigid	1
1y, 2a, 3a, 4, 5i	111202	Back Upholstery Assembly Tension Adjustable with Standard Cover 18" Wide, 11-12 Height, Rigid	1
1y, 2b, 3b, 4, 5i	506475	Back Upholstery Assembly Tension Adjustable with Standard Cover 18" Wide, 13-14 Height, Rigid	1
1z, 2b, 3b, 4, 5i	506476	Back Upholstery Assembly Tension Adjustable with Standard Cover 18" Wide, 15-16 Height, Rigid	1
1aa, 2c, 3c, 4, 5i	506477	Back Upholstery Assembly Tension Adjustable with Standard Cover 18" Wide, 17-20 Height, Rigid	1
1ab, 2a, 3a, 4, 5j	111203	Back Upholstery Assembly Tension Adjustable with Standard Cover 19" Wide, 11-12 Height, Rigid	1
1ab, 2b, 3b, 4, 5j	506478	Back Upholstery Assembly Tension Adjustable with Standard Cover 19" Wide, 13-14 Height, Rigid	1
1ac, 2b, 3b, 4, 5j	506479	Back Upholstery Assembly Tension Adjustable with Standard Cover 19" Wide, 15-16 Height, Rigid	1
1ad, 2c, 3c, 4, 5j	506480	Back Upholstery Assembly Tension Adjustable with Standard Cover 19" Wide, 17-20 Height, Rigid	1
1ae, 2a, 3a, 4, 5k	111204	Back Upholstery Assembly Tension Adjustable with Standard Cover 20" Wide, 11-12 Height, Rigid	1
1ae, 2b, 3b, 4, 5k	506481	Back Upholstery Assembly Tension Adjustable with Standard Cover 20" Wide, 13-14 Height, Rigid	1
1af, 2b, 3b, 4, 5k	506482	Back Upholstery Assembly Tension Adjustable with Standard Cover 20" Wide, 15-16 Height, Rigid	1
1ag, 2c, 3c, 4, 5k	506483	Back Upholstery Assembly Tension Adjustable with Standard Cover 20" Wide, 17-20 Height, Rigid	1
1ai, 2b, 3b, 4, 5l	506485	Back Upholstery Assembly Tension Adjustable with Standard Cover 21" Wide, 13-16 Height, Rigid	1
1aj, 2c, 3c, 4, 5l	506486	Back Upholstery Assembly Tension Adjustable with Standard Cover 21" Wide, 17-20 Height, Rigid	1

Item Number	Part Number	Description	Quantity
1a, 2b, 3b, 4, 5m	506488	Back Upholstery Assembly Tension Adjustable with Standard Cover 22" Wide, 13-16 Height, Rigid	1
1am, 2c, 3c, 4, 5m	506489	Back Upholstery Assembly Tension Adjustable with Standard Cover 22" Wide, 17-20 Height, Rigid	1
8b, 2a, 3a, 4, 5c	111205	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 12" Wide, 11-12 Height, Rigid	1
8b, 2b, 3b, 4, 5c	107788	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 12" Wide, 13-14 Height, Rigid	1
8c, 2b, 3b, 4, 5c	107789	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 12" Wide, 15-16 Height, Rigid	1
8d, 2c, 3c, 4, 5c	107790	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 12" Wide, 17-20 Height, Rigid	1
8f, 2a, 3a, 4, 5d	111206	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 13" Wide, 11-12 Height, Rigid	1
8f, 2b, 3b, 4, 5d	107791	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 13" Wide, 13-14 Height, Rigid	1
8g, 2b, 3b, 4, 5d	107792	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 13" Wide, 15-16 Height, Rigid	1
8i, 2c, 3c, 4, 5d	107793	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 13" Wide, 17-20 Height, Rigid	1
8k, 2a, 3a, 4, 5e	111207	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 14" Wide, 11-12 Height, Rigid	1
8k, 2b, 3b, 4, 5e	107794	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 14" Wide, 13-14 Height, Rigid	1
8l, 2b, 3b, 4, 5e	107795	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 14" Wide, 15-16 Height, Rigid	1
8n, 2c, 3c, 4, 5e	107796	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 14" Wide, 17-20 Height, Rigid	1
8p, 2a, 3a, 4, 5f	111208	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 15" Wide, 11-12 Height, Rigid	1
8p, 2b, 3b, 4, 5f	107797	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 15" Wide, 13-14 Height, Rigid	1
8q, 2b, 3b, 4, 5f	107798	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 15" Wide, 15-16 Height, Rigid	1
8s, 2c, 3c, 4, 5f	107799	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 15" Wide, 17-20 Height, Rigid	1
8u, 2a, 3a, 4, 5g	111209	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 16" Wide, 11-12 Height, Rigid	1

Item Number	Part Number	Description	Quantity
8u, 2b, 3b, 4, 5g	107800	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 16" Wide, 13-14 Height, Rigid	1
8v, 2b, 3b, 4, 5g	107801	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 16" Wide, 15-16 Height, Rigid	1
8x, 2c, 3c, 4, 5g	107802	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 16" Wide, 17-20 Height, Rigid	1
8z, 2a, 3a, 4, 5h	111210	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 17" Wide, 11-12 Height, Rigid	1
8z, 2b, 3b, 4, 5h	107803	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 17" Wide, 13-14 Height, Rigid	1
8aa, 2b, 3b, 4, 5h	107804	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 17" Wide, 15-16 Height, Rigid	1
8ac, 2c, 3c, 4, 5h	107805	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 17" Wide, 17-20 Height, Rigid	1
8ae, 2a, 3a, 4, 5i	111211	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 18" Wide, 11-12 Height, Rigid	1
8ae, 2b, 3b, 4, 5i	107806	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 18" Wide, 13-14 Height, Rigid	1
8af, 2b, 3b, 4, 5i	107807	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 18" Wide, 15-16 Height, Rigid	1
8ah, 2c, 3c, 4, 5i	107808	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 18" Wide, 17-20 Height, Rigid	1
8aj, 2a, 3a, 4, 5j	111212	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 19" Wide, 11-12 Height, Rigid	1
8aj, 2b, 3b, 4, 5j	107809	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 19" Wide, 13-14 Height, Rigid	1
8ak, 2b, 3b, 4, 5j	107810	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 19" Wide, 15-16 Height, Rigid	1
8am, 2c, 3c, 4, 5j	107811	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 19" Wide, 17-20 Height, Rigid	1
8ao, 2a, 3a, 4, 5k	111213	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 20" Wide, 11-12 Height, Rigid	1
8ao, 2b, 3b, 4, 5k	107812	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 20" Wide, 13-14 Height, Rigid	1
8ap, 2b, 3b, 4, 5k	107813	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 20" Wide, 15-16 Height, Rigid	1
8ar, 2c, 3c, 4, 5k	107814	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 20" Wide, 17-20 Height, Rigid	1

Item Number	Part Number	Description	Quantity
8at, 2b, 3b, 4, 5l	107815	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 21" Wide, 13-16 Height, Rigid	1
8au, 2c, 3c, 4, 5l	107816	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 21" Wide, 17-20 Height, Rigid	1
8aw, 2b, 3b, 4, 5m	107817	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 22" Wide, 13-16 Height, Rigid	1
8ax, 2c, 3c, 4, 5m	107818	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 22" Wide, 17-20 Height, Rigid	1
1a	002486	Backrest Cover 10" Rigid x 11-14"	1
1b	002487	Backrest Cover 10" Rigid x 13-16"	1
1c	002488	Backrest Cover 10" Rigid x 15-18"	1
1d	002490	Backrest Cover 11" Rigid x 11-14"	1
1e	002491	Backrest Cover 11" Rigid x 13-16"	1
1f	002492	Backrest Cover 11" Rigid x 15-18"	1
1g	002494	Backrest Cover 12" Rigid x 11-14"	1
1h	002495	Backrest Cover 12" Rigid x 13-16"	1
1i	002496	Backrest Cover 12" Rigid x 15-18"	1
1j	002498	Backrest Cover 13" Rigid / 14" Cat x 11-14"	1
1k	002499	Backrest Cover 13" Rigid / 14" Cat x 13-16"	1
1l	003043	Backrest Cover 13" Rigid - 14" CAT/Liberty x 17-20"	1
1m	002502	Backrest Cover 14" Rigid / 15" Cat x 11-14"	1
1n	002503	Backrest Cover 14" Rigid / 15" Cat x 13-16"	1
1o	003044	Backrest Cover 14" Rigid - 15" CAT/Liberty x 17-20"	1
1p	002506	Backrest Cover 15" Rigid / 16" Cat x 11-14"	1
1q	002507	Backrest Cover 15" Rigid / 16" Cat x 13-16"	1
1r	003045	Backrest Cover 15" Rigid - 16" CAT/Liberty x 17-20"	1
1s	002510	Backrest Cover 16" Rigid / 17" Cat x 11-14"	1
1t	002511	Backrest Cover 16" Rigid / 17" Cat x 13-16"	1
1u	003046	Backrest Cover 16" Rigid - 17" CAT/Liberty x 17-20"	1
1v	003049	Backrest Cover 17" Rigid / 18" Cat x 11-14"	1

Item Number	Part Number	Description	Quantity
1w	003050	Backrest Cover 17" Rigid / 18" Cat x 13-16"	1
1x	003052	Backrest Cover 17" Rigid - 18" CAT/Liberty x 17-20"	1
1y	003055	Backrest Cover 18" Rigid / 19" Cat x 11-14"	1
1z	003056	Backrest Cover 18" Rigid / 19" Cat x 13-16"	1
1aa	003058	Backrest Cover 18" Rigid - 19" CAT/Liberty x 17-20"	1
1ab	003061	Backrest Cover 19" Rigid / 20" Cat x 11-14"	1
1ac	003062	Backrest Cover 19" Rigid / 20" Cat x 13-16"	1
1ad	003064	Backrest Cover 19" Rigid - 20" CAT/Liberty x 17-20"	1
1ae	003067	Backrest Cover 20" Rigid / 21" Cat x 11-14"	1
1af	003068	Backrest Cover 20" Rigid / 21" Cat x 13-16"	1
1ag	003070	Backrest Cover 20" Rigid - 21" CAT/Liberty x 17-20"	1
1ah	003072	Backrest Cover 21" Rigid / 22" Cat x 13-16"	1
1ai	003073	Backrest Cover 21" Rigid - 22" CAT/Liberty x 15-18"	1
1aj	003074	Backrest Cover 21" Rigid - 22" CAT/Liberty x 17-20"	1
1ak	003076	Backrest Cover 22" Rigid x 13-16"	1
1al	003077	Backrest Cover 22" Rigid x 15-18"	1
1am	003078	Backrest Cover 22" Rigid x 17-20"	1
2a	003093	Tension Adjustable Upholstery Wing 3.5" Height Left	1
2b	003095	Tension Adjustable Upholstery Wing 5.5" Height Left	1
2c	003097	Tension Adjustable Upholstery Wing 9.5" Height Left	1
3a	003092	Tension Adjustable Upholstery Wing 3.5" Height Right	1
3b	003094	Tension Adjustable Upholstery Wing 5.5" Height Right	1
3c	003096	Tension Adjustable Upholstery Wing 9.5" Height Right	1
4	100973	Back Upholstery D-Ring	2
5a	003483	Back Upholstery Strap 7.3" <i>Used on 10" chair widths</i>	3
5b	003484	Back Upholstery Strap 9.3" <i>Used on 11" chair widths</i>	3

Item Number	Part Number	Description	Quantity
5c	003485	Back Upholstery Strap 11.3" <i>Used on 12" chair widths</i>	3
5d	003486	Back Upholstery Strap 13.3" <i>Used on 13" chair widths</i>	3
5e	003487	Back Upholstery Strap 15.3" <i>Used on 14" chair widths</i>	3
5f	002532	Back Upholstery Strap 17.3" <i>Used on 15" chair width</i>	3
5g	002533	Back Upholstery Strap 19.3" <i>Used on 16" chair widths</i>	3
5h	002534	Back Upholstery Strap 21.3" <i>Used on 17" chair widths</i>	3
5i	002535	Back Upholstery Strap 23.3" <i>Used on 18" chair widths</i>	3
5j	002536	Back Upholstery Strap 25.3" <i>Used on 19" chair widths</i>	3
5k	002537	Back Upholstery Strap 27.3" <i>Used on 20" chair widths</i>	3
5l	002538	Back Upholstery Strap 29.3" <i>Used on 21" chair widths</i>	3
5m	002539	Back Upholstery Strap 31.3" <i>Used on 22" chair widths</i>	3
6a	001748	Tension Strap Folding Handle 9-11 Wide	1
6b	001749	Tension Strap Folding Handle 12-13 Wide	1
6c	001750	Tension Strap Folding Handle 14-15 Wide	1
6d	001751	Tension Strap Folding Handle 16-17 Wide	1
6e	001752	Tension Strap Folding Handle 18-19 Wide	1
6f	001753	Tension Strap Folding Handle 20-22 Wide	1
7	100734	M5 x 16 PHMS ZC	2
8a	003892	Air Mesh Backrest Cover 12R Width x 9-12" Height	1
8b	003893	Air Mesh Backrest Cover 12R Width x 11-14" Height	1
8c	003894	Air Mesh Backrest Cover 12R Width x 13-16" Height	1
8d	003895	Air Mesh Backrest Cover 12R Width x 15-18" Height	1

Item Number	Part Number	Description	Quantity
8e	003896	Air Mesh Backrest Cover 14C 13R Width x 9-12" Height	1
8f	003897	Air Mesh Backrest Cover 14C 13R Width x 11-14" Height	1
8g	003898	Air Mesh Backrest Cover 14C 13R Width x 13-16" Height	1
8h	003899	Air Mesh Backrest Cover 14C 13R Width x 15-18" Height	1
8i	003913	Air Mesh Backrest Cover 14C 13R Width x 17-20" Height	1
8j	003900	Air Mesh Backrest Cover 15C 14R Width x 9-12" Height	1
8k	003901	Air Mesh Backrest Cover 15C 14R Width x 11-14" Height	1
8l	003902	Air Mesh Backrest Cover 15C 14R Width x 13-16" Height	1
8m	003903	Air Mesh Backrest Cover 15C 14R Width x 15-18" Height	1
8n	003914	Air Mesh Backrest Cover 15C 14R Width x 17-20" Height	1
8o	003904	Air Mesh Backrest Cover 16C 15R Width x 9-12" Height	1
8p	003905	Air Mesh Backrest Cover 16C 15R Width x 11-14" Height	1
8q	003906	Air Mesh Backrest Cover 16C 15R Width x 13-16" Height	1
8r	003907	Air Mesh Backrest Cover 16C 15R Width x 15-18" Height	1
8s	003915	Air Mesh Backrest Cover 16C 15R Width x 17-20" Height	1
8t	003909	Air Mesh Backrest Cover 17C 16R Width x 9-12" Height	1
8u	003910	Air Mesh Backrest Cover 17C 16R Width x 11-14" Height	1
8v	003911	Air Mesh Backrest Cover 17C 16R Width x 13-16" Height	1
8w	003912	Air Mesh Backrest Cover 17C 16R Width x 15-18" Height	1
8x	003916	Air Mesh Backrest Cover 17C 16R Width x 17-20" Height	1
8y	003918	Air Mesh Backrest Cover 18C 17R Width x 9-12" Height	1
8z	003919	Air Mesh Backrest Cover 18C 17R Width x 11-14" Height	1
8aa	003920	Air Mesh Backrest Cover 18C 17R Width x 13-16" Height	1
8ab	003921	Air Mesh Backrest Cover 18C 17R Width x 15-18" Height	1
8ac	003922	Air Mesh Backrest Cover 18C 17R Width x 17-20" Height	1
8ad	003924	Air Mesh Backrest Cover 19C 18R Width x 9-12" Height	1
8ae	003925	Air Mesh Backrest Cover 19C 18R Width x 11-14" Height	1

Item Number	Part Number	Description	Quantity
8af	003926	Air Mesh Backrest Cover 19C 18R Width x 13-16" Height	1
8ag	003927	Air Mesh Backrest Cover 19C 18R Width x 15-18" Height	1
8ah	003928	Air Mesh Backrest Cover 19C 18R Width x 17-20" Height	1
8ai	003930	Air Mesh Backrest Cover 20C 19R Wide x 9-12" Height	1
8aj	003931	Air Mesh Backrest Cover 20C 19R Width x 11-14" Height	1
8ak	003932	Air Mesh Backrest Cover 20C 19R Width x 13-16" Height	1
8al	003933	Air Mesh Backrest Cover 20C 19R Width x 15-18" Height	1
8am	003934	Air Mesh Backrest Cover 20C 19R Width x 17-20" Height	1
8an	003936	Air Mesh Backrest Cover 21C 20R Width x 9-12" Height	1
8ao	003937	Air Mesh Backrest Cover 21C 20R Width x 11-14" Height	1
8ap	003938	Air Mesh Backrest Cover 21C 20R Width x 13-16" Height	1
8aq	003939	Air Mesh Backrest Cover 21C 20R Width x 15-18" Height	1
8ar	003940	Air Mesh Backrest Cover 21C 20R Width x 17-20" Height	1
8as	003942	Air Mesh Backrest Cover 22C 21R Width x 13-16" Height	1
8at	003943	Air Mesh Backrest Cover 22C 21R Width x 15-18" Height	1
8au	003944	Air Mesh Backrest Cover 22C 21R Width x 17-20" Height	1
8av	003946	Air Mesh Backrest Cover 22R Width x 13-16" Height	1
8aw	003947	Air Mesh Backrest Cover 22R Width x 15-18" Height	1
8ax	003948	Air Mesh Backrest Cover 22R Width x 17-20" Height	1