


Liberty Pull to Lock Wheel Locks

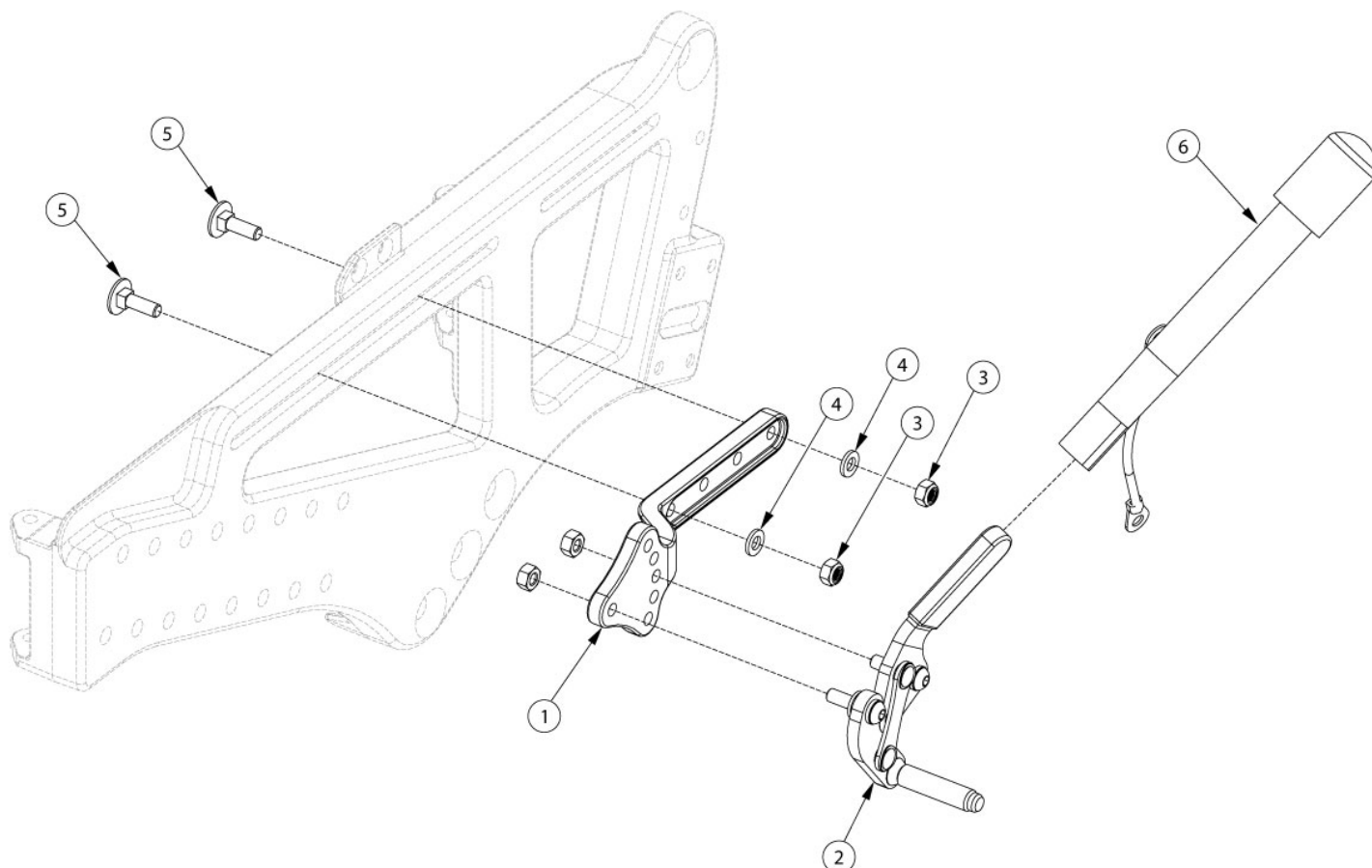

Item Number	Part Number	Description	Quantity
1, 2a, 3-5	506183	Wheel Lock Assembly, Pull Right	1
1, 2b, 3-5	506184	Wheel Lock Assembly, Pull Left	1
1, 2a, 2b, 3 - 5	124915	Wheel Lock Assembly, Pull PAIR	1
1, 2c, 3-6	506185	Wheel Lock Assembly, Pull with Extension Right	1
1, 2d, 3-6	506186	Wheel Lock Assembly, Pull with Extension Left	1
1, 2d, 2c, 3-6	124910	Wheel Lock Assembly, Pull with Extension PAIR	1
1	003329	Mount, Wheel Lock	1
2a	003676	Wheel Lock, Pull, R	1
2b	003677	Wheel Lock, Pull, L	1
2c, 6	003678	Wheel Lock Pull with Extension Right	1
2d, 6	003679	Wheel Lock Pull with Extension Left	1

Item Number	Part Number	Description	Quantity
3	100658	M6 Nylock Nut Blk Zn	2
4	100746	M6 Flat Washer Blk Zn	2
5	103032	M6 x 20 Carriage Bolt, Blk Zn	2
6	200369	Wheel Lock Extension	1