


Catalyst Half Folding Backrest

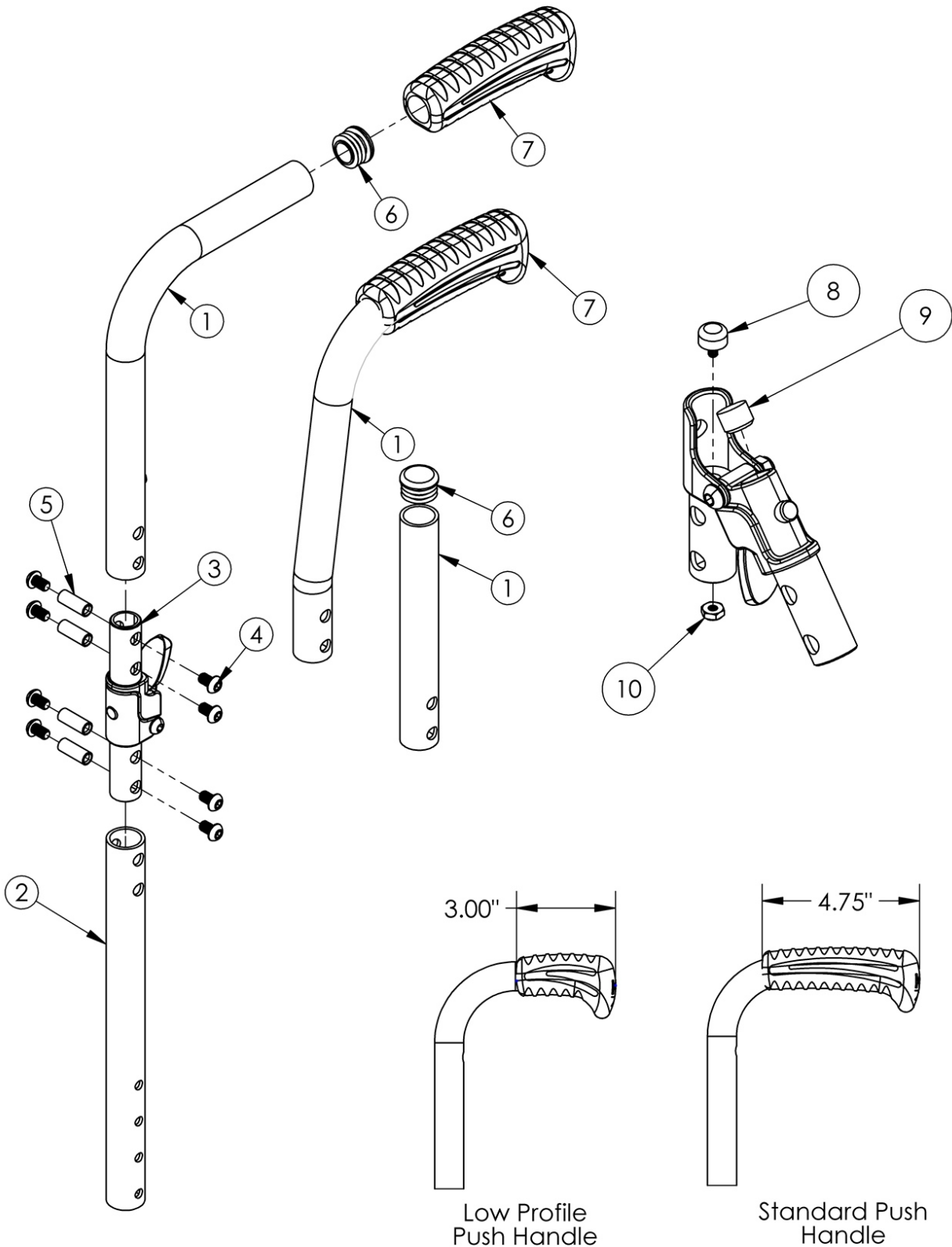

Not available with Transit option, Heavy Duty option, Flip Back Armrest or Angle Adjustable Locking Extendable Flip Up Armrest <p>
 Low Profile Push Handles are not available with Drum Brake <p>

If changing from another style backrest and the Drum Brake is present, a new Drum Brake Cable Kit may be required

Item Number	Part Number	Description	Quantity	Retail
3a, 4, 5, 8-10	504599	Half Folding Backrest Hinge Latch Assembly Left	1	\$80.44
3b, 4, 5, 8-10	504600	Half Folding Backrest Hinge Latch Assembly Right	1	\$80.44
3a, 3b, 4, 5, 8-10	504601	Half Folding Backrest Hinge Latch Assembly Pair	1	\$160.86
1a, 2, 3a, 4-6, 7a, 8-10	504602	Half Folding Back Post Assembly Short Straight w/ Standard Push Handle Left	1	\$122.29
1a, 2, 3b, 4-6, 7a, 8-10	504603	Half Folding Back Post Assembly Short Straight w/ Standard Push Handle Right	1	\$122.29
1a, 2, 3a, 3b, 4-6, 7a, 8-10	504604	Half Folding Back Post Assembly Short Straight w/ Standard Push Handle Pair	1	\$256.23
1b, 2, 3a, 4-6, 7a, 8-10	504605	Half Folding Back Post Assembly Tall Straight w/ Standard Push Handle Left	1	\$122.29
1b, 2, 3b, 4-6, 7a, 8-10	504606	Half Folding Back Post Assembly Tall Straight w/ Standard Push Handle Right	1	\$122.29
1b, 2, 3a, 3b, 4-6, 7a, 8-10	504607	Half Folding Back Post Assembly Tall Straight w/ Standard Push Handle Pair	1	\$256.23
1c, 2, 3a, 4-6, 7b, 8-10	504608	Half Folding Back Post Assembly Short Straight w/ Low Profile Push Handle Left	1	\$122.29
1c, 2, 3b, 4-6, 7b, 8-10	504609	Half Folding Back Post Assembly Short Straight w/ Low Profile Push Handle Right	1	\$122.29
1c, 2, 3a, 3b, 4-6, 7b, 8-10	504610	Half Folding Back Post Assembly Short Straight w/ Low Profile Push Handle Pair	1	\$256.23
1d, 2, 3a, 4-6, 7b, 8-10	504611	Half Folding Back Post Assembly Tall Straight w/ Low Profile Push Handle Left	1	\$122.29
1d, 2, 3b, 4-6, 7b, 8-10	504612	Half Folding Back Post Assembly Tall Straight w/ Low Profile Push Handle Right	1	\$122.29
1d, 2, 3a, 3b, 4-6, 7b, 8-10	504613	Half Folding Back Post Assembly Tall Straight w/ Low Profile Push Handle Pair	1	\$256.23
1e, 2, 3a, 4-6, 8-10	504614	Half Folding Back Post Assembly Short Straight Left	1	\$114.57
1e, 2, 3b, 4-6, 8-10	504615	Half Folding Back Post Assembly Short Straight Right	1	\$114.57
1e, 2, 3a, 3b, 4-6, 8-10	504616	Half Folding Back Post Assembly Short Straight Pair	1	\$240.05
1f, 2, 3a, 4-6, 8-10	504617	Half Folding Back Post Assembly Tall Straight Left	1	\$114.57
1f, 2, 3b, 4-6, 8-10	504618	Half Folding Back Post Assembly Tall Straight Right	1	\$114.57
1f, 2, 3a, 3b, 4-6, 8-10	504619	Half Folding Back Post Assembly Tall Straight Pair	1	\$240.05
1g, 2, 3a, 4-6, 7a, 8-10	504620	Half Folding Back Post Assembly Short 8 Degree Bend with Standard Push Handle Left	1	\$131.34
1g, 2, 3b, 4-6, 7a, 8-10	504621	Half Folding Back Post Assembly Short 8 Degree Bend with Standard Push Handle Right	1	\$131.34
1g, 2, 3a, 3b, 4-6, 7a, 8-10	504622	Half Folding Back Post Assembly Short 8 Degree Bend with Standard Push Handle Pair	1	\$275.19

Item Number	Part Number	Description	Quantity	Retail
1h, 2, 3a, 4-6, 7a, 8-10	504623	Half Folding Back Post Assembly Tall 8 Degree Bend with Standard Push Handle Left	1	\$131.34
1h, 2, 3b, 4-6, 7a, 8-10	504624	Half Folding Back Post Assembly Tall 8 Degree Bend with Standard Push Handle Right	1	\$131.34
1h, 2, 3a, 3b, 4-6, 7a, 8-10	504625	Half Folding Back Post Assembly Tall 8 Degree Bend with Standard Push Handle Pair	1	\$275.19
1i, 2, 3a, 4-6, 7b, 8-10	504626	Half Folding Back Post Assembly Short 8 Degree Bend with Low Profile Push Handle Left	1	\$131.34
1i, 2, 3b, 4-6, 7b, 8-10	504627	Half Folding Back Post Assembly Short 8 Degree Bend with Low Profile Push Handle Right	1	\$131.34
1i, 2, 3a, 3b, 4-6, 7b, 8-10	504628	Half Folding Back Post Assembly Short 8 Degree Bend with Low Profile Push Handle Pair	1	\$275.19
1j, 2, 3a, 4-6, 7b, 8-10	504629	Half Folding Back Post Assembly Tall 8 Degree Bend with Low Profile Push Handle Left	1	\$131.34
1j, 2, 3b, 4-6, 7b, 8-10	504630	Half Folding Back Post Assembly Tall 8 Degree Bend with Low Profile Push Handle Right	1	\$131.34
1j, 2, 3a, 3b, 4-6, 7b, 8-10	504631	Half Folding Back Post Assembly Tall 8 Degree Bend with Low Profile Push Handle Pair	1	\$275.19
4	101184	M6 x 10 BTN SHCS BLKZC w/Patch	8	\$1.58
5	000669	M6 x 20.5 Threaded Barrel	4	\$4.88
6	100803	Tube End Plug 7/8"	1	\$1.58
7a	109433	Ergo Push Handle Grip 7/8" Standard with Instruction	1	\$7.14
7b	109431	Ergo Push Handle Grip 7/8" Low Profile with Instruction	1	\$7.14