


Arc Dual Hand Tilt Mechanism

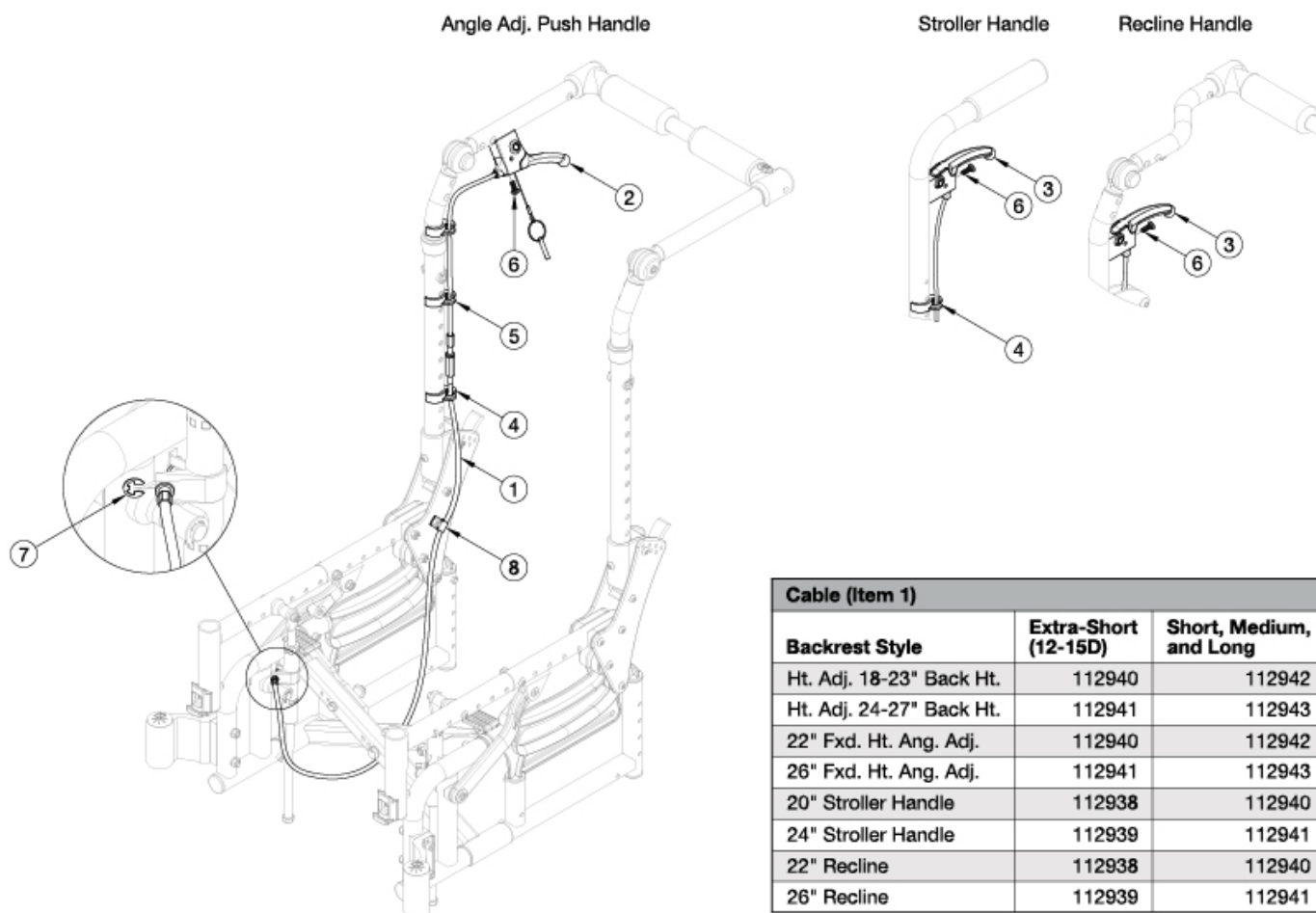

Please note cable kits 112938 - 112943 contain two cables and two e-rings.

Item Number	Part Number	Description	Quantity
1c, 2, 4-8	112988	Complete Dual Hand Tilt, Height Adjustable, 18-23" Height, Extra Short Depth Range, Arc	1
1e, 2, 4-8	112989	Complete Dual Hand Tilt, Height Adjustable, 18-23" Height, Short/Med/Long Depth Range, Arc	1
1d, 2, 4-8	112990	Dual Hand Tilt, Height Adjustable, 24-27" Height, Extra Short Depth Range, Arc	1
1f, 2, 4-8	112991	Dual Hand Tilt, Height Adjustable, 24-27" Height, Short/Med/Long Depth Range, Arc	1
1c, 3, 4, 6-8	112992	Dual Hand Tilt, Fixed Height, 22" Height, Extra Short Depth Range, Arc	1
1e, 3, 4, 6-8	112993	Dual Hand Tilt, Fixed Height, 22" Height, Short/Med/Long Depth Range, Arc	1
1d, 3, 4, 6-8	112994	Dual Hand Tilt, Fixed Height, 26" Height, Extra Short Depth Range, Arc	1
1f, 3, 4, 6-8	112995	Dual Hand Tilt, Fixed Height, 26" Height, Short/Med/Long Depth Range, Arc	1

Item Number	Part Number	Description	Quantity
1a, 3, 4, 6-8	112996	Dual Hand Tilt, Stroller Handle, 20" Height, Extra Short Depth Range, Arc	1
1c, 3, 4, 6-8	112997	Dual Hand Tilt, Stroller Handle, 20" Height, Short/Med/Long Depth Range, Arc	1
1b, 3, 4, 6-8	112998	Dual Hand Tilt, Stroller Handle, 24" Height, Extra Short Depth Range, Arc	1
1d, 3, 4, 6-8	112999	Dual Hand Tilt, Stroller Handle, 24" Height, Short/Med/Long Depth Range, Arc	1
1a, 3, 4, 6-8	113000	Dual Hand Tilt, Reclining, 22" Height, Extra Short Depth Range, Arc	1
1c, 3, 4, 6-8	113001	Dual Hand Tilt, Reclining, 22" Height, Short/Med/Long Depth Range, Arc	1
1b, 3, 4, 6-8	113002	Dual Hand Tilt, Reclining, 26" Height, Extra Short Depth Range, Arc	1
1d, 3, 4, 6-8	113003	Dual Hand Tilt, Reclining, 26" Height, Short/Med/Long Depth Range, Arc	1
1a, 7	112938	Cable, Tilt, Dual, Short, Arc	1
1b, 7	112939	Cable, Tilt, Dual, Medium, Arc	1
1c, 7	112940	Cable, Tilt, Dual, Long, Arc	1
1d, 7	112941	Cable, Tilt, Dual, XL, Arc	1
1e, 7	112942	Cable, Tilt, Dual, XXL, Arc	1
1f, 7	112943	Cable, Tilt, Dual, XXXL, Arc	1
2	002801	Lever Assembly Horizontal Tilt <i>Used with Height Adjustable Backrests</i>	2
3	002800	Lever Assembly Vertical Tilt <i>Used with Stroller Handle and Reclining Backrests</i>	2
4	001117	Cable Clip 1"	2
5	001118	Cable Clip 7/8" <i>Used with Height Adjustable Backrests</i>	2
6	100734	M5 x 16 PHMS ZC	2
7	102822	E-Ring, M6, YZ	2
8	001239	P-Clip 3/16"	2