


Catalyst / Spark Seat Pan (Non Transit)

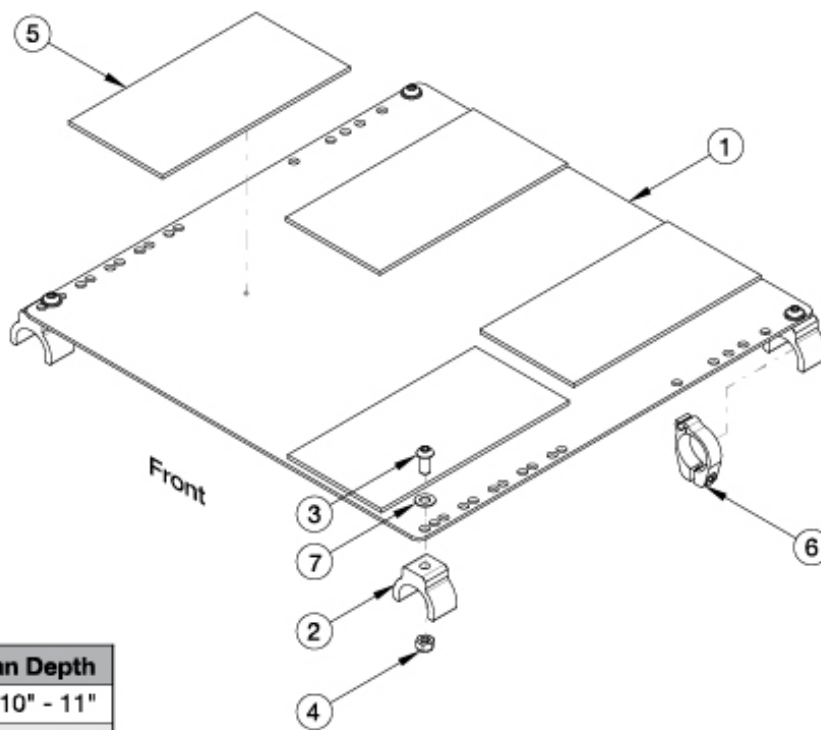

Part Number	Tape Size	Seat Pan Depth
004458	3" x 4"	10" - 11"
004458 & 004459	3" x 4" & 3" x 5"	12"
004459	3" x 5"	13"
004460	3" x 6"	14" - 22"

The seat pan width and depth measure approximately 0.5" less than the chair size. For a complete seat pan assembly, select the seat pan size needed and hardware kit.

Item Number	Part Number	Description	Quantity
2, 3a, 4, 6	115613	Spark / Catalyst Aluminum Seat Pan Hardware, 10"-14" Wide <i>Use with 10"-14" Wide chairs</i>	1
2, 3b, 4, 6, 7	124270	Spark / Catalyst Aluminum Seat Pan Hardware, 15"-18" Wide <i>Use with 15"-18" Wide chairs</i>	1
2, 3b, 4, 6	124271	Spark / Catalyst Aluminum Seat Pan Hardware, 19"-22" Wide <i>Use with 19"-22" Wide chairs</i>	1
1a, 5a	506009	Seat Pan Catalyst / Spark with AB Tape 10 x 10	1
1b, 5a	506010	Seat Pan Catalyst / Spark with AB Tape 10 x 11	1
1c, 5a, 5b	506011	Seat Pan Catalyst / Spark with AB Tape 10 x 12	1
1d, 5b	506012	Seat Pan Catalyst / Spark with AB Tape 10 x 13	1
1e, 5c	506013	Seat Pan Catalyst / Spark with AB Tape 10 x 14	1

Item Number	Part Number	Description	Quantity
1f, 5c	506014	Seat Pan Catalyst / Spark with AB Tape 10 x 15	1
1g, 5c	506015	Seat Pan Catalyst / Spark with AB Tape 10 x 16	1
1h, 5c	506016	Seat Pan Catalyst / Spark with AB Tape 10 x 17	1
1i, 5c	506017	Seat Pan Catalyst / Spark with AB Tape 10 x 18	1
1j, 5a	506018	Seat Pan Catalyst / Spark with AB Tape 11 x 10	1
1k, 5a	506019	Seat Pan Catalyst / Spark with AB Tape 11 x 11	1
1l, 5a, 5b	506020	Seat Pan Catalyst / Spark with AB Tape 11 x 12	1
1m, 5b	506021	Seat Pan Catalyst / Spark with AB Tape 11 x 13	1
1n, 5c	506022	Seat Pan Catalyst / Spark with AB Tape 11 x 14	1
1o, 5c	506023	Seat Pan Catalyst / Spark with AB Tape 11 x 15	1
1p, 5c	506024	Seat Pan Catalyst / Spark with AB Tape 11 x 16	1
1q, 5c	506025	Seat Pan Catalyst / Spark with AB Tape 11 x 17	1
1r, 5c	506026	Seat Pan Catalyst / Spark with AB Tape 11 x 18	1
1s, 5a	506027	Seat Pan Catalyst / Spark with AB Tape 12 x 10	1
1t, 5a	506028	Seat Pan Catalyst / Spark with AB Tape 12 x 11	1
1u, 5a, 5b	506029	Seat Pan Catalyst / Spark with AB Tape 12 x 12	1
1v, 5b	506030	Seat Pan Catalyst / Spark with AB Tape 12 x 13	1
1w, 5c	506031	Seat Pan Catalyst / Spark with AB Tape 12 x 14	1
1x, 5c	506032	Seat Pan Catalyst / Spark with AB Tape 12 x 15	1
1y, 5c	506033	Seat Pan Catalyst / Spark with AB Tape 12 x 16	1
1z, 5c	506034	Seat Pan Catalyst / Spark with AB Tape 12 x 17	1
1aa, 5c	506035	Seat Pan Catalyst / Spark with AB Tape 12 x 18	1
1ab, 5a	506036	Seat Pan Catalyst / Spark with AB Tape 13 x 10	1
1ac, 5a	506037	Seat Pan Catalyst / Spark with AB Tape 13 x 11	1
1ad, 5a, 5b	506038	Seat Pan Catalyst / Spark with AB Tape 13 x 12	1
1ae, 5c	506039	Seat Pan Catalyst / Spark with AB Tape 13 x 13	1
1af, 5c	506040	Seat Pan Catalyst / Spark with AB Tape 13 x 14	1
1ag, 5c	506041	Seat Pan Catalyst / Spark with AB Tape 13 x 15	1

Item Number	Part Number	Description	Quantity
1ah, 5c	506042	Seat Pan Catalyst / Spark with AB Tape 13 x 16	1
1ai, 5c	506043	Seat Pan Catalyst / Spark with AB Tape 13 x 17	1
1aj, 5c	506044	Seat Pan Catalyst / Spark with AB Tape 13 x 18	1
1ak, 5a	506045	Seat Pan Catalyst / Spark with AB Tape 14 x 10	1
1al, 5a	506046	Seat Pan Catalyst / Spark with AB Tape 14 x 11	1
1am, 5a, 5b	506047	Seat Pan Catalyst / Spark with AB Tape 14 x 12	1
1an, 5b	506048	Seat Pan Catalyst / Spark with AB Tape 14 x 13	1
1ao, 5c	505780	Seat Pan Catalyst / Spark with AB Tape 14 x 14	1
1ap, 5c	505781	Seat Pan Catalyst / Spark with AB Tape 14 x 15	1
1aq, 5c	505782	Seat Pan Catalyst / Spark with AB Tape 14 x 16	1
1ar, 5c	505783	Seat Pan Catalyst / Spark with AB Tape 14 x 17	1
1as, 5c	505784	Seat Pan Catalyst / Spark with AB Tape 14 x 18	1
1at, 5c	505785	Seat Pan Catalyst / Spark with AB Tape 14 x 19	1
1au, 5c	505786	Seat Pan Catalyst / Spark with AB Tape 14 x 20	1
1av, 5a	506049	Seat Pan Catalyst / Spark with AB Tape 15 x 10	1
1aw, 5a	506050	Seat Pan Catalyst / Spark with AB Tape 15 x 11	1
1ax, 5a, 5b	506051	Seat Pan Catalyst / Spark with AB Tape 15 x 12	1
1ay, 5b	506052	Seat Pan Catalyst / Spark with AB Tape 15 x 13	1
1az, 5c	505787	Seat Pan Catalyst / Spark with AB Tape 15 x 14	1
1ba, 5c	505788	Seat Pan Catalyst / Spark with AB Tape 15 x 15	1
1bb, 5c	505789	Seat Pan Catalyst / Spark with AB Tape 15 x 16	1
1bc, 5c	505790	Seat Pan Catalyst / Spark with AB Tape 15 x 17	1
1bd, 5c	505791	Seat Pan Catalyst / Spark with AB Tape 15 x 18	1
1be, 5c	505792	Seat Pan Catalyst / Spark with AB Tape 15 x 19	1
1bf, 5c	505793	Seat Pan Catalyst / Spark with AB Tape 15 x 20	1
1bg, 5a	506053	Seat Pan Catalyst / Spark with AB Tape 16 x 10	1
1bh, 5a	506054	Seat Pan Catalyst / Spark with AB Tape 16 x 11	1
1bi, 5a, 5b	506055	Seat Pan Catalyst / Spark with AB Tape 16 x 12	1

Item Number	Part Number	Description	Quantity
1bj, 5b	506056	Seat Pan Catalyst / Spark with AB Tape 16 x 13	1
1bk, 5c	505794	Seat Pan Catalyst / Spark with AB Tape 16 x 14	1
1bl, 5c	505795	Seat Pan Catalyst / Spark with AB Tape 16 x 15	1
1bm, 5c	505796	Seat Pan Catalyst / Spark with AB Tape 16 x 16	1
1bn, 5c	505797	Seat Pan Catalyst / Spark with AB Tape 16 x 17	1
1bo, 5c	505798	Seat Pan Catalyst / Spark with AB Tape 16 x 18	1
1bp, 5c	505799	Seat Pan Catalyst / Spark with AB Tape 16 x 19	1
1bq, 5c	505800	Seat Pan Catalyst / Spark with AB Tape 16 x 20	1
1br, 5c	505801	Seat Pan Catalyst / Spark with AB Tape 16 x 21	1
1bs, 5c	505802	Seat Pan Catalyst / Spark with AB Tape 16 x 22	1
1bt, 5c	505803	Seat Pan Catalyst / Spark with AB Tape 17 x 14	1
1bu, 5c	505804	Seat Pan Catalyst / Spark with AB Tape 17 x 15	1
1bv, 5c	505805	Seat Pan Catalyst / Spark with AB Tape 17 x 16	1
1bw, 5c	505806	Seat Pan Catalyst / Spark with AB Tape 17 x 17	1
1bx, 5c	505807	Seat Pan Catalyst / Spark with AB Tape 17 x 18	1
1by, 5c	505808	Seat Pan Catalyst / Spark with AB Tape 17 x 19	1
1bz, 5c	505809	Seat Pan Catalyst / Spark with AB Tape 17 x 20	1
1ca, 5c	505810	Seat Pan Catalyst / Spark with AB Tape 17 x 21	1
1cb, 5c	505811	Seat Pan Catalyst / Spark with AB Tape 17 x 22	1
1cc, 5c	505812	Seat Pan Catalyst / Spark with AB Tape 18 x 14	1
1cd, 5c	505813	Seat Pan Catalyst / Spark with AB Tape 18 x 15	1
1ce, 5c	505814	Seat Pan Catalyst / Spark with AB Tape 18 x 16	1
1cf, 5c	505815	Seat Pan Catalyst / Spark with AB Tape 18 x 17	1
1cg, 5c	505816	Seat Pan Catalyst / Spark with AB Tape 18 x 18	1
1ch, 5c	505817	Seat Pan Catalyst / Spark with AB Tape 18 x 19	1
1ci, 5c	505818	Seat Pan Catalyst / Spark with AB Tape 18 x 20	1
1cj, 5c	505819	Seat Pan Catalyst / Spark with AB Tape 18 x 21	1
1ck, 5c	505820	Seat Pan Catalyst / Spark with AB Tape 18 x 22	1

Item Number	Part Number	Description	Quantity
1cl, 5c	505821	Seat Pan Catalyst / Spark with AB Tape 19 x 14	1
1cm, 5c	505822	Seat Pan Catalyst / Spark with AB Tape 19 x 15	1
1cn, 5c	505823	Seat Pan Catalyst / Spark with AB Tape 19 x 16	1
1co, 5c	505824	Seat Pan Catalyst / Spark with AB Tape 19 x 17	1
1cp, 5c	505825	Seat Pan Catalyst / Spark with AB Tape 19 x 18	1
1cq, 5c	505826	Seat Pan Catalyst / Spark with AB Tape 19 x 19	1
1cr, 5c	505827	Seat Pan Catalyst / Spark with AB Tape 19 x 20	1
1cs, 5c	505828	Seat Pan Catalyst / Spark with AB Tape 19 x 21	1
1ct, 5c	505829	Seat Pan Catalyst / Spark with AB Tape 19 x 22	1
1cu, 5c	505830	Seat Pan Catalyst / Spark with AB Tape 20 x 14	1
1cv, 5c	505831	Seat Pan Catalyst / Spark with AB Tape 20 x 15	1
1cw, 5c	505832	Seat Pan Catalyst / Spark with AB Tape 20 x 16	1
1cx, 5c	505833	Seat Pan Catalyst / Spark with AB Tape 20 x 17	1
1cy, 5c	505834	Seat Pan Catalyst / Spark with AB Tape 20 x 18	1
1cz, 5c	505835	Seat Pan Catalyst / Spark with AB Tape 20 x 19	1
1da, 5c	505836	Seat Pan Catalyst / Spark with AB Tape 20 x 20	1
1db, 5c	505837	Seat Pan Catalyst / Spark with AB Tape 20 x 21	1
1dc, 5c	505838	Seat Pan Catalyst / Spark with AB Tape 20 x 22	1
1dd, 5c	505839	Seat Pan Catalyst / Spark with AB Tape 21 x 14	1
1de, 5c	505840	Seat Pan Catalyst / Spark with AB Tape 21 x 15	1
1df, 5c	505841	Seat Pan Catalyst / Spark with AB Tape 21 x 16	1
1dg, 5c	505842	Seat Pan Catalyst / Spark with AB Tape 21 x 17	1
1dh, 5c	505843	Seat Pan Catalyst / Spark with AB Tape 21 x 18	1
1di, 5c	505844	Seat Pan Catalyst / Spark with AB Tape 21 x 19	1
1dj, 5c	505845	Seat Pan Catalyst / Spark with AB Tape 21 x 20	1
1dk, 5c	505846	Seat Pan Catalyst / Spark with AB Tape 21 x 21	1
1dl, 5c	505847	Seat Pan Catalyst / Spark with AB Tape 21 x 22	1
1dm, 5c	505848	Seat Pan Catalyst / Spark with AB Tape 22 x 14	1

Item Number	Part Number	Description	Quantity
1dn, 5c	505849	Seat Pan Catalyst / Spark with AB Tape 22 x 15	1
1do, 5c	505850	Seat Pan Catalyst / Spark with AB Tape 22 x 16	1
1dp, 5c	505851	Seat Pan Catalyst / Spark with AB Tape 22 x 17	1
1dq, 5c	505852	Seat Pan Catalyst / Spark with AB Tape 22 x 18	1
1dr, 5c	505853	Seat Pan Catalyst / Spark with AB Tape 22 x 19	1
1ds, 5c	505854	Seat Pan Catalyst / Spark with AB Tape 22 x 20	1
1dt, 5c	505855	Seat Pan Catalyst / Spark with AB Tape 22 x 21	1
1du, 5c	505856	Seat Pan Catalyst / Spark with AB Tape 22 x 22	1
2	000990	Seat Pan Clip CAT	4
3a	100667	M6 x 12 BTN SHCS BLKZC w/Patch	4
3b	101708	M6 x 14 BTN SHCS BLKZC w/Patch	4
4	100658	M6 Nylock Nut Blk Zn	4
5a	004458	AB Tape, 3" x 4"	4
5b	004459	AB Tape, 3" x 5"	4
5c	004460	AB Tape, 3" x 6"	4
6	102856	Seat Pan Stop Collar	2
7	102803	Washer Flat .250 X .500 X .027 Hx Blz	4